


Assembly Instructions for PROSCOPE™ Teaching Stethoscopes  
(Models #671 and #661)

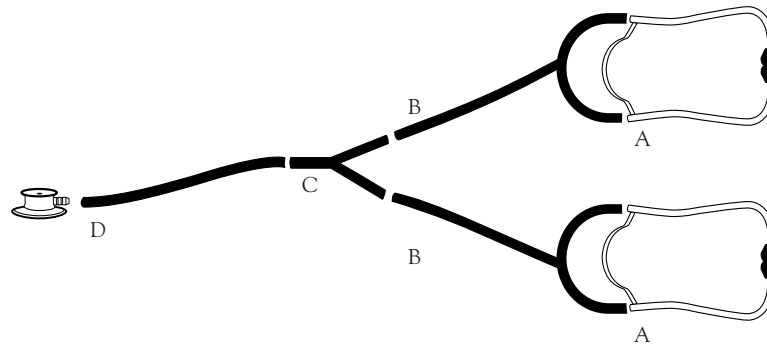

**TO ASSEMBLE**

- A. Attach "Y" tubing (#660-05BK) to each set of the binaurals (660-05) as shown in the illustration.
  - B. Attach free end of each "Y" tubing to the double pair end of the "Y" connector (#611-20).
  - C. Connect one end of the straight tubing (#671-05TBK) to single end of the "Y" connector.
  - D. Connect remaining end of the straight tubing to stem of chestpiece (#660-01 or #670-01).
- Make sure all tubing connections are firm.

## PROSCOPE™ Teaching Stethoscopes

(Models #671 and #661)

### Replacement Parts Included

Part Number	Description	Qty.
660-01	PROSCOPE™ Nurse Chestpiece	1*
670-01	PROSCOPE™ Dual Head Chestpiece	1*
611-20	"Y" Connector	1
671-05TBK	PVC Tubing, Straight, 10", Black	1
660-05TBK	PVC "Y" tubing, 22", Black	2
660-05	Binaurals	2

**\*NOTE:** PROSCOPE™ Model #671 includes the 670-01 chestpiece only  
PROSCOPE™ Model #661 includes the 660-01 chestpiece only



 ADC  
55 Commerce Drive  
Hauppauge, NY 11788

tel: 631-273-9600  
1-800-232-2670  
fax: 631-273-9659

 ADC (UK) Ltd.  
Unit 6, PO14 1TH  
United Kingdom

[www.adctoday.com](http://www.adctoday.com)  
email: [info@adctoday.com](mailto:info@adctoday.com)



 Attention, see  
instructions for use.

IB p/n 9345-00 - rev 3

Printed in the U.S.A.