

Lif-O-Gen®

Emergency Portable Oxygen Kits

Refillable or Disposable: The Choice is Yours.

Lif-O-Gen emergency oxygen kits from Allied Healthcare provide a simple, safe and effective means of delivering life-saving oxygen in the critical moments between a medical incident and the arrival of medical professionals.

Lif-O-Gen emergency oxygen kits are designed to be installed in office spaces, factories, malls, at swimming pools, or in the home ... any place sudden respiratory distress may occur. These kits can be deployed in seconds by a co-worker, family member, or even by the victim of the emergency.



Lif-O-Gen Automated Wall Mount Emergency Oxygen

- Completely automated deployment
- Refillable oxygen cylinder
- Up to 45 minutes of continuous treatment
- Integrated flow indicator



Lif-O-Gen Disposable Portable Emergency Oxygen Kit

- Convenient disposable oxygen cylinders
- Lightweight and portable
- Up to 30 minutes of treatment



Lif-O-Gen[®] Automated

Wall Mount Emergency Oxygen Kit with Refillable Cylinder

When you can't breathe, nothing else matters. In those critical seconds it is crucial to have the simplest and quickest method of oxygen deployment possible.

When pulled from its cradle, the Lif-O-Gen automated system will immediately begin to deliver 6 lpm oxygen USP through the integrated mask and tubing. Oxygen will continue to be delivered for up to 45 minutes. Simply place the unit back in the cradle to stop the flow of oxygen, or alternatively, insert the attached regulator pin.

The unit can be re-used until depleted with only a replacement of the mask train. The tank can be re-used or refilled when depleted.



Integrated mask storage compartment

- Operation** Simply Pull the unit from the wall and oxygen will automatically begin flowing
- Flow Rate** 6 liters per minute
- Capacity** 45 minutes from one refillable cylinder
- Mounting** Sturdy aluminum wall mount cradle secures delivery system and mask assembly



31-01-0600

Contents

- 31-01-0600 45-minute Refillable Emergency Oxygen Kit:** one refillable oxygen cylinder, oxygen regulator, mask & tubing, wall mount cradle; 45 minutes of medical oxygen USP
- 31-01-0610 Spare Oxygen Delivery Assembly,** without cradle, for immediate replacement of a depleted unit
- 31-01-0100 Replacement Mask and tubing assembly**

Lif-O-Gen® Disposable

Portable Emergency Oxygen Kit with Disposable Cylinders

The Lif-O-Gen disposable oxygen kits provide the same immediate access to life saving oxygen, but feature disposable cylinders that eliminate the need for refilling tanks. When a cylinder is depleted, it can simply be disposed of in regular waste per your local guidelines and replaced with a new cylinder.

The system comes in two configurations, the 15 minute single cylinder kit, and the 30-minute twin pac, which contains two 15-minute disposable oxygen cylinders in a high-impact resistant case.



Operation	15 minute kit: Thread the regulator onto the cylinder to initiate the flow of oxygen 30 minute kit: Thread the regulator onto the cylinder and press the metal button to initiate the flow of oxygen
Flow Rate	6 liters per minute
Capacity	15 minutes or 30 minutes, depending on model
Mounting	Plastic case with handle and wall hook

Contents

- 31-01-0500 15-minute Disposable Emergency Oxygen Kit:** one disposable oxygen cylinder, oxygen regulator, mask & tubing, 15 minutes of medical oxygen USP
- 31-01-0555 30-minute Disposable Emergency Oxygen Kit:** two disposable oxygen cylinders, oxygen regulator, mask & tubing, carry case, 30 minutes of medical oxygen USP
- 31-01-0510 Replacement 15 minute Oxygen Cylinder** for 31-01-0500 and 31-01-0555

Refilling is Easy, With Lif-O-Gen®

Regardless of whether you prefer the disposable or refillable system, Lif-O-Gen has taken the hassle out of the refilling process.

With the Lif-O-Gen Automated System, getting the tank filled is as easy as calling the number on the inside panel of the kit (and listed below). Our technical service department will provide shipping details, and once it arrives at Allied the refill will be shipped back to you within 7 days, all for a standard price.

Lif-O-Gen Disposable Refills are even easier. Once a tank is used, just dispose of it per your local guidelines and purchase a replacement tank (31-01-0510) from your Lif-O-Gen distributor.

KIT ORDERING INFORMATION

31-01-0600 45-minute Refillable Emergency Oxygen Kit: one refillable oxygen cylinder, oxygen regulator, mask & tubing, wall mount cradle, 45 minutes of medical oxygen USP

31-01-0555 30-minute Disposable Emergency Oxygen Kit: two disposable oxygen cylinders, oxygen regulator, mask & tubing, carry case, 30 minutes of medical oxygen USP

31-01-0500 15-minute Disposable Emergency Oxygen Kit: one disposable oxygen cylinder, oxygen regulator, mask and tubing, 15 minutes of medical oxygen USP

REFILL ORDERING INFORMATION

Tank Refill *To have a depleted Lif-O-Gen automated tank refilled, call 800-411-5136*

31-01-0610 Spare Oxygen Delivery Assembly, without cradle, for immediate replacement of a depleted unit

31-01-0510 Replacement Oxygen Cylinder for 31-01-0500 and 31-01-0555; 15 minutes of medical oxygen USP

REPLACEMENT PARTS & ACCESSORIES

31-01-0900 Wall Mount Emergency Oxygen Cabinet with Alarm

31-01-0910 Wall Mount Emergency Oxygen/AED Cabinet with Alarm

31-80-0420 Lif-O-Gen Disposable Replacement Oxygen Regulator, 6 lpm fixed flow, with contents gauge, for use with 31-01-0500

31-80-0505 Lif-O-Gen Disposable Replacement Oxygen Regulator, 6 lpm fixed flow, with contents gauge, push-button type, for use with 31-01-0555

31-80-2058 Replacement Carry Case for Disposable Twin Pac 31-01-0555

31-01-0100 Replacement mask assembly and tubing for all Lif-O-Gen kits

For additional information on Lif-O-Gen® products and accessories visit us at www.lifogen.com



Allied Healthcare Products, Inc. ▪ 1720 Sublette Avenue ▪ St. Louis, MO 63110 USA
314/771-2400 ▪ 800/444-3954 ▪ FAX: 314/771-0650 ▪ www.alliedhpi.com

U.S.
Toll Free: 800/444-3940
FAX: 800/477-7701

Canada & Mexico
Toll Free: 800/446-0552
FAX: 800/246-6201

International
Telephone: 314/268-1683
FAX: 314/771-5183

ISO 13485: 2003

©Allied Healthcare Products, Inc.®

All specifications are nominal and subject to change without notice.

Form 31-00-0002
October 2012