


BD SafetyGlide™ Shielding **Hypodermic Needle**

OUICK REFERENCE CARD



- Attach the BD SafetyGlide Shielding Hypodermic Needle to any standard BD Luer-Lok[™] or luer-slip hypodermic syringe
- Twist until firmly seated
- Pull shield straight off needle to avoid damaging needle point



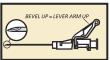

 Administer injection following established technique. For user convenience, the needle bevel-up position is oriented to the lever arm



syringe as per your usual technique

NOTE: If necessary to transport filled syringe to point of administration, use a safe, passive, onehanded recapping technique (as per OSHA standards) to cover the needle before transport to point of use

BD SafetyGlide™ Shielding Hypodermic Needle

QUICK REFERENCE CARD



 After injection, immediately apply a single-finger stroke to the activation assisted lever arm to activate the shielding mechanism

NOTE: Activate away from self and others. Listen for click and visually confirm needle tip is fully covered

 After activation, discard immediately into the nearest sharps container

Things to Remember:

- When using SafetyGlide, be sure to push firmly with finger to activate shield
- Aseptic technique and proper skin preparation of the site are essential
- Discard SafetyGlide needle and syringe in the nearest sharps collector. Follow your institution's policy for safe disposal of all medical waste