

Advanced Wound Care Dressings



dynarex

WE CARE LIKE FAMILY

Reorder No. 3024
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
FoamFlex™
Non-Adhesive Waterproof Foam Dressing
Sterile
INSTRUCTIONS FOR USE
1. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
2. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.

Reorder No. 3027
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
DynaGinate™
Calcium Alginate Dressing
Sterile
INSTRUCTIONS FOR USE
1. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
2. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.

Reorder No. 3031
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
DynaGinate™ AG
Silver Calcium Alginate
Sterile
INSTRUCTIONS FOR USE
1. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
2. (OPTIONAL) If necessary, border with adhesive tape.

Reorder No. 3037
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
DynaFoam™
Waterproof Composite Dressing
Sterile
INSTRUCTIONS FOR USE
1. Remove backing paper from dressing.
2. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
3. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.
Store in a dry place at room temperature.

Reorder No. 3035
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
DynaGuard™
Waterproof Composite Dressing
Sterile
INSTRUCTIONS FOR USE
1. Remove backing paper from dressing.
2. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
3. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.
Store in a dry place at room temperature.

Reorder No. 3034
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
ActiGuard™
Hydrogel Sheet
Sterile
INSTRUCTIONS FOR USE
1. Remove backing paper from dressing.
2. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
3. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.
Store in a dry place at room temperature.

Reorder No. 3012
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
CuraFoam™
Foam Dressing
Sterile
INSTRUCTIONS FOR USE
1. Remove backing paper from dressing.
2. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
3. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.
Store in a dry place at room temperature.

Reorder No. 3018
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
DynaDerm™
Hydrocolloid Dressing
Sterile • X-Thin
INSTRUCTIONS FOR USE
1. Remove backing paper from dressing and hold in place for several seconds to improve adhesion.
2. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.
Store in a dry place at room temperature.

Reorder No. 3021
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
DynaLevin™
Waterproof Adhesive Bordered Foam
Sterile
INSTRUCTIONS FOR USE
1. Remove backing paper from dressing and hold in place for several seconds to improve adhesion.
2. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.

Reorder No. 3039
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
Petroleum Gauze Dressing
Non-Woven • Sterile
Petroleum Non-Woven Gauze Pad, Neutral Wound Dressing
Contents sterile unless package is opened or damaged.
Single use only.
Not made with natural rubber latex.
CAUTION: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Reorder No. 3049
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
DynaFoam™ AG
Silver Foam Dressing
Bordered • Sterile
INSTRUCTIONS FOR USE
1. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
2. (OPTIONAL) If necessary, border with adhesive tape.
Not made with natural rubber latex.
Sterile unless opened or damaged.
Store in a dry place at room temperature.
CAUTION: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Reorder No. 3047
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
CuraFoam™ AG
Silver Foam Dressing • Sterile
INSTRUCTIONS FOR USE
1. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
2. (OPTIONAL) If necessary, border with adhesive tape.
Not made with natural rubber latex.
Sterile unless opened or damaged.
Store in a dry place at room temperature.
CAUTION: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Reorder No. 3059
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
SiliGentle™
Self-Adherent Silicone Bordered Foam Dressing
Sterile
INSTRUCTIONS FOR USE
1. Remove backing paper from dressing.
2. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
3. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.
Store in a dry place at room temperature.

Reorder No. 3065
Manufactured for: Dynarex Corporation, Orangeburg, NY 10962
www.dynarex.com
Made in China
DynaGel™
Hydrogel Impregnated Gauze
Sterile
INSTRUCTIONS FOR USE
1. Remove backing paper from dressing.
2. Apply dressing gently to wound area and hold in place for several seconds to improve adhesion.
3. (OPTIONAL) If necessary, border with adhesive tape.
Sterile unless opened or damaged.
Store in a dry place at room temperature.
Not made with natural rubber latex.

Comparison Guide



dynarex® is your one-stop medical supply company since 1967

Dynarex Clinical Advocate
Janette Ostrander, RN, BSN

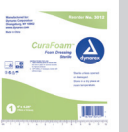
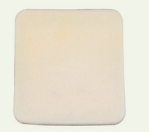
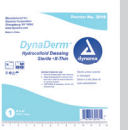













Dynarex Wound Care Dressings are comparable to all name brand wound care dressings.







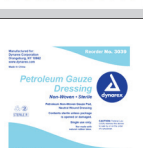

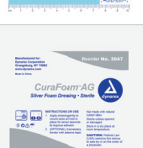
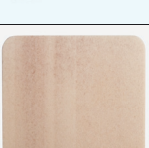





WE CARE LIKE FAMILY

ADVANCED WOUND CARE DRESSING PRODUCT LINE

Product	Reorder No.'s and Sizes	Description	Uses	Wound Stages (Pressure Ulcers)	Absorbency Level	Comparable Products	HCPCS Code		
		CuraFoam™ Foam Dressing	3011 – 2" × 2" 3012 – 4" × 4 1/4" 3013 – 6" × 6"	Maintains a moist wound environment, prevents bacterial contamination and absorbs up to 20 times its own weight.	• Manage chronic and acute wounds • Lower extremity ulcers, pressure ulcers, donor sites, and full thickness wounds • Low to moderate exuding wounds • Partial and full thickness wounds	II, III, IV	High	• HydraFoam • Medline OptiFoam • Kendall Hydrasorb	3011–A6209 3012–A6210 3013–A6210
		DynaDerm™ Hydrocolloid Dressing (Thin; Extra-Thin; Standard)	3014 – 2" × 2" (Thin) 3015 – 4" × 4" (Thin) 3016 – 6" × 6" (Thin) 3017 – 6" × 7" (Sacral) 3018 – 4" × 4" (Extra Thin) 3019 – 6" × 6" (Extra Thin) 3020 – 4" × 4" (Standard Dressing)	Maintains a moist wound environment that remains permeable to moisture vapor and oxygen. Can be safely worn for several days, depending on the wound condition and exudate levels.	• Non-infected wounds • Partial thickness ulcers • Full-thickness ulcers	II, III, IV	Low	• DermaRite DermaFilm • Kendall Ultec • ConvaTec Duoderm • Medline ExuDerm	3014–A6234 3015–A6234 3016–A6238 3017–A6238 3018–A6234 3019–A6235 3020–A6209
		DynaLevin™ Waterproof Adhesive Bordered Foam	3021 – 4" × 4" 3022 – 6" × 6"	Maintains a moist wound environment. Absorbs four times more than hydrocolloids of the same size and will not adhere to wound bed.	• Manage chronic and acute wounds • Low to moderate exuding, partial and full thickness wounds	II, III, IV	Medium	• DermaRite DermaLevin • Smith & Nephew Allevyn Adhesive	3021–A6212 3022–A6212
		Foam Flex™ Non-Adhesive Waterproof Foam Dressing	3023 – 2" × 2" 3024 – 4" × 4" 3025 – 6" × 6"	Sterile foam dressing maintains a moist wound environment, has a semi-permeable film and absorbs 4 times more than hydrocolloids. Also prevents bacterial contamination and conforms to awkward-to-dress areas.	• Manage chronic and acute wounds • Lower extremity ulcers, pressure ulcers, donor sites, and full thickness wounds • Low to moderate exuding wounds • Partial and full thickness wounds	II, III, IV	High	• DermaRite DermaFoam • Smith & Nephew Allevyn Foam	3023–A6209 3024–A6209 3025–A6210
		DynaGinate™ Calcium Alginate Dressing	3026 – 2" × 2" 3027 – 4 1/2" × 4 1/2" 3028 – 4" × 8" 3029 – 2g × 30cm	Used as a topical wound dressing for moderate to high exuding wounds absorbing up to 17 times its own weight. Conforms readily to any wound bed. Gels on contact with moisture, and assists in maintaining a moist wound healing environment.	• Pressure ulcers • Diabetic ulcers • Leg ulcers • Donor sites • Surgical wounds • Lacerations and abrasions	III, IV	High	• DermaRite DermaGinate	3026–A6196 3027–A6197 3028–A6197 3029–A6199
		DynaGinate AG™ Silver Calcium Alginate	3030 – 2" × 2" 3031 – 4" × 5" 3032 – 4" × 8" 3033 – 2g × 30cm	Same as Calcium Alginate dressing, with antibacterial Silver. Provides 3 days of continuous antibacterial protection. Absorbs 12 times its own weight in fluid.	• Infected wounds and wounds suspected of possible infection • Pressure ulcers • Leg ulcers • Diabetic wounds • Donor sites • Moderate to heavy draining • Partial and full thickness wounds	N/A	High	• DermaRite DermaGinate AG • ConvaTec Aquacel AG • Johnson & Johnson Silvercel	3030–A6196 3031–A6197 3032–A6197 3033–A6199
		ActiGuard™ HydroGel Sheet	3034 – 4" × 4"	Non-adhering dressing for use on all partial and full thickness wounds (except 3rd degree burns). Absorbs up to 3 times its own weight in moisture. Permeable to air and moisture vapor, allowing excess moisture to evaporate and allowing the wound to breathe. Transparent for easy monitoring.	• Diabetic ulcers • Pressure ulcers • Donor sites • Selected surgical sites • Burns (1st and 2nd degree) • Skin tears	II, III, IV	Low	• DermaRite AquaDerm • Medline DermaGel • Medline Skintegrity • Cardinal Healthcare Hydrogel sheet • Smith & Nephew Flexigel	3034–A6242

Designed to promote a moist environment, thereby accelerating the healing of many difficult-to-treat wounds or chronic wounds. Substances that help to provide ideal moisture conditions include alginate fiber, foam, and hydrocolloid dressings.

ADVANCED WOUND CARE DRESSING PRODUCT LINE

Product		Reorder No.'s and Sizes	Description	Uses	Wound Stages (Pressure Ulcers)	Absorbency Level	Comparable Products	HCPCS Code	
		DynaGuard™ Waterproof Composite Dressing	3035 – 4" × 4" 3036 – 6" × 6"	Sterile multi-layered dressing. Low adherent layer protects the wound as its semi-occlusive layer keeps external contamination from seeping through.	<ul style="list-style-type: none"> • Manage chronic or secondary dressing • Manage chronic and acute wounds • Lower extremity ulcers • Pressure ulcers • Donor Sites • Partial and full thickness wounds • Low to moderate exuding wounds 	II, III, IV	Medium	<ul style="list-style-type: none"> • DermaRite • DermaDress • Smith & Nephew CovrSite Plus 	3035–A6203 3036–A6203
		DynaFoam™ Waterproof Bordered Foam	3037 – 4" × 4" 3038 – 6" × 6"	Primary foam dressing with a waterproof composite secondary layer. Foam pad absorbs up to 20 times its own weight in fluid while the secondary composite keeps the foam in place. Maintains a moist wound environment while protecting and cushioning the wound.	<ul style="list-style-type: none"> • Manage chronic and acute wounds • Lower extremity ulcers, pressure ulcers, donor sites, and full thickness wounds • Low to moderate exuding wounds • Partial and full thickness wounds 	II, III, IV	High	<ul style="list-style-type: none"> • DermaRite Bordered Foam • DeRoyal Polyderm Plus 	3037–A6212 3038–A6213
		Petroleum Gauze Dressing	3039 – 3" × 9" 3040 – 3" × 9"	Non-adherent primary dressing that maintains a moist wound bed for healing. Cotton gauze impregnated with petroleum for a smooth and conforming dressing.	<ul style="list-style-type: none"> • Used for non-draining wounds such as tube sites, skin grafts, circumcisions, burns, non-infected wounds, pressure ulcers and abrasions 	II, III, IV	Medium	<ul style="list-style-type: none"> • Curad Sterile Petroleum Gauze • Covidien Vaseline Petrolatum Gauze • DermaSciences Petrolatum Gauze Non-Adhering Dressing 	3039–A6223 3040–A6223
		CuraFoam™ AG Silver Foam Dressing	3046 – 2" × 2" 3047 – 4" × 4" 3048 – 6" × 6"	Foam dressing infused with silver ions for antimicrobial protection. Ionic silver inhibits microbial growth, preventing colonization on the dressing to help manage the wound. Absorbs up to 5 times its own weight.	<ul style="list-style-type: none"> • Pressure ulcers • 1st and 2nd degree burns • Surgical wound sites • Diabetic ulcers • Donor sites 	III, IV	High	<ul style="list-style-type: none"> • DermaRite HydraFoam AG • Smith & Nephew Allevyn AG • Medline Optifoam AG 	3046–A4649 3047–A4649 3048–A4649
		DynaFoam™ AG Bordered Silver Foam Dressing	3049 – 4" × 4" 3050 – 6" × 6"	Self-adherent primary foam dressing with silver ions and a secondary adhesive border. Absorbs up to 5x its own weight. Ionic silver inhibits microbial growth, preventing colonization on the dressing to help manage the wound. Vapor permeable border is waterproof and leakproof, providing effective protection against contamination.	<ul style="list-style-type: none"> • Pressure ulcers • 1st and 2nd degree burns • Surgical wound sites • Diabetic ulcers • Donor sites 	III, IV	High	<ul style="list-style-type: none"> • DermaRite HydraFoam AG • Smith & Nephew Allevyn AG • Medline Optifoam AG • Coloplast Biatain AG 	3049–A4649 3050–A4649
		SiliGentle™ Silicone Foam Dressing and Silicone Bordered Foam Dressing	<i>Non-Bordered</i> 3055 – 2" × 2" 3056 – 4" × 4" 3057 – 6" × 6" <i>Bordered</i> 3058 – 2" × 2" 3059 – 4" × 4" 3060 – 6" × 6" 3061 – Heel 4" × 4" 3062 – Sacral 6" × 6" 3063 – Lg Sacral 6" × 6"	Consists of a silicone wound contact layer, super absorbent pad layer and a vapor permeable and waterproof film. The gentle silicone layer can be moved without losing its adherence. Can minimize pain and trauma to wound and its surrounding skin. Provides superior absorption and locks exudates away.	Indicated for the management of partial and full thickness wounds, including: <ul style="list-style-type: none"> • Pressure Ulcers • Leg and foot ulcers • Diabetic ulcers • Venous ulcers • Pressure sores • Traumatic wounds • Donor sites • Skin tears 	II, III, IV	High	<ul style="list-style-type: none"> • DermaRite ComfortFoam and ComfortFoam Border • Coloplast Biatain Silicone Foam • Molnlycke Health Care Meliplex Border • Smith & Nephew Allevyn Gentle and Allevyn Gentle Border 	HCPCS – COMING SOON!
		DynaGel™ Hydrogel Impregnated Gauze Dressing	3065 – 4" × 4"	Sterile gauze impregnated with hydrogel. Convenient for gentle filling of dry or minimally exuding wounds. Helps soften eschar and necrotic tissue.	Used as a primary dressing for the management of: <ul style="list-style-type: none"> • Acute or chronic partial and full thickness wounds/ulcers that are dry or with minimal exudate • Wounds with tunneling 	II, III, IV	None	<ul style="list-style-type: none"> • DermaRite DermaGauze • Skintegrity Hydrogel Impregnated Gauze 	HCPCS – COMING SOON!

NEW!

NEW!



Find us on:



For more information contact your local Dynarex® distributor sales representative.

Or call **888-396-2739** to find your nearest distributor.

10 Glenshaw Street • Orangeburg, NY 10962 • www.dynarex.com

Ordering Information

Reorder No.	CuraFoam™ Description	Quantity
3011	2" x 2" – Foam Dressing	12/10/Cs
3012	4" x 4 1/4" – Foam Dressing	12/10/Cs
3013	6" x 6" – Foam Dressing	12/10/Cs

3011 HCPCS: A6209, 3012 & 3013 HCPCS: A6210

Reorder No.	DynaDerm™ Description	Quantity
3014	2" x 2" – Hydrocolloid Dressing – Thin	12/20/Cs
3015	4" x 4" – Hydrocolloid Dressing – Thin	12/10/Cs
3016	6" x 6" – Hydrocolloid Dressing – Thin	12/5/Cs
3017	6" x 7" – Hydrocolloid Dressing – Sacral	12/5/Cs
3018	4" x 4" – Hydrocolloid Dressing – Extra Thin	12/10/Cs
3019	6" x 6" – Hydrocolloid Dressing – Extra Thin	12/5/Cs
3020	4" x 4" – Hydrocolloid Dressing – Standard	12/5/Cs

3014, 3015 & 3018 HCPCS: A6234, 3016 & 3017 HCPCS: A6238, 3019 HCPCS: A6235, 3020 HCPCS: A6209

Reorder No.	DynaLevin™ Description	Quantity
3021	4" x 4" – Waterproof Adhesive Bordered Foam	12/10/Cs
3022	6" x 6" – Waterproof Adhesive Bordered Foam	12/10/Cs

3021 & 3022 HCPCS: A6212

Reorder No.	FoamFlex™ Description	Quantity
3023	2" x 2" – Non-Adhesive Waterproof Foam	12/10/Cs
3024	4" x 4" – Non-Adhesive Waterproof Foam	12/10/Cs
3025	6" x 6" – Non-Adhesive Waterproof Foam	12/10/Cs

3023 & 3024 HCPCS: A6209, 3025 HCPCS: A6210

Reorder No.	DynaGinate™ Description	Quantity
3026	2" x 2" – Calcium Alginate Dressing	12/10/Cs
3027	4 1/4" x 4 1/4" – Calcium Alginate Dressing	12/10/Cs
3028	4" x 8" – Calcium Alginate Dressing	12/5/Cs
3029	2 g x 30 cm – Calcium Alginate Rope Dressing	12/5/Cs

3026 HCPCS: A6196, 3027 & 3028 HCPCS: A6197, 3029 HCPCS: A6199

Reorder No.	DynaGinate™ AG Description	Quantity
3030	2" x 2" – Silver Calcium Alginate Dressing	10/10/Cs
3031	4" x 5" – Silver Calcium Alginate Dressing	10/10/Cs
3032	4" x 8" – Silver Calcium Alginate Dressing	10/5/Cs
3033	2 g x 30 cm – Silver Calcium Alginate Rope Dressing	10/5/Cs

3030 HCPCS: A6196, 3031 & 3032 HCPCS: A6197, 3033 HCPCS: A6199

Reorder No.	ActiGuard™ Description	Quantity
3034	4" x 4" – Hydrogel Sheet	12/10/Cs

3034 HCPCS: A6242

Reorder No.	DynaGuard™ Description	Quantity
3035	4" x 4" – Waterproof Composite Dressing	12/10/Cs
3036	6" x 6" – Waterproof Composite Dressing	12/10/Cs

3035 & 3036 HCPCS: A6203

Reorder No.	DynaFoam™ Description	Quantity
3037	4" x 4" – Waterproof Bordered Foam	12/10/Cs
3038	6" x 6" – Waterproof Bordered Foam	12/10/Cs

3037 HCPCS: A6212, 3038 HCPCS: A6213

Reorder No.	Petroleum Gauze Description	Quantity
3039	3" x 9" – Petroleum Gauze Dressing	4/50/Cs
3040	3" x 9" – Petroleum Gauze Dressing	10/12/Cs

3039 & 3040 HCPCS: A6223

Reorder No.	CuraFoam™ AG Description	Quantity
3046	2" x 2" – Silver Foam Dressing	10/10/Cs
3047	4" x 4" – Silver Foam Dressing	10/10/Cs
3048	6" x 6" – Silver Foam Dressing	10/10/Cs

3046, 3047 & 3048 HCPCS: A4649

Reorder No.	DynaFoam™ AG Description	Quantity
3049	4" x 4" – Bordered Silver Foam Dressing	10/10/Cs
3050	6" x 6" – Bordered Silver Foam Dressing	10/10/Cs

3049 & 3050 HCPCS: A4649

Reorder No.	SiliGentle™ Description	Quantity
3055	2" x 2" – Silicone Foam Dressing	12/10/Cs
3056	4" x 4" – Silicone Foam Dressing	12/10/Cs
3057	6" x 6" – Silicone Foam Dressing	12/10/Cs
3058	2" x 2" – Silicone Bordered Foam Dressing	12/10/Cs
3059	4" x 4" – Silicone Bordered Foam Dressing	12/10/Cs
3060	6" x 6" – Silicone Bordered Foam Dressing	12/10/Cs
3061	Heel, 10" x 6" – Silicone Bordered Foam Dressing	12/5/Cs
3062	Sacral, 7" x 7" – Silicone Bordered Foam Dressing	12/5/Cs
3063	Lg Sacral, 9" x 9" – Silicone Bordered Foam Dressing	12/5/Cs

3055-3063 HCPCS: COMING SOON!

Reorder No.	DynaGel™ Description	Quantity
3065	4" x 4" – Hydrogel Impregnated Gauze Dressing	12/15/Cs

3065 HCPCS: COMING SOON!

dynarex® is your one-stop medical supply company since 1967

For more information contact your local Dynarex® distributor sales representative.

Or call **888-396-2739** to find your nearest distributor.

10 Glenshaw Street • Orangeburg, NY 10962 • www.dynarex.com



Find us on:

