



Cleansing Enema Set

Cleans and Relieves to Help you Feel Better

Kit assembled in a compact package for single use. Saves time in the E.R. and elsewhere.

- Clean out the contents of the bowels and flush away waste in the colon
- Ease digestion by taking stress off the bowel
- Relieve constipation

Contents:

- (1) 1500ml Enema Bag
- (1) Inlet Tubing (60" length)
- (1) Tubing Clamp
- (1) Protection Cap
- (1) Castile Soap Concentrate
- (1) Moisture Proof Drape, 45x42cm

For more information
 contact your Dynarex sales representative
 888-396-2739

Reorder No. 4711

1

Cleansing Enema Set

Single Use • Disposable
Clean, Ready to Use



Lot No. _____

Contents:

- 1 - 1500ml Enema Bag
- 1 - Inlet Tubing (60" length)
- 1 - Tubing Clamp
- 1 - Protection Cap
- 1 - Castile Soap Concentrate
- 1 - Moisture Proof Drape, 45x42cm

SUGGESTED PROCEDURE

1. Slide tubing to "off" position.
2. Place two fingers under blue pull tab, pull forward to open top of bag.
3. Add desired quantity of selected solution.
4. "Squeegee" across the valve closure with thumb and forefinger to seal bag.
5. Suspend bag for administration.
6. Position moisture proof drape under patient.
7. Remove lube cap from tip of tube.
8. Proceed with administration using approved technique.
9. Drain unused solution from bag and discard.

Manufactured for:
 Dynarex Corporation
 Orangeburg, NY 10962
 www.dynarex.com
 Made in China

Reorder No.	Cleansing Enema Set	Quantity
4711	Cleansing Enema Set	50/cs

Already mobile?
 Scan your smart phone here for more information.



Visit our website to see our Flush Away Flushable Wipes (#1321-1324)



*Depending on market demand and Dynarex stock, we will be happy to address any orders for items currently not shown.