

Minor Procedure Fenestrated Drape - 30" x 30"

(No video available)



1. Orient drape with "Top" arrow pointing toward head of patient.




2. Remove adhesive strip liner and position drape with fenestration over operative site.



3. Unfold laterally, then toward head, and then toward feet.



4. Firmly seal fenestration to patient's skin to complete draping.

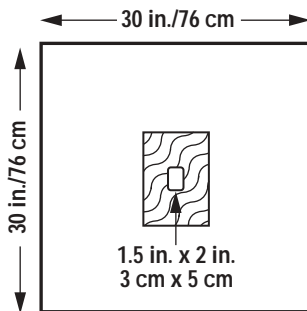

Features

- 30" x 30"
- 1.5" x 2" Fenestration
- SURROUND absorbent fabric reinforcement

Stock #

89206

Description

Minor Procedure Fenestrated Drape  
Sterile, 1 per package, 25 packages / 1 dispenser

Items Per Case

100 (25 x 4)

79206♦

Minor Procedure Fenestrated Drape  
Non-Sterile, Bulk Pack

250

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom-tray supplier

† As with any drape or pack recommendation, it is the hospital's decision as to the appropriate draping technique.

\* Registered Trademark or Trademark of Kimberly-Clark Worldwide or its affiliates.

© 2003 KCWW. All rights reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement.

