

## Laparotomy Drape with Troughs

(89234 depicted)



1. Orient body stamp on drape with patient and remove adhesive strip liners clockwise.



2. Center drape over operative site.




3. Unfold drape, first laterally, then toward head to form an anesthesia screen and extend armboard covers laterally.



4. Unfold drape toward feet and firmly seal edge of fenestration to patient's skin. Insert surgical lines and tubing through the tube holders.



5. Open troughs to help contain excess fluids to complete draping.

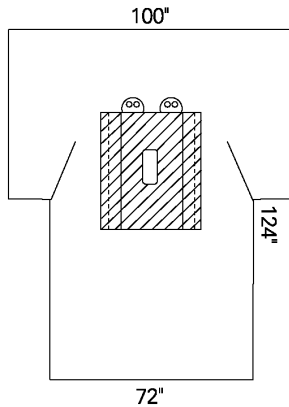

### Features

- Armboard Covers
- 100" x 72" x 124"
- 4" x 12" Fenestration
- Absorbent Troughs
- CONTROL\* PLUS Fabric Reinforcement
- Tube Holders

### Stock #

### Description

### Items Per Case

89234

Laparotomy Drape with Troughs, Sterile, 1 per package

13

79234 ♦

Laparotomy Drape with Troughs, Non-Sterile, Bulk Pack

15

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.