

Lower Extremity Drape



1. With an Impervious Split Drape and an Impervious Stockinette in place, orient body stamp on drape with patient.



2. Unfold drape once and guide extremity through the conformable fenestration.



(89278 depicted)


3. Pull drape to above the operative site. Unfold drape, first laterally, then toward head to form an anesthesia screen.



4. Unfold toward feet.



5. Insert surgical lines and tubing through the tube holders to complete draping.

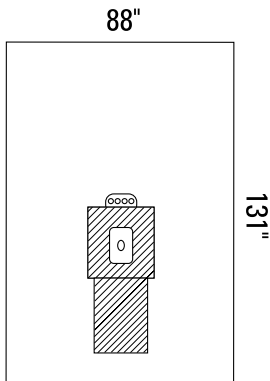

Features

- 88" x 131"
- 2½" Conformable Circular Fenestration
- CONTROL* Fabric Reinforcement
- Tube Holders

Draping Hints

- This drape has no armboard covers which is different from the drape depicted above.

Stock #	Description	Items Per Case
89276	Lower Extremity Drape, Sterile, 1 per package	14
79276 ♦	Lower Extremity Drape, Non-Sterile, Bulk Pack	16

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

Pack Stock #
88431

Description
Ortho Pack III – Lower Extremity

Packs/Case
5

Component Stock #

Sterile Non-Sterile ♦

Contents:

89618	79498	1 Back Table Cover, Zone-Reinforced, 44" x 90"
95521	70521	1 ULTRA Specialty Surgical Gown, X-Large, with Towel
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"
89821		1 Impervious Stockinette, Large, 12" x 48"
89331	79330	1 Impervious Split Drape, 60" x 70", 4" x 21" Split
89276	79276	1 Lower Extremity Drape, 88" x 131" <ul style="list-style-type: none">• 2½" Conformable Circular Fenestration• CONTROL* Fabric Reinforcement• Tube Holders
		3 Drape Tape Strips, 5" x 23½"

Pack Stock #
88461

Description
Ortho Pack VI – Lower Extremity

Packs/Case
5

Component Stock #

Sterile Non-Sterile ♦

Contents:

89618	79498	1 Back Table Cover, Zone-Reinforced, 44" x 90"
95111	70111	1 ULTRA Surgical Gown, Large, in Overwrap with Towel
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"
		1 Suture Bag
89701	79720	1 Absorbent Towel, 15" x 26"
89731	79731	2 Utility Drapes, 26" x 15", with Tape
89331	79330	1 Impervious Split Drape, 60" x 70", 4" x 21" Split
89121	79121	1 Large Drape, 60" x 76"
89276	79276	1 Lower Extremity Drape, 88" x 131" <ul style="list-style-type: none">• 2½" Conformable Circular Fenestration• CONTROL* Fabric Reinforcement• Tube Holders

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. *Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.