



**8675/8677/8678**  
**OWNER'S MANUAL**



**SERIAL NUMBER:** \_\_\_\_\_

**DATE OF MANUFACTURE:** \_\_\_\_\_

**8675 MAXIMUM PATIENT WEIGHT: 250 LBS.**

**8677/8678 MAXIMUM PATIENT WEIGHT: 375 LBS.**

## **IMPORTANT INFORMATION**

Safety First: The primary concern of UMF Medical is that this equipment be operated and maintained with the safety of the patient and doctor in mind

- 1 Read this manual before operating your new UMF Medical equipment.
- 2 It is the responsibility of the customer to insure that appropriate personnel are informed of the contents of this manual.
- 3 Be sure that you understand the instructions contained in this manual before operating this equipment.
- 4 This manual should remain permanently affixed or near the equipment for convenient reference.

## **SAFETY SYMBOLS**

### **WARNING**

The warning symbol identifies special instructions or procedures, which if not correctly followed could result in personal injury.

### **CAUTION**

The caution symbol identifies special instructions or procedures, which if not correctly followed could result in danger or damage of equipment.

## **INTRODUCTION**

### **UNPACKING**

Do not use knives or other sharp objects to unpack table as damage may occur to upholstery or painted surfaces.

### **CONCEALED DAMAGE**

If any concealed damages are found after delivery, immediately request inspection by the delivery carrier. Notification of concealed damage must be made within fifteen (15) days of delivery. All claims for shortages not occurring during shipping must be made within fifteen (15) days after the reception of material to UMF Medical.

### **DESIGN CHANGES**

UMF Medical reserves the right to make changes in design or improvements or to modify the patterns of our products. The right is also reserved to modify our products that may be consistent with the latest trends and designs, factors beyond our control, and/or government regulations. Such changes may be made without notice.

## **INSTALLATION**

The table is secured to a wood skid for shipment. Remove table from skid and move to desired location. Adjust leveling feet to eliminate any rocking of table due to uneven floors. In order to operate the table, the table power cord must be plugged into a matching three pronged, grounded non-isolated, correctly polarized 120 volt outlet.

**WARNING: ALL USERS OF THIS TABLE SHOULD BE FAMILIAR WITH THE CONTROLS AND OPERATION OF ALL TABLE FUNCTIONS AND POSITIONS BEFORE USING THE TABLE FOR THE EXAMINATION OR TREATMENT OF PATIENTS.**

**WARNING: FOR PATIENT SAFETY, ENTRY AND EXITING OF THE TABLE SHOULD ONLY OCCUR WITH THE TABLE IN ITS LOWEST UPRIGHT POSITION WITH THE SEAT BACK SECTION UPRIGHT AND THE LEG SECTION IN A LOCKED POSITION.**

**WARNING: DO NOT USE THE LEG REST TO SUPPORT PATIENT'S ENTIRE WEIGHT. LEG REST WEIGHT CAPACITY = 50 LBS.**

**WARNING: DO NOT USE THE BACK SECTION AS A SEAT. ONCE ELEVATED, IT SHOULD ONLY BE USED TO SUPPORT A PATIENT'S BACK WITH THEIR BODY SITTING ON THE SEAT SECTION.**

## **8677/8678**

Hi-low and back positioning is accomplished by 24-volt DC actuator units, which are activated by a hand control (standard) or foot control (optional). **Note: The controls are sealed units and should not be opened.**



## **8675**

The UMF Medical 8675 Chair is capable of positioning from an upright chair position to a horizontal position. Positioning of the chair back is accomplished by lifting upward on back release handle to unlock and lower to desired position, releasing hand locks position. To raise the chair back to upright position simply lift upward on back, downward pressure will lock in place.


Chair height adjustment is accomplished by pumping downward on the foot lever to desired height, lifting upward on foot lever locks height into position. To lower chair simply push foot lever to full down position and hold, chair slowly returns to low position, remember to lift upward on foot lever and lock position before allowing patient exit from chair.

**033 – FOLD AWAY TABLE**

Locking folding table provides convenient place for supplies while performing procedures. Can be easily folded down and positioned out of the way during patient entry and exit.



**062 – 2 FUNCTION NON-PROGRAMMABLE FOOT CONTROL (5060 ONLY)**

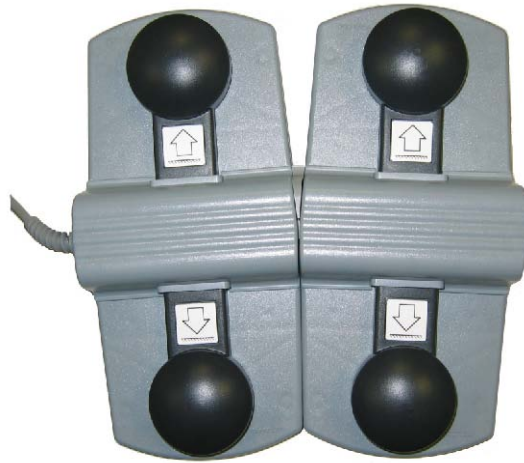

**032 – ARTICULATING HEAD REST**



### **Care of Upholstery**

The upholstery material used on the top, leg rest, and headrest is resistant to most medical stains. Care should be used where solvents or dyes are used. Mild soap and water should be used for daily cleaning of upholstery. UMF Medical recommends the use of Murphy's oil soap or Ivory liquid. If dirt or stains still remain, use rubbing alcohol.

**Note:** Immediately remove any fluid spilled on upholstery surface.

**CAUTION: WHEN THE USE OF RUBBING ALCOHOL IS NECESSARY, TEST AN INCONSPICUOUS AREA TO ASSURE THAT DAMAGE TO UPHOLSTERY WILL NOT OCCUR.**

### **Care of Painted surface:**

A chemical acid resistant paint is used, but extreme care must be taken not to use ammonia based cleaners or discoloration of paint may occur. A damp cloth or mild soap solution should be sufficient. UMF Medical recommends the use of Murphy's oil soap or Ivory liquid.

### **Care of Bright Metal Surfaces:**

All non-painted surfaces, chrome plated or stainless steel should be wiped weekly with a clean damp cloth then buffed to a lustrous shine with a soft dry cloth.

### **Care of Base:**


The protective base is easily washable with mild soap and water. A soft bristle brush may be used on scuffed or stained areas. UMF Medical recommends the use of Murphy's oil soap or Ivory liquid.

# ELECTRICAL SCHEMATICS

## ELECTRICAL SPECS

VOLTAGE: 115 VAC  
FREQUENCY: 60 Hz CURRENT  
DRAW: 5 Amp (MAX.)

**8677/8678**

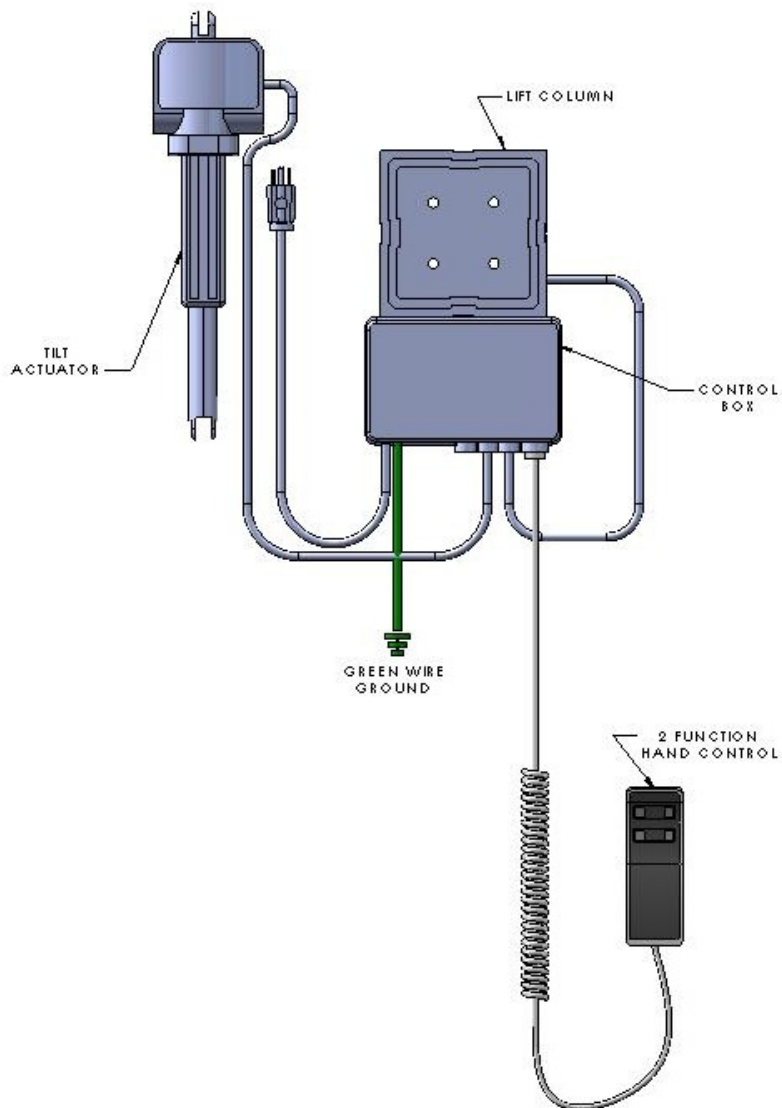



## **WARRANTY INFORMATION**

United Metal Fabricators Inc. d.b.a. UMF Medical warrants its products to the original purchaser at retail to be free from defective material and workmanship under normal use and service for a period of three (3) years. United Metal Fabricators Inc. d.b.a. UMF Medical warrants its Signature Series Examination Tables (51xx model numbers) and Treatment and Orthopedic Tables (55xx model numbers) to the original purchaser at retail to be free from defective material and workmanship under normal use and service for a period of five (5) years.

### ***What is not covered?***

Products which have been subjected to abuse, accident, alteration, modification, tampering, negligence, freight damage, misuse or if repaired or serviced by anyone other than a service facility authorized by United Metal Fabricators Inc. d.b.a. UMF Medical to render such service, or if the model or serial number has been altered, tampered with, defaced or removed. Except as it may otherwise specifically agree in writing, United Metal Fabricators Inc. d.b.a. UMF Medical shall not be liable for transportation, labor or other charges for adjustment, repairs, replacement parts, installation or other work which may be done with such products. United Metal Fabricators Inc. d.b.a. UMF Medical shall not be liable for changes in color caused by natural or artificial light, matching of color, grain or texture except to commercially acceptable standards

### ***What we will do:***

United Metal Fabricators Inc. d.b.a. UMF Medical will replace all defective parts free of charge that are covered by this limited warranty. For warranty service information, contact the dealer through whom the item was purchased or our customer service department.

### **What you must do for the warranty service:**

Box or crate the product carefully, preferably in the original carton. Ship insured with a copy of your bill of sale plus a letter of authorization or return goods from United Metal Fabricators Inc. d.b.a. UMF Medical, to the address given below, FOB Factory.

No other warranty is applicable to this product. The duration of any implied warranties, including the implied warranty of merchantability, is limited to the duration of the express warranty herein. United Metal Fabricators Inc. d.b.a. UMF Medical shall not be liable for the loss of the use of this product inconvenience, loss or other damages, direct or consequential, arising out of the use of, or inability to use, this product or for any breach of any express or implied warranty of merchantability applicable to this product.

Some states do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so these limitations or exclusions may not apply to you. This gives you specific legal rights and you may also have other rights, which may vary from state to state.