

# 850 Treatment Table Parts List





## PARTS LIST

| Image Number | Note | Part Number    | Description                                    | QTY per Product |
|--------------|------|----------------|--|-----------------|
| 1            | A    | <b>U3003XX</b> | <b>850 UPHOLSTERED TOP</b>                     | 1               |
| 1A           | B    | 222700         | TABLE FRAME 850/851/855                        | 2               |
| 1A           | B    | 701010         | 1X2X18 RECT INSERT CAP/GLIDE LT.GRAY           | 4               |
| 1A           | B    | 566519         | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC | 4               |
| 1A           | B    | 566528         | SCREW-TAP 10 X 2 3/4 PAN PHIL T/S ZINC         | 4               |
| 3            |      | 222720         | 850/851/855/856 WELDED BRACE ASSM.             | 1               |
| 3            |      | 566401         | BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC              | 4               |
| 3            |      | 566204         | NUT-1/4-20 HEX NYLON LOCKNUT ZINC              | 4               |
| 4            | C    | 222730         | WELDED, 850-851-856 REINF. BRACE ASSM WELDED   | 1               |
| 4            |      | 566401         | BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC              | 4               |
| 4            |      | 566204         | NUT-1/4-20 HEX NYLON LOCKNUT ZINC              | 4               |

## NOTES

| Note | Description  |
|------|--|
| A    | XX- Use color code for specific part number i.e. 02 - Royal Blue   |
| B    | These parts are included with the above listed UPHOLSTERED TOPS  |
| C    | For tables with optional shelf, cabinet, or drawer. Must order double-sided tape listed on <b>Page 3</b> |

# 850 Treatment Table Parts List

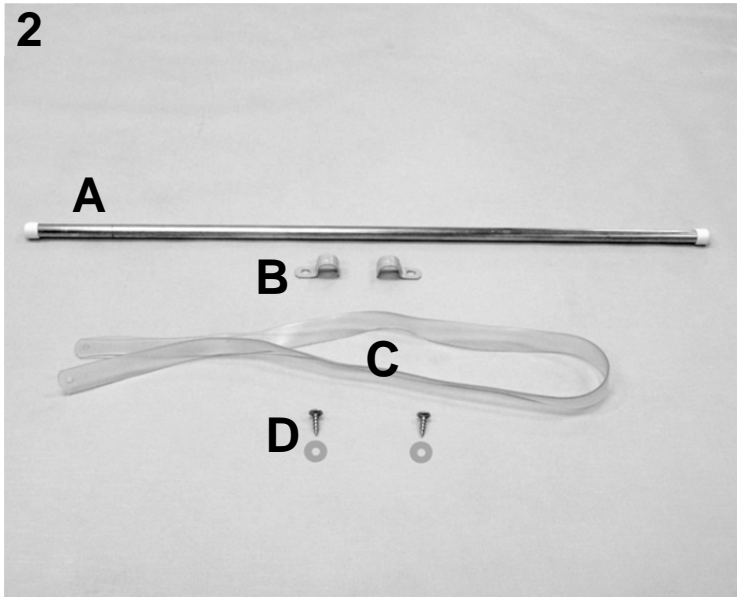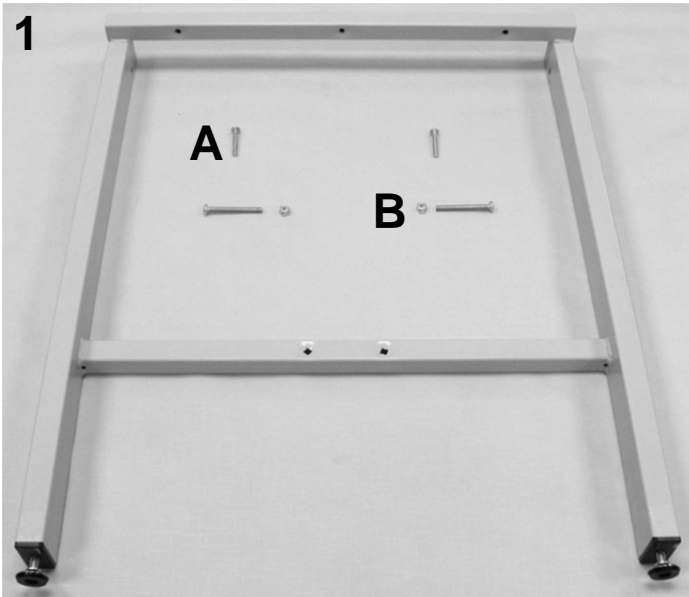

## PARTS LIST

| Image Number | Note  | Part Number    | Description  | QTY per Product |
|--------------|---|----------------|--|-----------------|
| 1            |   | <b>222708K</b> | <b>WELDED LEG ASSM. KIT 19"</b>                        | 2               |
| 1            | A   | 222708         | WELDED LEG ASSM. 19"                                   | 1               |
| 1            |   | <b>222709K</b> | <b>WELDED LEG ASSM. KIT 25" (850 style)</b>            | 2               |
| 1            | A   | 222709         | WELDED LEG ASSM. 25" (850 style)                       | 1               |
| 1            |   | <b>222710K</b> | <b>WELDED LEG ASSM. KIT 30" (850 style)</b>            | 2               |
| 1            | A   | 222710         | WELDED LEG ASSM. 30" (850 style)                       | 1               |
| 1            |   | <b>222711K</b> | <b>WELDED LEG ASSM. KIT 36" (850 style)</b>            | 2               |
| 1            | A   | 222711         | WELDED LEG ASSM. 36" (850 style)                       | 1               |
| 1            | A   | 500350         | GLIDE NON-SWIVEL ZINC 5/16X1" NP LN                    | 2               |
| 1            | A   | 500105         | INSERT, BLK HDPE, RIBBED 16-18 ga 1X2 , 5/16-18 THREAD | 2               |
| 1A           | A   | 566506         | SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC            | 2               |
| 1B           | A   | 566519         | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC         | 2               |
| 1B           | A   | 566204         | NUT-1/4-20 HEX NYLON LOCKNUT ZINC                      | 2               |
| 2A           |   | <b>222705K</b> | <b>PAPER ROD HOLDER ASSM. KIT (850 STYLE)</b>          | 1               |
| 2A           | A   | 222705         | 850/851/855/856 ROD, PAPER HOLDER 1/2" ALUM            | 1               |
| 2A           | A   | 700650         | 1/2" PLASTIC CAP, WHITE                                | 2               |
| 2B           |   | 201810         | PAPER ROD CLAMP, GRAY                                  | 2               |
| 2C           |   | 271500         | VINYL TABLE STRAP                                      | 1               |
| 2D           |   | 566302         | WASHER-#10 SAE F/W ZINC                                | 2               |
| 2D           |   | 566534         | SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC              | 2               |
| <b>NOTES</b> |   |                |  |                 |
| <b>Note</b>  | <b>Description</b>                                |                |  |                 |
| A            | These parts are included in ALL above listed kit. |                |  |                 |

# 850 Treatment Table Parts List



| PARTS LIST   |   |                |  |                 |
|--------------|---|----------------|--|-----------------|
| Image Number | Note  | Part Number    | Description                                    | QTY per Product |
| 1            |   | <b>263960K</b> | <b>850 SERIES PLAIN SHELF KIT</b>              | 1               |
| 1            | A   | 263960         | 850 SERIES PLAIN SHELF                         | 1               |
| 1            | A   | 705600         | DOUBLE SIDED FOAM TAPE, 3/4" X 36YDS           | 1 ft.           |
| 2            |   | 263950         | 850,851,856 MODULE SHELF                       | 1               |
| 3            |   | <b>266780K</b> | <b>850/851/856 CABINET MODULE ASSM. KIT</b>    | 1               |
| 3A           | A   | 266780         | 850/851/856 CABINET MODULE                     | 1               |
| 3A           | A   | 566519         | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC | 4               |
| 3A           | A   | 566204         | NUT-1/4-20 HEX NYLON LOCKNUT ZINC              | 4               |
| 4            |   | <b>266760K</b> | <b>850/851/856 MODULE DRAWER ASSM. KIT</b>     | 1               |
| 4            | A   | 266760         | 850/851/856 MODULE DRAWER ASSM                 | 1               |
| 4A           | A   | 566519         | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC | 4               |
| 4A           | A   | 566204         | NUT-1/4-20 HEX NYLON LOCKNUT ZINC              | 4               |
| NOTES        |   |                |  |                 |
| Note         | Description                                       |                |  |                 |
| A            | These parts are included in the above listed kit. |                |  |                 |
|              |   |                |  |                 |