


.....

PNEUMATIC HEIGHT ADJUSTABLE

# EXAM STOOL

WITH BACKREST

.....

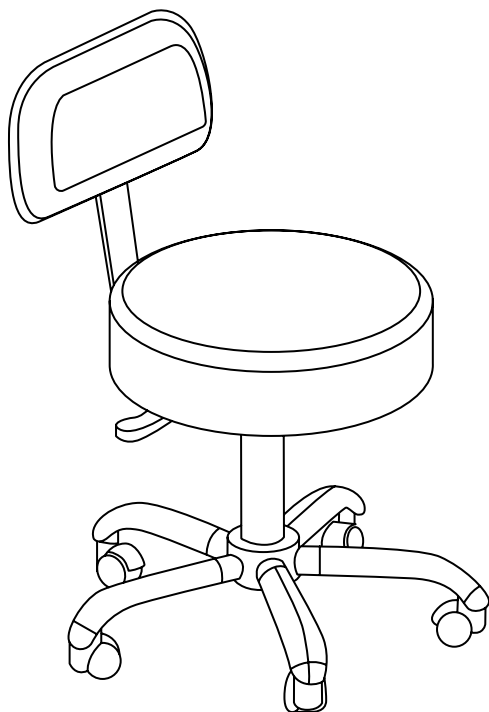

**USER GUIDE**

ADI910-03



# Thank you for your purchase.

.....

**Viva Comfort maintains a  
strong practice of ongoing  
development of products and  
commercial research.**

.....


**We recognize that starting and maintaining a medical  
practice is expensive. We'll help make the process cost-  
effective and elegant. Additionally, our team of specialists  
will ensure everything you need works as efficiently as  
possible, so you can effectively treat your clients.**

# Table of Contents

- 2** Parts List
- 3** Hardware List
- 4** Assembly Instructions
- 10** Care & Maintenance Guide
- 17** Warranty


# Parts List

**A**

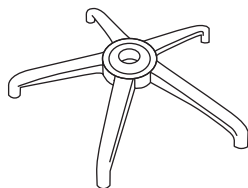

Base

**B**

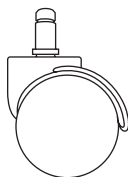


Caster Wheel (x5)

**C**




Hydraulic Lift Adjustment

**D**

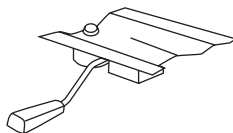


Hydraulic Lever Bracket

**E**

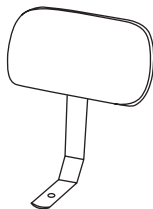






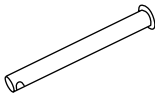
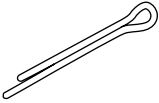
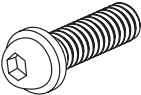
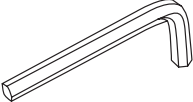
Backrest

**F**

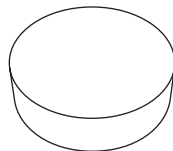

Padded Stool Seat


# Hardware List
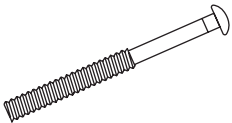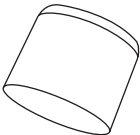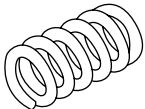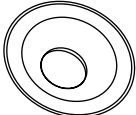
<b>1</b>  Screw 1 (M10*L105mm)	<b>2</b>  Screw Top Cover
<b>3</b>  Spring	<b>4</b>  Spring Washer
<b>5</b>  Screw Bottom Cover	<b>6</b>  Latch (M7)
<b>7</b>  Pin	<b>8</b>  Screw 2(M6*26mm) (x4)
<b>9</b>  Allen Key	

# Assembly Instructions


## Step 1:

**A.** Insert the five Caster Wheels (B) into the bottom of the Base (A).

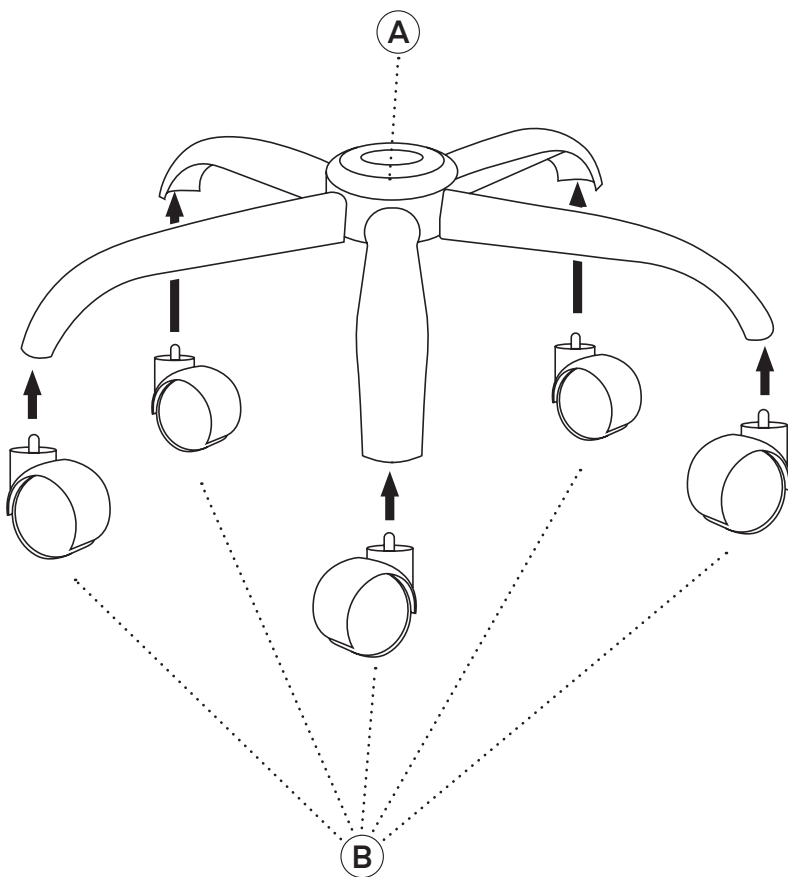



**Step 2:**

- A.** Insert the Hydraulic Lift Adjustment (C) into the Base (A), as pictured below.

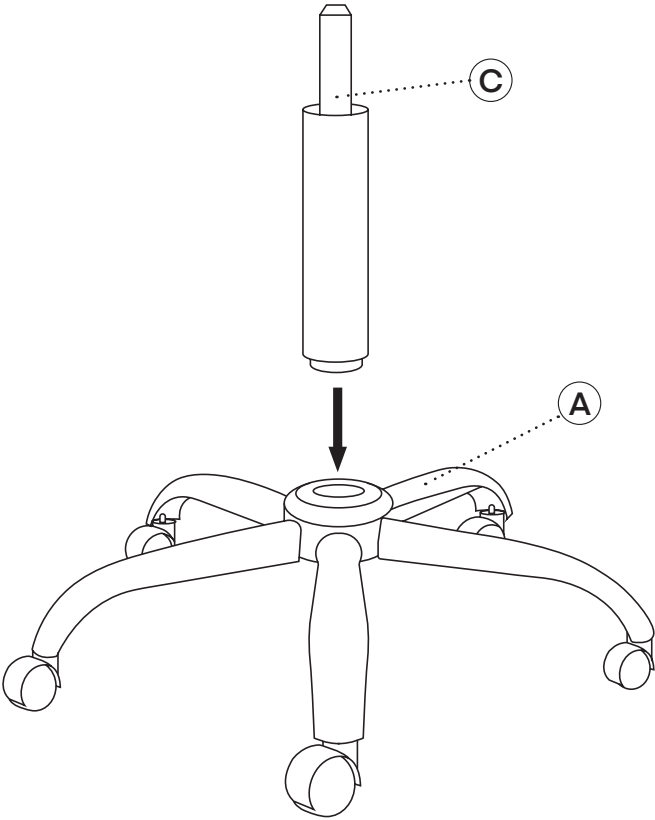

### Step 3:



- A.** Put the Hydraulic Lever Bracket (D) into the top Hydraulic Lift Adjustment (C) you installed in Step 2.

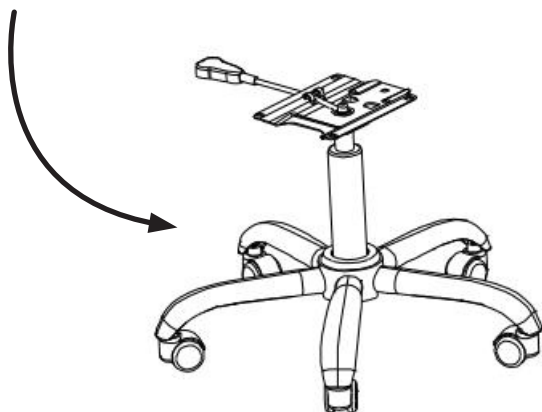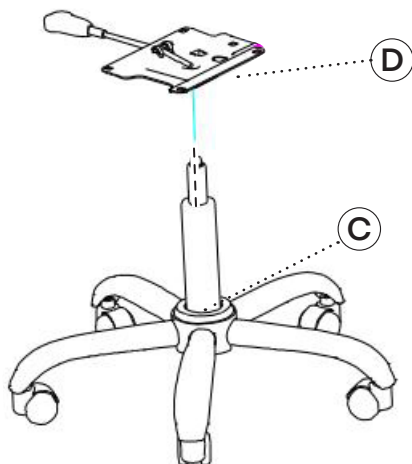




#### Step 4:

- A.** Place the Backrest (E) under the two holes of the Hydraulic Lever Bracket (D).
- B.** Insert Latch (6) so that it comes through the Backrest (F) holes.
- C.** Insert the Pin (7) into the small holes at the front of the Latch (6), as pictured.

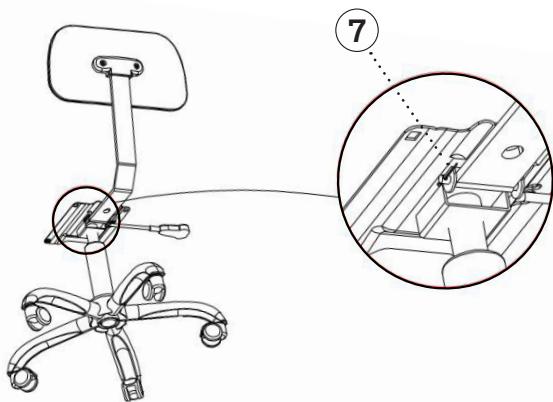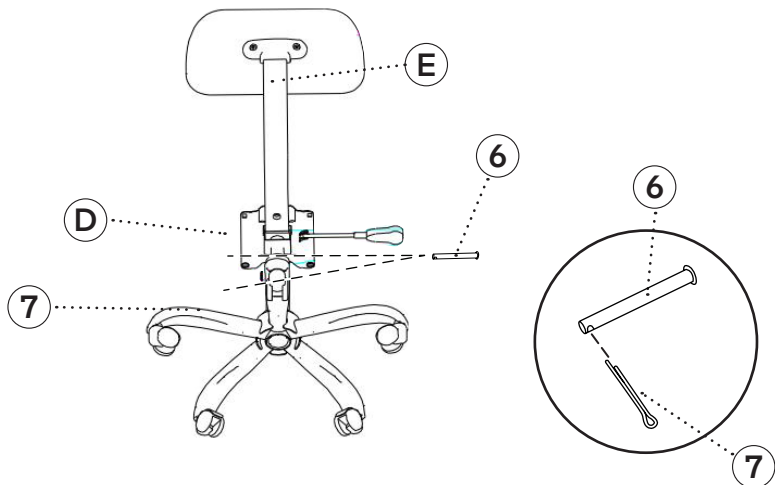


## Step 5:

- A.** Insert Screw 1 (1) into small square holes of the Hydraulic Lift Bracket (D), as well as come through the hole at the front of the Backrest (E).
- B.** Come through the hole of the Screw Top Cover (2), and then add the Spring (3) and Washer (4).
- C.** Take the Screw Bottom Cover (5) off. Make sure the holes could match with Screw 1 (1), and then tighten.

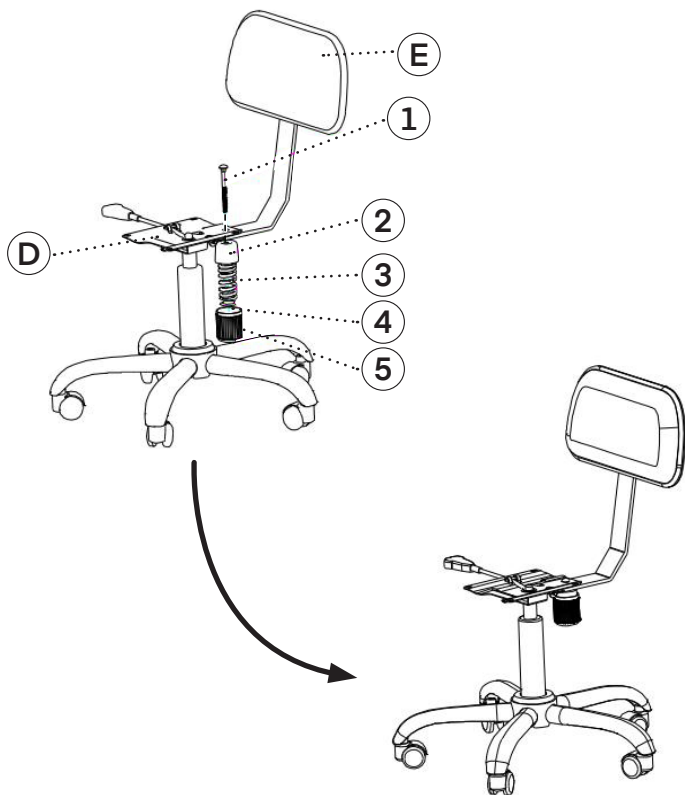

## Step 6:

- A.** Place the Padded Stool Seat (F) on the flat surface of the Hydraulic Lever Bracket (D) and align the holes on the cushion with the ones on the bracket.
- B.** Insert the Screw 2's (8) and tighten with a Philips head screwdriver.
- C.** Your Pneumatic Height Adjustable Exam Stool with Backrest is ready to use.

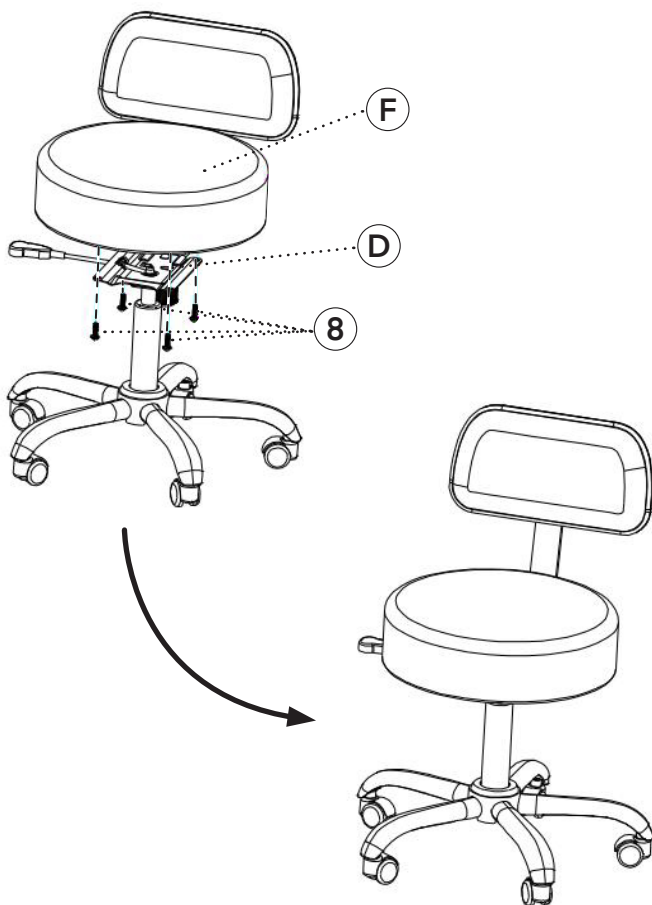

# Care & Maintenance Guide

**Disclaimer:** Viva Comfort will not be held responsible for any damage to the upholstery or surfaces of our products due to cleaning attempts made, whether correctly or improperly conducted. We will not be liable for any damage to the upholstery or the product surfaces, such as discoloration or deterioration, from attempting to clean.

Thank you for purchasing a Viva Comfort product. We are honored to have your business and serve your medical equipment needs. To ensure your product(s) stay in optimal condition, even with consistent use, we have crafted a Care & Maintenance Guide to help you properly maintain and sanitize your product. Please carefully read these guidelines and follow the recommendations listed. Please contact our customer service representatives if you have any questions regarding these instructions.

**Note:** Be mindful when using sharp or pointed objects in the vicinity of upholstery and exam tables. They can cause damage to the exam table surfaces and puncture or tear the upholstery.

**Note: DO NOT** use pumice stones, scouring pads, steel wool, or textured cleaning pads to clean any surface of the exam table or the upholstery. We recommend a microfiber towel or a soft cloth.

**Note: DO NOT** store steel wool on/in your exam tables or on your upholstery.

# Upholstery Cleaning:

Viva Comfort uses premium antimicrobial upholstery on all our exam tables. Consistent care should be taken to maintain the upholstery's integrity. We recommend that you always test the solutions below on an inconspicuous spot before using them on the entire upholstery. The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our upholstery.

**The following Cleaners, Cleaning Materials, and Chemical Makeups will deteriorate and cause permanent damage to your upholstery.**

**Please Note:** Should anything spill on your upholstery, wipe it up immediately and rinse it several times with water to avoid damage.



- Avoid using acidic or abrasive cleaners, as they can cause damage to your upholstery.
- Avoid using dyes (hair, textile, food) near your exam tables as they will permanently stain your upholstery. Should a dye spill occur, wipe it up as soon as possible with dishwashing detergent or an all-purpose cleaner. This is not guaranteed to solve the problem, it but can help reduce the chance of staining.
- Avoid using drainer cleaners or rust removers near your upholstery. Should drain cleaner spill, wipe it up as soon as possible, and rinse it several times with water. Should rust remover spill, wipe off all residue immediately, wash the area with dishwashing soap, and rinse it several times. Ensure you thoroughly dry the area.

# Upholstery Cleaning:


- Avoid using the following products near or on your exam tables, as they will cause damage to the surface that may result in discoloration and/or deterioration.

CHEMICAL COMPOUND	AFFILIATED NAMES
<b>Silicon Dioxide</b>	Pumice/Pumice Stones, Degreaser
<b>Sodium Hydroxide</b>	Caustic Soda, Caustic, Lye, Soda Lye, Degreaser, Drainer Cleaner
<b>Hydrofluoric Acid</b>	Rust Remover, Toilet Bowl Cleaner, Concrete Cleaner, Metal Polishes
<b>Phosphoric Acid</b>	Tile Cleaner, Toilet Bowl Cleaner, Rust Remover, Concrete Cleaner
<b>Sulfuric Acid</b>	Oleic Acid, Oil of Vitriol, Oleum, Drain Cleaner, Heavy-Duty Acidic Cleaner
<b>Hydrochloric Acid</b>	Muriatic Acid, Hydrogen Chloride, Toilet Bowl Cleaner, Calcium, Lime and Rust Remover

Below are the recommended cleaning instructions for our antimicrobial upholstery.

**Please Note: DO NOT** use any alcohol-based cleaning agents, as this will degrade the upholstery.

### MILD SOIL LEVEL

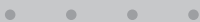

If the surface is lightly soiled, we recommend using a solution comprised of mild dish soap (10%) mixed in warm water (90%). Saturate a soft towel and clean the upholstery gently. In a separate bucket, have clean water and a soft towel. Thoroughly rinse the upholstery with clean water, as well as dry it completely.

### MODERATE SOIL LEVEL



If the surface is moderately soiled, we recommend using a solution comprised of Fantastik® (50%) and warm water (50%). Saturate a soft towel and clean the upholstery gently. In a separate bucket, have clean water and a soft towel. Thoroughly rinse the upholstery with clean water, as well as dry it thoroughly.

### SEVERE SOIL LEVEL



If the surface is severely soiled, we recommend using a solution comprising of bleach (10%) and warm water (90%). Saturate a soft, **white** towel and clean the upholstery gently. In a separate bucket, have clean water and a soft, **white** towel. Thoroughly rinse the upholstery with clean water, as well as dry it completely. Since this solution utilizes bleach, please only use it when necessary; ensure all the bleach solution is removed from the bed, do not let the solution sit on the upholstery, and understand that discoloration may occur.

- 
- Fantastik® is a registered trademark of DowBrands, Inc.®, Registered trademark of the Dow Chemical Co.®

## Exam Bed Surface Cleaning:

The following Cleaners, Cleaning Materials, and Chemical Makeups will deteriorate and cause permanent damage to your exam beds' surfaces.

**Please Note:** Should anything spill on your exam table's surface, wipe it up immediately and rinse it several times with water to avoid damage.



- Avoid using acidic or abrasive cleaners, as they can cause damage to your exam table and its upholstery.
- Avoid using dyes (hair, textile, food) near your exam tables as they will permanently stain your table. Should a dye spill occur, wipe it up as soon as possible with dishwashing detergent or an all-purpose cleaner. This is not guaranteed to solve the problem, it but can help reduce the chance of staining.
- Avoid using drainer cleaners or rust removers near your exam table. Should drain cleaner spill, wipe it up as soon as possible, and rinse it several times with water. Should rust remover spill, wipe off all residue immediately, wash the area with dishwashing soap, and rinse it several times. Ensure you thoroughly dry the area.
- Avoid using the following products near or on your exam tables, as they will cause damage to the surface that may result in discoloration and/or deterioration.



## Compatible Household Cleaners

### For Exam Bed Surfaces

Below is a list of compatible cleaners to use on the exam bed surfaces. We recommend that you always test the solutions below on an inconspicuous spot before using them on the entire surface. The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our exam bed surfaces. We will not be liable for any damage to your product surfaces, such as discoloration or deterioration, from attempting to clean.

(These cleaner recommendations are only for exam bed surfaces; DO NOT use the below cleaners on the upholstery. For instructions on how to clean your upholstery, refer to our "Upholstery Cleaning" section.)



Dawn®  
Fantastik®  
Windex®

Clorox®  
Lysol®  
Formula 409®

Glass Plus®  
Lestoil®

- 
- Fantastik® is a registered trademark of DowBrands, Inc.®, Registered trademark of the Dow Chemical Co.®
  - Dawn® is a registered trademark of The Procter & Gamble Company®.
  - Windex® is a registered trademark of S.C. Johnson & Son, Inc.®.
  - Clorox® is a registered trademark of The Clorox Company®.
  - Lysol® is a registered trademark of Reckitt Benckiser LLC®.
  - Formula 409® is a registered trademark of The Clorox Company®.
  - Glass Plus® is a registered trademark of Reckitt Benckiser LLC®.
  - Lestoil® is a registered trademark of The Clorox Company®.

# Laminate Surface Cleaning:

**Note:** Though the tips below are recommended for cleaning soiled and stained areas, we will not be liable for any damage to the product surfaces, such as discoloration or deterioration, from attempting to clean.

**Note:** The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our product.

**Note:** Be sure to utilize proper protective equipment before handling the below cleaning solutions.

<div>MILD SOIL LEVEL</div> <div><div></div><div></div><div></div><div></div></div>	<p>If the surface is lightly soiled, we recommend using a soft cloth or microfiber towel dampened with a mild soap and water mixture to clean the area.</p>
<div>MODERATE SOIL LEVEL</div> <div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>	<p>If the surface is moderately soiled, we recommend using a solution comprised of a mild household cleaner and baking soda. Create a paste consistency with the recommended cleaning agents, take a nylon bristle brush, and lightly scrub the affected area. <b>DO NOT</b> use a metal bristle brush nor aggressively scrub, or you will cause permanent damage to the surface. Thoroughly rinse the area with clean water, as well as dry it thoroughly.</p>
<div>SEVERE SOIL LEVEL</div> <div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>	<p>If the surface is severely soiled, we recommend using undiluted bleach. Use a cotton pad/ball with bleach on it to gently rub the affected area for a minute or so. Thoroughly rinse the area with clean water, as well as dry it completely. Since this solution utilizes bleach, please only use it when necessary; ensure all the bleach solution is removed from the area, do not let the solution sit on the surface, and understand that discoloration may occur.</p>

# Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.



**ADI910-03**

© 2023 Viva Comfort

**vivacomfort.com**