




.....

ARISTO H-BRACE TABLE

TILTED MATTRESS

.....

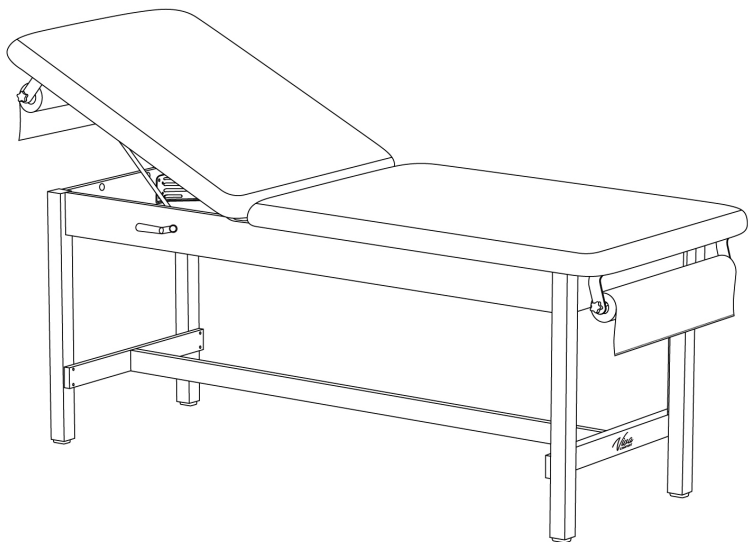

USER GUIDE

SKU: ADI996-03-SERIES



Thank you for your purchase.

.....

Viva Comfort maintains a strong practice of ongoing development of products and commercial research.


.....

We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.


Table of Contents

3	Parts List
4	Hardware List
4	Tools Needed (Not Included)
5	Preparation and Inspection Before Use
5	Assembly Instructions
9	Shelf Installation
11	Shelf Cabinet Installation
19	Double Drawer Installation
23	Paper Roll Dispenser Installation
25	Backrest Angle Adjustment
27	Care & Maintenance Guide
28	Safety Warning
30	Warranty
31	Notes


Below are the optional configurations available in the Aristo Line. Assembly instructions for the table and select optional accessories are included; however, not all steps may apply depending on the specific components or configuration purchased.

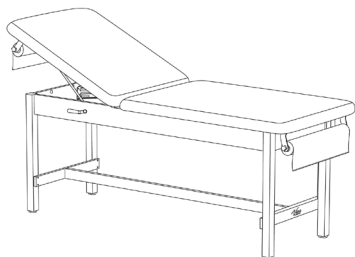


Aristo H-Brace Table
Tilted Mattress

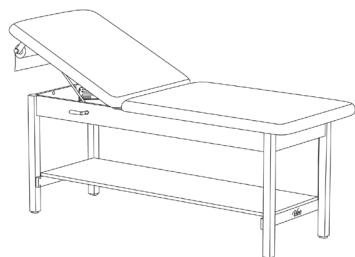

Aristo H-Brace Table
with Shelf
Tilted Mattress

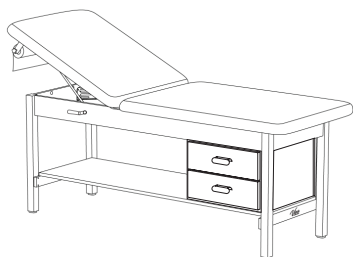


Aristo H-Brace Table
with Shelf and Double Drawer
Tilted Mattress

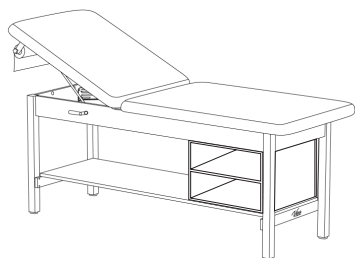

Aristo H-Brace Table
with Shelf and Shelf Cabinet
Tilted Mattress

Parts List

A

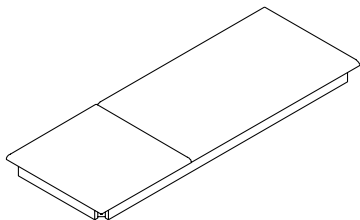

Tabletop

B


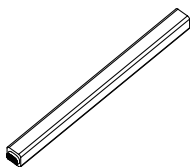


Table Legs (x4)

C

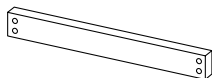



Short Beam (x2)

D





Long Beam

Hardware List

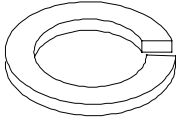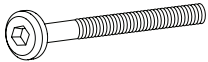1  T30 Torx Bolt (x8) (M8*85 mm)	2  M8 Washer (x8)	3  Torx Bolt (x12) (1/4*42 mm)
--	---	---

Tools Needed (Not Included)

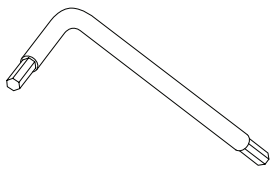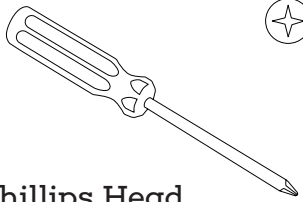 T30 Torx L-Wrench	 Phillips Head Screwdriver
---	--

Preparation and Inspection Before Use

- A.** Confirm whether a product manual and warranty card accompany the product.
- B.** Check whether the installation hardware package is attached.
- C.** Check whether the components are deformed or damaged.
- D.** If any abnormalities occur during installation and use, please get in touch with the dealer in time.


Assembly Instructions

Step 1:

- A.** Please take the product out of the packaging and place it on a horizontal ground with the Tabletop (A) facing down and the frame facing up.
 - a.** Please lay a blanket on the ground to prevent the leather of the table from being scratched, and make sure the ground is clean and free of pollutants or sharp objects.

Step 2:


- A.** Position the Table Legs (B) so they stand vertically on the ground, ensuring the end with the installation holes faces the table surface.
- a.** Align these holes with the metal connection angle on the Tabletop (A) frame.
- B.** Insert the eight T30 Torx Bolts (1) with M8 Washers (2) through the holes in the metal connection angle and tighten securely to attach the Table Legs (B) to the Tabletop (A) frame.

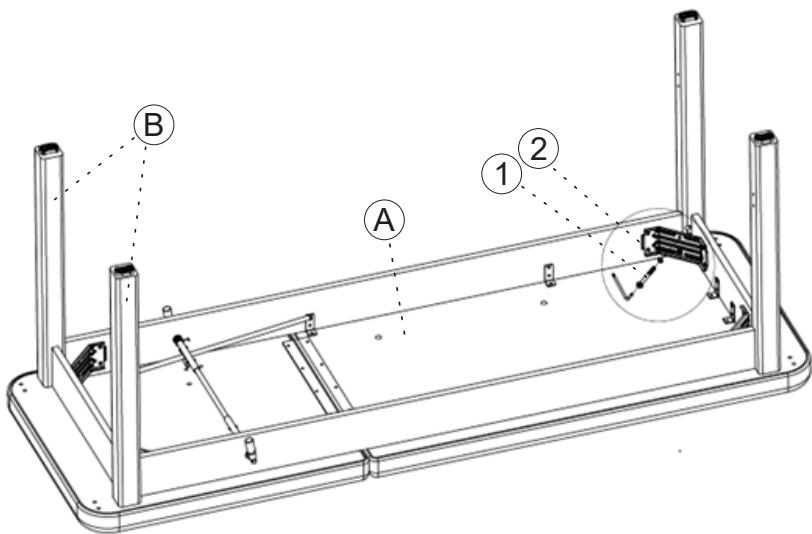

Note: If the thread of the built-in nut inside the table leg is misaligned with the pre-drilled hole, use a flat-blade screwdriver to gently adjust the angle of the nut until the threads align properly with the installation hole.


Step 3:

- A.** Use eight Torx Bolts (3), as shown, to attach both ends of the Short Beams (C) to the inner sides of the adjacent wooden Table Legs (B), as pictured, using a Torx Wrench.

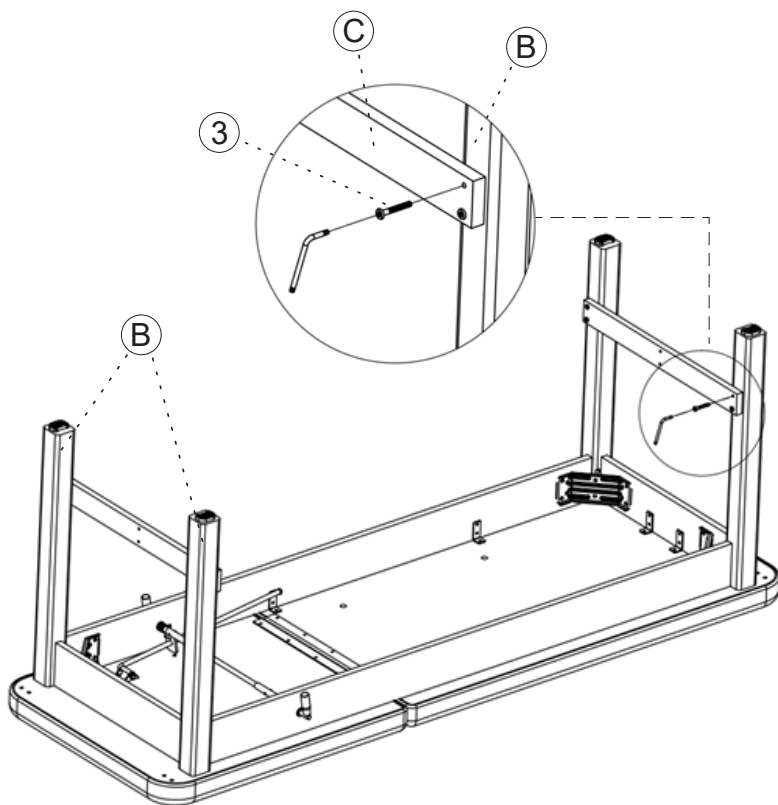


Step 4:

- A.** Use four Torx Bolts (3), as shown, to secure both ends of the Long Beam (D) to the left and right Short Beams (C), using a Torx Wrench.

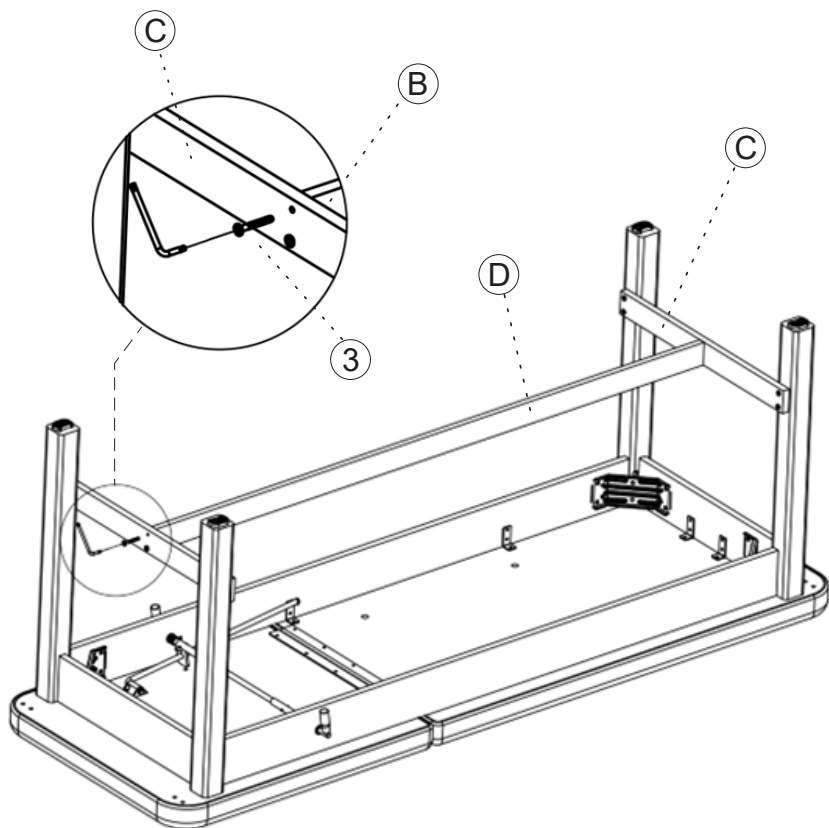


Shelf Installation (Optional)

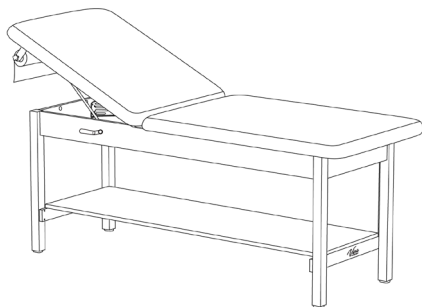

Parts List


E

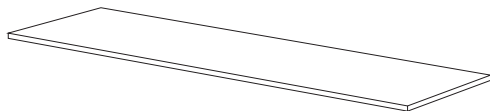

Shelf


Hardware List

4

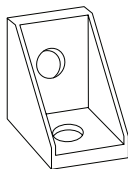

Corner Brace
Bracket (x4)

5

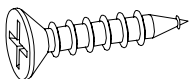


Phillips Screw
(x8)

Step 5:


- A.** Turn the table over so that the tabletop is facing up and the legs are facing downward.
- a.** Ensure the table is standing securely on the ground.

Step 6:







- A.** Place the Shelf (E) in the correct position.
- B.** Use Phillips Screws (5) to attach all four of the Corner Brace Brackets (4) to both the Shelf (E) and the Short Beam (C) located beneath it.

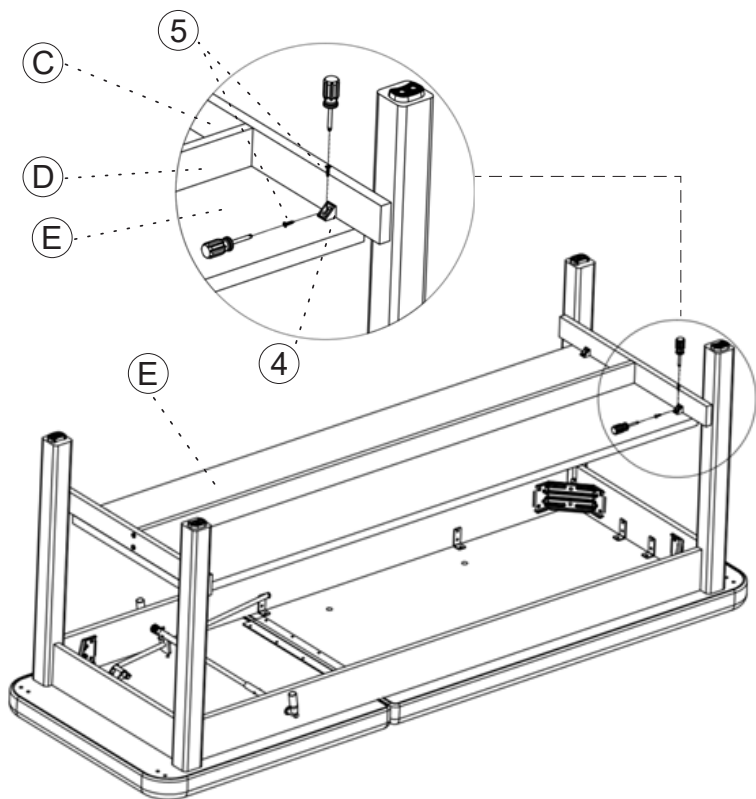








Shelf Cabinet Installation (Optional)

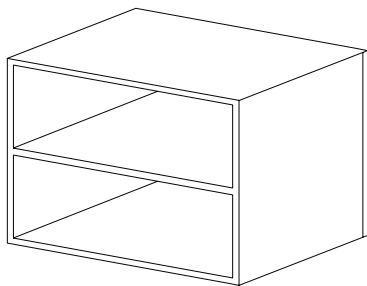

Parts List

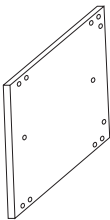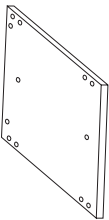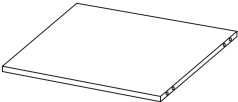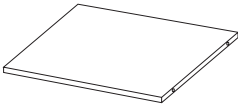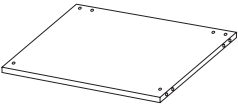
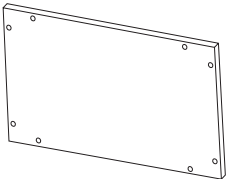
H  Left Panel	I  Right Panel	J  Top Panel
K  Middle Shelf	L  Bottom Panel	M  Back Panel

Hardware List

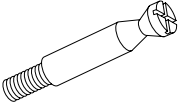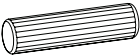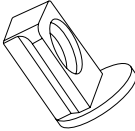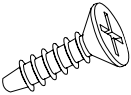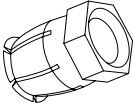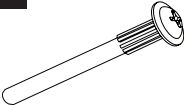
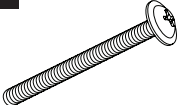
6  Drawer Connecting Dowel (x18)	7  Wooden Dowel (x9) (6*30mm)	8  Shelf Support (x4) (20*10*14)	9  Shelf Support Screw (x5) (20*10*14)
10  Drawer Connecting Nut (x2)	11  Drawer Connecting Fastening Nut (x2)	12  Drawer Connecting Screw Bolt (x18)	13  Shelf Support Hardware (x5) (20*10*14)
14  Interlocking Bolt 1 (x9) (5*43-M4)	15  Interlocking Bolt 2 (x9) (5*43-M4)		

Step 7:


- A.** Insert all Wooden Dowels (17) into the pre-drilled holes on the panels to be assembled.
- B.** Install the Drawer Connecting Screw Bolts (13) into the Drawer Connecting Fastening Nuts (11), then insert them into the adjacent holes near the Drawer Connecting Dowel (1) on the designated panels.
- C.** This product uses a four-in-one connector. Use a screwdriver to rotate the embedded lock cap so that the slot is aligned parallel with the axis of the Drawer Connecting Dowel (12).

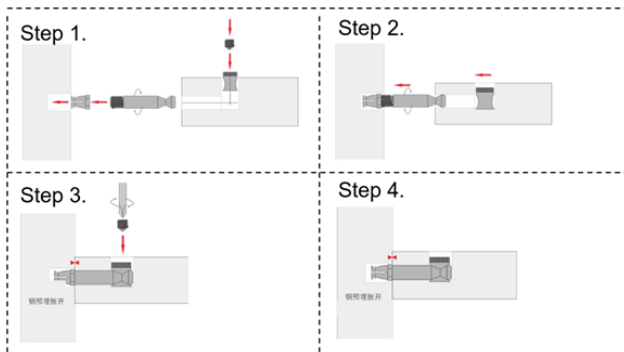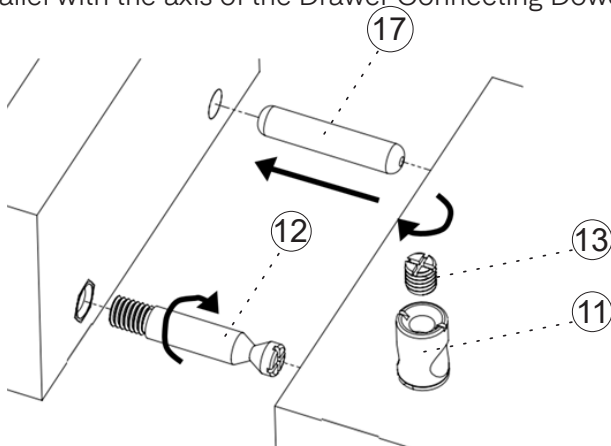


⑥ Connecting Dowels

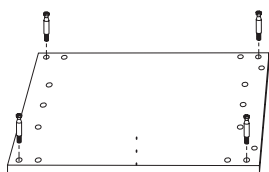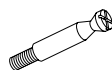

H

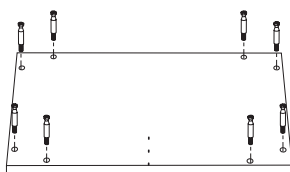


M

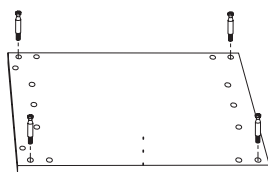


I

⑦ Wooden Dowel Pins

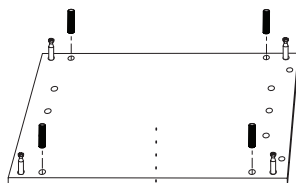


H

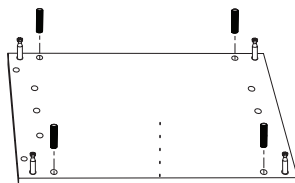

I

Step 8:


- A.** Using the Shelf Support Screws (9), secure the four Shelf Supports(8) into the pre-drilled holes on the inner sides of the Left Panel (H) and Right Panel (I) at your desired shelf height, as pictured.

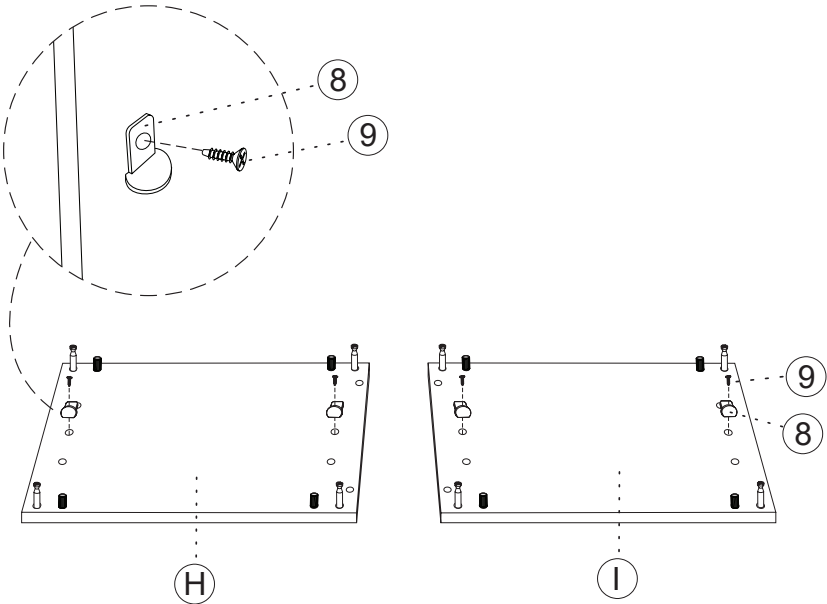

Step 9:


A. Attach the Left Panel (H), Back Panel (M), and Right Panel (I) to the Bottom Panel (L) and Top Panel (J).

a. Push all panels together until they are fully flush, then secure in place.

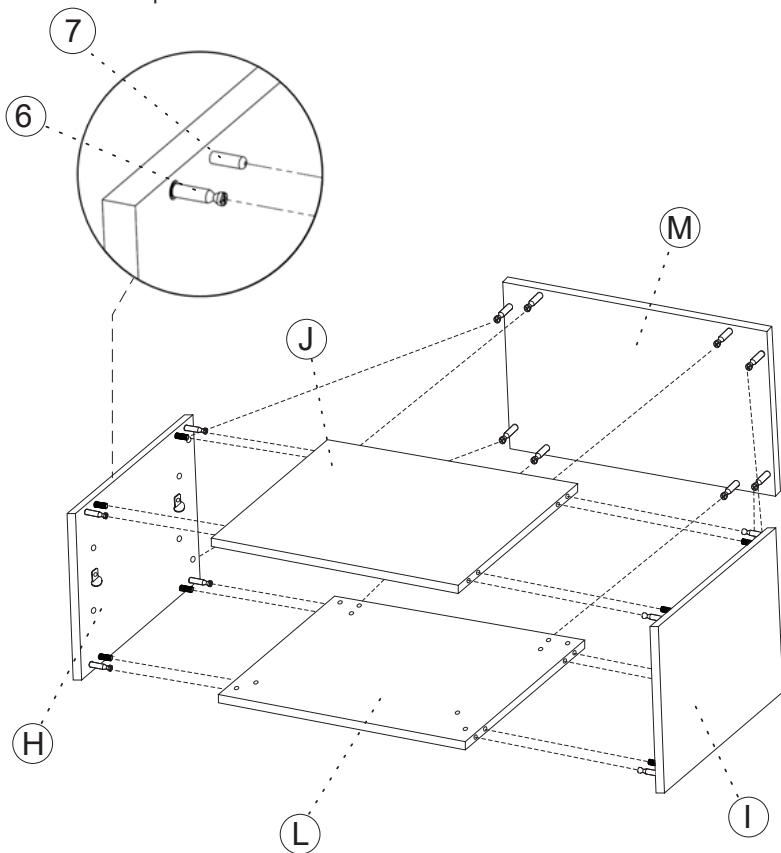



Step 10:

- A.** Slide the cabinet from the side of the table and place it on the shelf, as shown in the figure.
- B.** Use the four Interlocking Bolt 1's (14) and Interlocking Bolt 's (15) to fix the cabinet to the shelf.

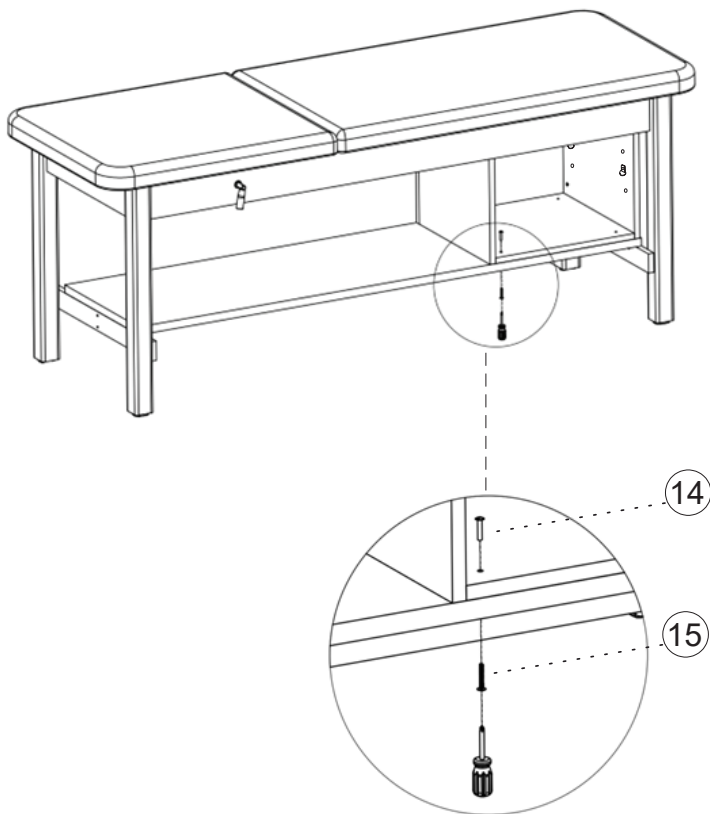


Step 11:


- A.** Slide the Middle Shelf (K) onto the installed Shelf Supports (8), ensuring it rests evenly and securely on all supports.

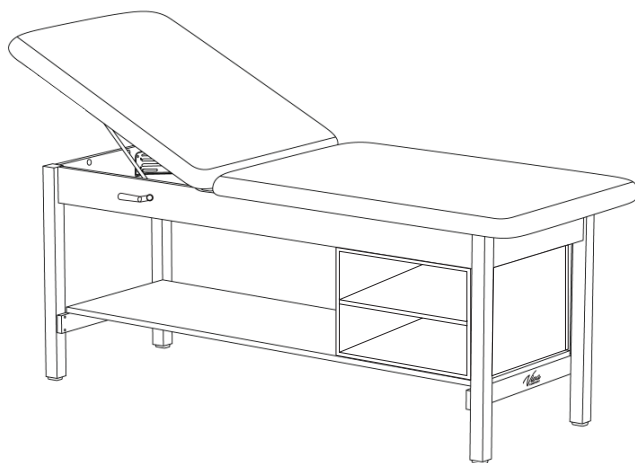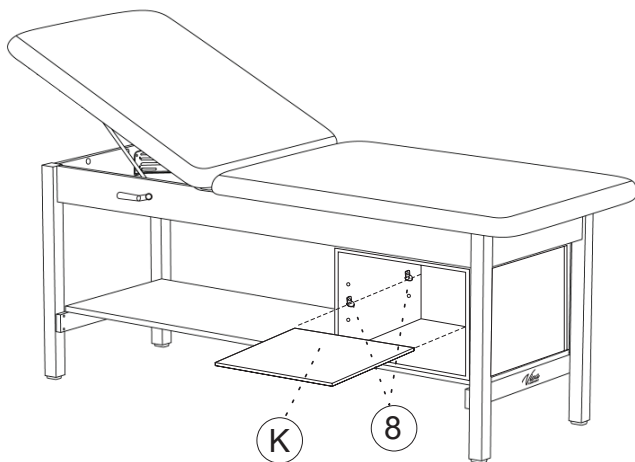


Double Drawer Installation (Optional)

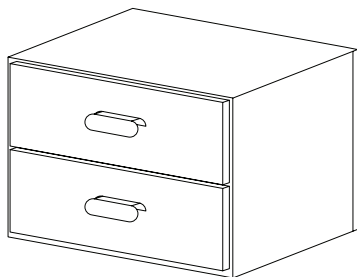

Parts List

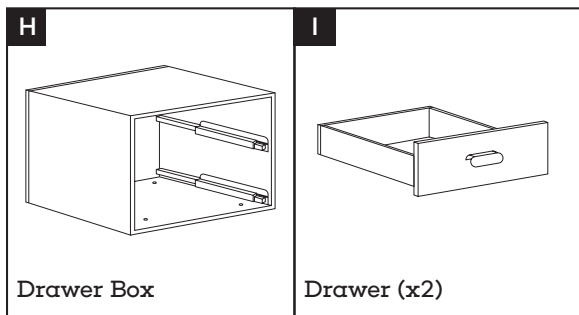


Hardware List

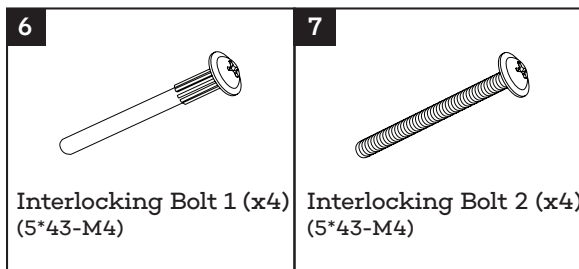


Step 12:

- A.** Remove the Drawers (I) from the Drawer Box (H).
- a.** When removing the Drawers (I), you must first pull out the guide pins on both sides of the bottom of the Drawers (I).

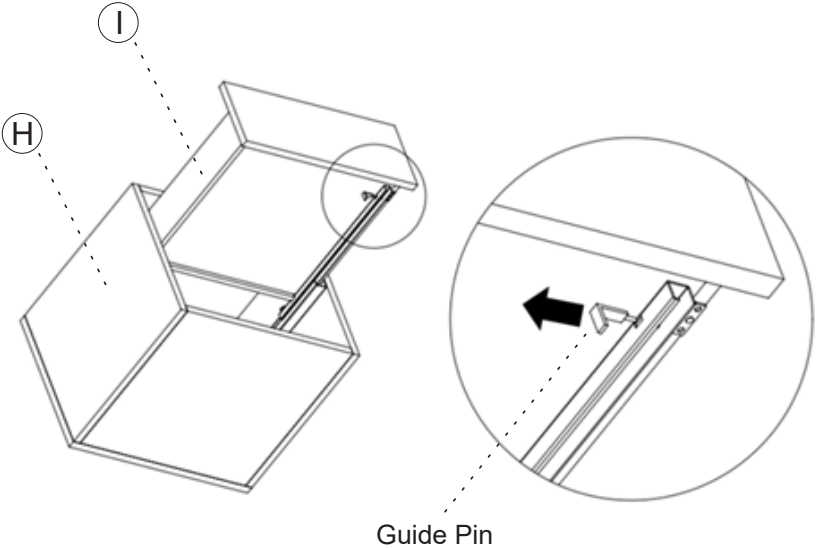



Step 13:

- A.** Slide the Drawer Box (H) from the side of the table and place it on the shelf, as shown in the figure.
- B.** Use the four Interlocking Bolt 1's (6) and the four Interlocking Bolt 2's (7) to fix the cabinet to the shelf.

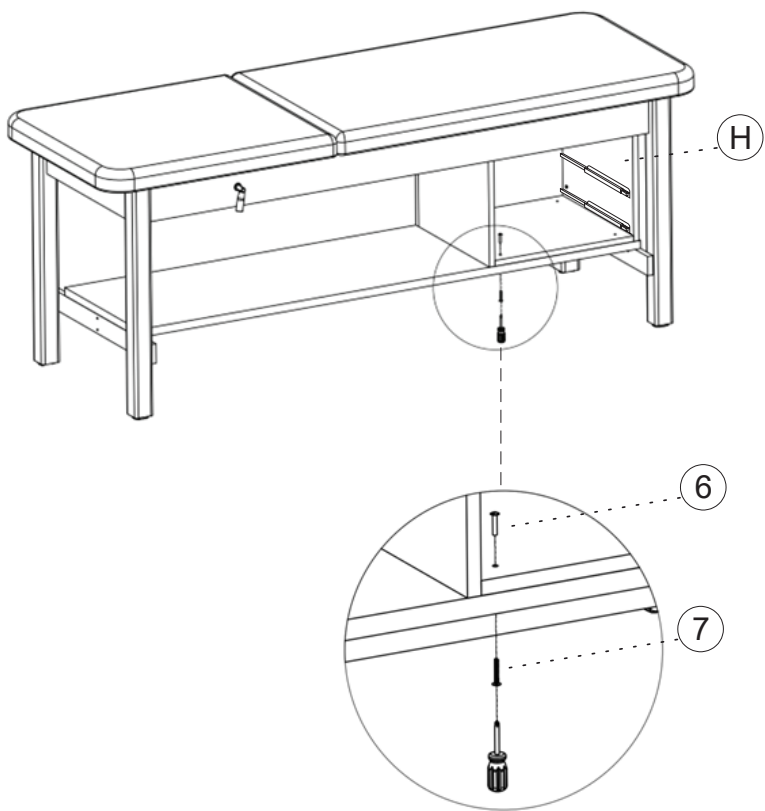


Step 14:

- A.** Install the two Drawers (I) back to the Drawer Box (H) by reinserting the guide pins to fix the guide rails to the Drawers (I).

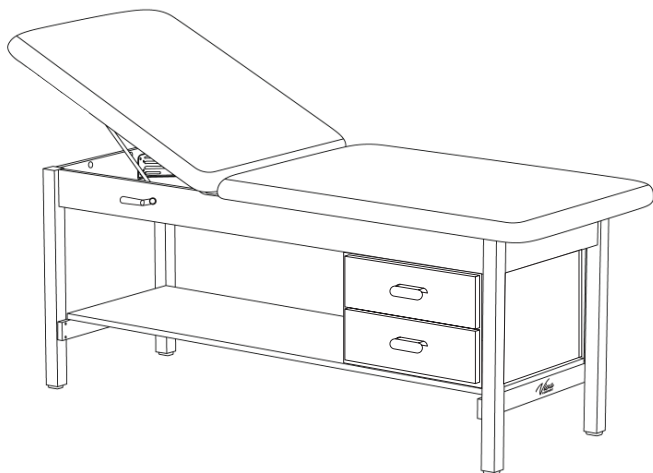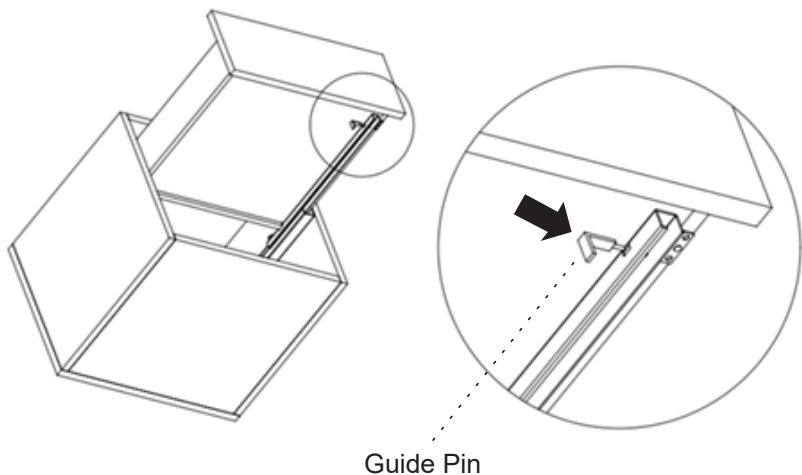


Paper Roll Dispenser Installation (Optional)

Parts List

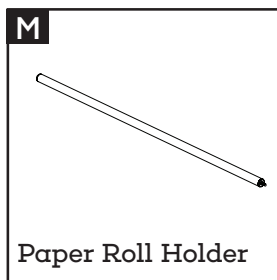


Hardware List

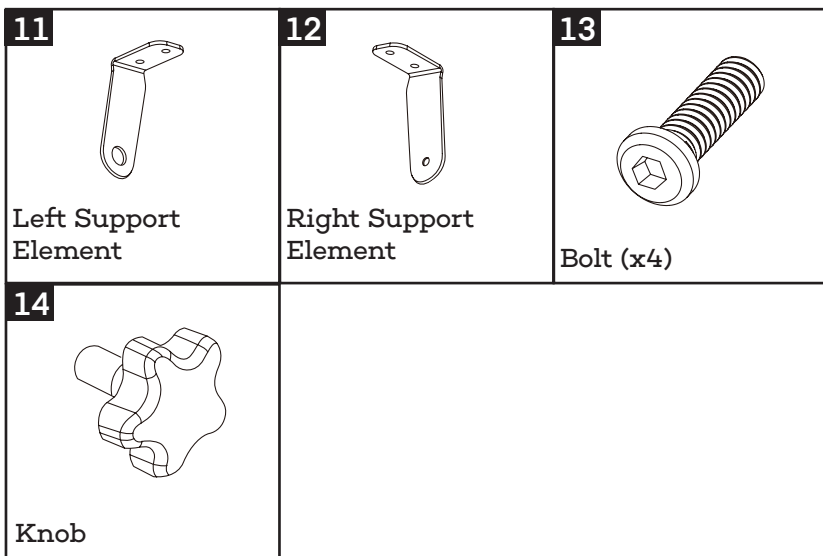

Step 15:


- A.** Use four Bolts (13) to secure the Left Support Element (11) and Right Support Element (12) to the back of the backrest as shown. Tighten using a Phillips head screwdriver.
- B.** Thread the Paper Roll Holder (M) through both ends of the installed supports. Then, secure one end using the included Knob (14).

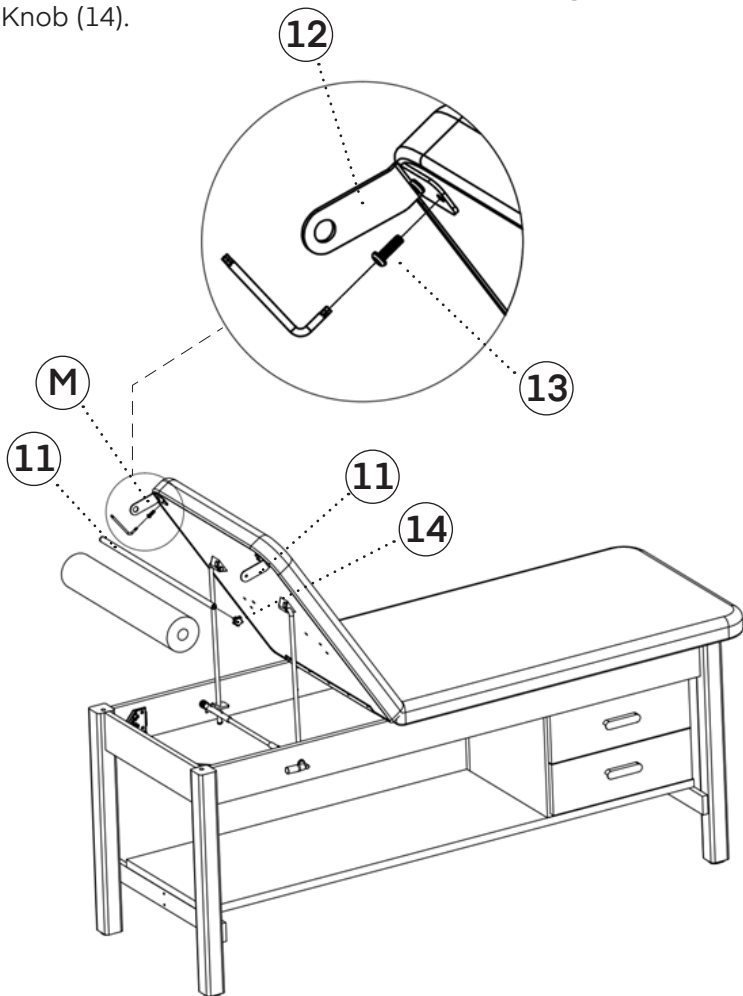

Backrest Angle Adjustment


Raising the Backrest Position:

- A.** To raise the backrest position, firmly hold the backrest and gently lift to your preferred angle. The mechanism will automatically engage and secure the position upon release.

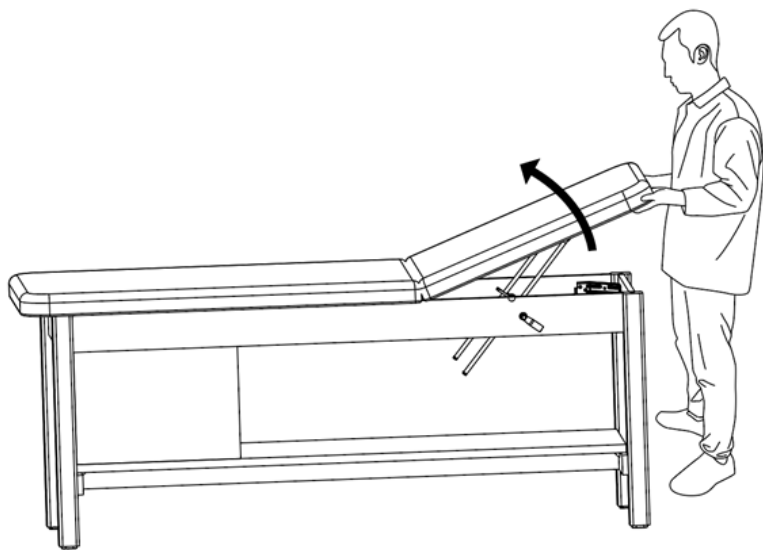

Lowering the Backrest Position:

- A.** To lower the backrest, lift the release handle, as pictured, upward and gently guide the backrest downward to the desired angle.
- B.** Once the backrest is in position, release the handle to engage the automatic locking mechanism.

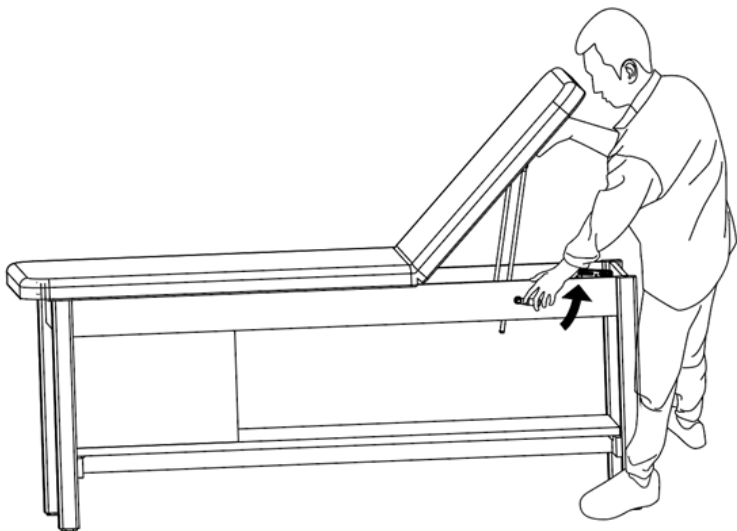

Note: Do not adjust the backrest angle while the patient is leaning against it. Doing so may cause the backrest to drop suddenly, posing a risk of injury—such as pinching or crushing the operator's arm.

Care & Maintenance Guide

A. Leather Maintenance:

a. It is recommended to use a small amount of neutral detergent/cleaner or wipe with a dry towel when wiping the leather, then wipe it clean with a dry cloth, dry it, and place it in a ventilated place to dry.

b. If disinfection is necessary, it is acceptable to use alcohol below 70% concentration to wipe it. It is prohibited to use hypochlorous acid disinfectant or other highly corrosive disinfectants for disinfection; otherwise, they may accelerate the aging of the leather, causing the leather to change color, deform, harden, become brittle, or even peel off the surface.

c. When not used for a long time, it is recommended that the table be covered with a table cover and stored in a dry environment. Avoid direct sunlight, rain, humidity, and other environments.

d. Too high or too low-temperature environments may accelerate the aging of leather.

e. Sharp objects are prohibited from touching the leather surface.

B. Wood Maintenance:

a. A dry towel is recommended to wipe metal and plates.

b. If disinfection is necessary, it is acceptable to use hypochlorous acid disinfectant with an alcohol concentration below 70% for wiping. It is prohibited to use Hypochlorous disinfectant or other highly corrosive disinfectants for disinfection; otherwise, the paint or wood veneer may be damaged.

C. Regular Maintenance:

a. Inspect the table surface every 3 months to check the stability of the table frame when it is pushed horizontally, vertically, or laterally. If there is any shaking, tighten the connecting bolts.

Safety Warning

A. The user is ultimately responsible for preventing product damage or injury to the patient and/or surgical staff.

B. Collapse risk:

 If the patient's weight exceeds the working load, the medical treatment table and the backrest may collapse if the patient's weight is not evenly distributed on the table surface.

C. Pinch risk:

 When the backrest of the table surface moves down to the end, the hand or finger is placed in the gap between the backrest and the table frame.

 When the drawer is closed, the hand or finger is placed in the gap between the drawer panel and the cabinet.

D. Infection risk:

When cleaning, please wear gloves, masks, and eye protection to prevent the cleaning spray from rebounding from the contaminated surface.

E. Table surface damage risk:

a. The ambient temperature requirement for using and storing the equipment is 0-40°C. If the equipment is exposed to an environment exceeding the specified temperature for a long time, the leather surface may crack or peel.

b. The humidity requirement for the equipment's transportation, use, and storage is 20%~80%. The leather surface may crack or peel if the equipment is exposed to an overly dry or humid environment for a long time.

c. The leather of the equipment table is strictly prohibited from contacting acidic and alkaline substances. Scrubbing the table with a non-neutral detergent may cause the leather to crack and peel prematurely, reducing its service life.

d. The leather of the table is strictly prohibited from contacting liquid water. Liquid water may penetrate the leather, causing the sponge inside the leather to degenerate, affect use, or crack and peel.

F. Hardware accessories rusting risk

a. The humidity requirement for equipment transportation, use, and storage environment is 20%~80%. If the equipment is exposed to an overly humid environment for a long time, the hardware accessories may rust.

b. The metal parts of the equipment are strictly prohibited from contacting acidic substances; otherwise, they may cause damage to the paint surface and then rust.

c. The metal parts of the equipment are strictly prohibited from contacting liquid water, which may cause metal materials to rust.

7. Other precautions:

a. It is strictly prohibited to approach fire sources.


b. Do not place it in direct sunlight.

c. Do not place it in high-temperature places such as heaters.

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

[illegible]

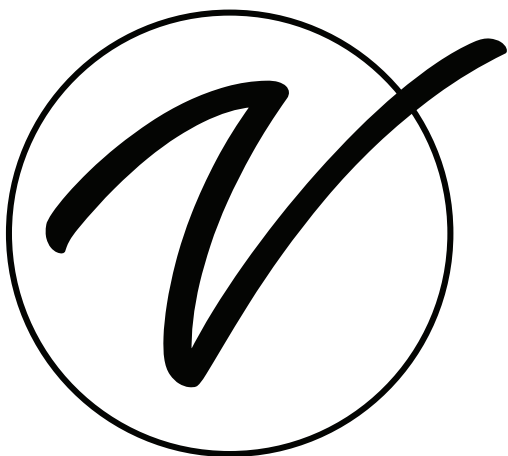

ADI996-03-SERIES

© 2025 Viva Comfort

vivacomfort.com