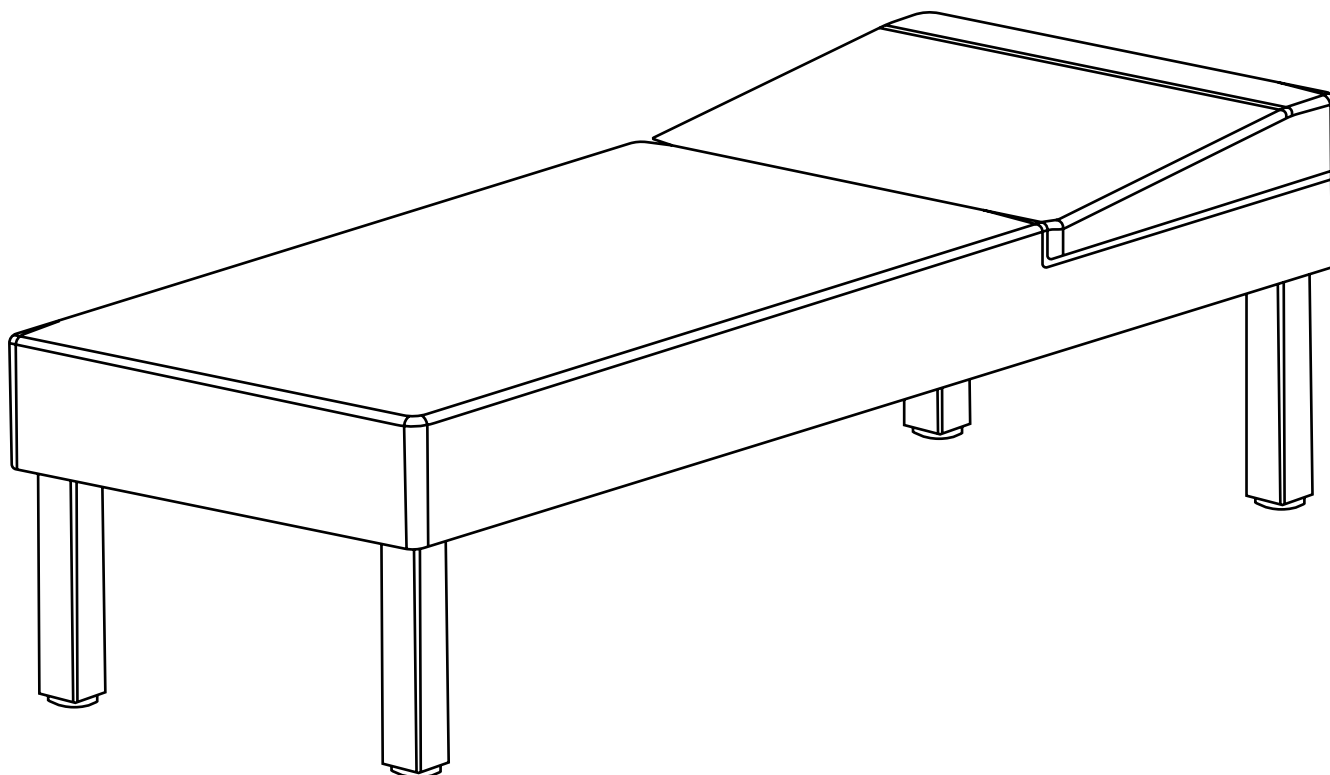


.....

LUCAS RECOVERY COUCH

.....



USER GUIDE

SKU: ADI996-11-SERIES



Thank you for your purchase.



**Viva Comfort maintains a
strong practice of ongoing
development of products and
commercial research.**








**We recognize that starting and maintaining a medical practice is
expensive. We'll help make the process cost-effective and elegant.
Additionally, our team of specialists will ensure everything you
need works as efficiently as possible, so you can effectively treat
your clients.**





Table of Contents

4	Parts List
5	Hardware List
6	Assembly Instructions
8	Adjusting The Table's Backrest Angle
9	Leather Maintenance
9	Regular Maintenance
10	Safety Warning
12	Warranty
13	Notes

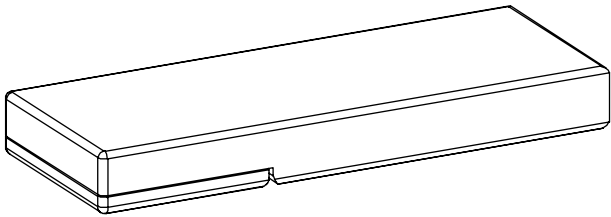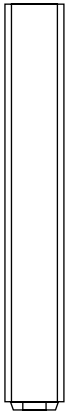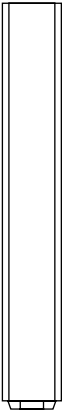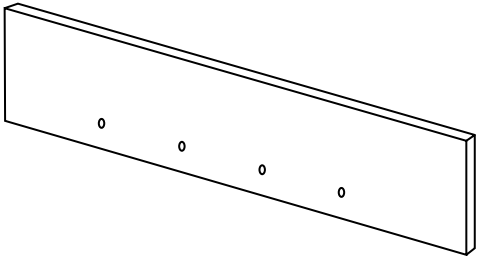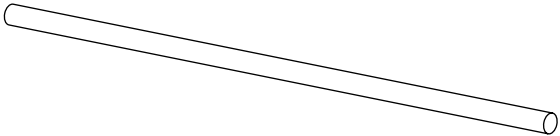Parts List

<div data-bbox="162 247 245 317">A</div> <div data-bbox="183 396 789 611">A perspective drawing of a rectangular table frame with a small notch on one of the longer sides.</div> <div data-bbox="183 684 397 720"><p>Table Frame</p></div>	<div data-bbox="810 247 893 317">B</div> <div data-bbox="1102 273 1166 680">A perspective drawing of a thin, vertical rectangular leg.</div> <div data-bbox="828 684 1050 720"><p>Left Leg (x2)</p></div>
<div data-bbox="162 756 245 825">C</div> <div data-bbox="440 779 501 1186">A perspective drawing of a thin, vertical rectangular leg, identical to the one in section B.</div> <div data-bbox="183 1192 435 1228"><p>Right Leg (x2)</p></div>	<div data-bbox="810 756 893 825">D</div> <div data-bbox="891 884 1364 1138">A perspective drawing of a long, thin rectangular supporting leg with four small circular holes spaced evenly along its length.</div> <div data-bbox="833 1192 1182 1228"><p>Supporting Leg (x2)</p></div>
<div data-bbox="162 1264 245 1333">E</div> <div data-bbox="204 1428 760 1562">A perspective drawing of a long, thin cylindrical rod.</div> <div data-bbox="183 1703 792 1738"><p>Paper Roll Holder (Sold Separately)</p></div>	

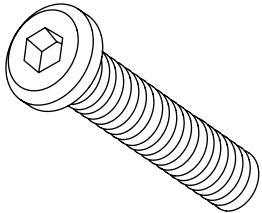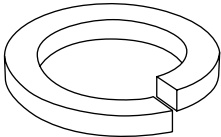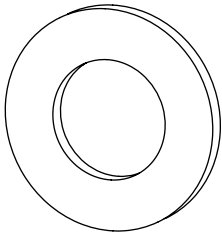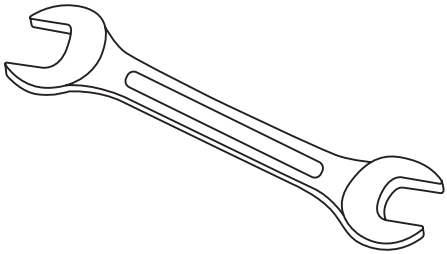Hardware List

<div>1</div> <div></div> <div>Bolt (x8) (M8 x 60 mm)</div>	<div>2</div> <div></div> <div>Spring Washer (x8) (Ø8.5 x Ø12.5 x 1.5 mm)</div>
<div>3</div> <div></div> <div>Flat Washer (x8) (Ø8.5 x Ø18 x 1 mm)</div>	<div>4</div> <div></div> <div>Wrench (13 x 88 mm)</div>

Assembly Instructions

Preparation and Inspection Before Use:

- A.** Confirm whether a product manual and warranty card accompany the product.
- B.** Confirm whether the installation hardware package is attached.
- C.** Confirm whether the components are deformed or damaged.
- D.** If any abnormalities occur during installation and use, please get in touch with the dealer in time.

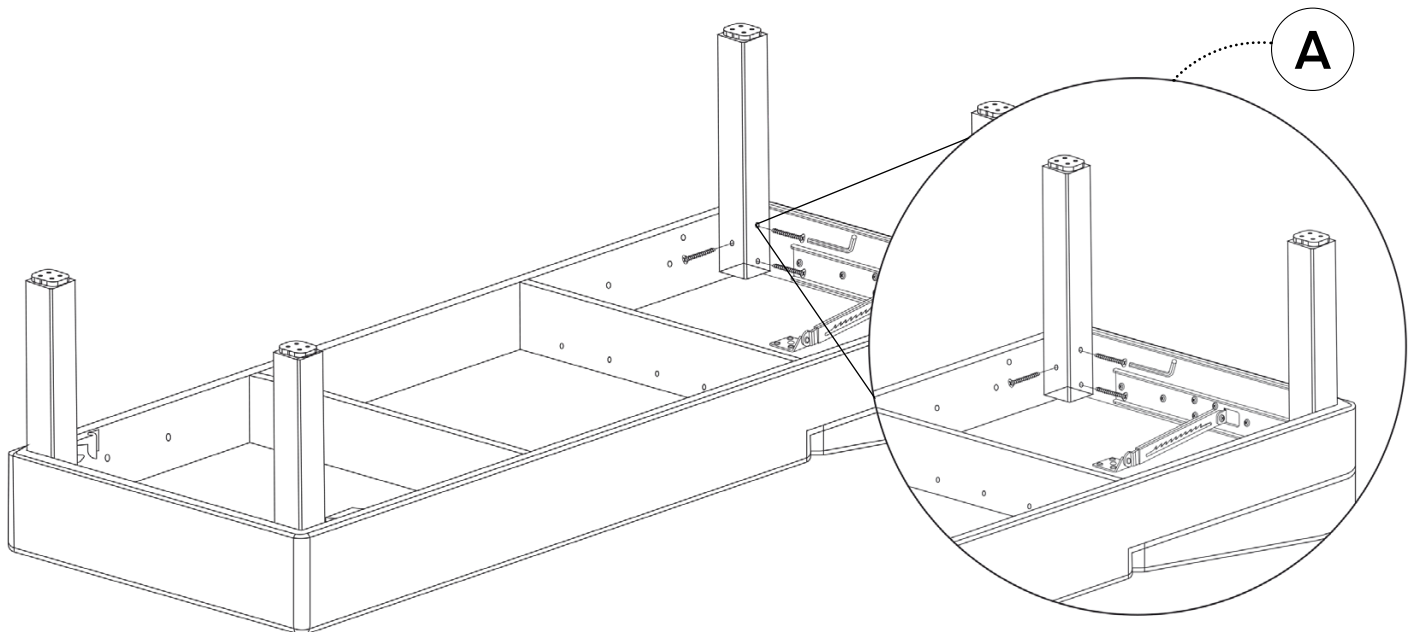
Installing the Table Legs:

Step 1:

- A.** Carefully remove the product from the packaging and place it on a flat surface with the tabletop facing down and the lower frame facing up.
- B.** Lay a blanket or soft cloth underneath the table to protect the leather surface from scratches. Ensure the ground is clean and free of dirt, debris, or sharp objects.

Step 2:

- A.** Identify the Left Legs (B) and Right Legs (C). Position each leg vertically at the corners of the Table Frame (A), ensuring they are perpendicular to the ground.
- B.** Align the side of each leg with two mounting holes to face the longer side of the Table Frame (A). The side with one mounting hole should face the shorter side.




- C.** Insert Bolts (1) through the leg's mounting holes and into the pre-installed threaded inserts in the Table Frame (A). Tighten the bolts clockwise using the included Wrench (4) until the legs are securely fastened.

Step 3:

- A.** Carefully turn the table upright so all legs rest evenly on the floor. Ensure the table is placed on a stable, level surface.


Replacing the Paper Roll:

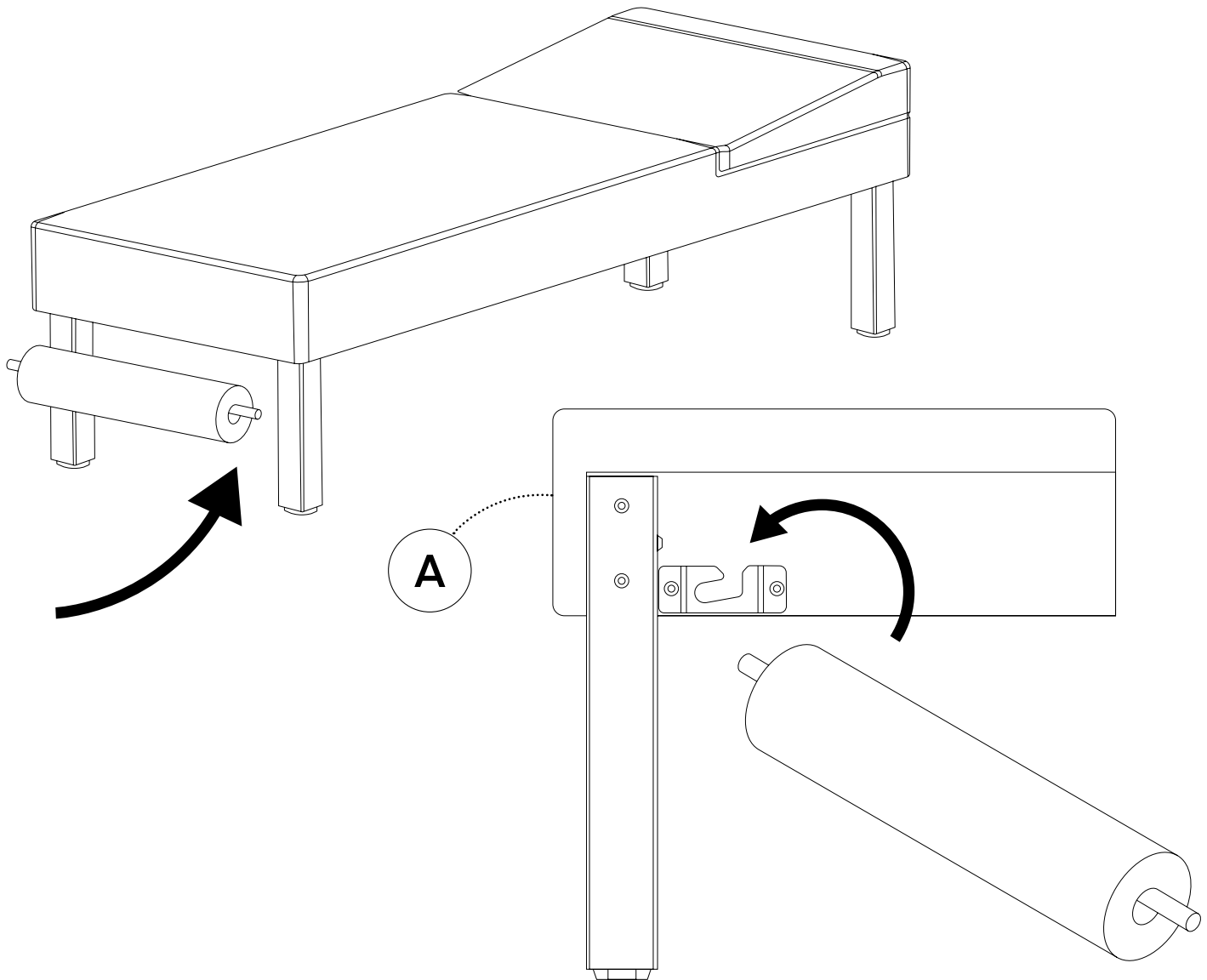- A.** Locate the Paper Roll Holder (E) beneath the Table Frame (A) on the side opposite the backrest.
- B.** Slide the holder away from you, then lift it upward to remove it.
- C.** Replace the empty roll with a new one, ensuring it rotates freely.
- D.** Reinsert the holder into its original position, as pictured below.



Adjusting the Table's Backrest Angle:

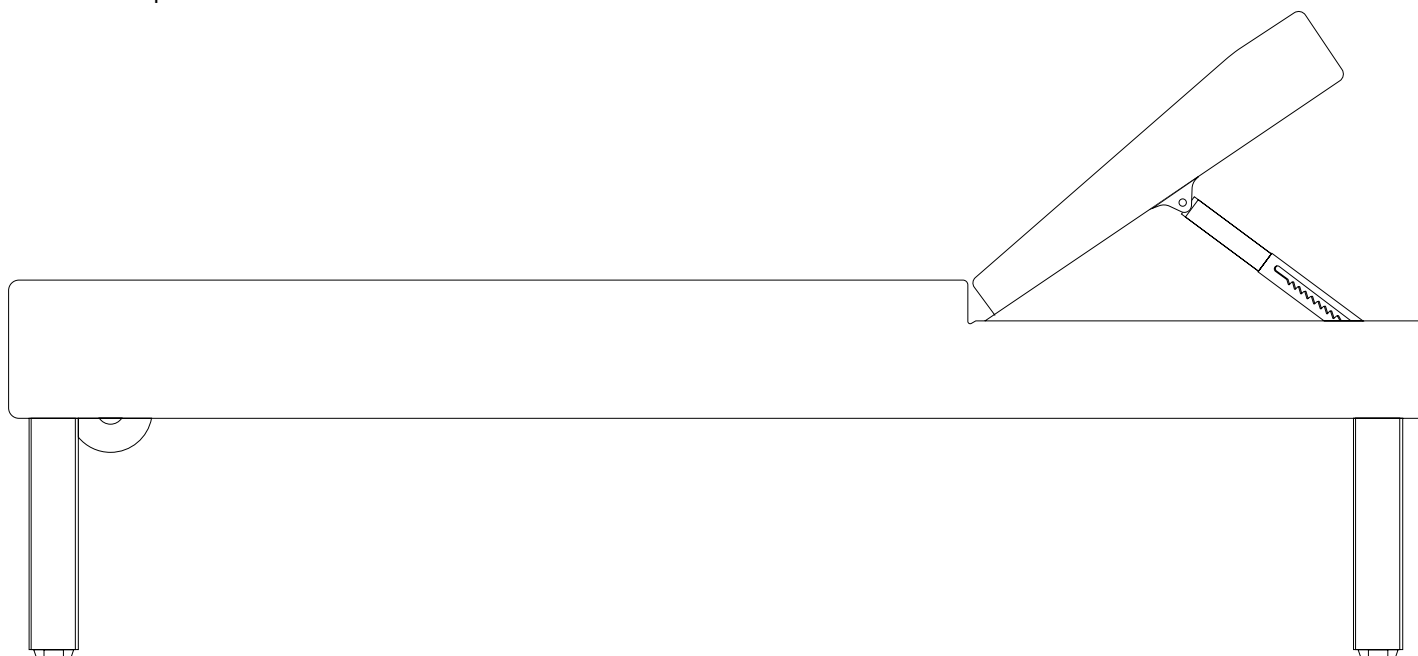- A.** Gently lift the backrest to your desired angle, then release it — the backrest will automatically lock into place.
- B.** To fully recline the backrest: Lift it to its highest position, then slowly lower it until it lies flat in a horizontal position.



Note: NEVER adjust the backrest angle while the patient is leaning on it. Sudden movement due to patient weight may cause injury to the patient or pinching the operator's arm.

Leather Maintenance

- A. Use a small amount of neutral detergent or a dry towel to gently wipe the leather. Then wipe it again with a clean, dry cloth and place the table in a well-ventilated area to dry naturally.
- B. If disinfection is required, use alcohol with a concentration below 70%.

Do not use hypochlorous acid or other strong corrosive disinfectants, as they may cause the leather to discolor, deform, harden, crack, or peel.

- C. If the table will not be used for an extended period, please cover it with a protective cloth or cover, and store it in a dry, ventilated environment, avoiding direct sunlight, rain, and humidity.
- D. Avoid exposing the table to extremely high or low temperatures, which may accelerate leather aging.
- D. Keep sharp objects away from the leather surface to prevent scratches and damage.






Regular Maintenance

Inspect the table frame every 3 months to test its stability by applying pressure in all directions (horizontal, vertical, and lateral).

If any looseness, shaking or instability is detected, check and tighten all connecting bolts to ensure secure fastening.

Safety Warning:

The user is responsible for preventing damage to this product and injury to patients and/or users.

	Warning: Indication of potential personal injury.
	Warning: Be careful of pinching fingers.
	Reminder: Keep away from moisture. This device needs to be moisture and waterproof.
	Reminder: Always place the packaging box in the correct direction during transportation, storage, and handling to avoid damage and safety problems caused by incorrect placement.
	Reminder: This product is fragile and needs to be handled with care.

Risk of collapse:



If the patient's weight exceeds the working load, the treatment table, backrest and leg support may collapse if the patient's weight is not evenly distributed on the table surface.

Pinch Risk:



When the table backrest moves down to the end, a hand or finger is placed in the gap between it and the table frame.

Infection Risk:

When cleaning, please wear gloves, masks, and eye protection to prevent the cleaning spray from rebounding from the contaminated surface.

Risk of Damage to the Table Surface:

- A.** The ambient temperature requirement for using and storing the equipment is: 0-40. If the equipment is exposed to an environment exceeding the specified temperature for a long time, it may cause the risk of cracking and peeling of the leather surface.

- B.** The humidity requirement for equipment transportation, use and storage environment is: 20%~80%. If the equipment is exposed to an overly dry or humid environment for a long time, the leather surface may crack or peel.
- C.** The leather of the equipment table is strictly prohibited from contacting acidic or alkaline substances. If a non-neutral detergent is used to scrub the table surface, the leather surface may crack or peel prematurely, reducing its service life.
- D.** The leather of the table surface is strictly prohibited from contacting liquid water. Liquid water may penetrate the leather, causing the sponge inside the leather to degenerate, affect its use, or cause the leather surface to crack or peel.

Hardware Accessories Rusting Risk:

- A.** The humidity requirement for equipment transportation, use, and storage environment is 20%~80%. If the equipment is exposed to an overly humid environment for a long time, the hardware accessories may rust.
- B.** The metal parts of the equipment are strictly prohibited from contacting acidic substances; otherwise, they may cause damage to the paint surface and then rust.
- C.** The metal parts of the equipment are strictly prohibited from contacting liquid water, which may cause metal materials to rust.

Other Precautions:

- A.** It is strictly prohibited to approach fire sources.
- B.** Do not place it in direct sunlight.
- C.** Do not place it in high-temperature places such as heaters.

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 5 years from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

[illegible]



ADI996-11-SERIES

© 2025 Viva Comfort

vivacomfort.com