


.....

# LUCAS PLUS

## RECOVERY COUCH

WITH ADJUSTABLE BACKREST

.....

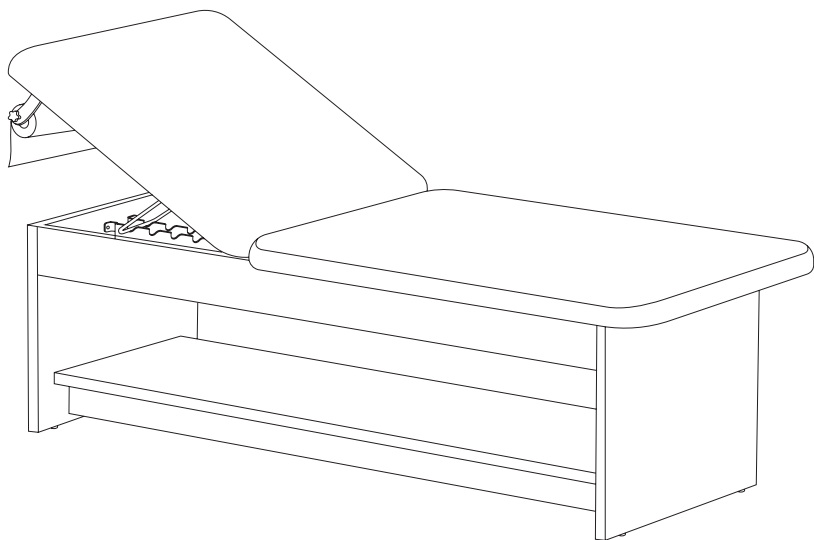

**USER GUIDE**

**SKU: ADI996-12-SERIES**



# Thank you for your purchase.



**Viva Comfort maintains a strong practice of ongoing development of products and commercial research.**









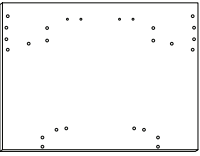
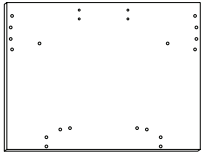
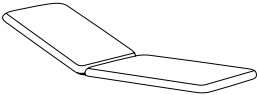



**We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.**

# Table of Contents

- 3** Parts List
- 4** Tools Needed (Not Included)
- 5** Hardware List
- 6** Preparation and Inspection Before Use
- 6** Assembly Instructions
- 16** Paper Roll Dispenser Installation
- 18** Backrest Angle Adjustment
- 19** Care & Maintenance Guide
- 20** Safety Warning
- 22** Warranty
- 23** Notes
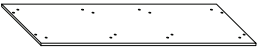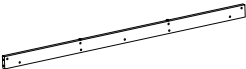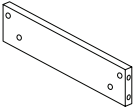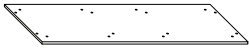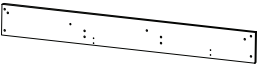
# Parts List

<p><b>A</b></p>  <p>Bottom Frame Board</p>	<p><b>B</b></p>  <p>Bottom Frame Long Beam (x2)</p>	<p><b>C</b></p>  <p>Bottom Frame Short Beam (x2)</p>
<p><b>D</b></p>  <p>Upper Frame Board</p>	<p><b>E</b></p>  <p>Upper Frame Long Beam (x2)</p>	<p><b>F</b></p>  <p>Upper Frame Middle Beam (x2)</p>
<p><b>G</b></p>  <p>Upper Frame Short Beam</p>	<p><b>H</b></p>  <p>Upper Frame Short Beam</p>	<p><b>I</b></p>  <p>Left Leg Panel</p>
<p><b>J</b></p>  <p>Right Leg Panel</p>	<p><b>K</b></p>  <p>Tabletop Base Board</p>	<p><b>L</b></p>  <p>Backrest Support Bracket</p>

# Hardware List

<b>1</b>  Wooden Dowel Pin (6*30mm) (x24)	<b>2</b>  Cam Lock Bolt (x60)	<b>3</b>  Connecting Nut (x)
<b>4</b>  Connecting Fastening Nut (x60)	<b>5</b>  Connecting Screw Bolt (x60)	<b>6</b>  Self-Tapping Screw (x14)
<b>7</b>  L-Shaped Connector (x6)	<b>8</b>  Hex Bolt (x10)	<b>9</b>  Backrest Support Arm
<b>10</b>  Backrest Support Rod	<b>Tools Needed</b> (Not Included)  	

# Preparation and Inspection Before Use

- A.** Confirm whether a product manual and warranty card accompany the product.
- B.** Check whether the installation hardware package is attached.
- C.** Check whether the components are deformed or damaged.
- D.** If any abnormalities occur during installation and use, please get in touch with the dealer in time.



## Assembly Instructions

### Step 1:

- A.** Please take the product out of the packaging and place it on a horizontal ground with the table facing down and the metal frame facing up.
  - a.** Please lay a blanket on the ground to prevent the leather of the table from being scratched, and make sure the ground is clean and free of pollutants or sharp objects.



## Step 2:

- A.** When installing, please insert all Wooden Dowel Pins (1) and Cam Lock Bolts (2) into the reserved holes on all the panels to be assembled.
- B.** Install all the Connecting Screw Bolts (5) on the reserved Connecting Fastening Nuts (4) on the panels to be constructed.
- C.** This product is installed with a four-in-one connector. Please use a screwdriver to adjust the direction of the embedded lock cap slot so that it is parallel to the axis of the Connecting Dowel (2).

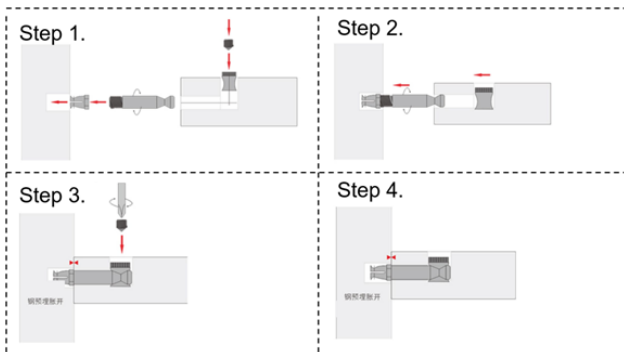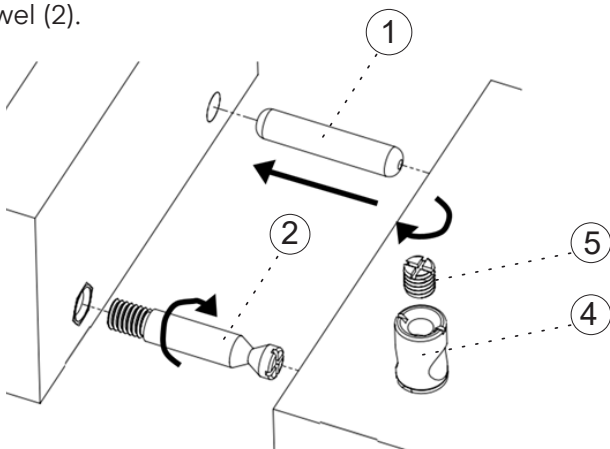


## Bottom Frames

### ② Connecting Cam Lock Bolts (2):

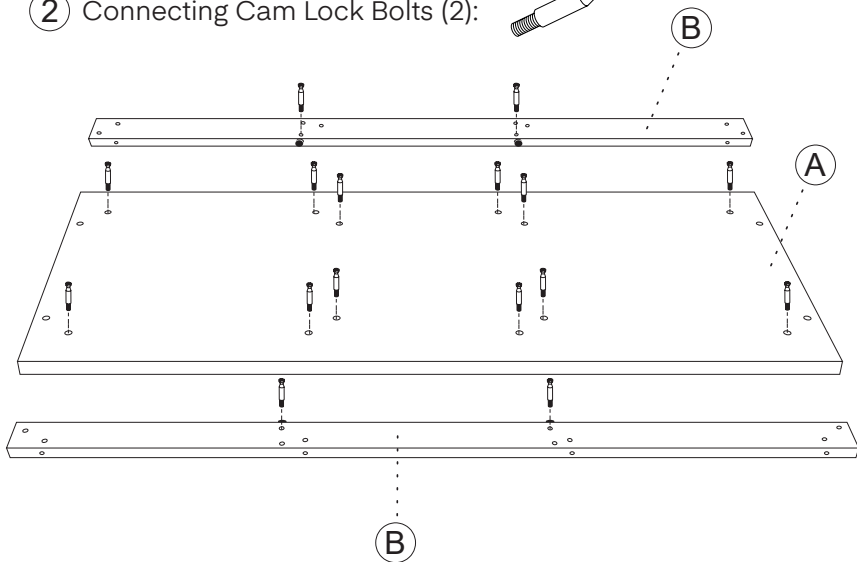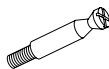


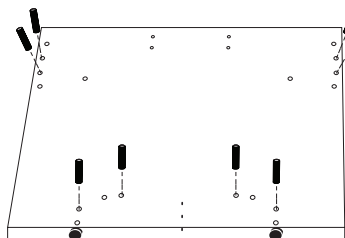
### ① Wooden Dowel Pins (1):

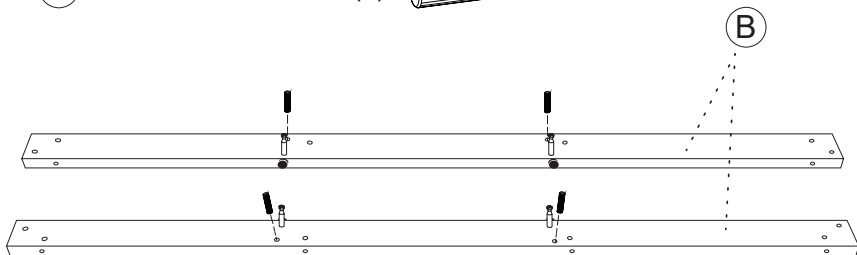




## Leg Panels

- ① Wooden Dowel Pins (1): 




J

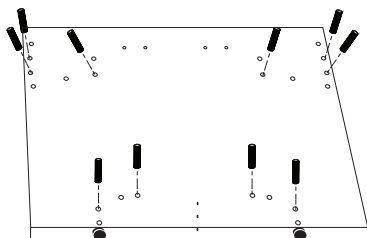


I

- ② Connecting Cam Lock Bolts (2): 

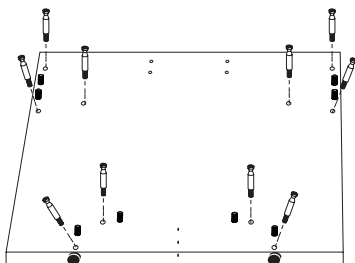



J

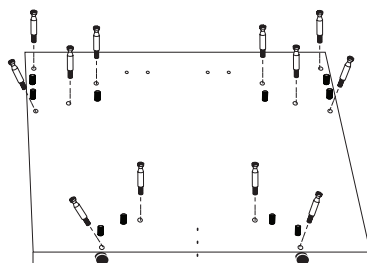


I

## Upper Frames

② Connecting Cam Lock Bolts (2):

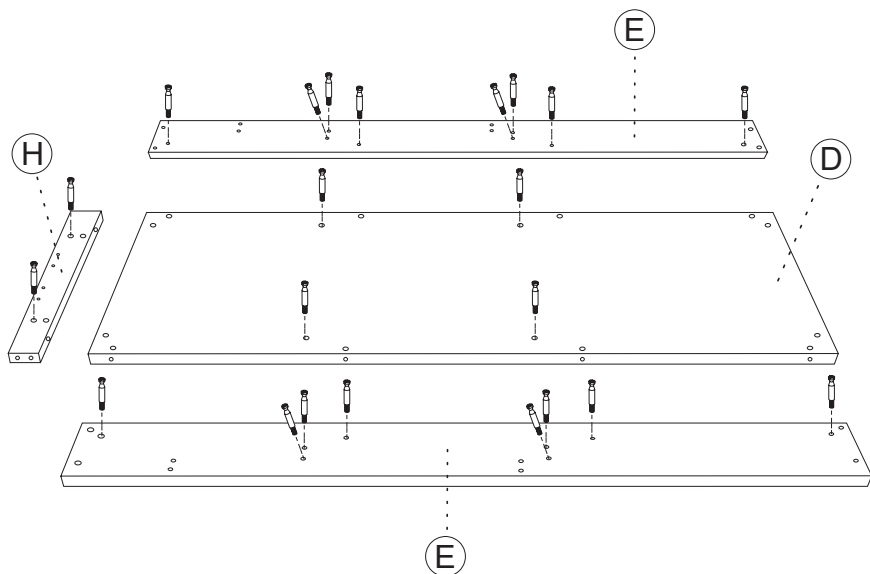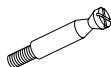


① Wooden Dowel Pins (1):

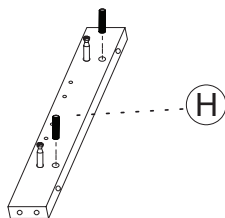


### Step 3:

- A.** Attach the Bottom Frame Long Beam (B) and Bottom Frame Short Beam (C) to the Bottom Frame Board (A).
- B.** Insert the Connecting Fastening Nuts (4) into the designated holes next to the pre-installed Cam Lock Bolts (2), then insert the Connecting Screw Bolts (5). Tighten the Connecting Screw Bolts (5) using a Phillips head screwdriver.

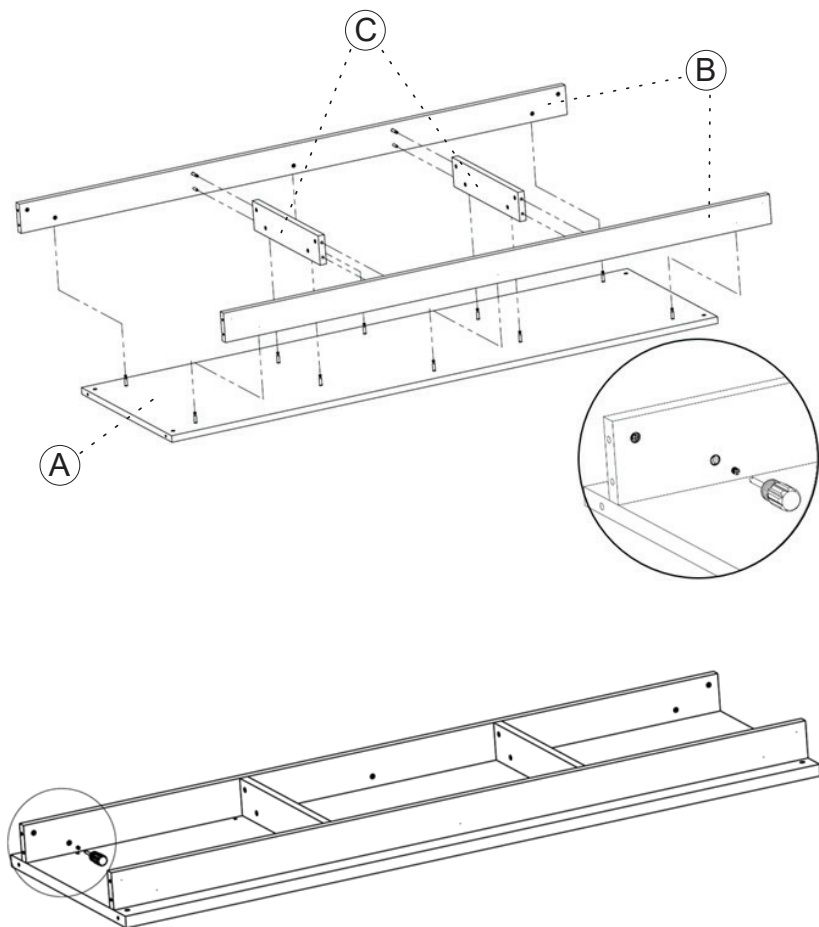

## Step 4:


- A.** As shown in the illustration, attach both Upper Frame Long Beams (E), both Upper Frame Middle Beams (F), and the Upper Frame Short Beam (H) to the Upper Frame Board (D).
- B.** Insert the Connecting Fastening Nuts (4) into the holes next to the pre-installed Cam Lock Bolts (2), followed by the Connecting Screw Bolts (5).
- a.** Tighten all Connecting Screw Bolts (5) using a Phillips head screwdriver.

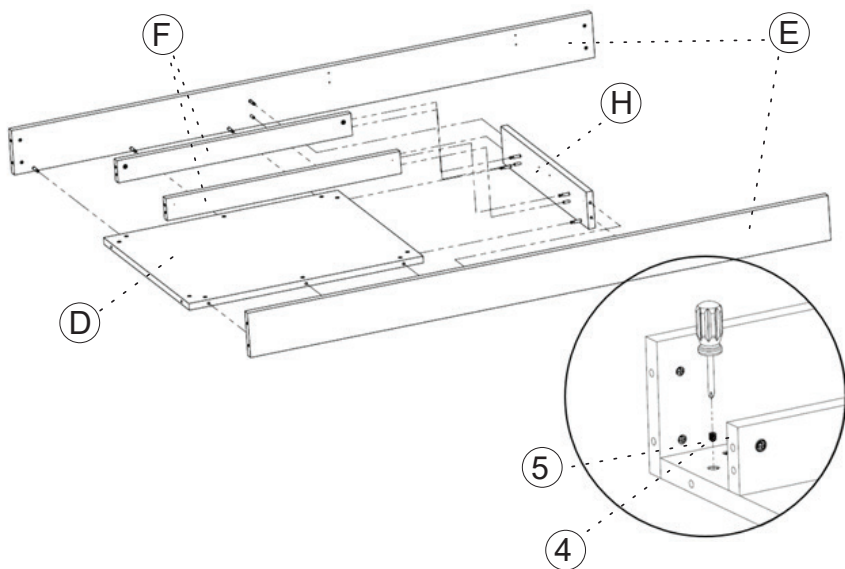

### Step 5:


**A.** Assemble the Upper Frame, the Lower Frame (assembled in Steps 3 and 4), the Left Leg Panel (I), and the Right Leg Panel (J) together. Insert the Connecting Fastening Nuts (4) into the holes adjacent to the pre-installed Cam Lock Bolts (2), then insert the Connecting Screw Bolts (5).

**a.** Tighten all Connecting Screw Bolts (5) using a Phillips head screwdriver.

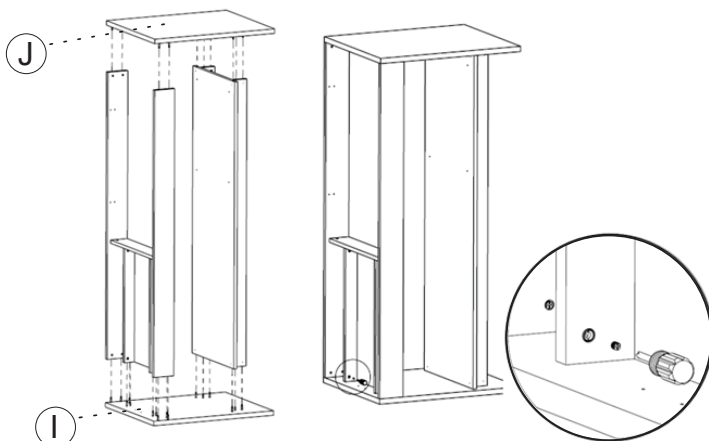


### Step 6:

**A.** Carefully turn the table over so that the tabletop faces upward and the legs face downward. Ensure the table is standing securely and evenly on the ground.

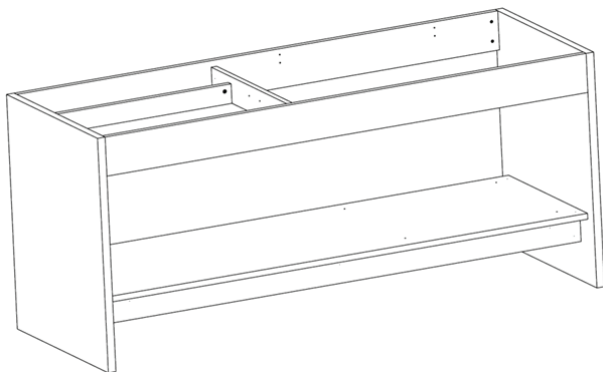


### Step 7:

- A.** Use twelve Self-Tapping Screws (6) to attach the six L-Shaped Connectors (7) to the pre-drilled holes along the inner sides of the Upper Frame.

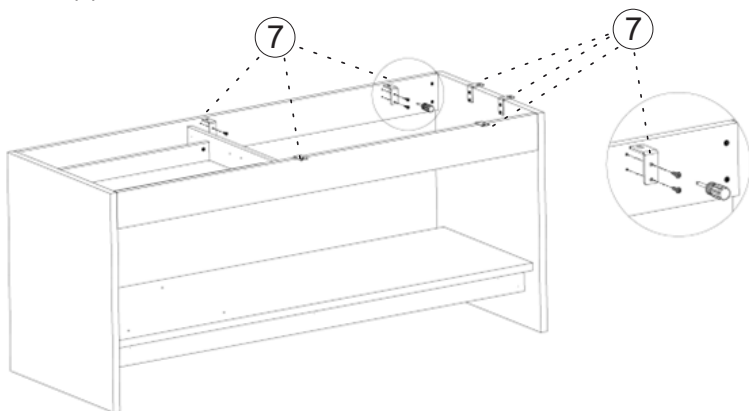


### Step 8:

- A.** Use six Bolts (8) to secure the L-Shaped Connectors (7) to the pre-drilled holes on the Tabletop Base Board (K).

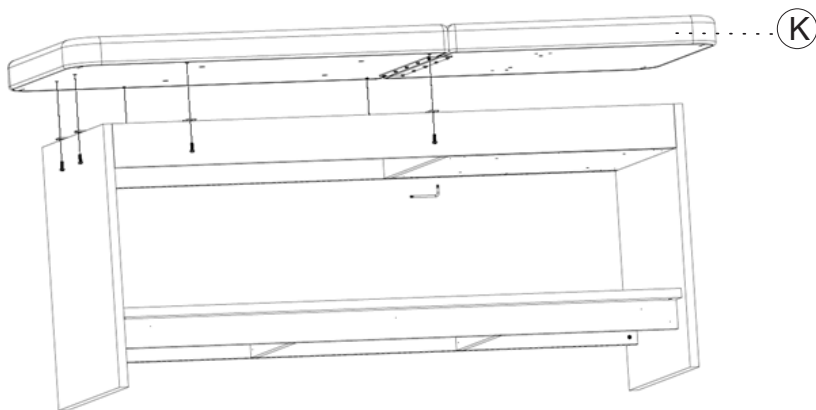


### Step 9:

- A.** Use two Self-Tapping Screws (6) to install the Backrest Support Bracket (L) onto the inner side of the Upper Frame, aligning with the pre-drilled holes as shown in the illustration below.

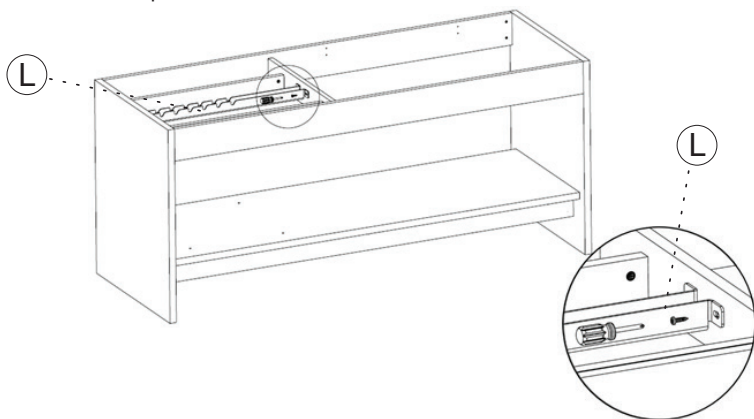


### Step 10:

- A.** Attach the Backrest Support Rod (10) to the Tabletop Base Board (K).
- B.** Then, use four Bolts (8) to secure the Backrest Support Arms (9) beneath the backrest, as shown in the illustration, using an L-Wrench.

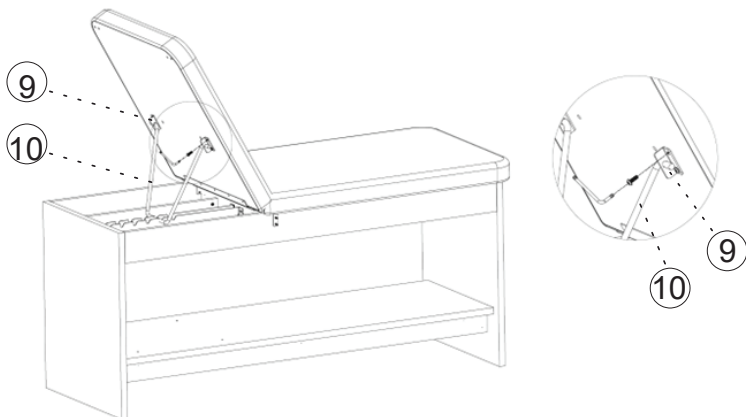

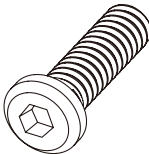
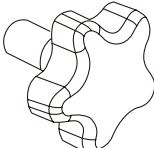


# Paper Roll Dispenser Installation

## Parts List

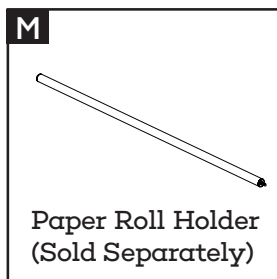


## Hardware List
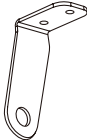
<p><b>11</b></p>  <p>Left Support Element</p>	<p><b>12</b></p>  <p>Right Support Element</p>	<p><b>13</b></p>  <p>Bolt (x4)</p>
<p><b>14</b></p>  <p>Knob (x#)</p>		



### Step 11:


- A.** Install the Paper Roll Holder (M) by securing it with four Bolts (13) underneath both sides of the table.

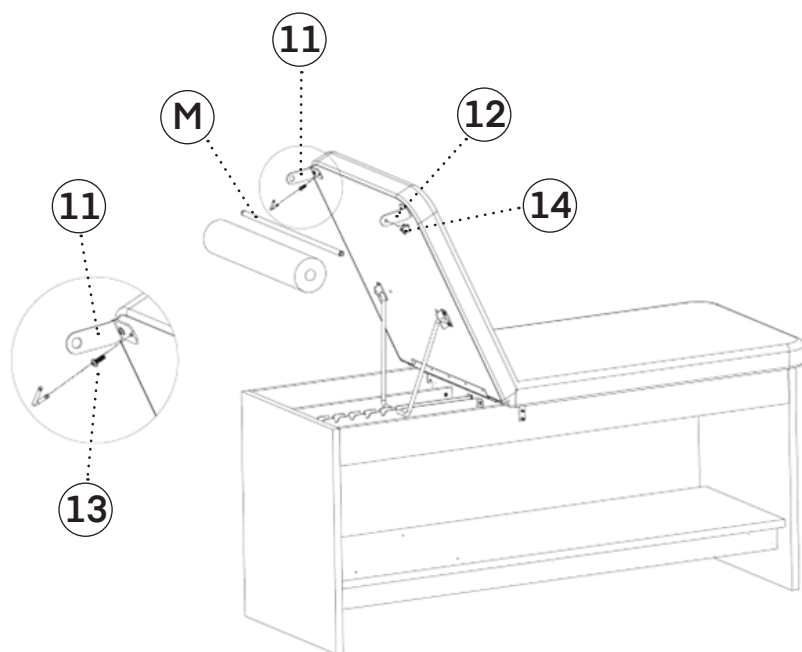

# Backrest Angle Adjustment

## Step 12:

- A.** Carefully lift the backrest upward by hand, then lower the support arm beneath it so that the lower end of the support arm fits securely into the slot on the table frame.

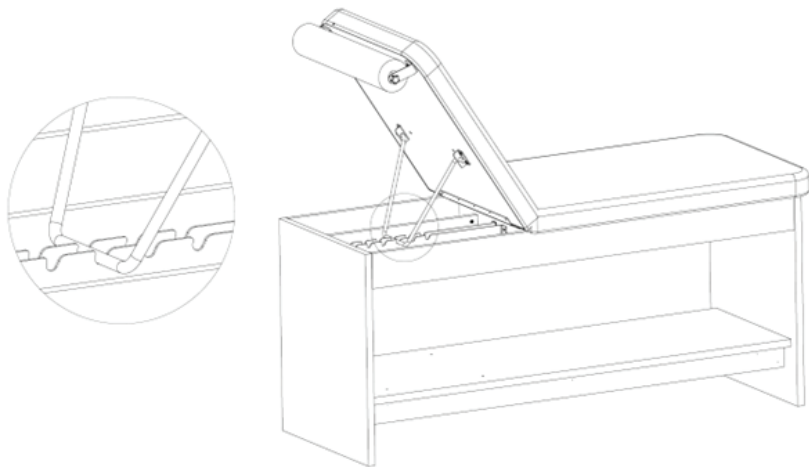

- B.** The backrest can be laid flat by reversing the operation above.

**Note:** Operating the backrest angle adjustment function is forbidden when the patient is leaning on it. For example, when the lower end of the support arm is out of the slot, if the patient's weight leans on the backrest, it can cause it to fall quickly, which may cause the operator's arm to be pinched or crushed.

# Care & Maintenance Guide

## A. Leather Maintenance:

**a.** It is recommended to use a small amount of neutral detergent/cleaner or wipe with a dry towel when wiping the leather, then wipe it clean with a dry cloth, dry it, and place it in a ventilated place to dry.

**b.** If disinfection is necessary, it is acceptable to wipe it with hypochlorous acid disinfectant (HClO) or alcohol with a concentration below 70%. Highly corrosive disinfectants are prohibited for disinfection; otherwise, they may accelerate the aging of the leather, causing the leather to change color, deform, harden, become brittle, or even peel off the surface.

**c.** When not used for a long time, it is recommended that the table be covered with a table cover and stored in a dry environment. Avoid direct sunlight, rain, humidity, and other environments.

**d.** Too high or too low-temperature environments may accelerate the aging of leather.

**e.** Sharp objects are prohibited from touching the leather surface.

## B. Metal and Plates Maintenance:

**a.** A dry towel is recommended to wipe metal and plates.

**b.** If disinfection is necessary, it is acceptable to use hypochlorous acid disinfectant or alcohol with a concentration below 70% to the maximum extent possible. Using highly corrosive disinfectants is prohibited for disinfection. Otherwise, the paint or wood veneer may be damaged. Using highly corrosive disinfectants is prohibited for disinfection. Otherwise, the paint or wood veneer may be damaged.


## C. Regular Maintenance:

**a.** Check the table frame every 3 months to test its stability when pushed horizontally/vertically/laterally. If there is any shaking, tighten the connecting bolts.


# Safety Warning


**A.** The user is ultimately responsible for preventing product damage or injury to the patient and/or surgical staff.

## **B. Collapse risk:**

 If the patient's weight exceeds the working load, the medical treatment table and the backrest may collapse if the patient's weight is not evenly distributed on the table surface.

## **C. Pinch risk:**

 When the backrest of the table surface moves down to the end, the hand or finger is placed in the gap between the backrest and the table frame.

 When the drawer is closed, the hand or finger is placed in the gap between the drawer panel and the cabinet.

## **D. Infection risk:**

When cleaning, please wear gloves, masks, and eye protection to prevent the cleaning spray from rebounding from the contaminated surface.

## **E. Table surface damage risk:**

**a.** The ambient temperature requirement for using and storing the equipment is 0°C-40°C. If the equipment is exposed to an environment exceeding the specified temperature for a long time, the leather surface may crack or peel.

**b.** The humidity requirement for the equipment's transportation, use, and storage is 20%~80%. The leather surface may crack or peel if the equipment is exposed to an overly dry or humid environment for a long time.

**c.** The leather of the equipment table is strictly prohibited from contacting acidic and alkaline substances. Scrubbing the table with a non-neutral detergent may cause the leather to crack and peel prematurely, reducing its service life.

**d.** The leather of the table is strictly prohibited from contacting liquid water. Liquid water may penetrate the leather, causing the sponge inside the leather to degenerate, affect use, or crack and peel.

## **F. Hardware accessories rusting risk**

**a.** The humidity requirement for equipment transportation, use, and storage environment is 20%~80%. If the equipment is exposed to an overly humid environment for a long time, the hardware accessories may rust.

**b.** The metal parts of the equipment are strictly prohibited from contacting acidic substances; otherwise, they may cause damage to the paint surface and then rust.

**c.** The metal parts of the equipment are strictly prohibited from contacting liquid water, which may cause metal materials to rust.

## **7. Other precautions:**

**a.** It is strictly prohibited to approach fire sources.


**b.** Do not place it in direct sunlight.

**c.** Do not place it in high-temperature places such as near heaters or fire places.

# Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 5 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

[illegible]

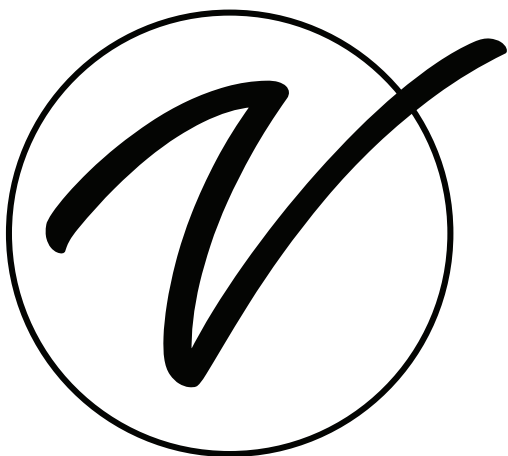

**ADI996-12-SERIES**

© 2025 Viva Comfort

**[vivacomfort.com](https://vivacomfort.com)**