


.....

SLIM LINE EXAM TABLE

WITH AN ADJUSTABLE BACKREST AND FULL BASE
.....

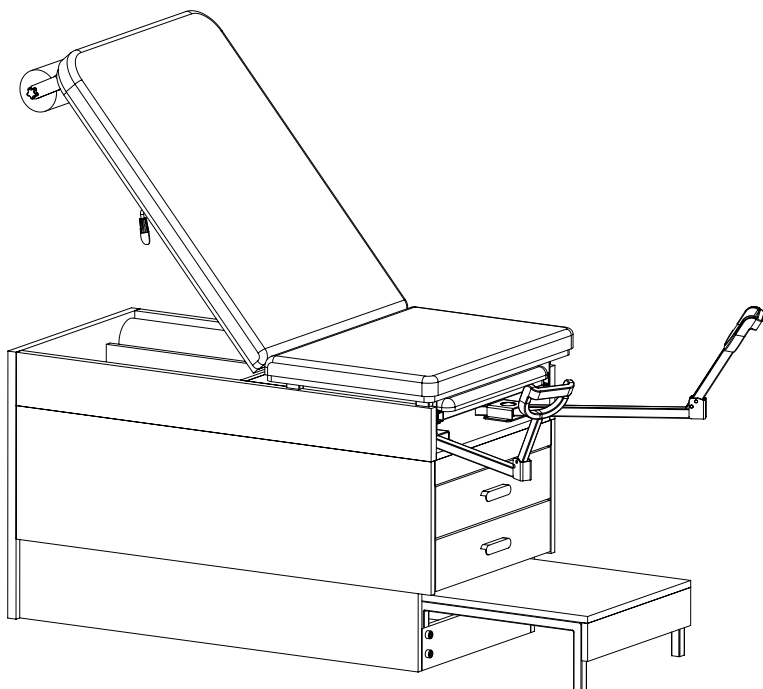

USER GUIDE

SKU: ADI996-FBASE-SERIES



Thank you for your purchase.



Viva Comfort maintains a strong practice of ongoing development of products and commercial research.




We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.

Table of Contents

- 4** Parts List
- 5** Hardware List
- 6** Tools Needed (Not Included)
- 7** Assembly Instructions
- 19** Care & Maintenance Guide
- 20** Safety Warning
- 22** Warranty
- 23** Notes


Parts List

A

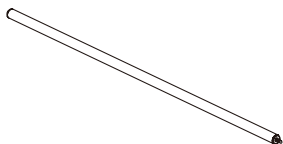

Paper Roll Holder

B

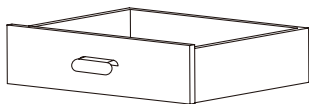

Drawer (x2)

C


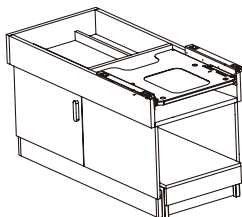



Table Base

D

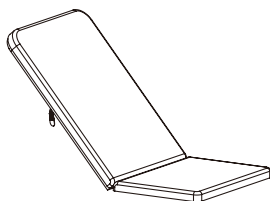


Mattress

E

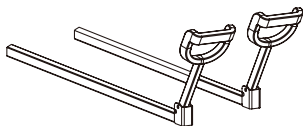


Stirrups (x2)

F

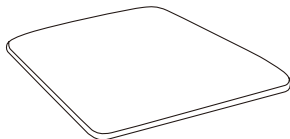



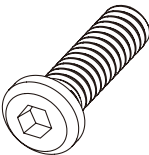
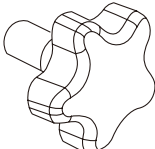
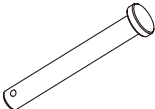




Leg Mattress

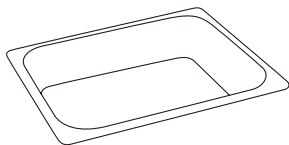G




Stainless Steel Basin

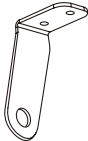
Hardware List

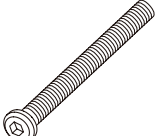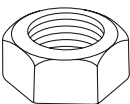1  Left Support Element	2  Right Support Element	3  Bolt (x#)
4  Knob (x#)	5  Metal Rod	6  Washer (x2)
7  Pin	8  Bolt (x#)	9  Screw Nut (x#)


Tools Needed (Not Included)

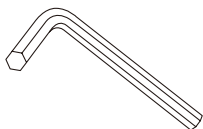

Allen Key

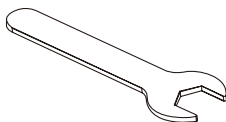

Wrench

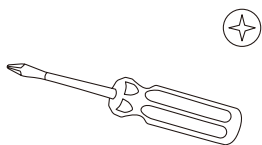

Phillips Head
Screwdriver

Assembly Instructions

Step 1:

Preparation and Inspection Before Use:

- A.** Confirm whether a product manual and warranty card accompany the product.
- B.** Confirm whether the installation hardware package is attached.
- C.** Confirm whether the components are deformed or damaged.
- D.** If any abnormalities occur during installation and use, please get in touch with the dealer in time.

Note:


The table top and the cabinet are packaged separately.

Please ensure that you have received the upholstery and cabinet.

Step 2:

Installing Paper Roll Hoder:


- A.** Use Bolts (3) to install the Left Support Element (1) and Right Support Element (2) under either side of the table.

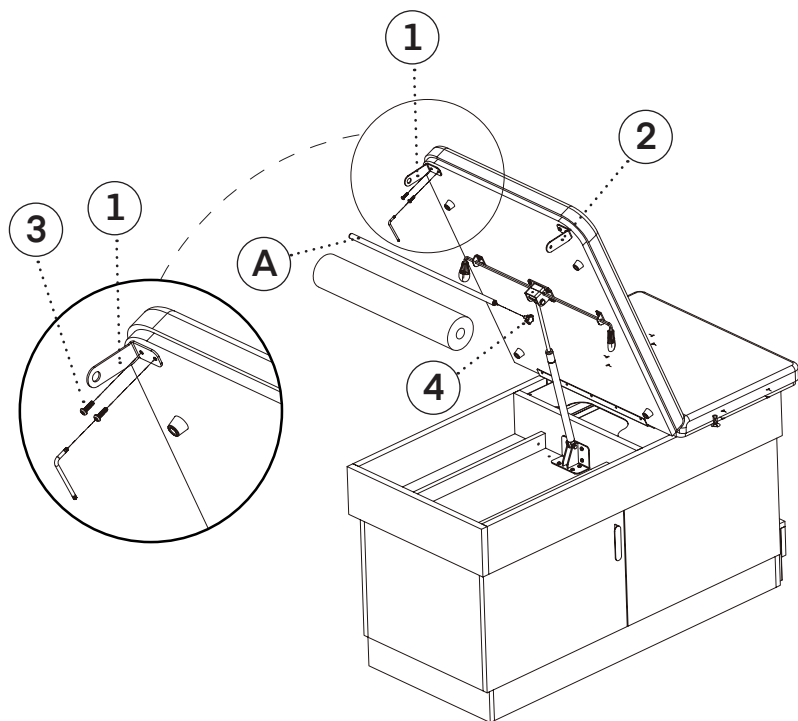

Step 3:

Installing Handles


- A.** To prevent damage during shipping, the Drawer's (B) handle is installed on the inside of the door. Before use, remove the handle, reposition it on the outside of the door, and securely fasten it using a Phillips head screwdriver.

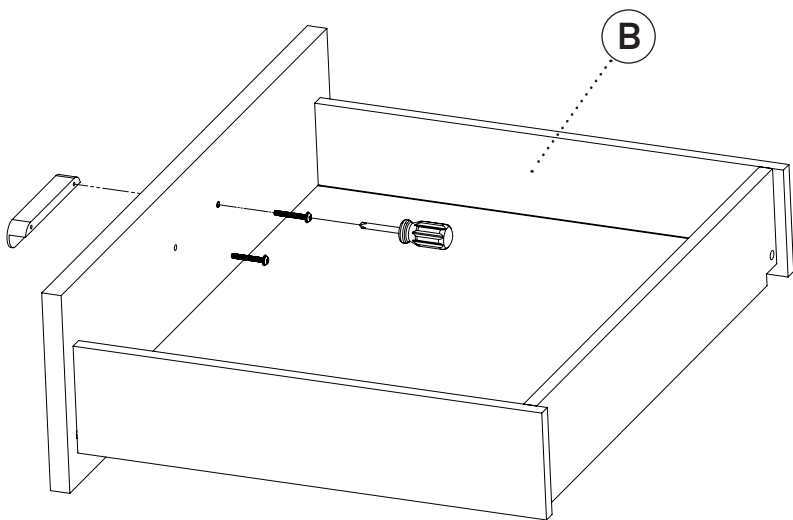

Step 4:

Installing Stirrups


- A.** Place the Stirrups (E) beneath the leg rest on the Table's Base (C), as shown.
- B.** Install and secure each stirrup using eight Bolts (8) and Screw Nuts (9). Tighten the bolts and nuts firmly with a wrench and Allen key until fully secured.

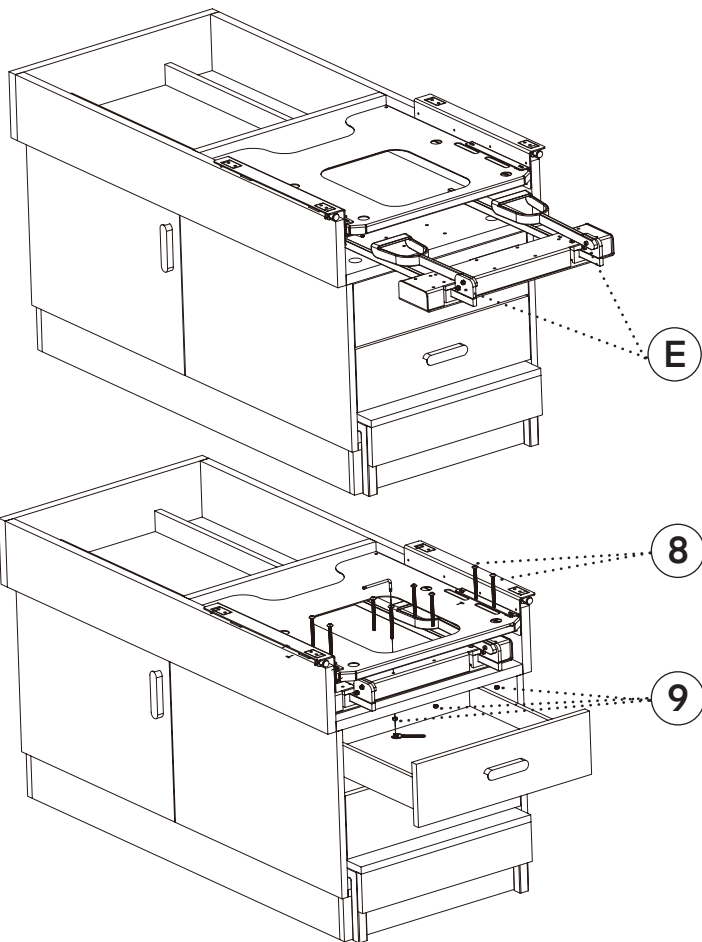

Step 5:


Installing the Table Top

- A.** Locate the Mattress (D) and Table Base (C).
- B.** With the tabletop in a V-shape, insert the metal hooks on the lower left and right sides of the Mattress (D) into the slots on the metal plates located on the upper left and right sides of the Table Base (C), as shown.

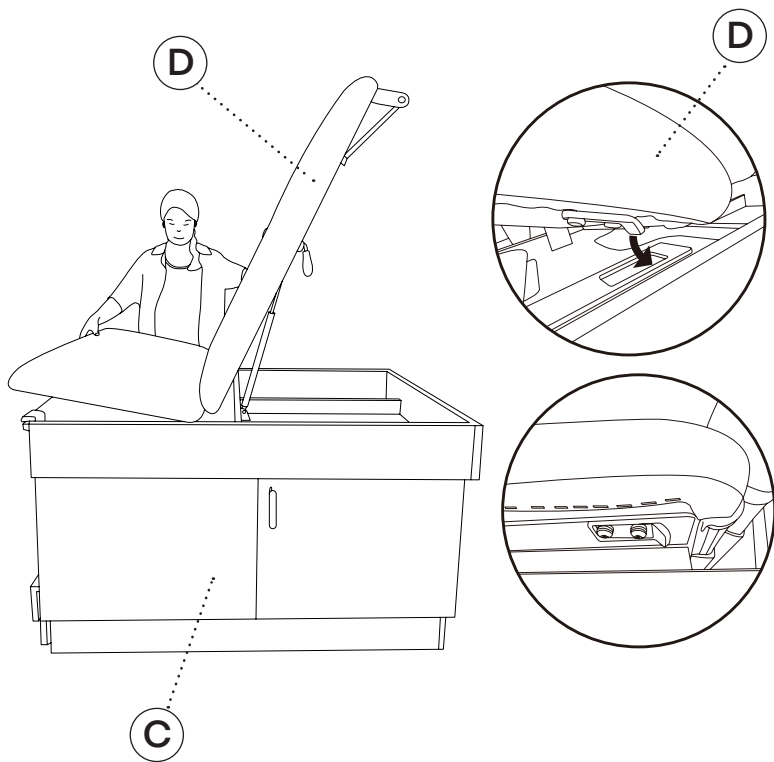



Step 6:

- A.** Pull the left and right pins located on the upper front of the Table's Base (C) outward.
- B.** Once fully extended, rotate each pin to either side to lock it in the open position.

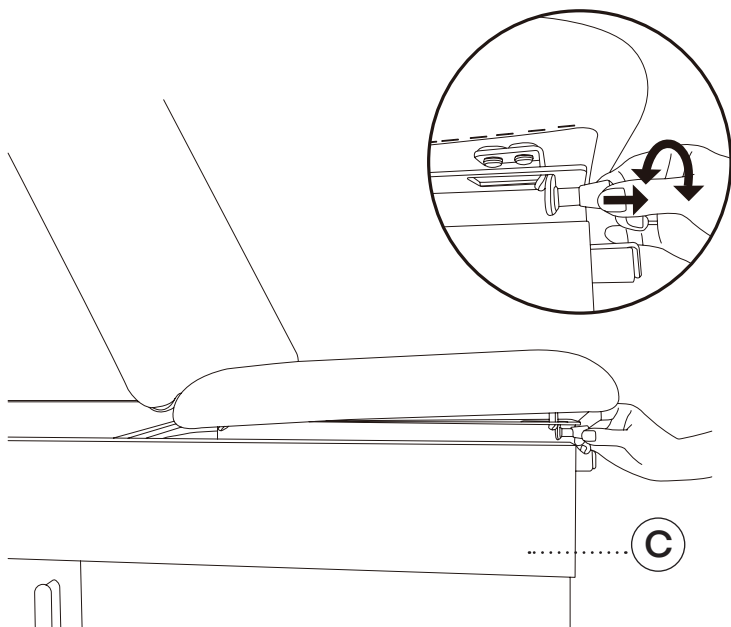

Step 7:


- A.** Firmly press down on the front of the seat cushion until it lowers. This will allow the metal hook beneath the front edge of the cushion to drop into the correct position, as shown.

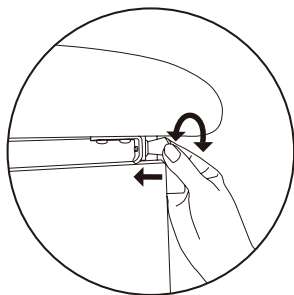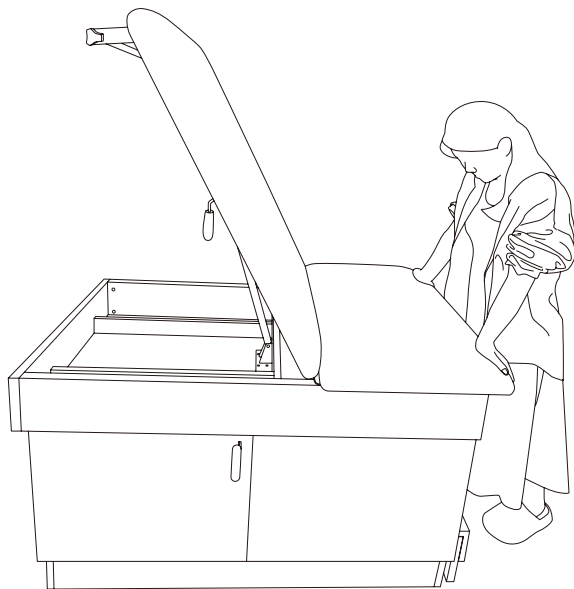

Step 8:

Installing Gas Springs



- A.** Adjust the backrest to the desired angle and hold it in place.
- B.** While maintaining this position, insert the end of the gas spring into the U-shaped mounting base on the table frame. Then, insert the Metal Rod (5) as shown, slide two Washers (6) onto the rod, and secure it using the included Pin (7).

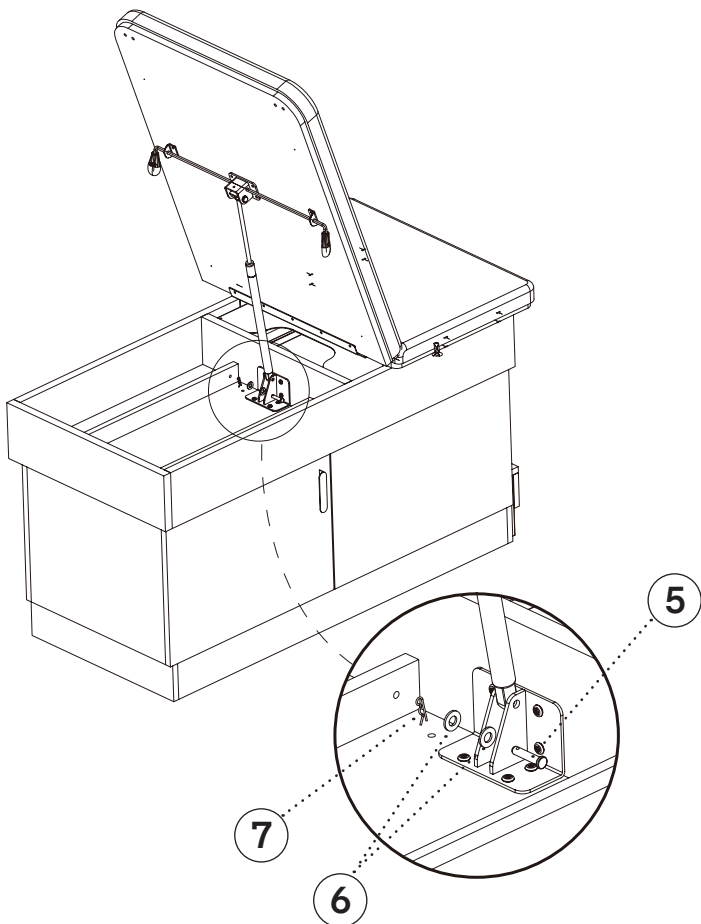

Step 9:

Leg Rest Adjustment Function


- A.** Pull the leg board outward to reveal the pre-installed, removable Stainless Steel Basin (G) and Leg Mattress (F), as shown. To retract the leg board, simply push it inward.
- B.** Lift the Leg Mattress (F) on the leg rest upward to access the Stainless Steel Basin underneath, as illustrated.

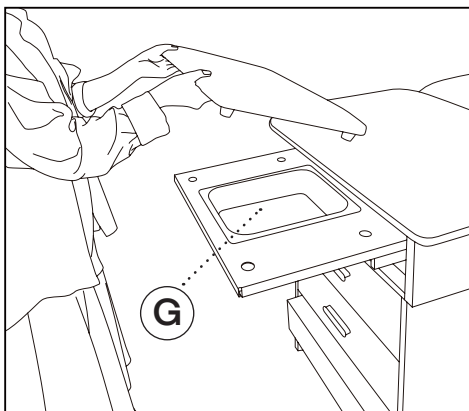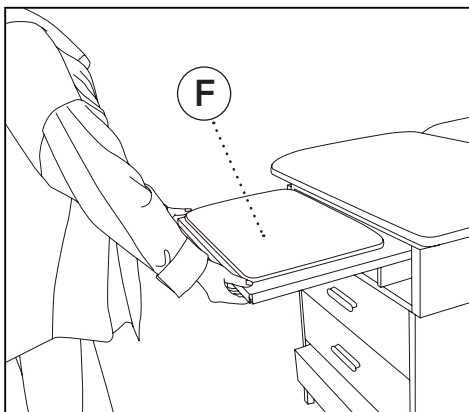

Step 10:

Pneumatic Backrest Adjustment

- A.** Use one hand to pull the gas spring brake wrench, and with the other hand, gently lift the backrest to the desired angle. Once positioned, release the gas spring brake wrench to lock it in place.

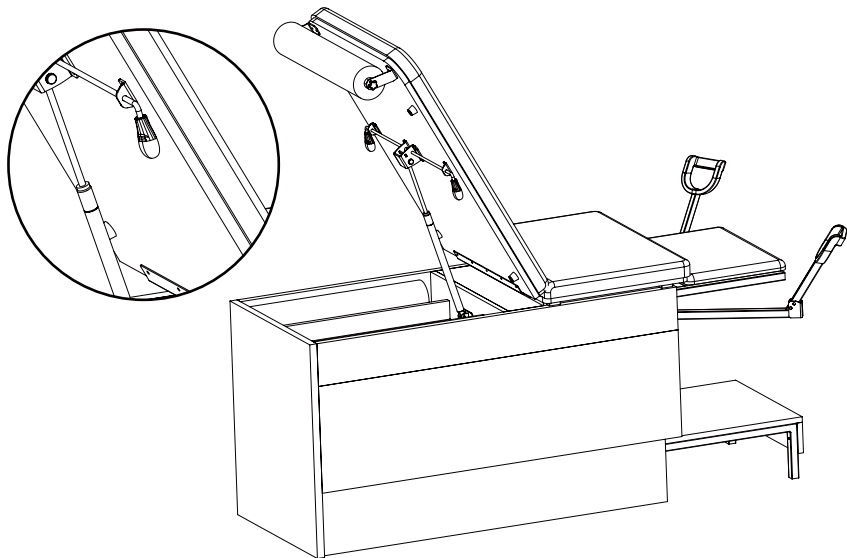


Note: Do not adjust the backrest angle while the patient is leaning on it. The patient's weight may cause the backrest to drop suddenly, risking injury to both the operator's arm and the patient.

Step 11:

Use of Stirrup Leg Support


- A.** Pull the Stirrup (E) leg support out from the side of the Table's Base (C), then swing it outward, fully, as pictured.
- B.** To retract, reverse the steps by swinging the Stirrup (E) back in and pushing it into the Table's Base (C).

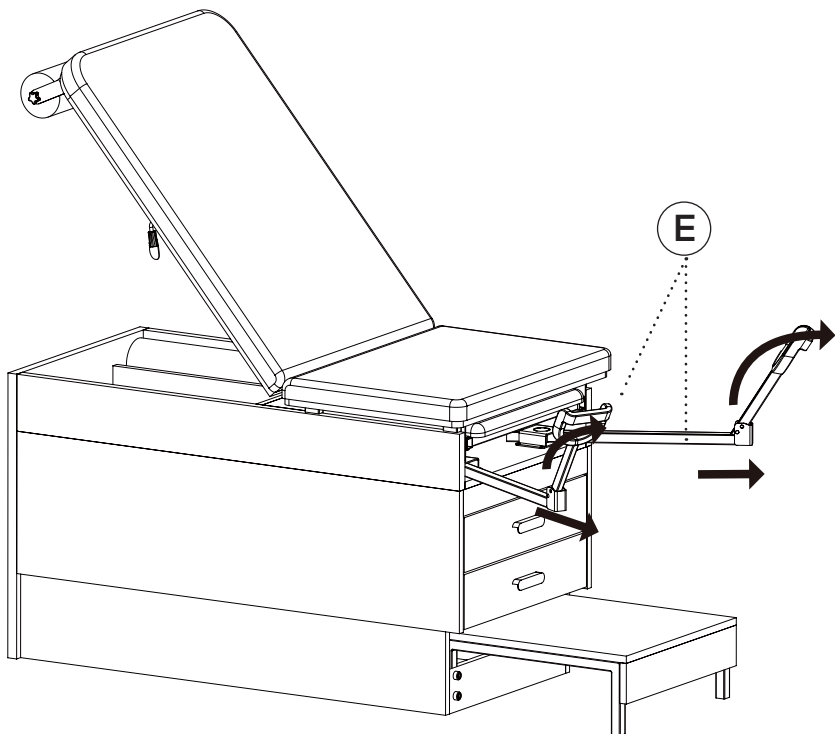

Step 12:

A. Adjust the angle of the Stirrups (E):

- a. The Stirrups (E) feature a four-position angle adjustment. To change the angle, lift the Stirrups (E) upward, rotate them left or right to the desired position, then press down to lock the

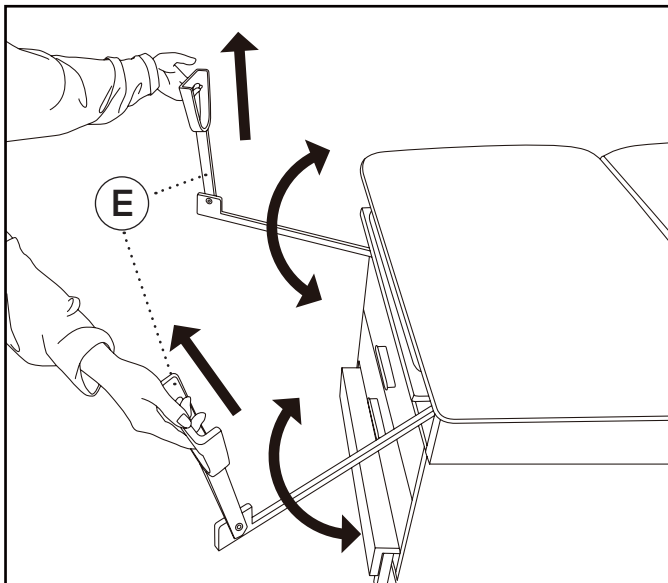

Note: When retracting the Stirrups (E) back into the table frame, be careful to avoid pinching your fingers in the narrow gap.

Care & Maintenance Guide

A. Leather Maintenance:

a. It is recommended to use a small amount of neutral detergent/cleaner or wipe with a dry towel when wiping the leather, then wipe it clean with a dry cloth, dry it, and place it in a ventilated place to dry.

b. If disinfection is necessary, it is acceptable to wipe it with hypochlorous acid disinfectant or alcohol below 70% concentration. 84 disinfectant or other highly corrosive disinfectants are prohibited for disinfection; otherwise, they may accelerate the aging of the leather, causing the leather to change color, deform, harden, become brittle, or even peel off the surface.

c. When not used for a long time, it is recommended that the table be covered with a table cover and stored in a dry environment. Avoid direct sunlight, rain, humidity, and other environments.

d. Too high or too low-temperature environments may accelerate the aging of leather.

e. Sharp objects are prohibited from touching the leather surface.

B. Metal and Plates Maintenance:

a. A dry towel is recommended to wipe metal and plates.

b. If disinfection is necessary, it is acceptable to use hypochlorous acid disinfectant or alcohol with a concentration below 70% to the maximum extent possible. Using BA-SI disinfectant or other highly corrosive disinfectants is prohibited for disinfection. Otherwise, the paint or wood veneer may be damaged.


C. Regular Maintenance:

a. Check the table frame every 3 months to test its stability when pushed horizontally/vertically/laterally. If there is any shaking, tighten the connecting bolts.


Safety Warning


A. The user is ultimately responsible for preventing product damage or injury to the patient and/or surgical staff.

B. Collapse risk:

 If the patient's weight exceeds the working load, the medical treatment table and the backrest may collapse if the patient's weight is not evenly distributed on the table surface.

C. Pinch risk:

 When the back-rest of the table surface moves down to the end, the hand or finger is placed in the gap between the backrest and the table frame.

 When the drawer is closed, the hand or finger is placed in the gap between the drawer panel and the cabinet.

D. Infection risk:

When cleaning, please wear gloves, masks, and eye protection to prevent the cleaning spray from rebounding from the contaminated surface.

E. Table surface damage risk:

a. The ambient temperature requirement for using and storing the equipment is 0-40°C. If the equipment is exposed to an environment exceeding the specified temperature for a long time, the leather surface may crack or peel.

b. The humidity requirement for the equipment's transportation, use, and storage is 20%~80%. The leather surface may crack or peel if the equipment is exposed to an overly dry or humid environment for a long time.

c. The leather of the equipment table is strictly prohibited from contacting acidic and alkaline substances. Scrubbing the table with a non-neutral detergent may cause the leather to crack and peel prematurely, reducing its service life.

d. The leather of the table is strictly prohibited from contacting liquid water. Liquid water may penetrate the leather, causing the sponge inside the leather to degenerate, affect use, or crack and peel.

F. Hardware accessories rusting risk:

a. The humidity requirement for equipment transportation, use, and storage environment is 20%~80%. If the equipment is exposed to an overly humid environment for a long time, the hardware accessories may rust.

b. The metal parts of the equipment are strictly prohibited from contacting acidic substances; otherwise, they may cause damage to the paint surface and then rust.

c. The metal parts of the equipment are strictly prohibited from contacting liquid water, which may cause metal materials to rust.

7. Other precautions:

a. It is strictly prohibited to approach fire sources.


b. Do not place it in direct sunlight.

c. Do not place it in high-temperature places such as heaters.

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

[illegible]

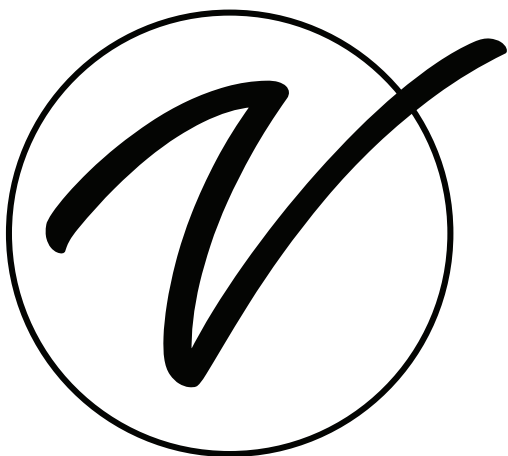

ADI996-FBASE-SERIES

© 2025 Viva Comfort

vivacomfort.com