




FLAT MATTRESS WOOD TREATMENT TABLE

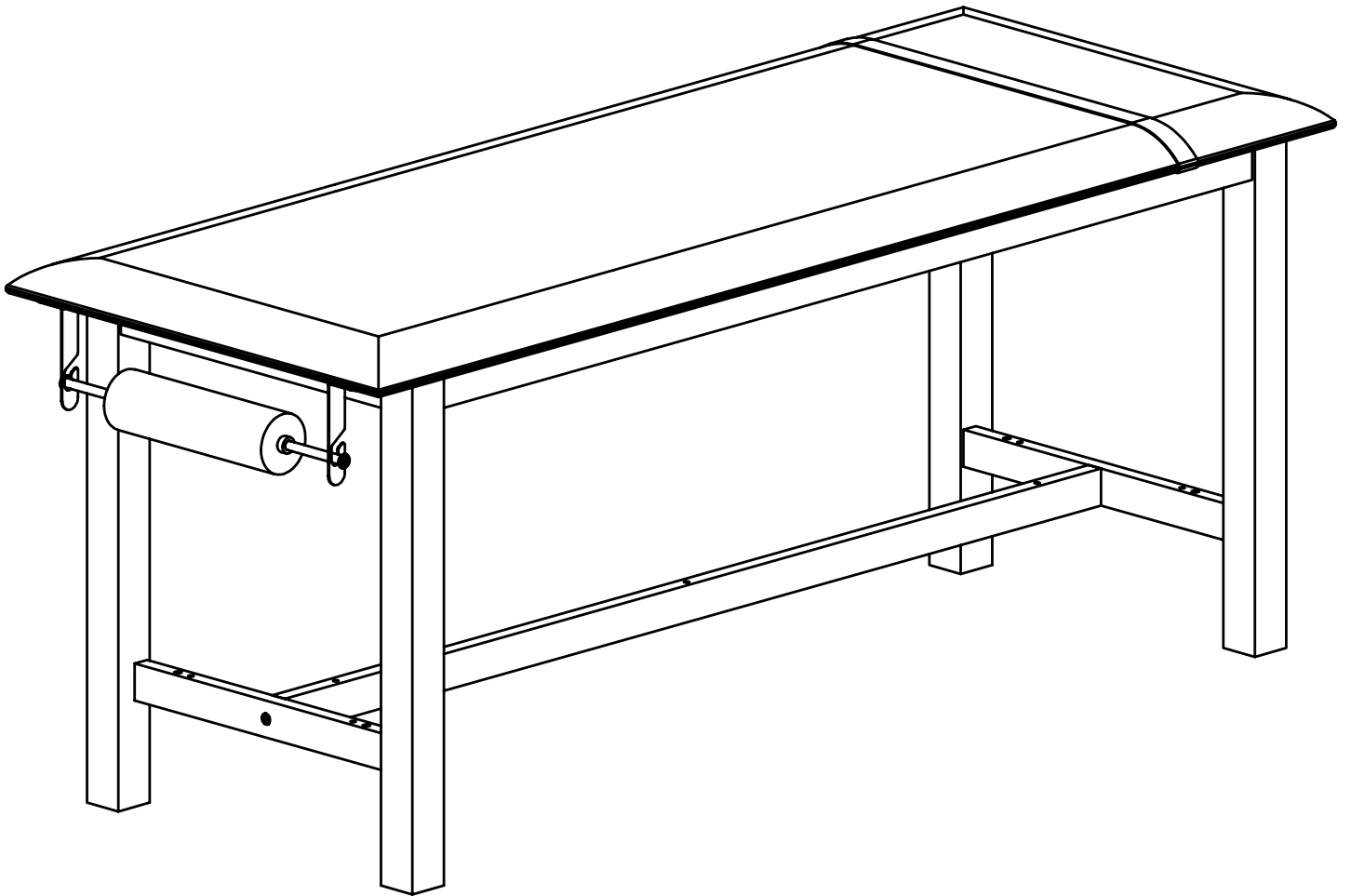

USER GUIDE

ADI996-HB-FM-SERIES



Thank you for your purchase.



Viva Comfort maintains a strong practice of ongoing development of products and commercial research.




We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.

Table of Contents

- 4** Treatment Table Parts List
- 5** Treatment Table Hardware List
- 6** Treatment Table Assembly Instructions
- 10** Shelf Parts List
- 11** Shelf Hardware List
- 12** Shelf Assembly Instructions
- 15** Drawer Parts List
- 16** Drawer Hardware List
- 17** Drawer Assembly Instructions
- 21** Paper Roll Dispenser Parts List
- 22** Paper Roll Dispenser Hardware List
- 23** Paper Roll Dispenser Assembly Instructions
- 26** Care And Maintenance Guide
- 30** Warranty
- 31** Notes


Treatment Table Parts List

A

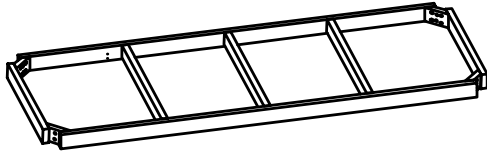


Frame

C

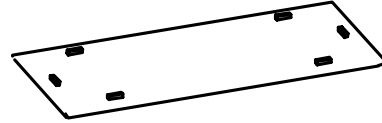

Mattress

D

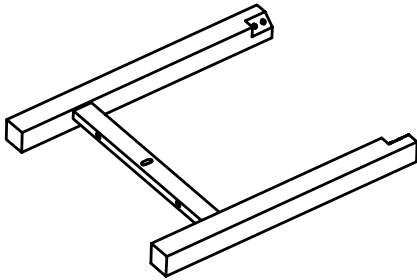




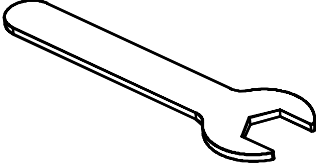
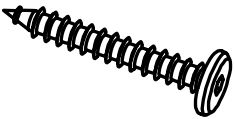

Leg (x2)

E





Support Brace

Hardware List
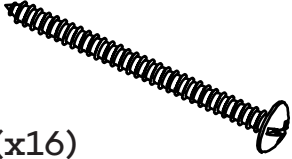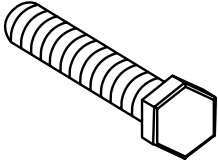
<p>1</p>  <p>Screw 1 (x16) 4 x 50mm</p>	<p>2</p>  <p>Bolt 1 (x8) 8 x 60mm</p>
<p>3</p>  <p>Flat Washer</p>	<p>4</p>  <p>Wrench</p>
<p>5</p>  <p>Screw 2 (x2) 7 x 50mm</p>	<p>6</p>  <p>Allen Key</p>

Tools Needed (But Not Included)

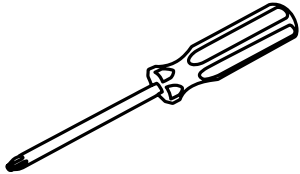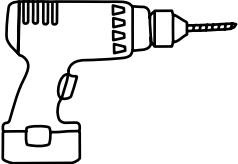 <p>Philips Head Screwdriver</p>	 <p>Electric Screwdriver</p>
---	--


Treatment Table Assembly Instructions

Please Note: We recommend utilizing at least two people for the assembly process.

Before assembling your table, ensure your working space is clean and debris-free. The assembly process has the upholstery side of the mattress facing down. Should the area not be clean and free of debris, you will risk ruining the upholstery and potentially causing puncture holes. A clean, white tarp or sheet should be laid on the floor before assembly.

Step 1:

- A. Locate the Mattress (C) and turn it so the upholstery faces down on the floor.
- B. Line the Frame (A) with the preinstalled wooden pieces on the Mattress (C).
- C. Locate the Screw 1's (1) and insert all sixteen screws into the predrilled holes of the Frame (A).
 - a. Secure them in place with a Philips head screwdriver or an electric screwdriver.

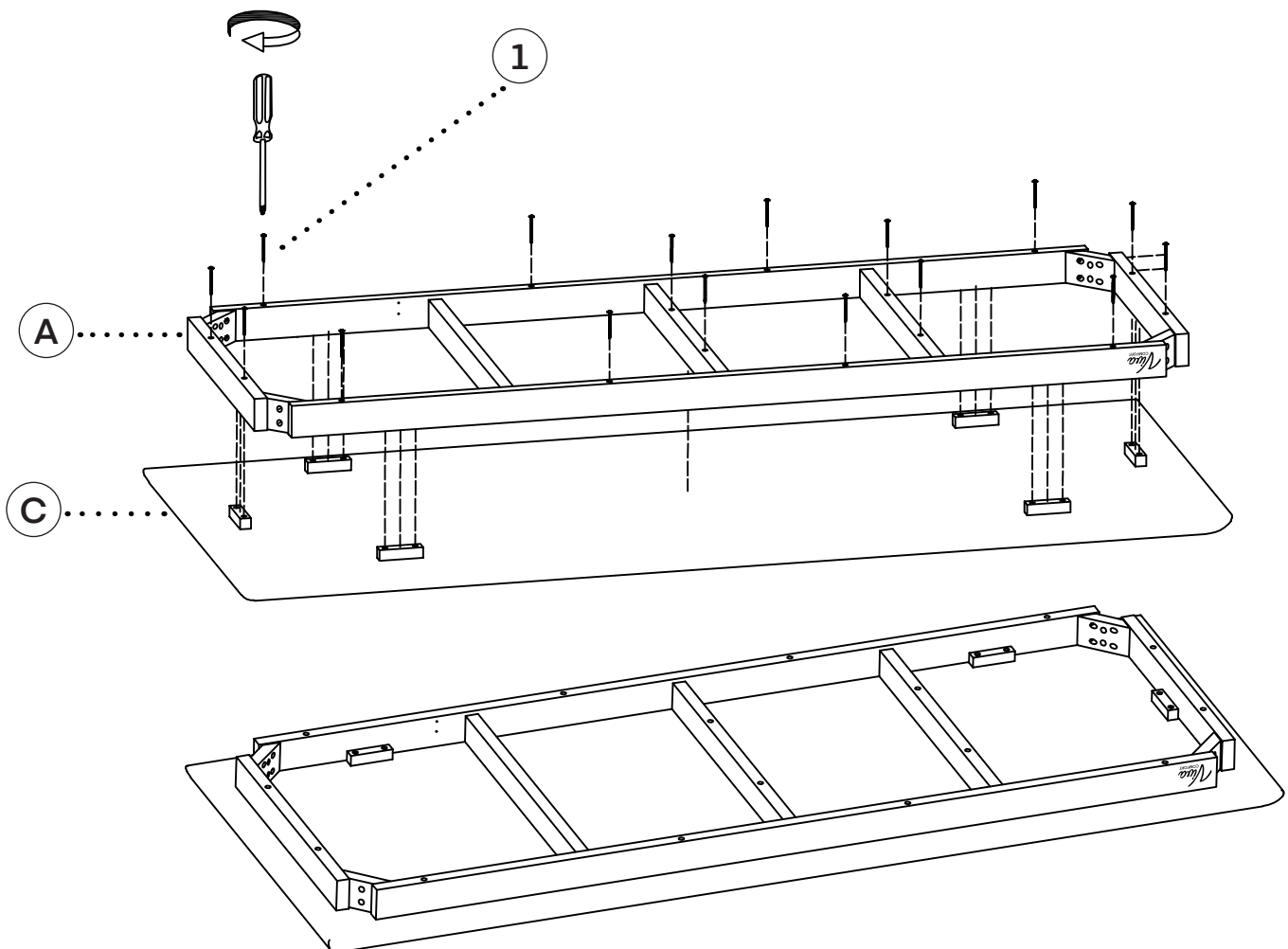

Step 2:

A. Take four of the Bolt 1's (2) and four of the Flat Washers (3).

a. Thread the Flat Washers (3) onto the Bolt 1's (2).


B. Locate one of the Leg's (D) and line it up with one of the ends of the table.

C. Insert two Bolt 1's (2) through the Frame (A) into one side of the Leg (D).

a. Secure in place with the included Wrench (4).


b. Repeat this for the other side of the Leg (D).

D. Repeat the above steps for the second Leg (D).

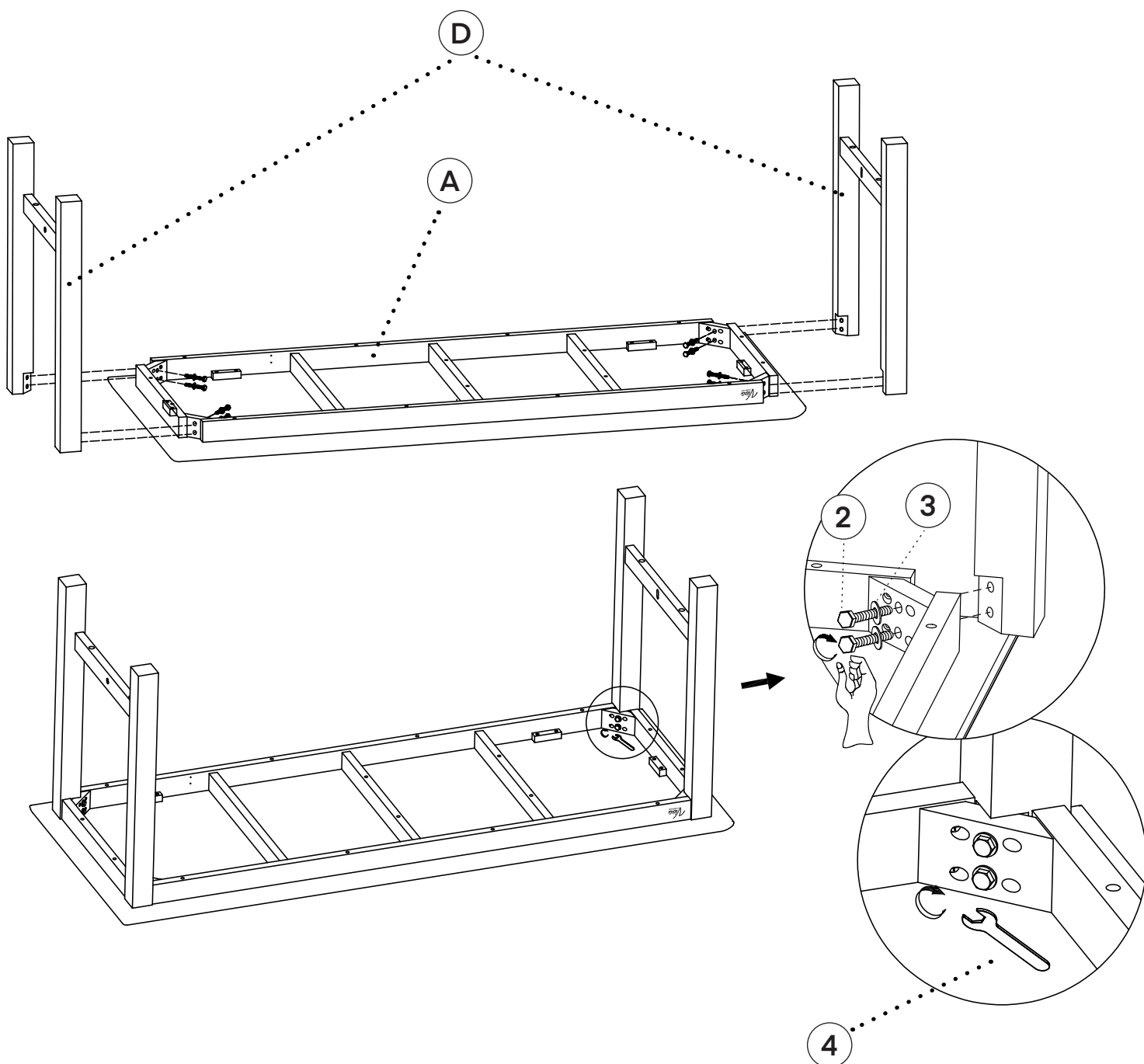

Step 3:

- A. Locate and take the Support Brace (E) and both Screw 2's (5).
 - a. We recommend utilizing two people for this step.
- B. Position the Support Brace (E) between the Legs (D).
- C. Insert both Screw 2's (5) through each side of the Legs (D) into the Support Brace (E).
- D. Secure the screws in place utilizing the included Allen Key (6).

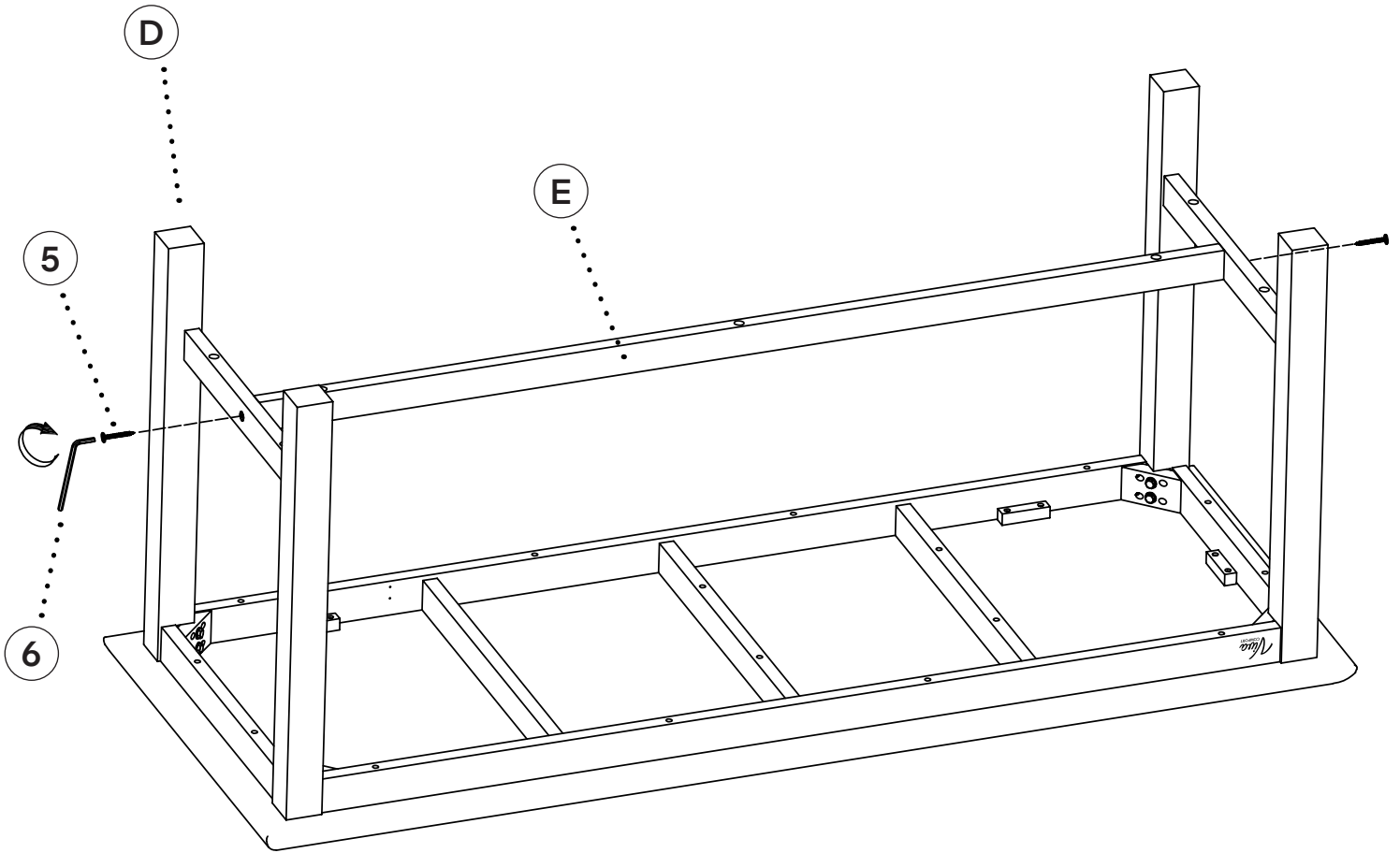

Step 4:

- A. Check all the hardware installed to ensure everything is secured in place.
- B. Using two people, turn the exam table over so it's resting on its Legs (D).
- C. Your Flat Mattress Wood Treatment Table is now ready to be used.



This section of the user guide is for if you purchased the Shelf option for your Flat Mattress Wood Treatment Table.




Follow the outlined assembly instructions. These steps should be completed after you fully assemble your treatment table.

*** If you also purchased the Drawer option, you must assemble your Shelf according to the position you would like your Drawers to be facing.**


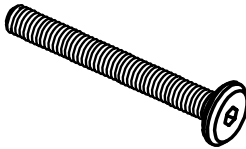
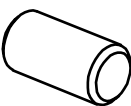
The predrilled holes on the surface of the Shelf should be oriented to the side of the table where you will be accessing the Drawers.

Drawer installation can be located on page 12.

Shelf Parts List

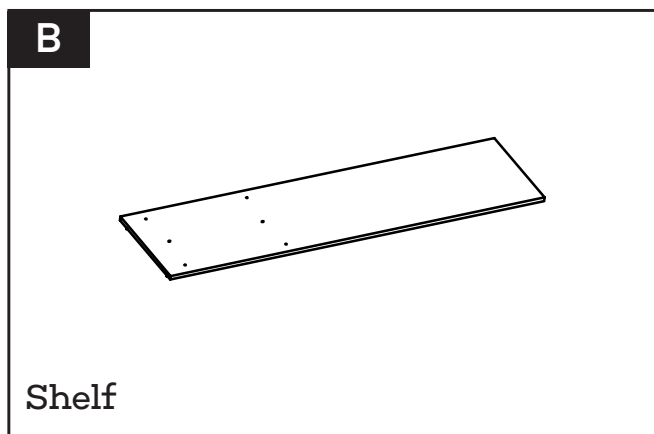


Shelf Hardware List

<p>6</p>  <p>Allen Key</p>	<p>7</p>  <p>Bolt 2 (x7) For Shelf</p>
<p>11</p>  <p>Wooden Dowel (x4)</p>	

Shelf Assembly Instructions

Step 1:


- A.** To install the Shelf (B), first ensure the table rests on its Legs (D).
- B.** Insert four Wooden Dowels (11) into the bottom of the Shelf (B).
- C.** Place the Shelf (B) on top of the Support Brace (E), with the predrilled holes facing down, and push down until secure.

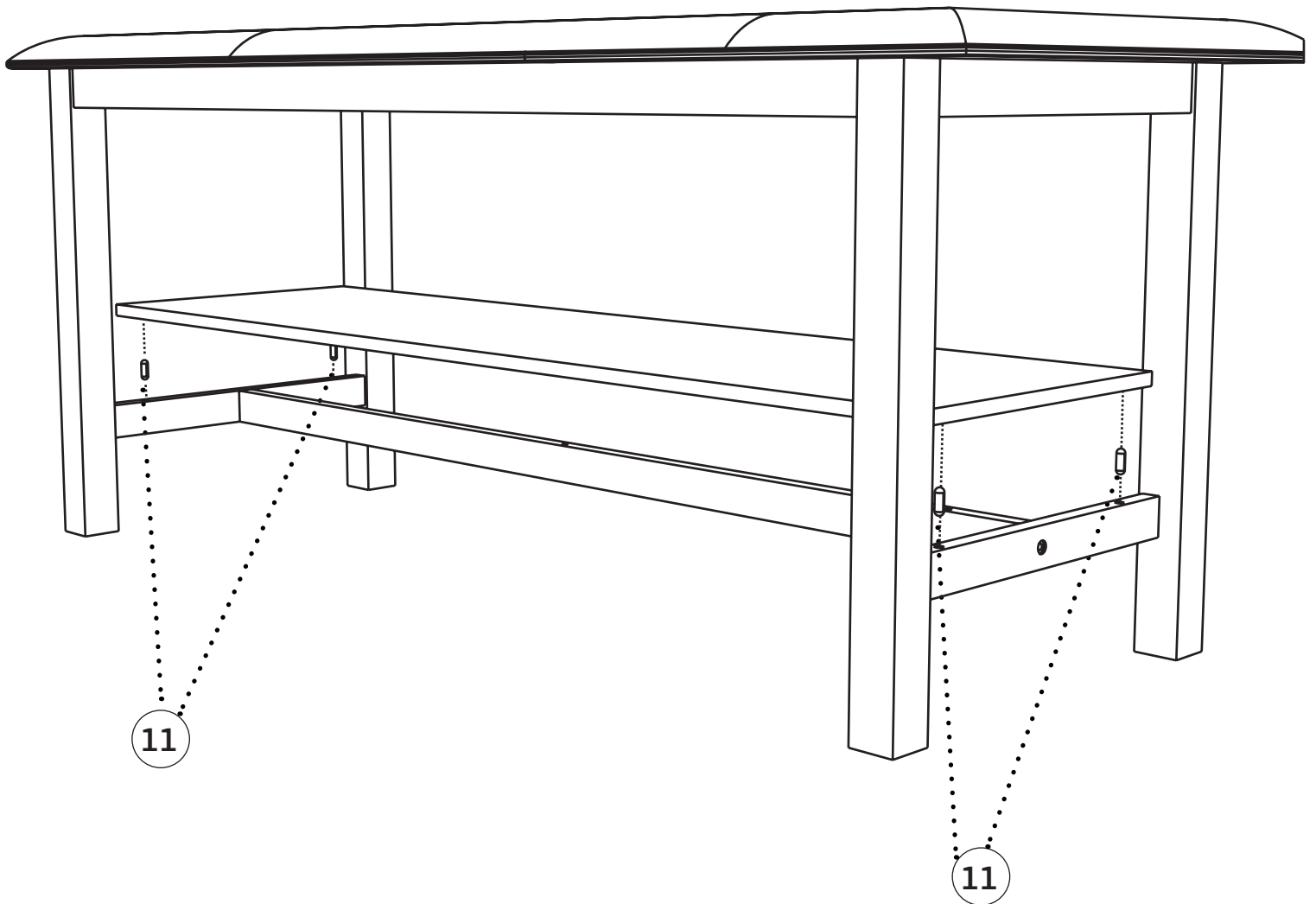

Shelf Assembly Instructions

Step 2:

- A. Insert seven Bolt 2's (7) through the Support Brace (E) and Legs (D) into the Shelf (B).
- B. Secure them in place using the included Allen Key (6).
- C. Your Wooden Treatment Table with Shelf is ready to be utilized.

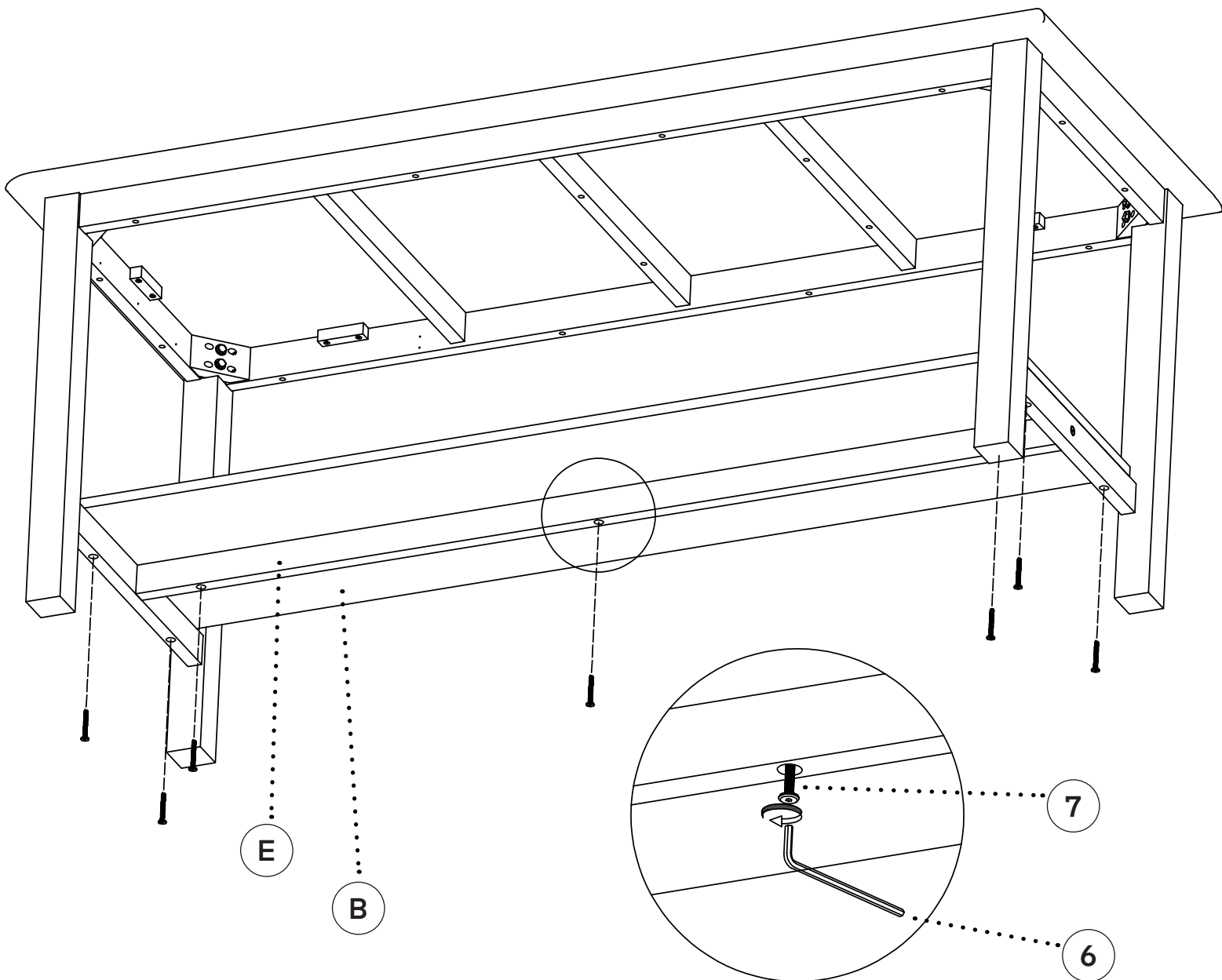




This section of the user guide is for if you purchased the Drawer option for your Flat Mattress Wood Treatment Table.






Follow the outlined assembly instructions. These steps should be completed after you fully assemble your treatment table.

*** If you purchased the Drawer option, you must assemble your Shelf according to the position you would like your Drawers to be facing. The predrilled holes on the Shelf's surface should be oriented to the side of the table where you will access the Drawers.**

Drawers Parts List

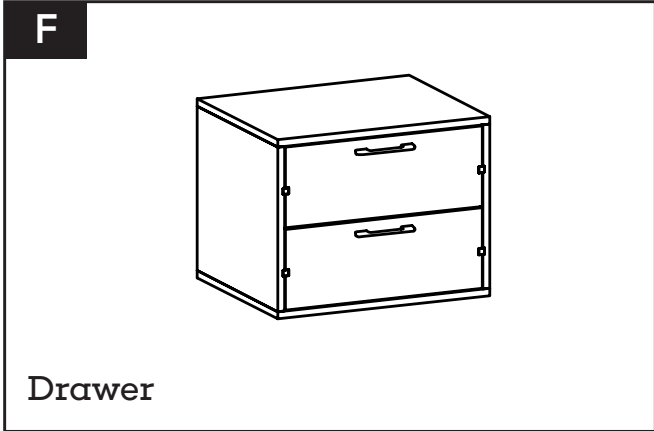


Drawer Hardware List
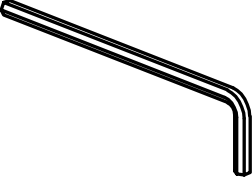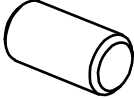
<p>6</p>  <p>Allen Key</p>	<p>8</p>  <p>Wooden Dowel (x2) For Drawers</p>
<p>9</p>  <p>Bolt 3 (x4) For Drawers</p>	

Drawers Assembly Instructions


Step 1:

- A.** To install the Drawers (F) onto the Shelf (B), first locate and insert the two Wooden Dowels (8) into the middle predrilled holes on the Shelf (B), as pictured.
- B.** Place the Drawers (F) on top of the Wooden Dowels (8) as pictured, and firmly press down until the base of the Drawers (F) are flush with the Shelf (B).

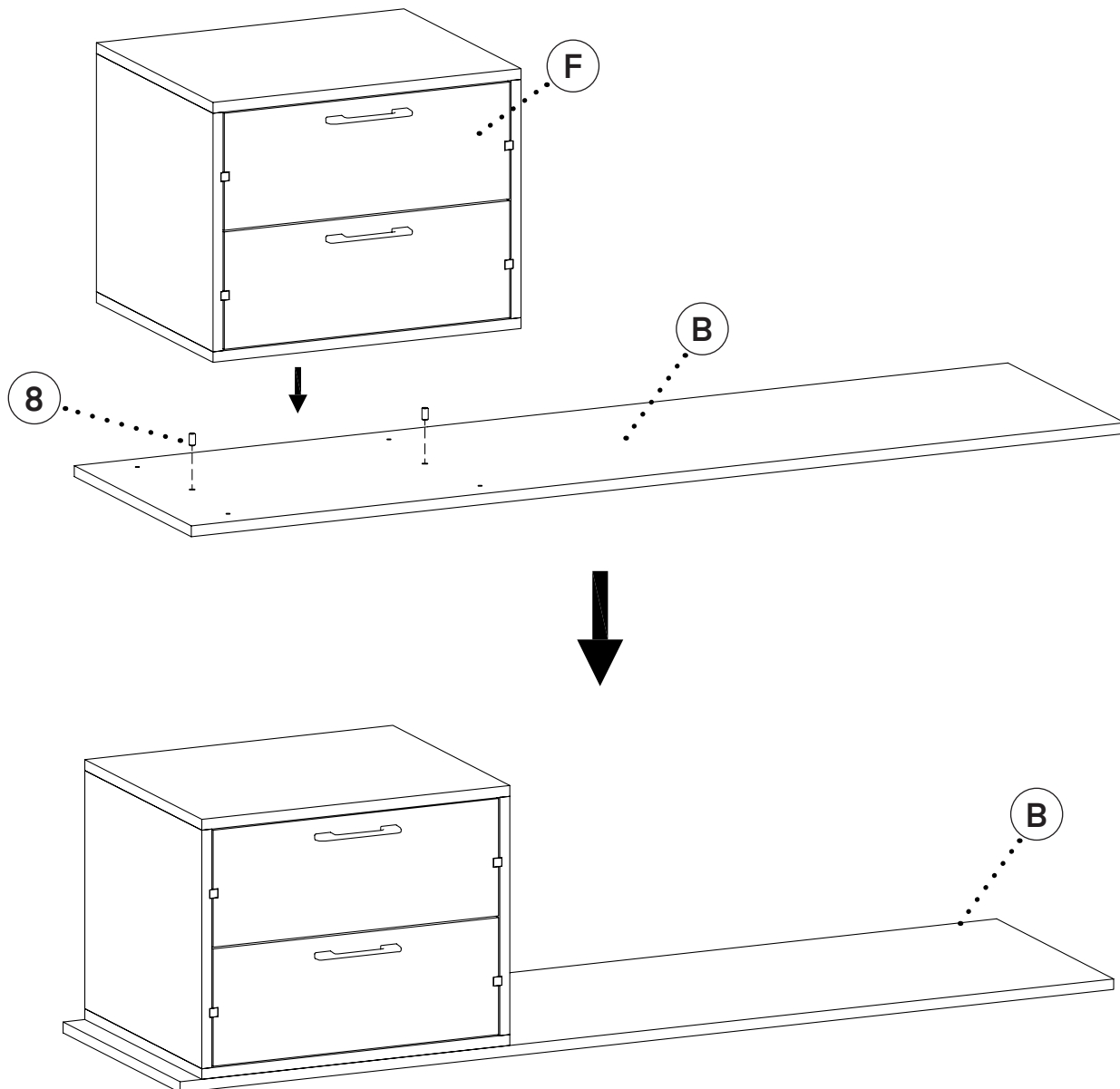

Step 2:


A. From under the Shelf (B), insert four Bolt 3s (9) into the Drawers (F), as pictured below.

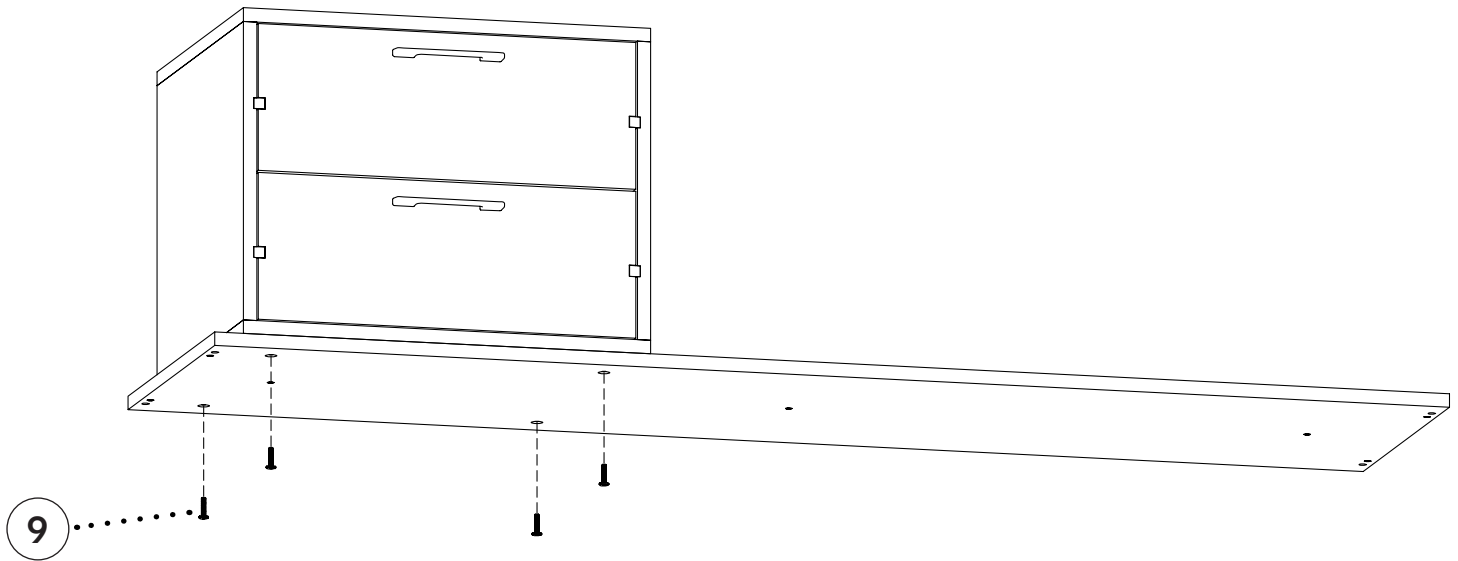

Step 3:

A. Secure them in place using the included Allen Key (6).

B. Your Wooden Treatment Table with Shelf and Drawers is now ready to be used.

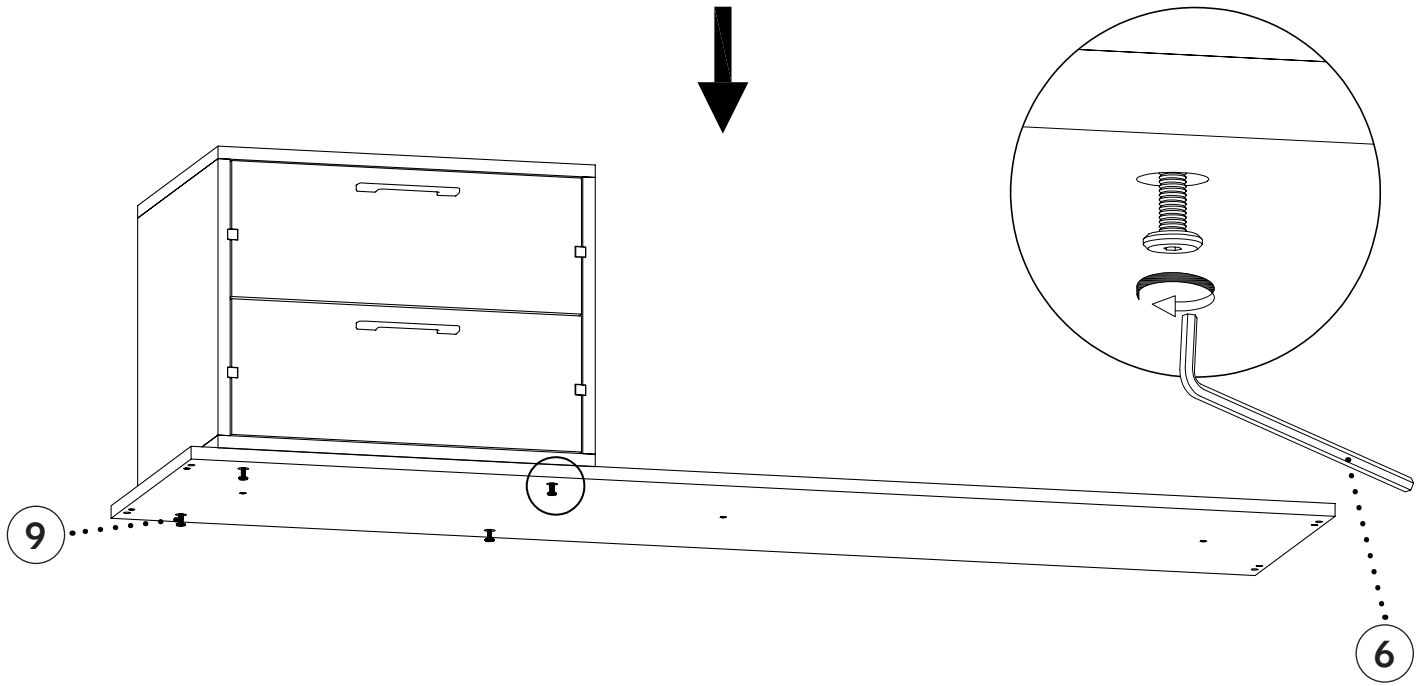




This section of the user guide is for if you purchased the Paper Roll Dispenser option for your Flat Mattress Wood Treatment Table.



Follow the outlined assembly instructions for the Paper Roll Dispenser. These steps should be completed after you fully assemble your treatment table.

Paper Roll Dispenser Parts List


G

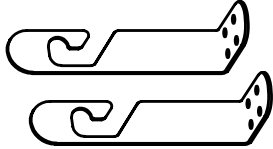

Support Element (x2)

The image shows two identical L-shaped support elements. Each element has a curved end on the left and a straight end on the right with three small circular holes. The top element is shown from a perspective view, while the bottom one is a flat 2D line drawing.


H

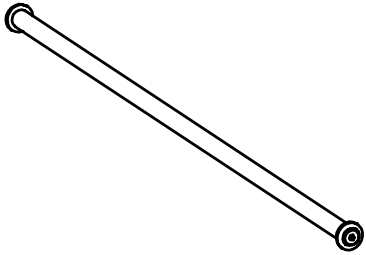

Paper Roll Holder

The image shows a long, thin cylindrical holder. It has a slightly larger diameter at one end, which is capped with a circular end cap. The other end is open.

I



Paper Cutter

The image shows a simple, flat, rectangular paper cutter. It has a slightly curved top edge and a small notch or cutout on the right side.

Paper Roll Dispenser Hardware List

10




M4x15 Screw (x 8)
For Paper Roll Dispenser

Paper Roll Dispenser Assembly Instructions

Step 1:


- A.** To affix the Paper Roll Dispenser, you can do this before or after installing the Mattress (C).
- B.** Locate the left Support Element (G) and affix it to the Mattress (C) utilizing four M4x15 Screws (10) and secure them in place with a Philips head screwdriver.
- C.** Repeat the above steps for the right Support Element (G).

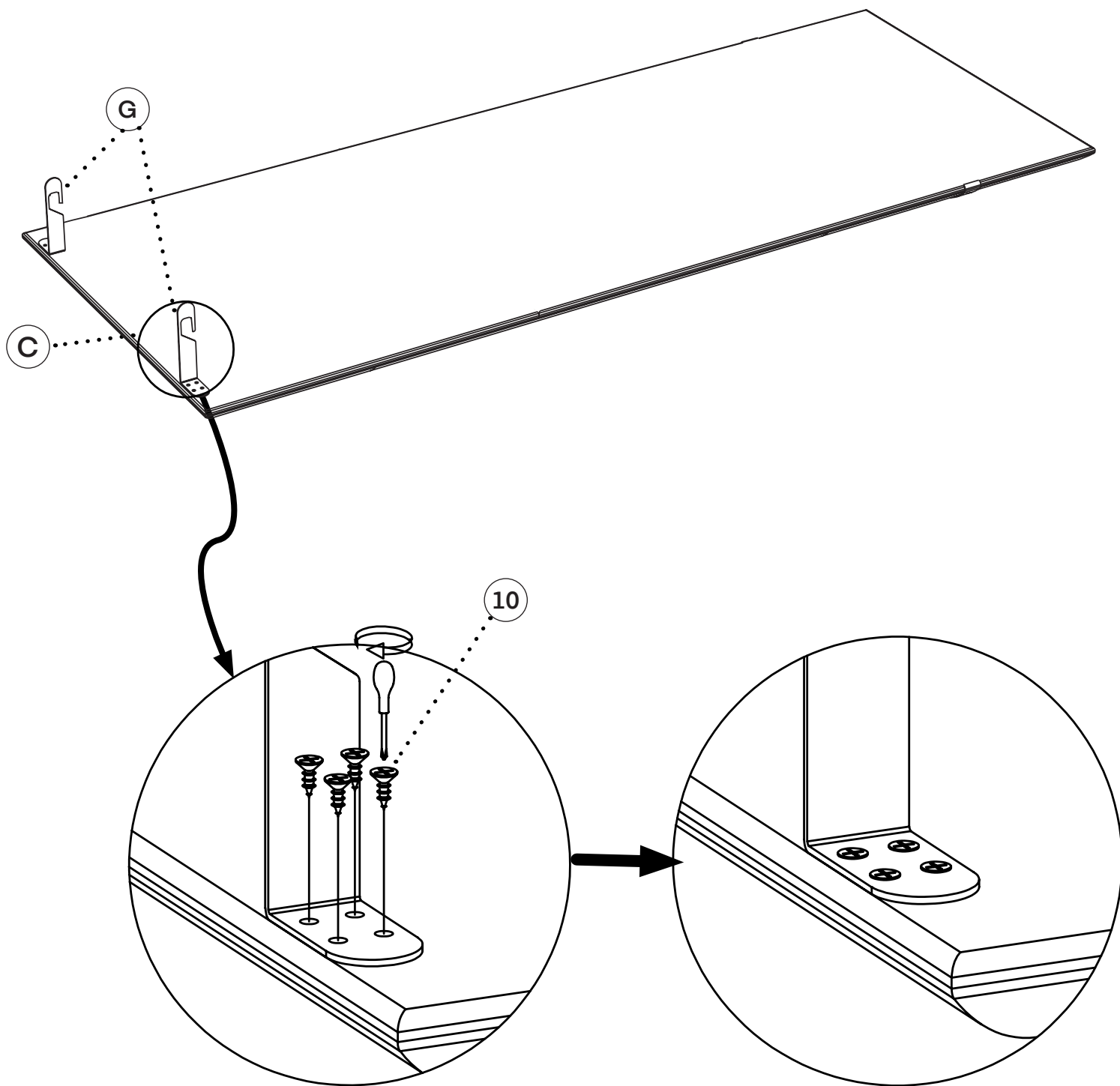

Step 2:


A. Place the Paper Roll Holder (H) between the left and right Support Element (G), as pictured.

B. You are now ready to utilize the Paper Roll Dispenser

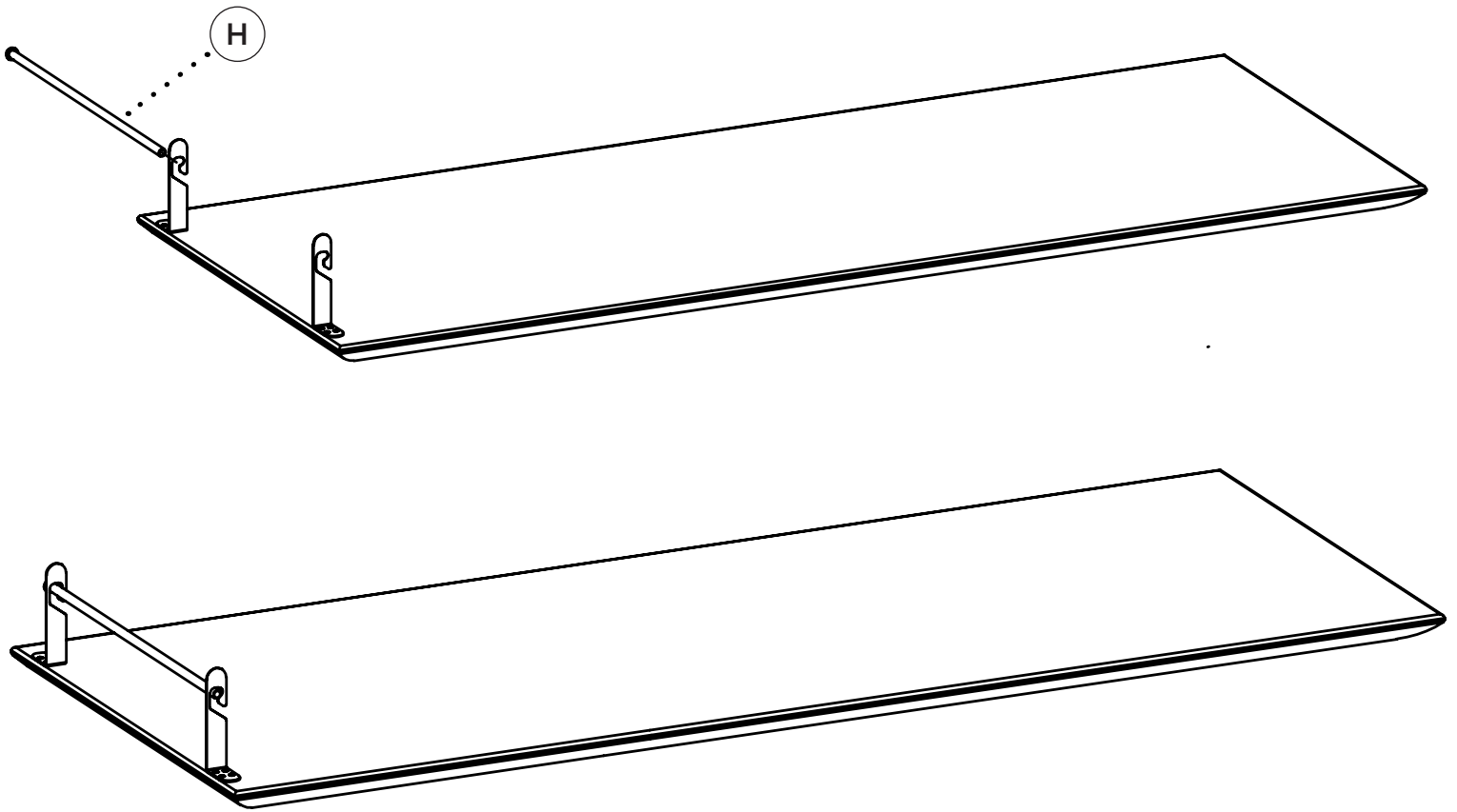

Step 3:

A. To install the Paper Cutter (I), slide it over the opposite end of the Mattress (C) until secure.

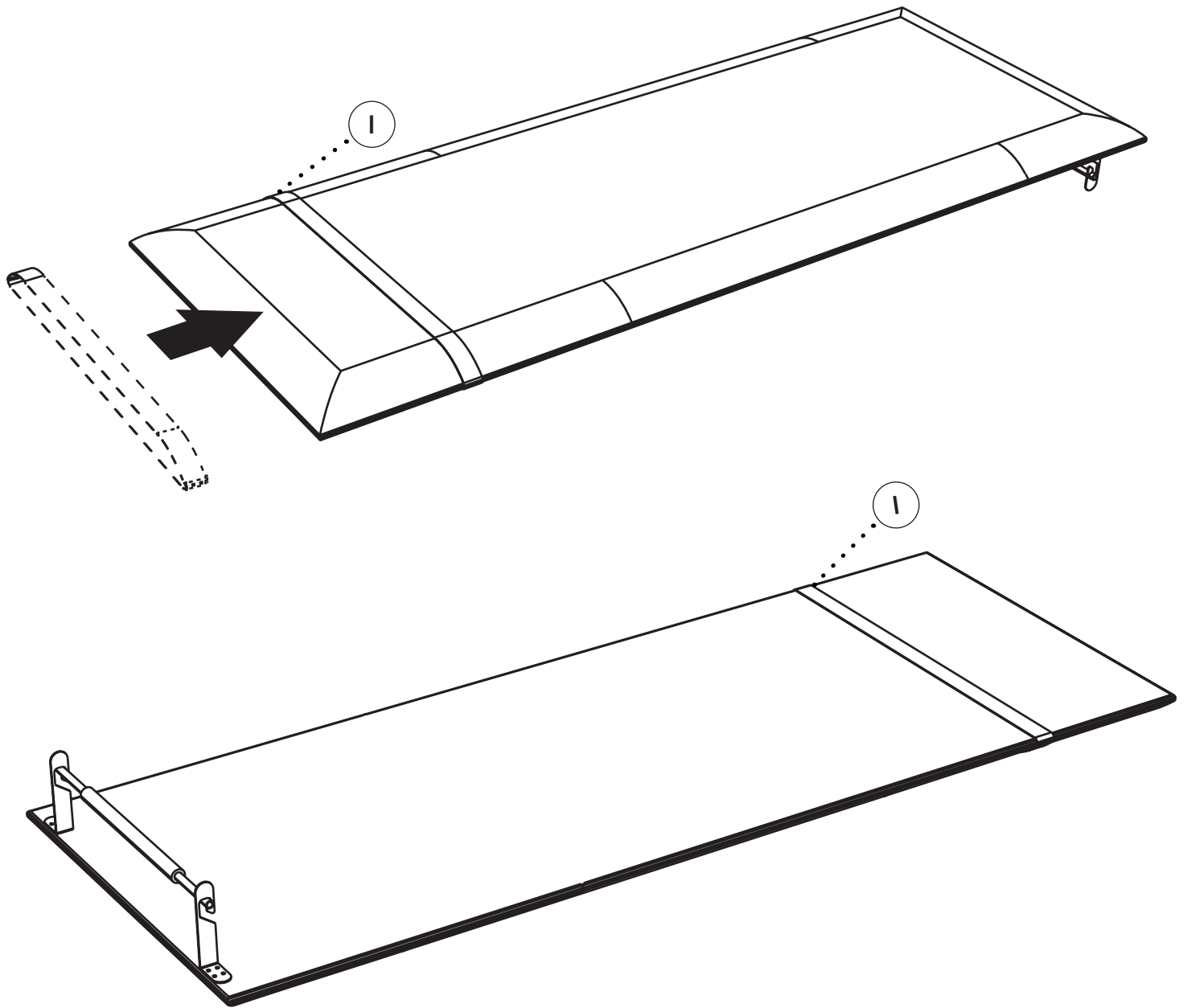

Care & Maintenance Guide

Disclaimer: Viva Comfort will not be held responsible for any damage to the upholstery or surfaces of our products due to cleaning attempts made, whether correctly or improperly conducted. We will not be liable for any damage to the upholstery or the product surfaces, such as discoloration or deterioration, from attempting to clean.

Thank you for purchasing a Viva Comfort product. We are honored to have your business and serve your medical equipment needs. To ensure your product(s) stay in optimal condition, even with consistent use, we have crafted a Care & Maintenance Guide to help you properly maintain and sanitize your product. Please carefully read these guidelines and follow the recommendations listed. Please contact our customer service representatives if you have any questions regarding these instructions.

Note: Be mindful when using sharp or pointed objects in the vicinity of upholstery and exam tables. They can cause damage to the exam table surfaces and puncture or tear the upholstery.

Note: DO NOT use pumice stones, scouring pads, steel wool, or textured cleaning pads to clean any surface of the exam table or the upholstery. We recommend a microfiber towel or a soft cloth.

Note: DO NOT store steel wool on/in your exam tables or on your upholstery.

Upholstery Cleaning:

Viva Comfort uses premium antimicrobial upholstery on all our exam tables. Consistent care should be taken to maintain the upholstery's integrity. We recommend that you always test the solutions below on an inconspicuous spot before using them on the entire upholstery. The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our upholstery.

The following Cleaners, Cleaning Materials, and Chemical Makeups will deteriorate and cause permanent damage to your upholstery.

Please Note: Should anything spill on your upholstery, wipe it up immediately and rinse it several times with water to avoid damage.






- Avoid using acidic or abrasive cleaners, as they can cause damage to your upholstery.
- Avoid using dyes (hair, textile, food) near your exam tables as they will permanently stain your upholstery. Should a dye spill occur, wipe it up as soon as possible with dishwashing detergent or an all-purpose cleaner. This is not guaranteed to solve the problem, it but can help reduce the chance of staining.
- Avoid using drainer cleaners or rust removers near your upholstery. Should drain cleaner spill, wipe it up as soon as possible, and rinse it several times with water. Should rust remover spill, wipe off all residue immediately, wash the area with dishwashing soap, and rinse it several times. Ensure you thoroughly dry the area.
- Avoid using the following products near or on your exam tables, as they will cause damage to the surface that may result in discoloration and/or deterioration.

CHEMICAL COMPOUND	AFFILIATED NAMES
Silicon Dioxide	Pumice/Pumice Stones, Degreaser
Sodium Hydroxide	Caustic Soda, Caustic, Lye, Soda Lye, Degreaser, Drainer Cleaner
Hydrofluoric Acid	Rust Remover, Toilet Bowl Cleaner, Concrete Cleaner, Metal Polishes
Phosphoric Acid	Tile Cleaner, Toilet Bowl Cleaner, Rust Remover, Concrete Cleaner
Sulfuric Acid	Oleic Acid, Oil of Vitriol, Oleum, Drain Cleaner, Heavy-Duty Acidic Cleaner
Hydrochloric Acid	Muriatic Acid, Hydrogen Chloride, Toilet Bowl Cleaner, Calcium, Lime and Rust Remover

Below are the recommended cleaning instructions for our antimicrobial upholstery.

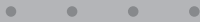Please Note: DO NOT use any alcohol-based cleaning agents, as this will degrade the upholstery.

<p>MILD SOIL LEVEL</p> 	<p>If the surface is lightly soiled, we recommend using a solution comprised of mild dish soap (10%) mixed in warm water (90%). Saturate a soft towel and clean the upholstery gently. In a separate bucket, have clean water and a soft towel. Thoroughly rinse the upholstery with clean water, as well as dry it completely.</p>
<p>MODERATE SOIL LEVEL</p> 	<p>If the surface is moderately soiled, we recommend using a solution comprised of Fantastik® (50%) and warm water (50%). Saturate a soft towel and clean the upholstery gently. In a separate bucket, have clean water and a soft towel. Thoroughly rinse the upholstery with clean water, as well as dry it thoroughly.</p>
<p>SEVERE SOIL LEVEL</p> 	<p>If the surface is severely soiled, we recommend using a solution comprising of bleach (10%) and warm water (90%). Saturate a soft, white towel and clean the upholstery gently. In a separate bucket, have clean water and a soft, white towel. Thoroughly rinse the upholstery with clean water, as well as dry it completely. Since this solution utilizes bleach, please only use it when necessary; ensure all the bleach solution is removed from the bed, do not let the solution sit on the upholstery, and understand that discoloration may occur.</p>

-
- Fantastik® is a registered trademark of DowBrands, Inc.®, Registered trademark of the Dow Chemical Co.®

Exam Bed Surface Cleaning:

The following Cleaners, Cleaning Materials, and Chemical Makeups will deteriorate and cause permanent damage to your exam beds' surfaces.

Please Note: Should anything spill on your exam table's surface, wipe it up immediately and rinse it several times with water to avoid damage.



- Avoid using acidic or abrasive cleaners, as they can cause damage to your exam table and its upholstery.
- Avoid using dyes (hair, textile, food) near your exam tables as they will permanently stain your table. Should a dye spill occur, wipe it up as soon as possible with dishwashing detergent or an all-purpose cleaner. This is not guaranteed to solve the problem, it but can help reduce the chance of staining.
- Avoid using drainer cleaners or rust removers near your exam table. Should drain cleaner spill, wipe it up as soon as possible, and rinse it several times with water. Should rust remover spill, wipe off all residue immediately, wash the area with dishwashing soap, and rinse it several times. Ensure you thoroughly dry the area.
- Avoid using the following products near or on your exam tables, as they will cause damage to the surface that may result in discoloration and/or deterioration.

CHEMICAL COMPOUND	AFFILIATED NAMES
Silicon Dioxide	Pumice/Pumice Stones, Degreaser
Sodium Hydroxide	Caustic Soda, Caustic, Lye, Soda Lye, Degreaser, Drainer Cleaner
Hydrofluoric Acid	Rust Remover, Toilet Bowl Cleaner, Concrete Cleaner, Metal Polishes
Phosphoric Acid	Tile Cleaner, Toilet Bowl Cleaner, Rust Remover, Concrete Cleaner
Sulfuric Acid	Oleic Acid, Oil of Vitriol, Oleum, Drain Cleaner, Heavy-Duty Acidic Cleaner
Hydrochloric Acid	Muriatic Acid, Hydrogen Chloride, Toilet Bowl Cleaner, Calcium, Lime and Rust Remover

Compatible Household Cleaners For Exam Bed Surfaces

Below is a list of compatible cleaners to use on the exam bed surfaces. We recommend that you always test the solutions below on an inconspicuous spot before using them on the entire surface. The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our exam bed surfaces. We will not be liable for any damage to your product surfaces, such as discoloration or deterioration, from attempting to clean.

(These cleaner recommendations are only for exam bed surfaces; DO NOT use the below cleaners on the upholstery. For instructions on how to clean your upholstery, refer to our "Upholstery Cleaning" section.)



Dawn®

Clorox®

Glass Plus®

Fantastik®

Lysol®

Lestoil®

Windex®




Formula 409®

Laminate Surface Cleaning:

Note: Though the tips below are recommended for cleaning soiled and stained areas, we will not be liable for any damage to the product surfaces, such as discoloration or deterioration, from attempting to clean.

Note: The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our product.

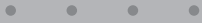Note: Be sure to utilize proper protective equipment before handling the below cleaning solutions.

MILD SOIL LEVEL 	If the surface is lightly soiled, we recommend using a soft cloth or microfiber towel dampened with a mild soap and water mixture to clean the area.
MODERATE SOIL LEVEL 	If the surface is moderately soiled, we recommend using a solution comprised of a mild household cleaner and baking soda. Create a paste consistency with the recommended cleaning agents, take a nylon bristle brush, and lightly scrub the affected area. DO NOT use a metal bristle brush nor aggressively scrub, or you will cause permanent damage to the surface. Thoroughly rinse the area with clean water, as well as dry it thoroughly.
SEVERE SOIL LEVEL 	If the surface is severely soiled, we recommend using undiluted bleach. Use a cotton pad/ball with bleach on it to gently rub the affected area for a minute or so. Thoroughly rinse the area with clean water, as well as dry it completely. Since this solution utilizes bleach, please only use it when necessary; ensure all the bleach solution is removed from the area, do not let the solution sit on the surface, and understand that discoloration may occur.

-
- Fantastik® is a registered trademark of Dow Brands, Inc.®, Registered trademark of the Dow Chemical Co.®
 - Dawn® is a registered trademark of The Procter & Gamble Company®.
 - Windex® is a registered trademark of S.C. Johnson & Son, Inc.®.
 - Clorox® is a registered trademark of The Clorox Company®.
 - Lysol® is a registered trademark of Reckitt Benckiser LLC®.
 - Formula 409® is a registered trademark of The Clorox Company®.
 - Glass Plus® is a registered trademark of Reckitt Benckiser LLC®.
 - Lestoil® is a registered trademark of The Clorox Company®.

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 3 years from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.



ADI996-HB-FM-SERIES

© 2023 Viva Comfort

vivacomfort.com