


PHYSICIAN BEAM SCALE

WITH HEIGHT ROD

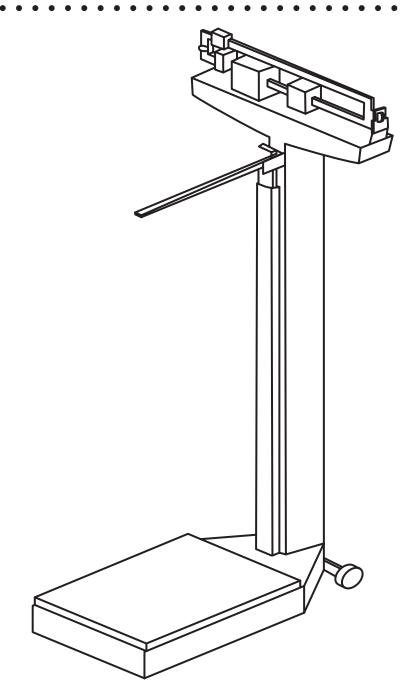

ADIME904-02 USER GUIDE



Thank you for your purchase.


Viva Comfort maintains a strong practice of ongoing development of products and commercial research.

We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.


Table of Contents

- 2 Parts List
- 3 Hardware List
- 4 Assembly Instructions
- 9 Warranty


Parts List

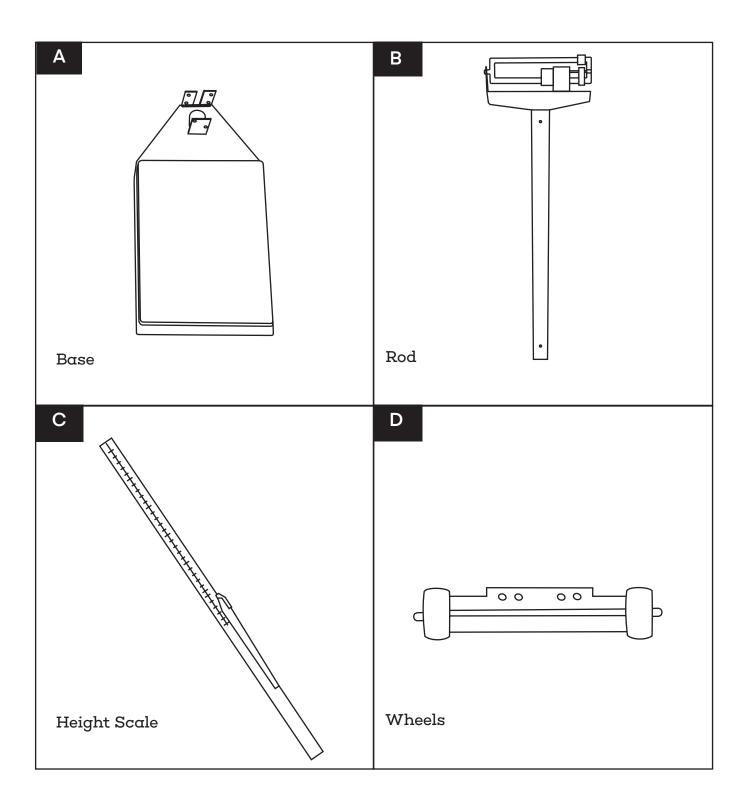

Hardware List

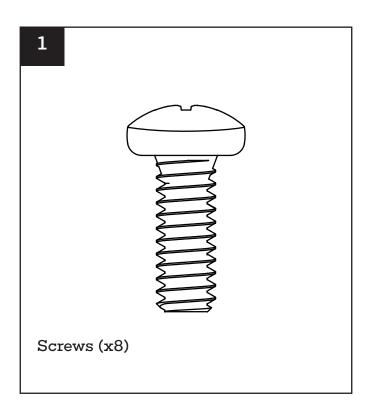

Tools Required (But Not Included):

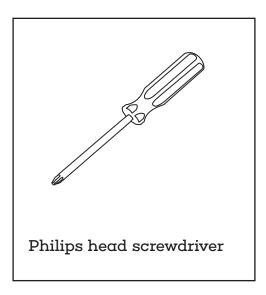


Assembly Instructions

Step 1:


A. Ensure you have all the necessary parts and hardware prior to assembly.

B. Locate the Base (A) and Rod (B) and ensure the Base (A) is on a flat surface.

C. The Rod (B) has a hook on the bottom of it that has to be connected to the Base (A).

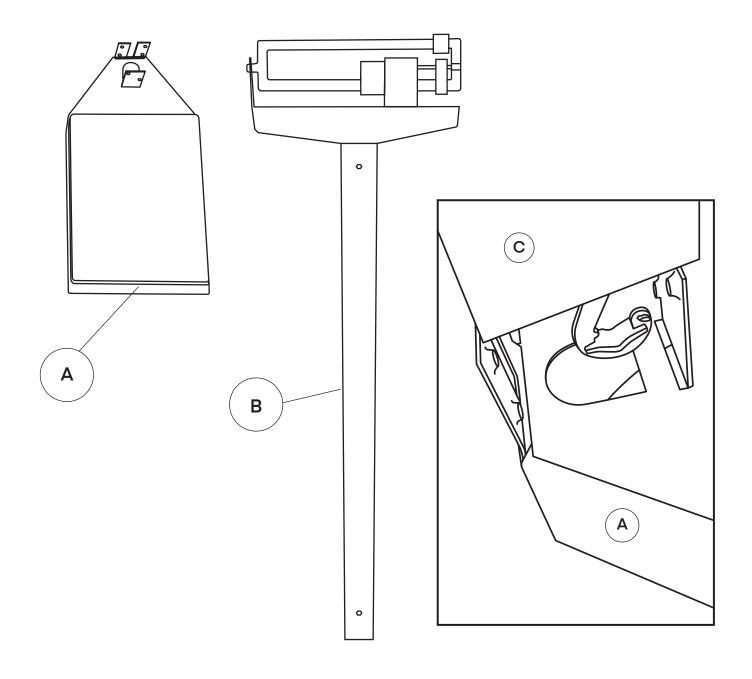

- **D.** Insert the Rod (B) into the hole on the Base (A), making sure the logo on the Rod (B) is facing inward.
- **E.** Ensure the hook catches on the affixed bar on the Base (A), as pictured.

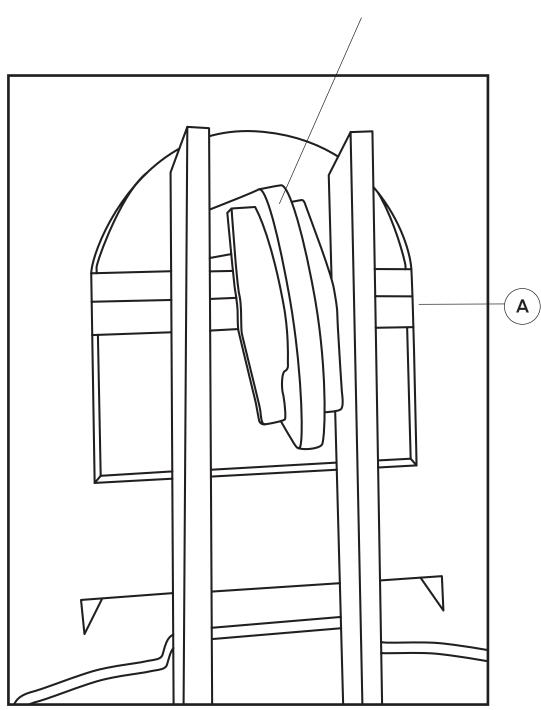


Hook

(View from the bottom)


Step 2:

- A. Once the hook is connected to the Base (A), it's time to insert 6 of the Screws (1) into the Rod (B).
- **B.** You will need to add four Screws (1) to the back of the Rod (B), as pictured.
- **C.** Secure them in the place with a Philips head screwdriver.
- **D.** Add two Screws (1) to the front, bottom of the Rod (B).
- E. Secure both in place with a Philips head screwdriver.

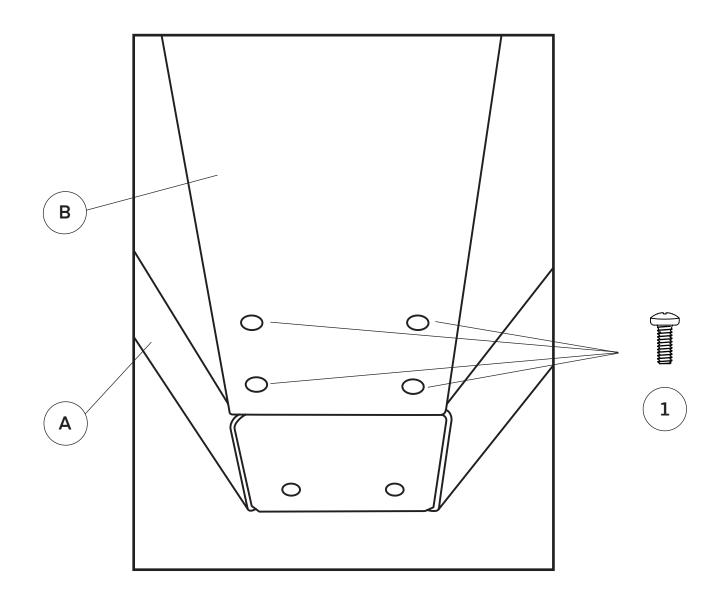


Step 3:

- A. Locate the Wheels (D) and the two remaining Screws (1).
- **B.** Line the Wheels (D) up with the two holes on the bottom, back of the Base (A).
- **C.** Insert the two Screws (1) through the holes on the Wheels (D) into the Base (A).
- **D.** Secure them in place with a Philips head screwdriver.

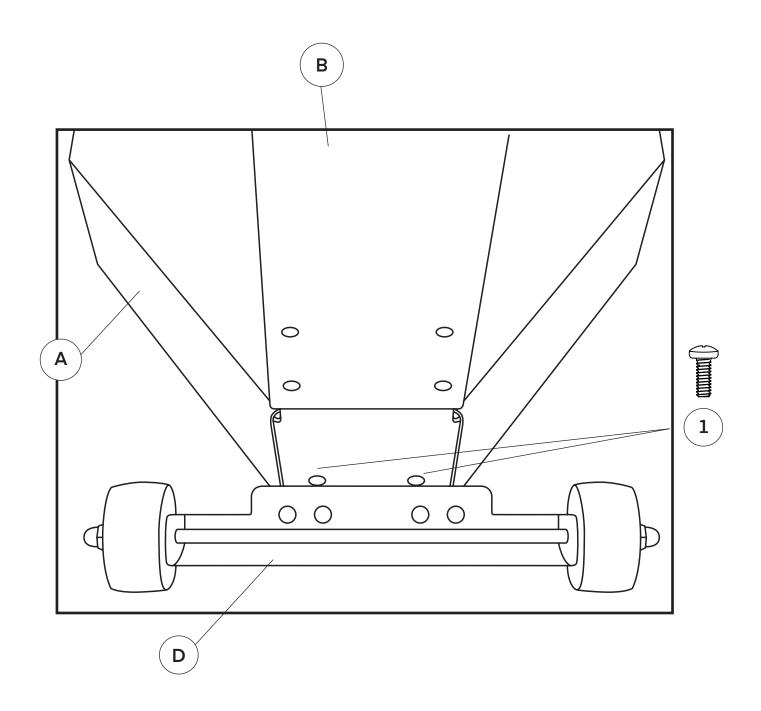

Step 4:

- A. Locate the Height Scale (C) and on the back of it, you will notice two key holes located on the top and bottom.
 - **a**. These are the points where you will adhere the Height Scale (C).
- **B.** There are two, preinstalled screws on the Rod (B), as pictured.
- C. Insert the two key holes over the preinstalled screws on the Rod (B).
- D. Slide the Height Scale (C) downward until it locks in place.
- E. Your Physician Beam Scale with Height Rod is now ready to be used.

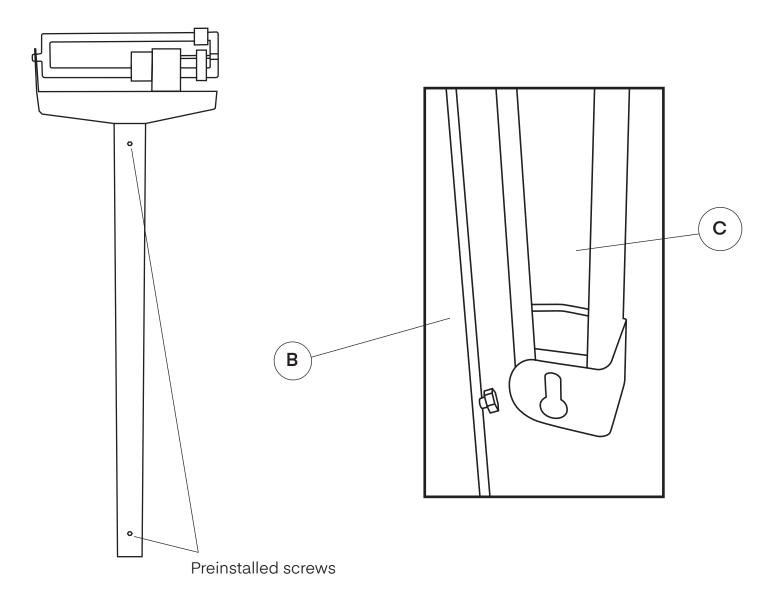

Warranty

- 1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
- 2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
- 3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
- 4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
- 5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
- 6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
- 7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
- 8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
- 9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
- 10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
- 11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.



ADIME904-02

© 2023 Viva Comfort vivacomfort.com