

.....

DIGITAL WHEELCHAIR

.....



USER GUIDE

SKU: ADIME904-07



Thank you for your purchase.



**Viva Comfort maintains a
strong practice of ongoing
development of products and
commercial research.**




**We recognize that starting and maintaining a medical practice is
expensive. We'll help make the process cost-effective and elegant.
Additionally, our team of specialists will ensure everything you
need works as efficiently as possible, so you can effectively treat
your clients.**

Table of Contents

2	Parts List
3	Hardware List
4	Summary
5	Main Technical Parameters
6	Assembly Method
7	Assembly Instructions
13	Key Function Description
14	BMI Introduction
15	Warranty
16	Notes

Parts List

1

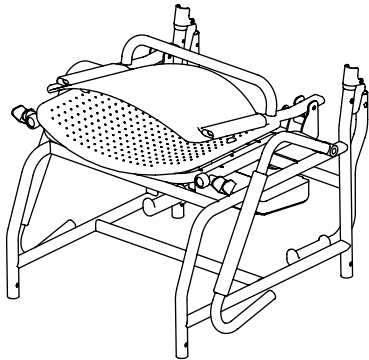


Chair Frame (x1)

2




Direction Wheel (x2)

3

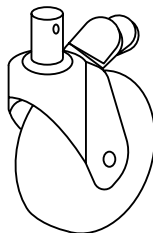





Swivel Wheel (x2)

4


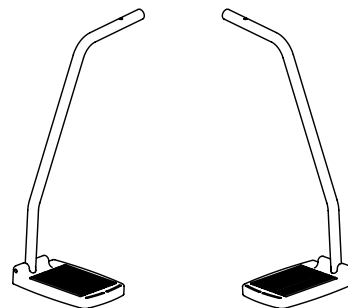


Leg Frame (x2)

Hardware List

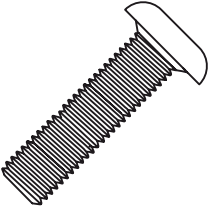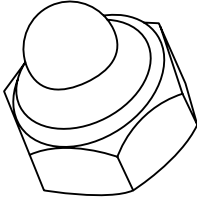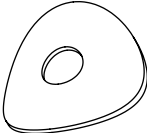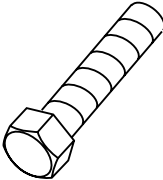A  Screws M6*33 mm (x4)	B  Nut M6 (x6)
C  Washer (x10)	D  Screws M6*30 mm (x2)

Tools Needed

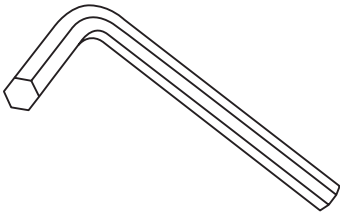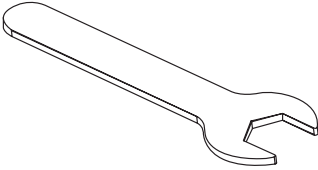E  Hex Key (x1)	F  Wrench (x1)
---	---

Summary

Thank you for choosing this Digital Wheelchair Scale. Before use, please read the manual thoroughly and keep it accessible for future reference.

This device features a high-precision load cell and advanced microcomputer control technology for accurate and reliable measurements. The large, easy-to-read LCD screen displays weight, tare, unit conversion, and BMI functions. Combining precise weighing capabilities with a comfortable, easy-to-manuever chair design, the Digital Wheelchair Scale is ideal for use in homes, hospitals, clinics, and other healthcare settings.

Important Safety Use Note

Battery Use Note

- This product contains a special rechargeable lithium battery. Do not use batteries from other brands, types, or models.
- When the LCD displays "LO," it means the battery is low—please charge it promptly.
- Always use the original charger provided. Using a different charger may cause damage or injury.

Weighing Use Note

When weighing:

- Always operate under the supervision of a nurse or trained caregiver.
- Place the scale on a level, hard surface. Do not use on slopes or uneven ground.
- Ensure the rear wheels are locked before use.
- Do not attempt to self-diagnose using the weight results. Please consult a medical professional for guidance


Other Note

- This product has been calibrated and tested before shipping. Do not open or modify it, as this may damage the equipment or result in inaccurate readings.
- This device must be used under the supervision of a nurse. It is not intended for other purposes (e.g., as a regular chair or for weighing non-human objects). Misuse may result in injury or damage.
- Keep the product away from electronics to avoid interference. Do not store or use it outdoors or near electromagnetic fields. Store it in a temperature- and humidity-controlled environment as specified.

Main Technical Parameters

Temperature	32°F – 104°F
Humidity	<85%
Accuracy Grade	C3
Max Weight	550 lbs.
Dim	36.6" × 24.4" × 38"
Tap Distance from floor	4.3"
Seat Distance from floor	21.3"
Net Weight	51.1 lbs.
Displayer	LCD displayer, 4.7" × 2.4"
Working Voltage	DC 7.4V (built-in rechargeable Li-ion battery)
Adaptor	Input AC 100-240V, Output DC 12v, 2.5A
Load Cell	Max Carry: 661 lbs, 5.9" × 1.7" × 1.6"

Basic Composition and External Node

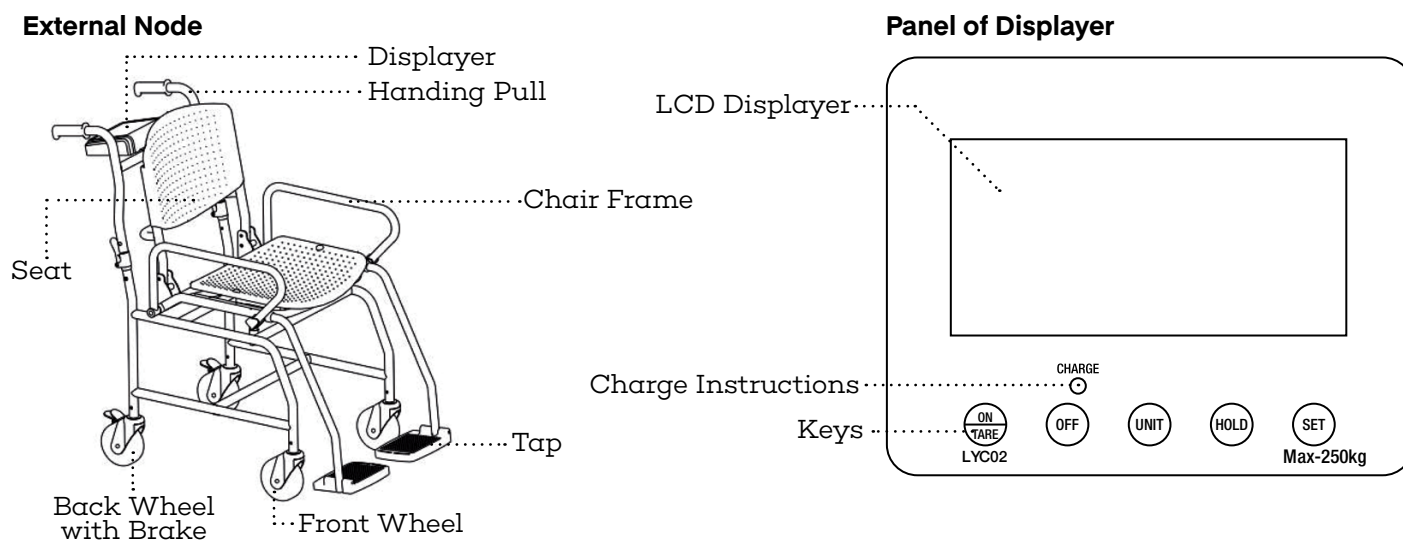

Assembly Method


- A.** Open the carton, check the packing list, and ensure all components are included. If anything is missing, please contact your local distributor.
- B.** Place the Digital Wheelchair Scale on a hard, level surface. Unfold the handle, adjust the footrest, and position the display screen for easy viewing (see Photo 4.1). The recommended clearance between the footrest and the floor is at least 4.3 inches (110 mm). Connect the wire from the load cell to the display unit.
Turn on the power switch located on the back of the display. Then press the "ON/TARE" button to activate the scale. The unit will perform a self-test before entering weighing mode. If the LCD does not read "0," press "ON/TARE" again to zero the scale.
- C.** Tare Function: After powering on and confirming the scale displays "0," place the item or person on the scale. Once the weight stabilizes, press "ON/TARE" to subtract the weight and return the display to "0."
- D.** Weighing Procedure: Before weighing, lock the rear wheels. Assist the individual into the chair. Once seated and the LCD displays a stable reading, the displayed value is the accurate weight.

Assembly Instructions


Note: When assembling the chair, do not over-tighten the screws. Finish the whole assembly and then tighten firmly.

Step 1:


A. Take off all packaging materials from the Chair Frame (1) and unfold it as shown in the picture.

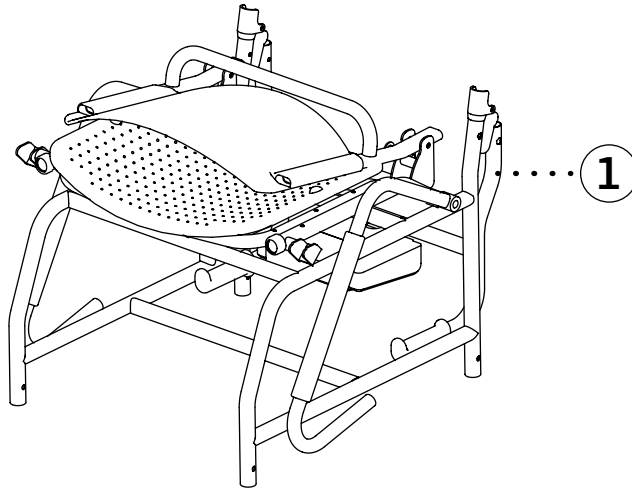

B. Turn both outfitted armrests counterclockwise and slide them into the slots until they are secure.

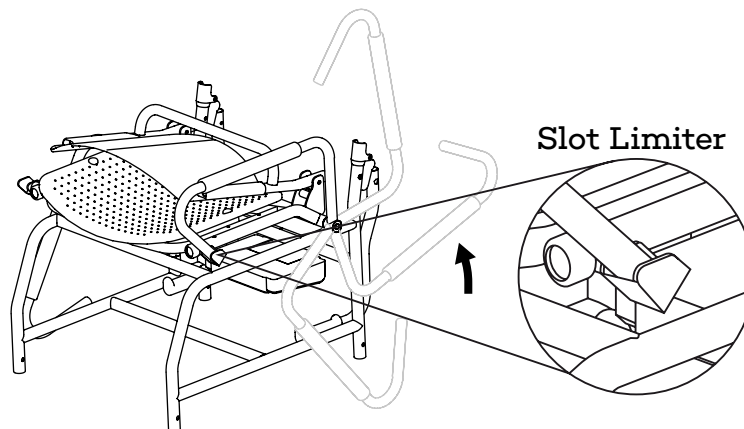


C. Lift the plastic backrest clockwise toward the back of the Chair Frame (1) until it locks into place.

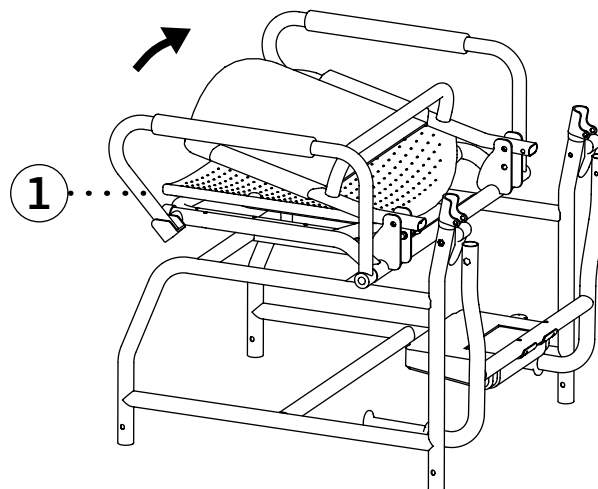


Step 2:

- A. While lifting the part, pull the integrated Spring-Loaded Pin outward using your fingers. as pictured.
- B. When the pin reaches the hole in the limit tube, let go of the pin so it locks into place.

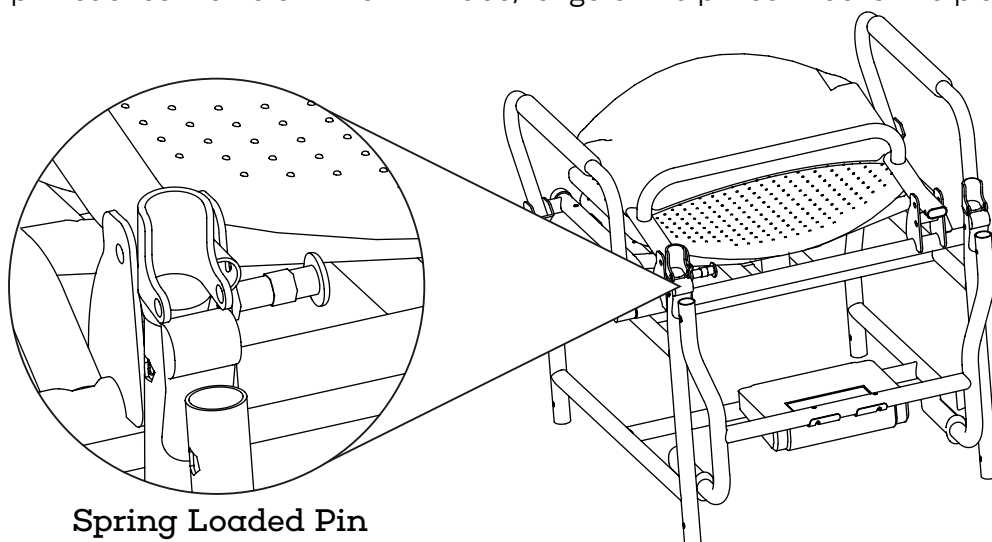

Step 3:

- A. Turn the Handle Frame counterclockwise and lift it toward the front of the Chair Frame (1) until you hear a "click" sound. This means it is locked in place.

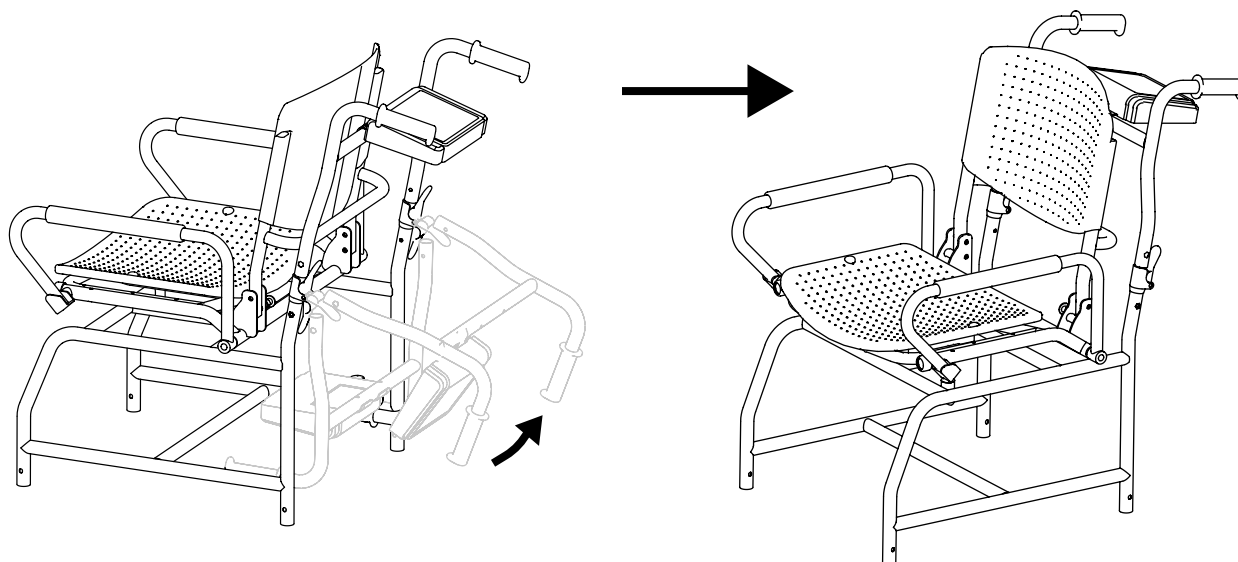


Step 4:

- A.** Gather the following parts: 2 Directional Wheels (2), 2 Swivel Wheels (3), 4 M6x33mm Screws (A), 4 M6 Nuts (B), 8 Washers (C), 1 Hex Key (E), 1 Open-End Wrench (F).
- B.** Insert one Directional Wheel (2) into each of the front legs of the Chair Frame (1), as pictured.




Step 5:

- A.** Take one M6x33mm Round Head Hex Socket Screw (A) and slide it through two Half-Round Washers (C) from the front side of the Chair Frame (1), as shown in the picture.
- B.** Repeat the same process on the other side.

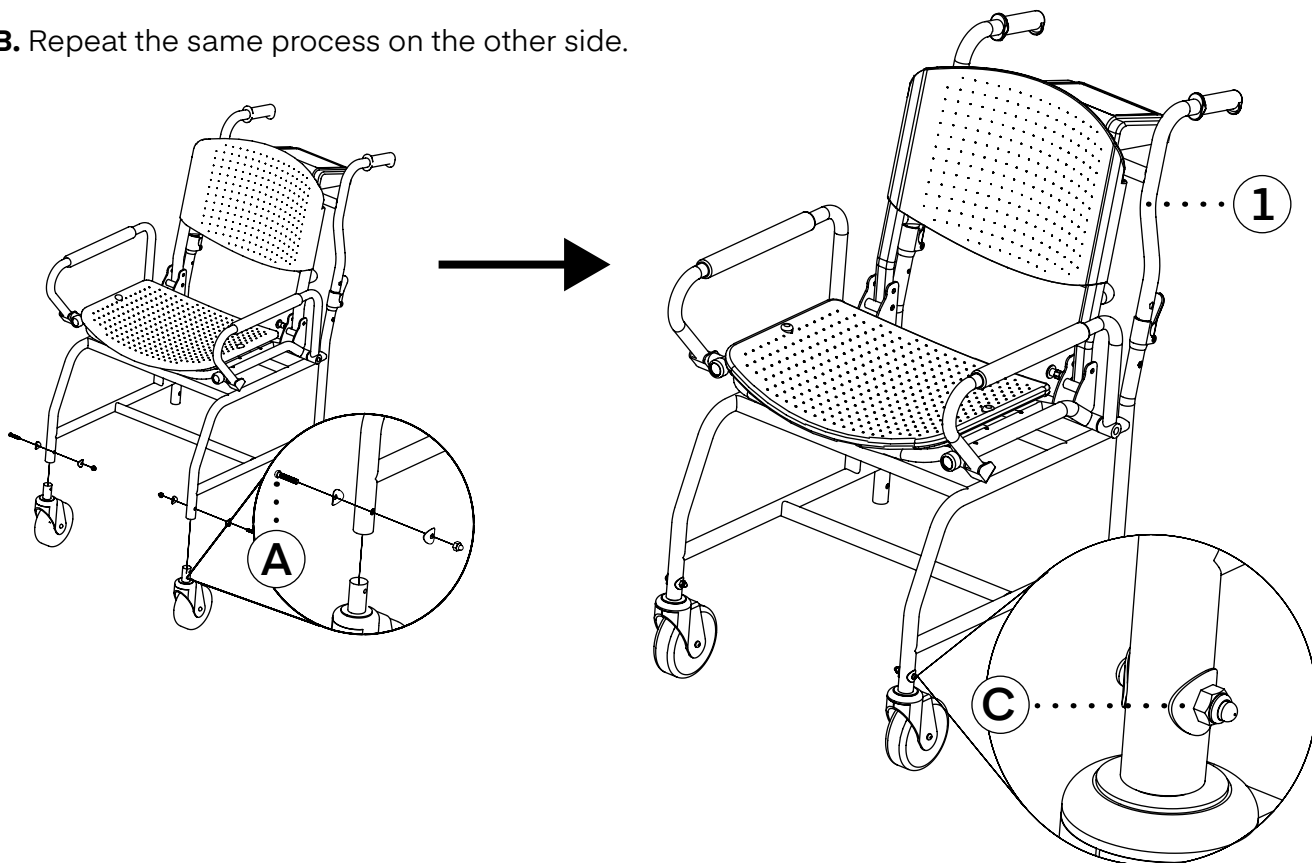


Step 6:

- A.** First, insert the Swivel Wheels (3) into the rear legs of the Chair Frame (1).
- B.** Slide two Half-Round Washers (C) onto one M6×33mm Round Head Hex Socket Screw (A), then secure it with one M6 Cap Nut (B).
- C.** Tighten the screw using the M4 Hex Key (E) and Open-End Wrench (F).
- D.** Repeat these steps to install all four wheels (Directional Wheels in front, Swivel Wheels in rear).

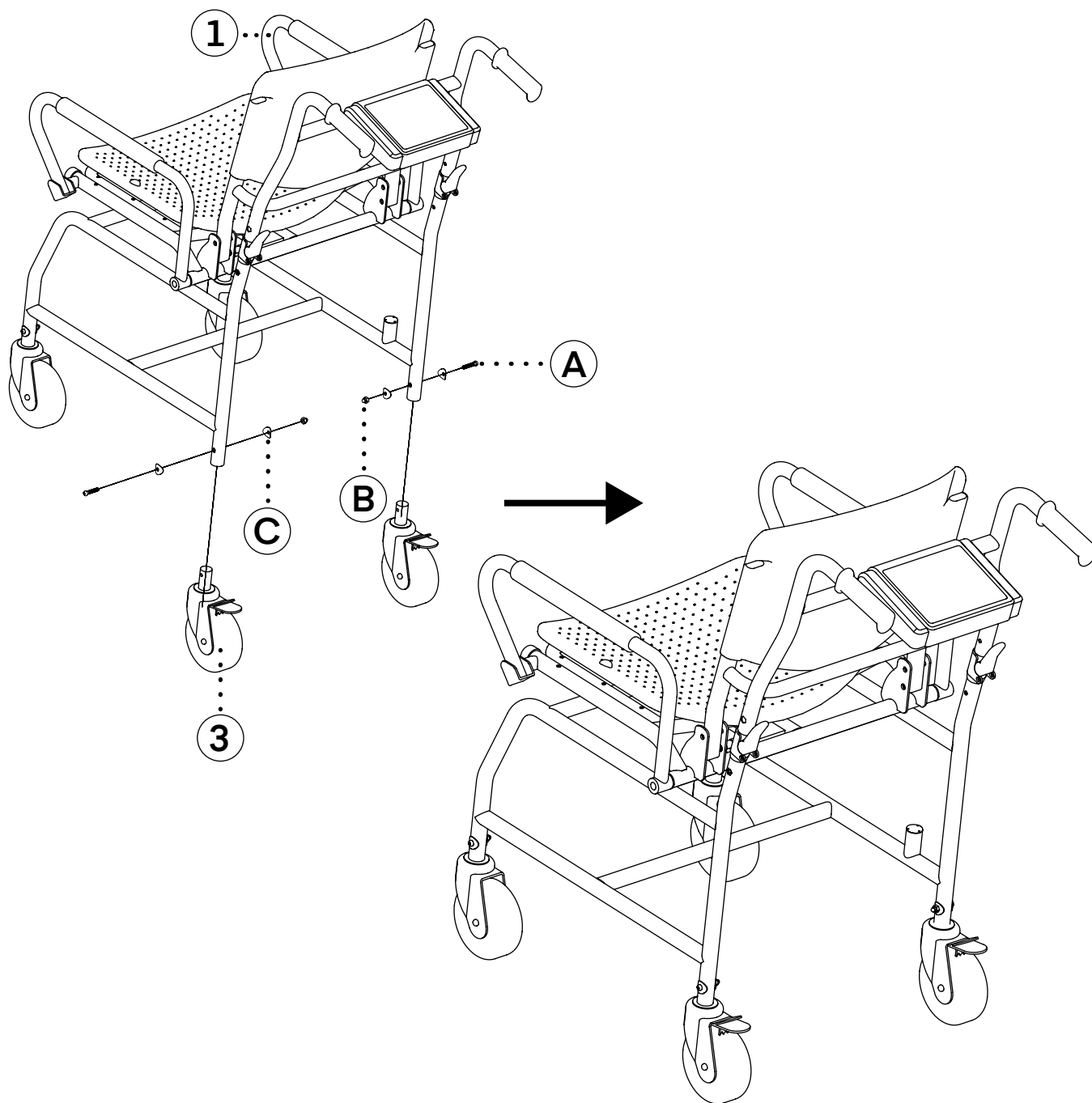


Step 7:

- A.** Find the following parts: 2 Leg Frames (4) (left and right), 2 M6×30mm Hex Head Flat Screws (D), 2 M6 Cap Nuts (B), 2 Half-Round Washers (C).
- B.** Insert the end of each Leg Frame (4) with holes into the tube under the seat panel, as shown in the picture, until the holes on the Leg Frame align with the small round holes under the seat panel.

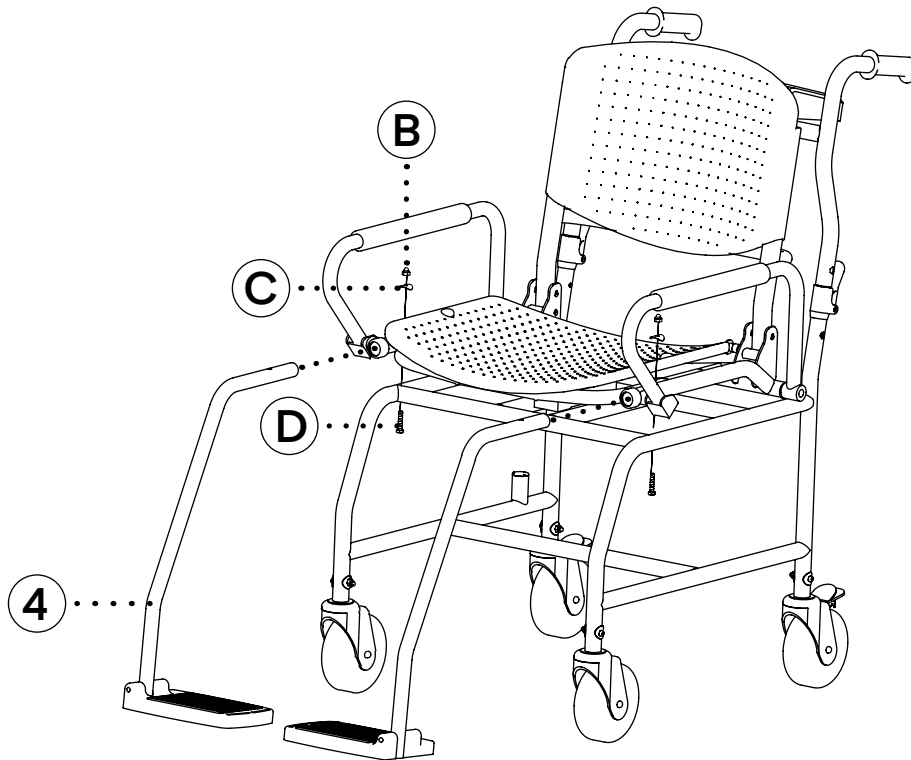

- C.** Insert one M6×30mm Hex Head Flat Screw (D) through the hexagonal hole under the seat panel and into the aligned hole on the Leg Frame (4).

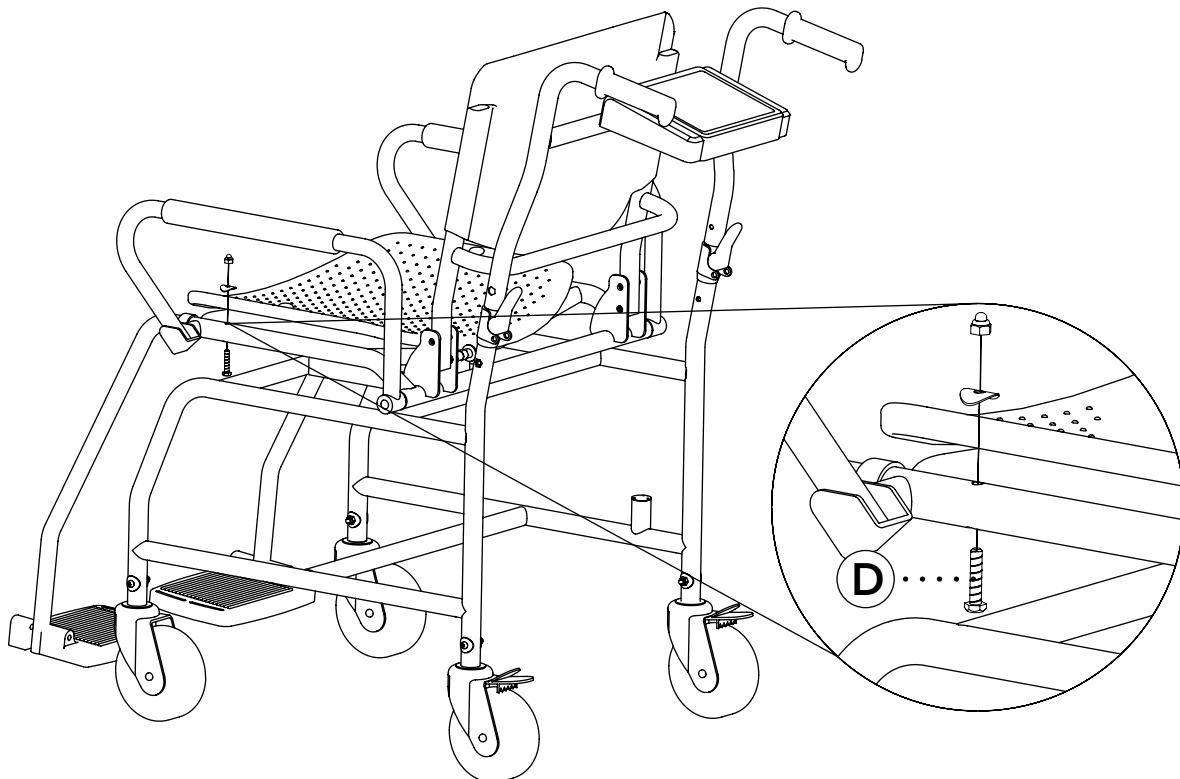

Step 8:

- A.** Slide one Half-Round Washer (G) onto the screw, then secure it with one M6 Cap Nut (F)..
- B.** Tighten the screw using the M4 Hex Key (I) to firmly fix it in place.
- C.** Repeat these steps for the other Leg Frame.
- D.** Once both sides are secured, the installation is complete.

Step 9:

- A.** Make sure all screws are fully tightened and secure.
- B.** Your Digital Wheelchair is now ready for use.



Key Function Description

1. "ON/TARE" key:

- A. When the scale is off, short press to turn it on.
- B. In weigh mode, short press to deduct tare weight.
- C. If the display doesn't show zero, short press to reset the weight to "0".
- D. In setting mode, this key increases the value.

2. "OFF" key:

- A. Short press to turn off the scale.

3. "UNIT" key:


- A. In weigh mode, short press to switch between kilograms (KG) and pounds (LB).
- B. In setting mode, this key decreases the value.

4. "HOLD" key:

- A. After zeroing, press this key to lock the displayed weight. The display will show "HOLD" and keep showing the weight even after the person leaves the chair. Press again to clear the hold and return to normal weighing.

5. "SET" key:

- A. Press to enter user information settings, such as the person's height.

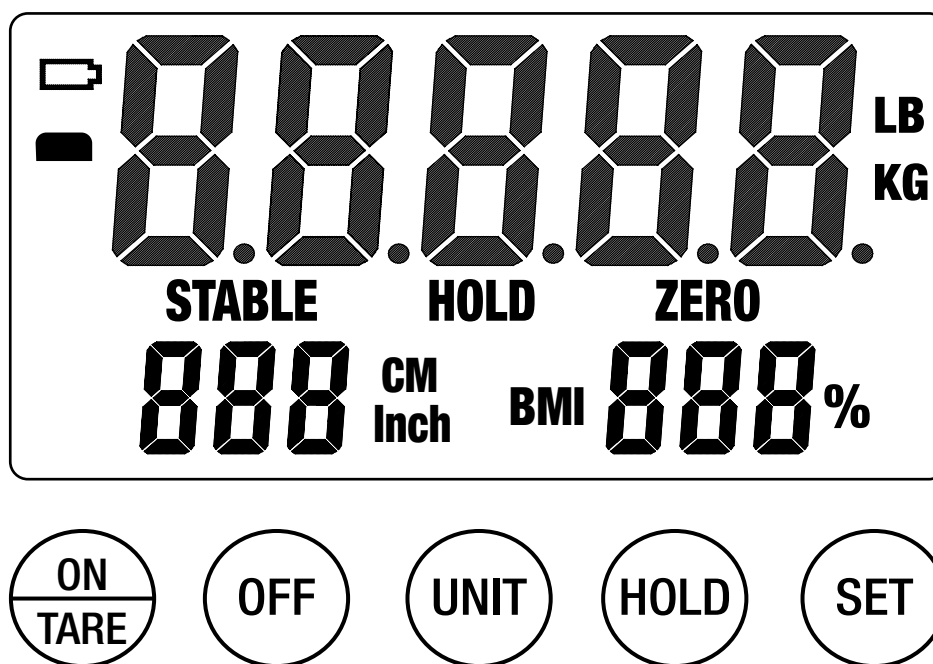

BMI Introduction

Universal Standard	<18.5	Thin
	18.5 - 24.9	Normal
	>25	Obesity
	>30	Partial Obesity
	>40	Severe Obesity
Asia Standard	<18.5	Thin
	18.5 - 22.9	Normal
	>23	Obesity
	>30	Partial Obesity
	>40	Severe Obesity


Care, Maintenance, and Troubleshooting

- A.** When the display shows "LO," recharge the scale promptly to avoid inaccurate readings or failure to weigh.
- B.** If the scale will not be used for an extended period, turn off the power switch located on the back of the display.
- C.** Always place the scale on a hard, level surface. Adjust as needed to ensure it is perfectly level for accurate results.
- D.** After each use, clean the scale and keep it dry.
- E.** Unauthorized disassembly or improper use (such as overloading or strong impact) may damage the load cell or PCB and will void the warranty. If overloaded, the display will show "-HI-."
- F.** For any questions or concerns, please contact your local distributor for assistance.

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 2 years from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

[illegible]

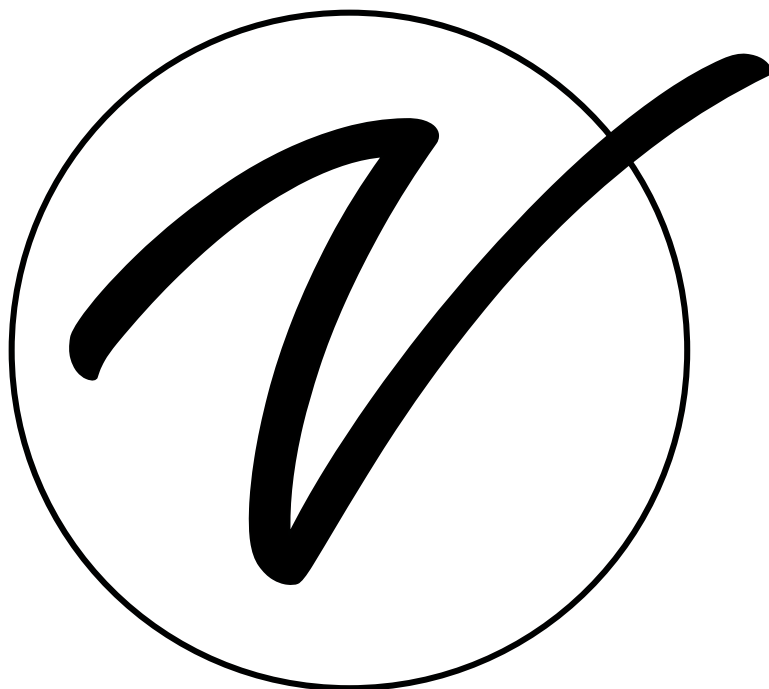

SKU: ADIME904-07

© 2025 Viva Comfort

vivacomfort.com