


REHAB MOBILE
TREATMENT MIRROR

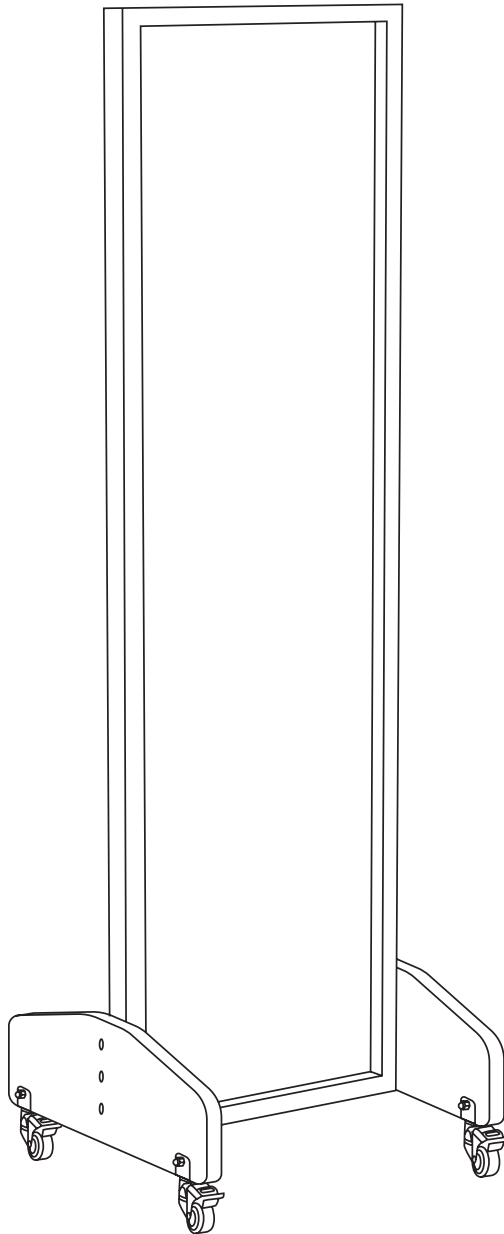

USER GUIDE

SKU: ADIME908-02



Thank you for your purchase.



Viva Comfort maintains a strong practice of ongoing development of products and commercial research.



We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.

Table of Contents

2	Specifications
2	Recommendation
3	Parts List
3	Hardware List
4	Preparation and Inspection Before Use
4	Assembly Instructions
7	Care & Maintenance Guide
7	Safety Warning
9	Warranty

Specifications

Product Dimensions: 72" H x 26.5" W x 3" D

Weight: 77 lbs.

Materials: Plywood Frame, Glass Mirror, Swivel Casters

Recommendation


Recommended Number of People Required for Assembly







2 persons

We advise using two people to prevent damage during assembly.

Parts List

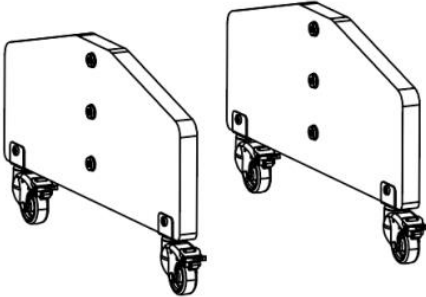A  Mirror Frame	B  Leg Plates (x2)
---	---

Hardware List

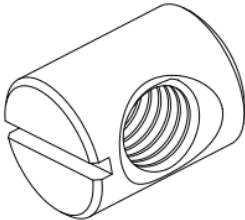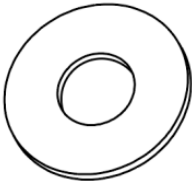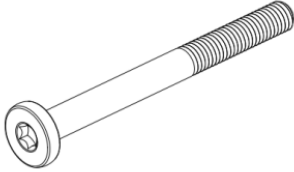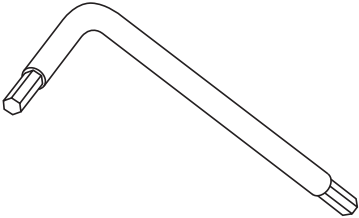1  Cylindrical Nut (x6)	2  Washer (x6)	3  Bolt (x6)
4  Wrench		


Preparation and Inspection Before Use

- A.** Verify that the product includes a manual.
- B.** Check that the installation hardware is provided and attached.
- C.** Inspect all components for any visible damage or deformation.
- D.** If you encounter any issues during installation or use, contact your seller promptly.
- E.** Carefully remove the product from its packaging and place it on a flat, stable surface.
- F.** Lay down a clean blanket or soft cloth to protect the floor and mirror from scratches or impact.
- G.** Ensure the area is free of debris, sharp objects, or contaminants.

Assembly Instructions


Step 1:

- A.** Locate and insert the Cylindrical Nut's (1) into the designated hole on the back of the Mirror Frame (A), as pictured.
- B.** Align the threaded hole of the Cylindrical Nut (1) with the outer hole of the Mirror Frame (A), as pictured.

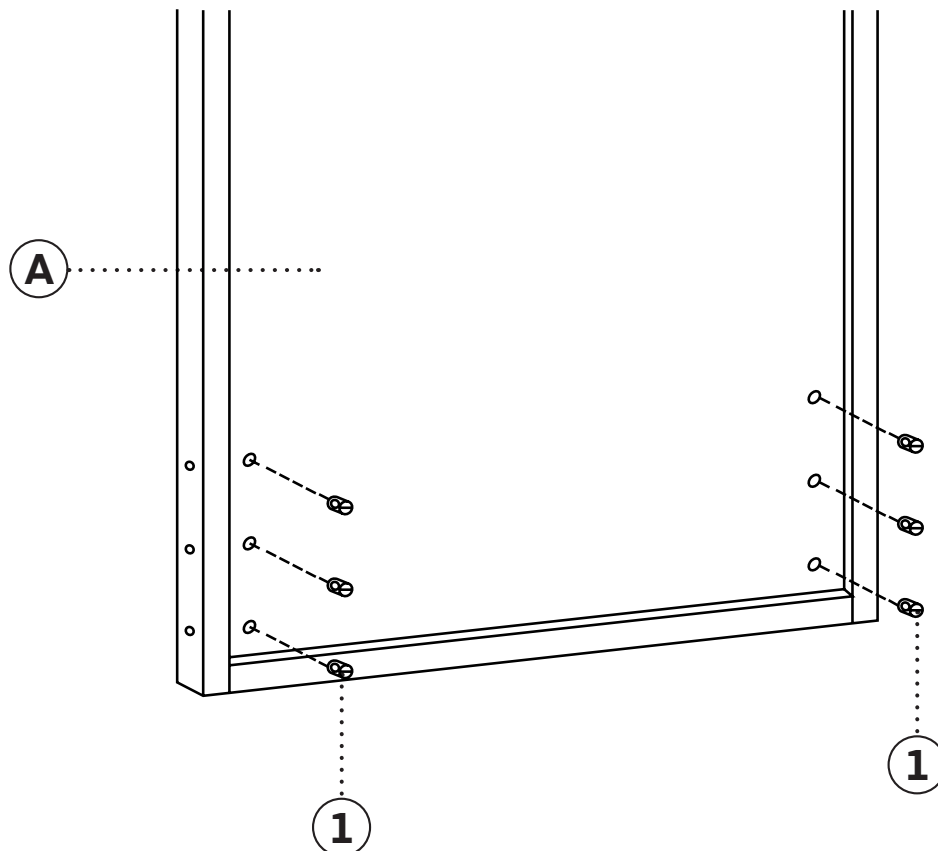


Step 2:

- A.** Position the left and right Leg Plates (B) on the corresponding sides of the Mirror Frame (A), ensuring all pre-drilled holes align.
- B.** Slide a Bolt (3) through a Washer (2), then insert it through the aligned holes on the Leg Plates (B) and Mirror Frame (A).
- C.** Use the Wrench (4) to turn the Bolt (3) clockwise and fully tighten.
 - a.** Repeat for the remaining Bolts (3) and Washers (2).

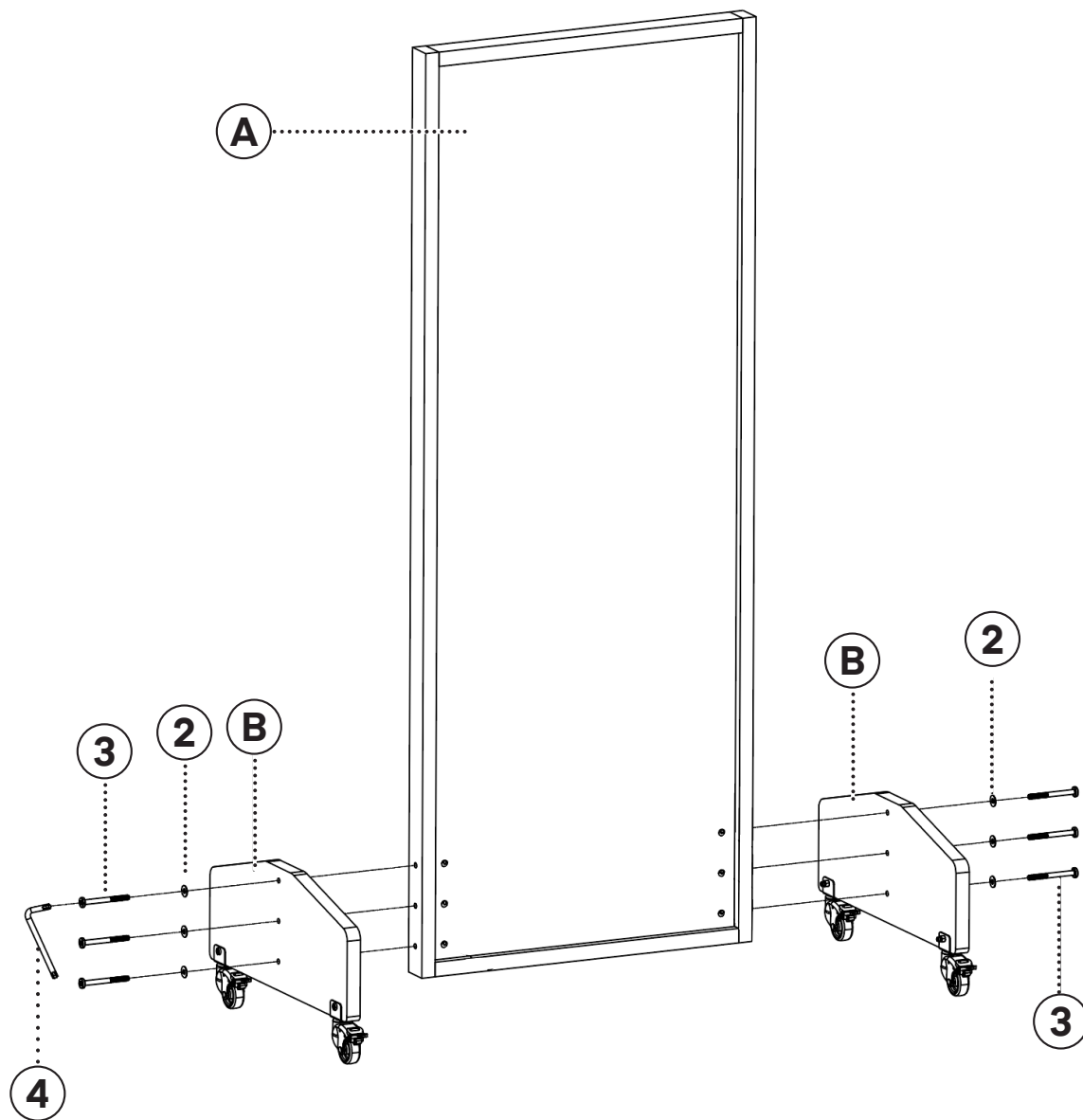

Step 3:

- A. Carefully stand the mirror upright, ensuring the casters face the ground and the frame faces upward.
- B. Place the mirror on a flat, level surface.
- C. Lock all four casters as shown to secure the mirror in place.

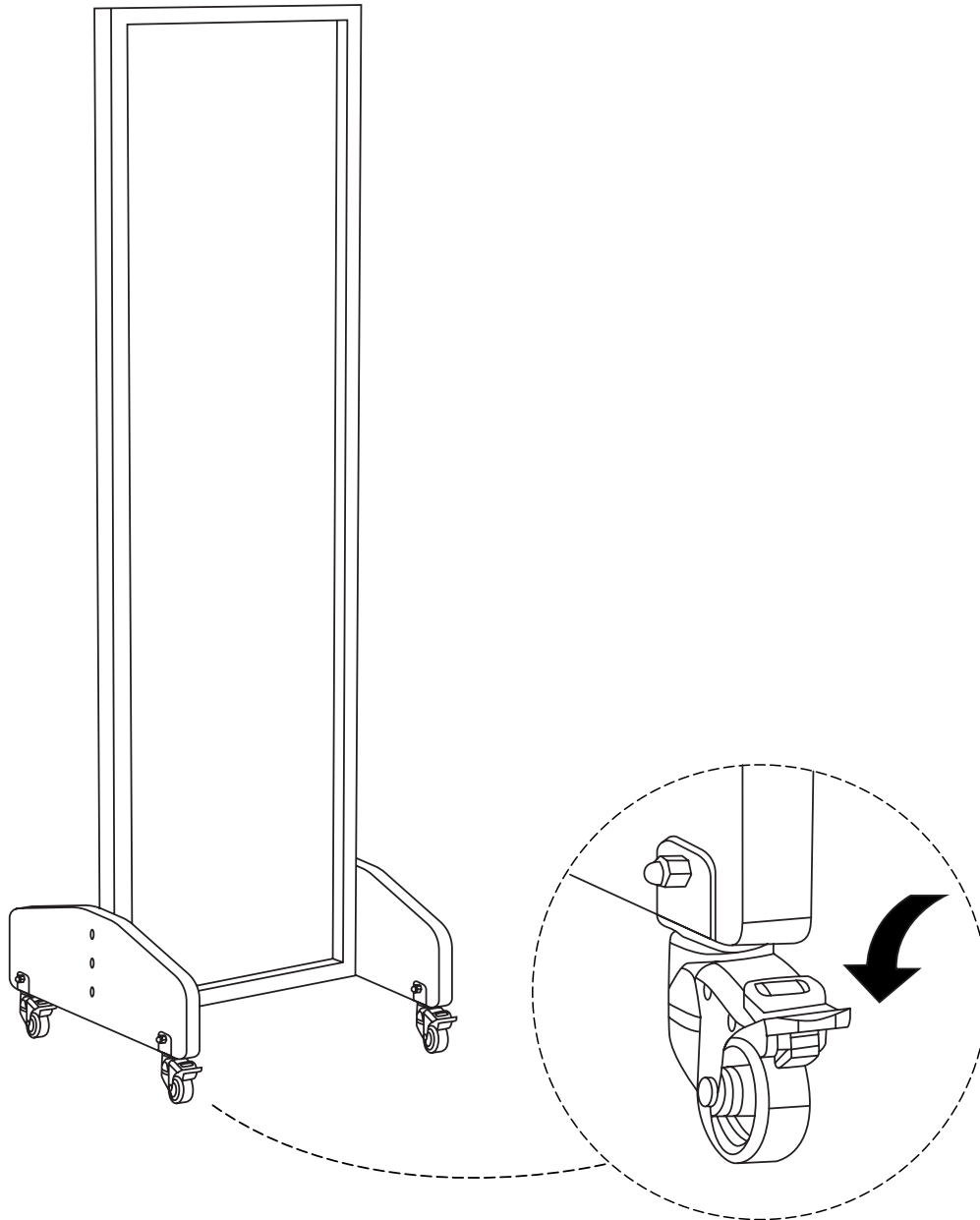

Care & Maintenance Guide

A. Maintenance of Metal and Plywood Frame:

- a. Wipe the metal and plywood surfaces with a dry, soft towel.
- b. For disinfection:
 - Use hypochlorous acid disinfectant or alcohol with less than 70% concentration.
 - Avoid highly corrosive disinfectants, as they can damage the plywood finish.

B. Regular Maintenance (Every 3 Months)

- a. Inspect the table structure, including the surface and frame.
- b. Ensure the left and right foot plates are secure and properly positioned.
- c. Tighten all bolts, including those securing the frame and wheels.

Safety Warning

A. The user is responsible for preventing damage to this product and injury to patients and/or users.

	Reminder: Keep away from moisture. This device needs to be moisture and waterproof.
	Reminder: Always place the packaging box in the correct direction during transportation, storage, and handling to avoid damage and safety problems caused by incorrect placement.
	Reminder: This product is fragile and needs to be handled with care.

B. Risk of Mirror Breakage:

-  Under no circumstances should you allow a mirror to fall over, as an accidental fall could result in the mirror glass shattering.
-  A hard object touching or striking the mirror can cause it to shatter.
-  Forgetting to lock the casters may cause the mirror to move or fall over.
-  Placing the mirror on a sloping surface may cause the mirror to move and fall.

C. Risk of Puncture:

-  After the mirror glass is broken, please wear thick protective gloves to carefully clean the glass fragments to avoid puncturing the skin.
-  After the mirror glass is broken, please use a vacuum cleaner to clean up all the fragments on the ground to avoid small glass fragments scattering on the ground and causing foot injuries.

Safety Warning (continued)

D. Infection Risk:

When cleaning, please wear gloves, masks, and eye protection to prevent the cleaning spray from rebounding from the contaminated surface.

E. Plywood Frame – Swelling and Moisture Warning

a. Do not allow the plywood frame to come into contact with water. Exposure to moisture may cause swelling and lead to surface peeling or delamination.

b. The recommended humidity range for transport, use, and storage is 20% to 80%. Prolonged exposure to excessively dry or humid environments can cause the surface to crack, peel, or warp.

c. When cleaning the glass, avoid over-spraying detergent. Excess liquid may seep into the gap between the glass and plywood, resulting in water damage and swelling.

F. Hardware – Rust Prevention Guidelines

a. Keep the product in a controlled environment with humidity between 20% and 80%. Extended exposure to high humidity can cause rusting of metal components.

b. Avoid contact with acidic substances on any metal parts, as they may damage protective coatings and lead to corrosion.

c. Do not expose metal parts to water. Prolonged or repeated contact can cause rust and degrade performance.

G. General Safety & Placement Guidelines

a. Keep away from open flames and heat sources.


b. Do not place the product in direct sunlight for extended periods.

c. Avoid placing near heaters or high-temperature areas.

d. Indoor use only – not suitable for outdoor environments or areas with extreme temperature or moisture conditions.

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

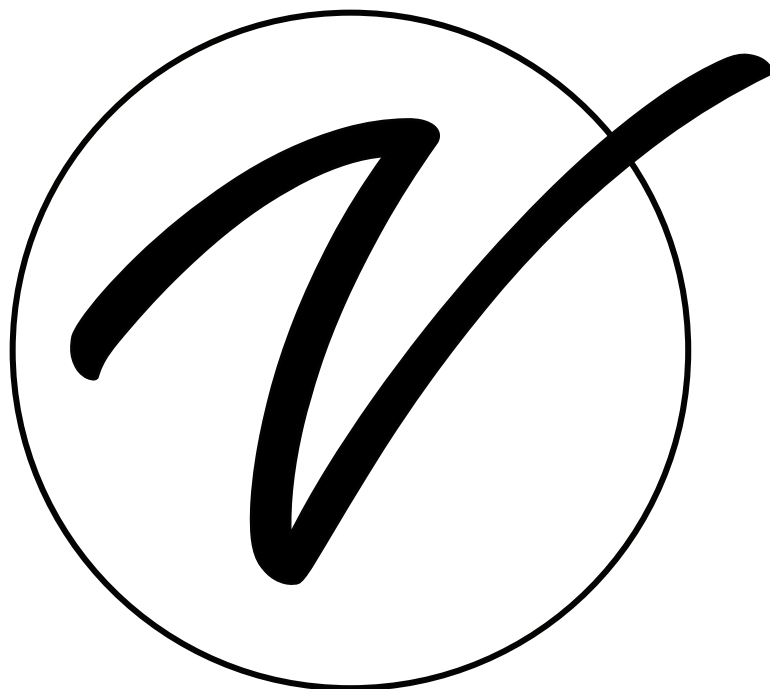

ADIME908-02

© 2025 Viva Comfort

vivacomfort.com