


.....

# MOBILE STAND

FOR WELCH ALLYN 4400 VITAL SIGNS MONITOR

.....

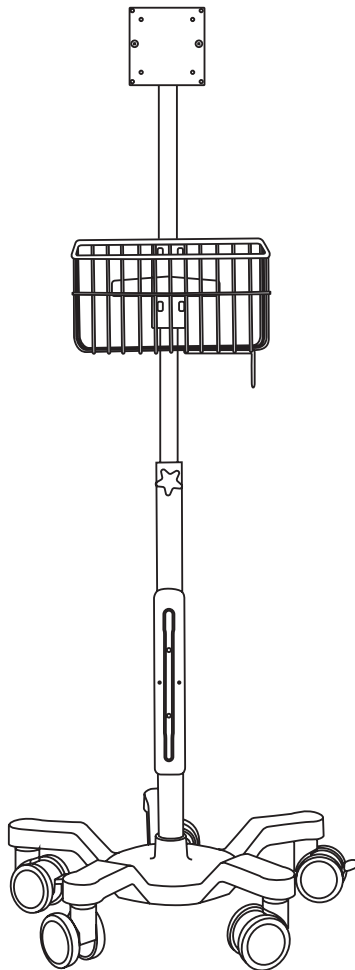

ADIME911-01

**USER GUIDE**



# Thank you for your purchase.



**Viva by AdirMed maintains  
a strong practice of ongoing  
development of products and  
commercial research.**




We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.

# Table of Contents

- 2** Parts List
- 2** Hardware List
- 3** Instructions
- 6** Installing Your Welch Allyn 4400 Vital Signs Monitor
- 7** Warranty

# Parts List

- 1. Pole
- 2. Concave Base
- 3. Pole Clamp
- 4. Basket
- 5. Handle
- 6. Mounting Plate
- 7. Stainless-Steel Pole

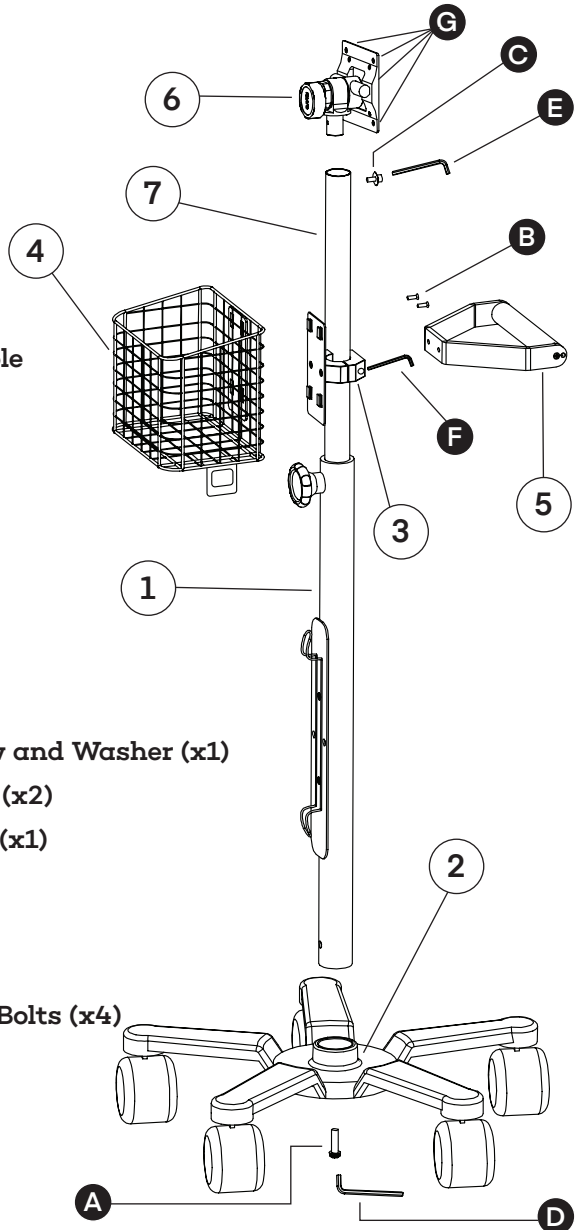


# Hardware List

- A. M8\*50 mm Screw and Washer (x1)
- B. M4\*10mm Screw (x2)
- C. M5\*12mm Screw (x1)
- D. M6 Allen Key
- E. M4 Allen Key
- F. M3 Allen Key
- G. M4\*20 Mounting Bolts (x4)

# Instructions


## Step 1:

- A.** Insert the Pole (1) into the Concave Base (2) with the screw pole end and affix it with a M8\*50mm Screw (A).
  - a.** Secure the screw using the provided M6 Allen Key (D).

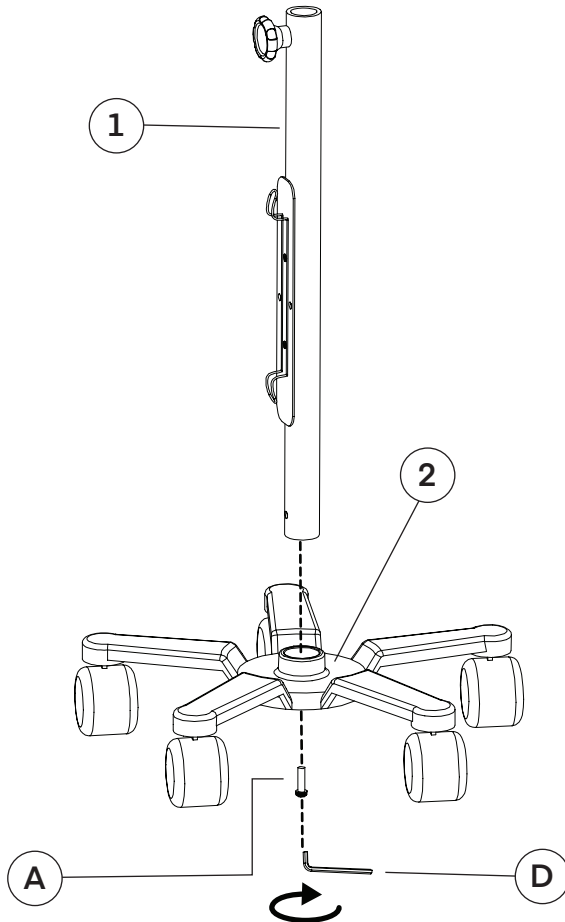


**Step 2:**

- A.** Put the Basket (4) on the plate affixed to the Pole Clamp (3) by sliding it above the clips and pushing down until secure.

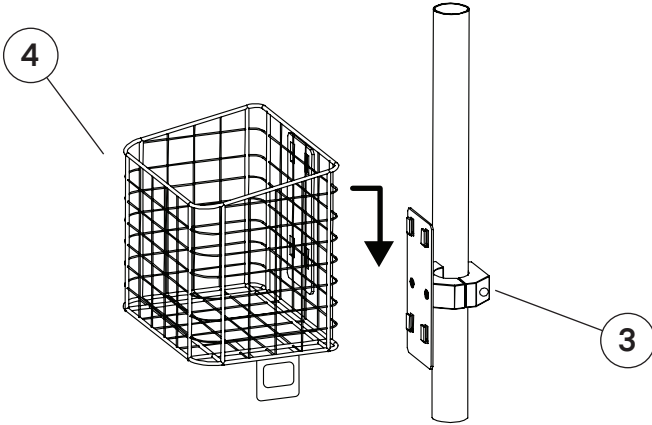


**Step 3:**

- A.** Install the Handle (5) on the Pole Clamp (3) with two M4\*10mm Screws (B), and secure them with the M3 Allen Key (F).
  - a.** The M4\*10mm Screws (B) are located on the Pole Clamp (3).

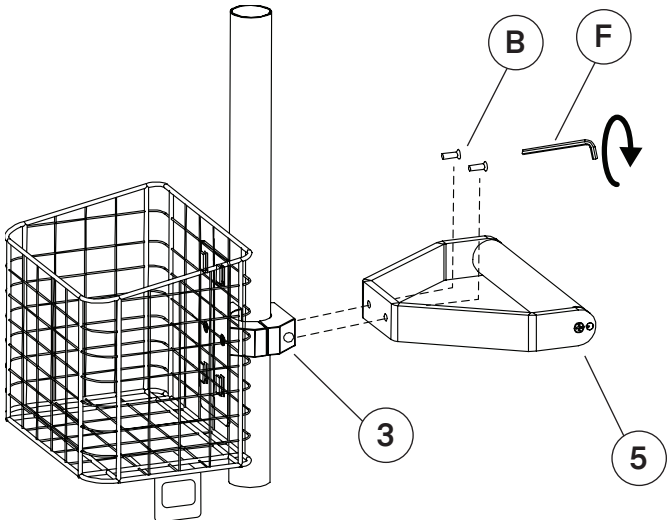

**Step 4:**


- A.** Insert the Mounting Plate (6) by setting it into the Stainless-Steel Pole (7) and affix it to the Pole (1) with a M5\*12mm Screw (C).
  - a.** You can locate this screw already on the mounting set.
  - b.** Secure it in place using the M4 Allen Key (E).
- B.** Your Mobile Stand for Welch Allyn 4400 Vital Signs Monitor is ready to be used.

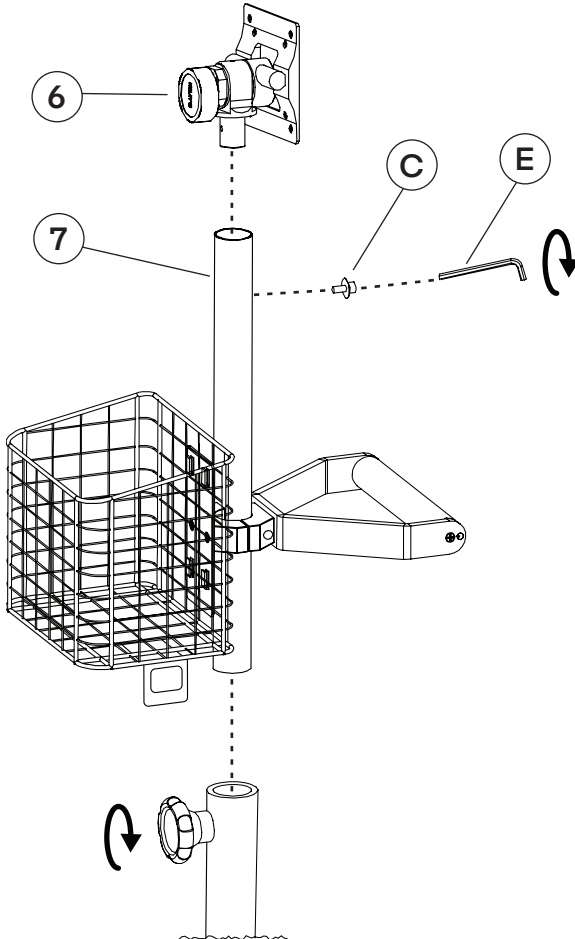

# Installing Your Welch Allyn 4400 Vital Signs Monitor

## Step 1:

- A.** Located on your Mounting Plate (6) are four included M4\*20 Mounting Bolts (G).
  - a.** Unscrew the M4\*20 Mounting Bolts (G) from the face of the plate.
- B.** Line the four holes on your monitor up with the four mounting holes on the Mounting Plate (6).
- C.** Secure the four M4\*20 Mounting Bolts (G) to the monitor going from the back into the monitor.

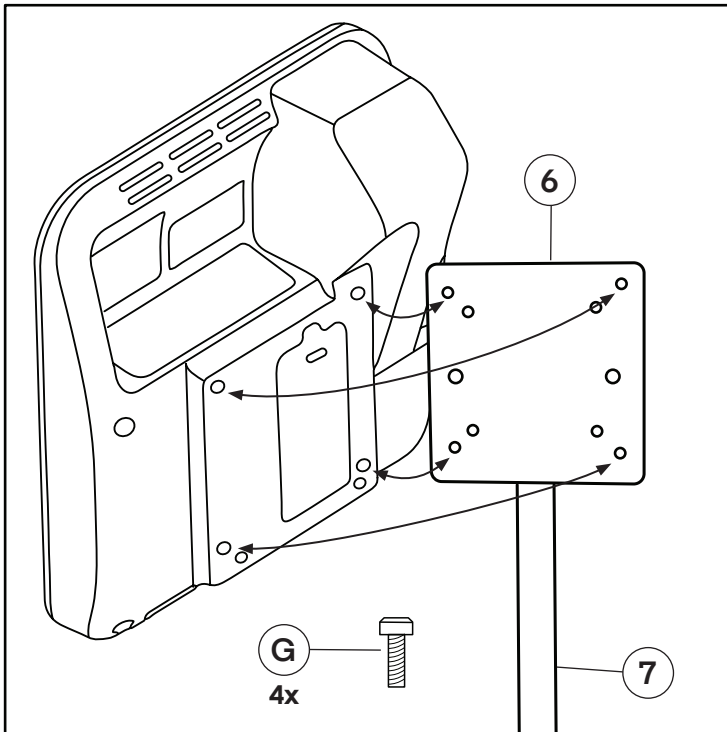



# Warranty

1. Adir Corp (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 6 months from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

*Viva*  
by **ADIR**Med

**ADIME911-01**

© 2023 AdirCorp  
**adircorp.com**