


.....

MEDICAL RECLINER CHAIR

.....

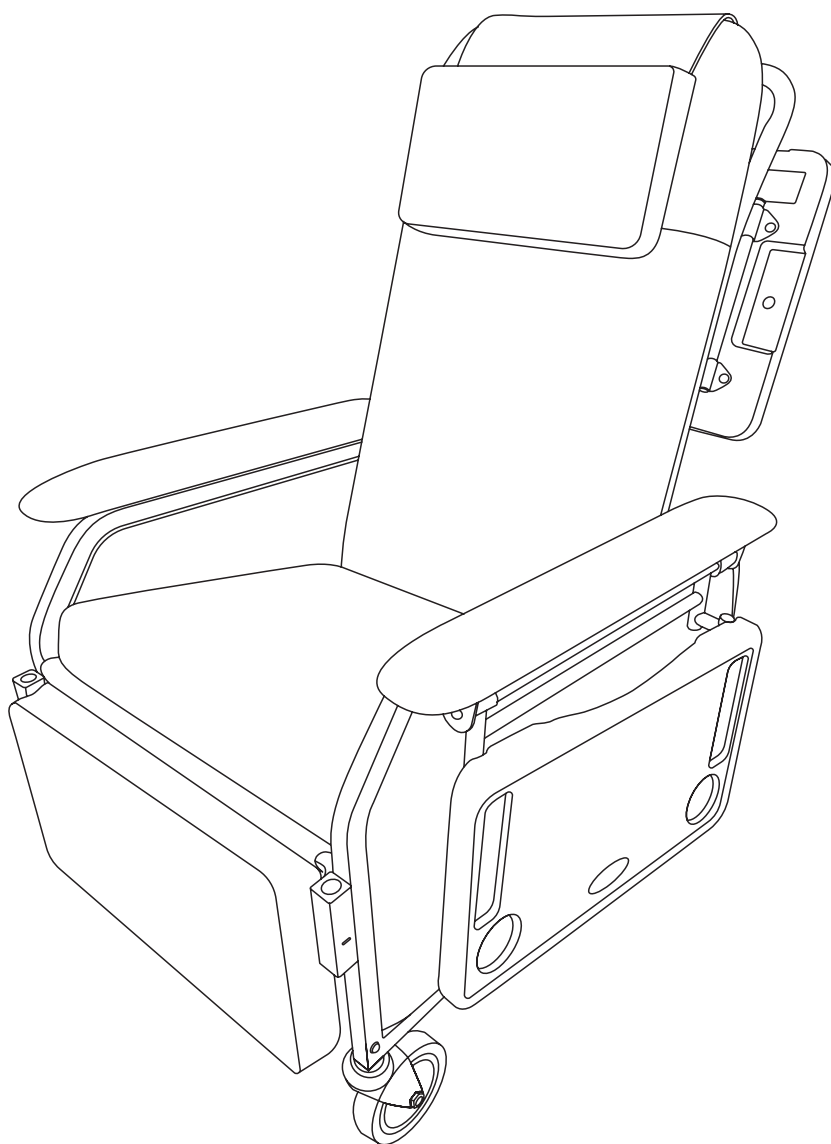

USER GUIDE

ADIME915-01-BLU



Thank you for your purchase.

.....

**Viva Comfort maintains a
strong practice of ongoing
development of products and
commercial research.**

.....

We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.

.....

Table of Contents

4	Parts List
5	Hardware List
5	Tools Needed For Assembly
6	Assembly Instructions
12	How To Adjust The Recliner Chair
13	How To Move The Recliner Chair
14	Care & Maintenance Guide
18	Warranty
19	Notes

Parts List

A




Directional Wheel (x2)

B




Swivel Wheel (x2)

C

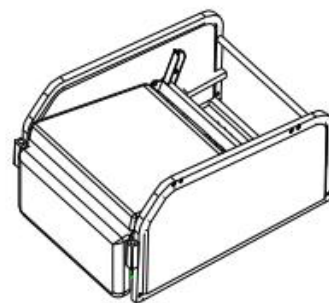

Seat Frame

D

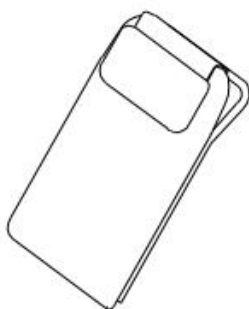


Backrest Cushion

E




Right Armrest

F

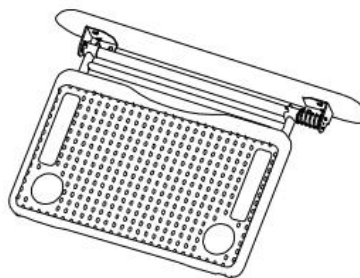






Left Armrest

G



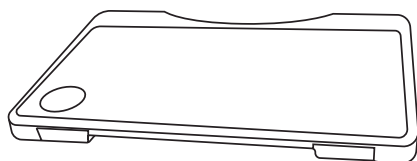


Front Tray

Hardware List
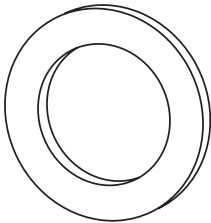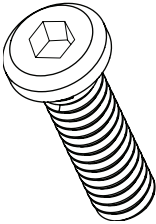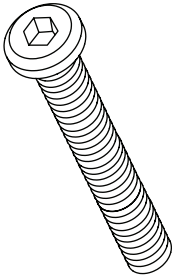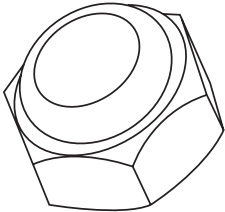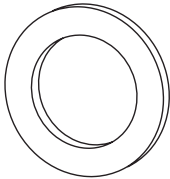
<div>1</div> <div></div> <div>Washer 1 (ø 36) (x4)</div>	<div>2</div> <div></div> <div>Hexagon Screw 1 (M8x18mm) (x8)</div>	<div>3</div> <div></div> <div>Hexagon Screw 2 (M6x33mm) (x4)</div>
<div>4</div> <div></div> <div>Nut (M6) (x4)</div>	<div>5</div> <div></div> <div>Washer 2 (ø 13) (x8)</div>	

Tools Needed For Assembly:

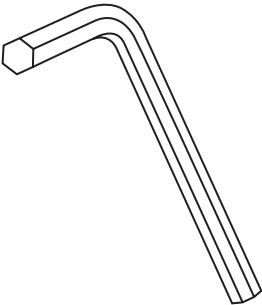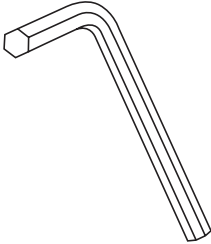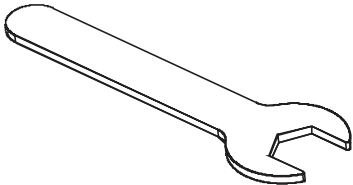
<div></div> <div>M5 Hexagonal Wrench</div>	<div></div> <div>M4 Hexagonal Wrench</div>	<div></div> <div>Wrench</div>
---	---	--

Assembly Instructions

Note: Before beginning, please note that both Directional Wheels (A) should be affixed to the front of the chair, and both Swivel Wheels (B) should be affixed to the back of the chair.


Step 1:

- A.** To install a wheel, locate either a Directional Wheel (A) or Swivel Wheel (B), place one Washer 1 (1) through the Wheel Rod (A) and insert the Wheel Rod (A) into the square hole at the bottom of the Seat Frame (C), as pictured.
- B.** Make one Hexagon Screw 2 (3) cross through one Washer 2 (5), and then insert the hole from the side of the frame.
- C.** Insert one Washer 2 (5) onto a Hexagon Screw 2 (3) and then secure a Nut (4) onto the screw.
- D.** Before proceeding with the remaining wheels, ensure both Directional Wheels (A) are in the front of the chair and both Swivel Wheels (B) are in the back.
- E.** Repeat the above steps for the remaining three wheels.
- F.** Secure all the screws and nuts with the included Wrench, as pictured.

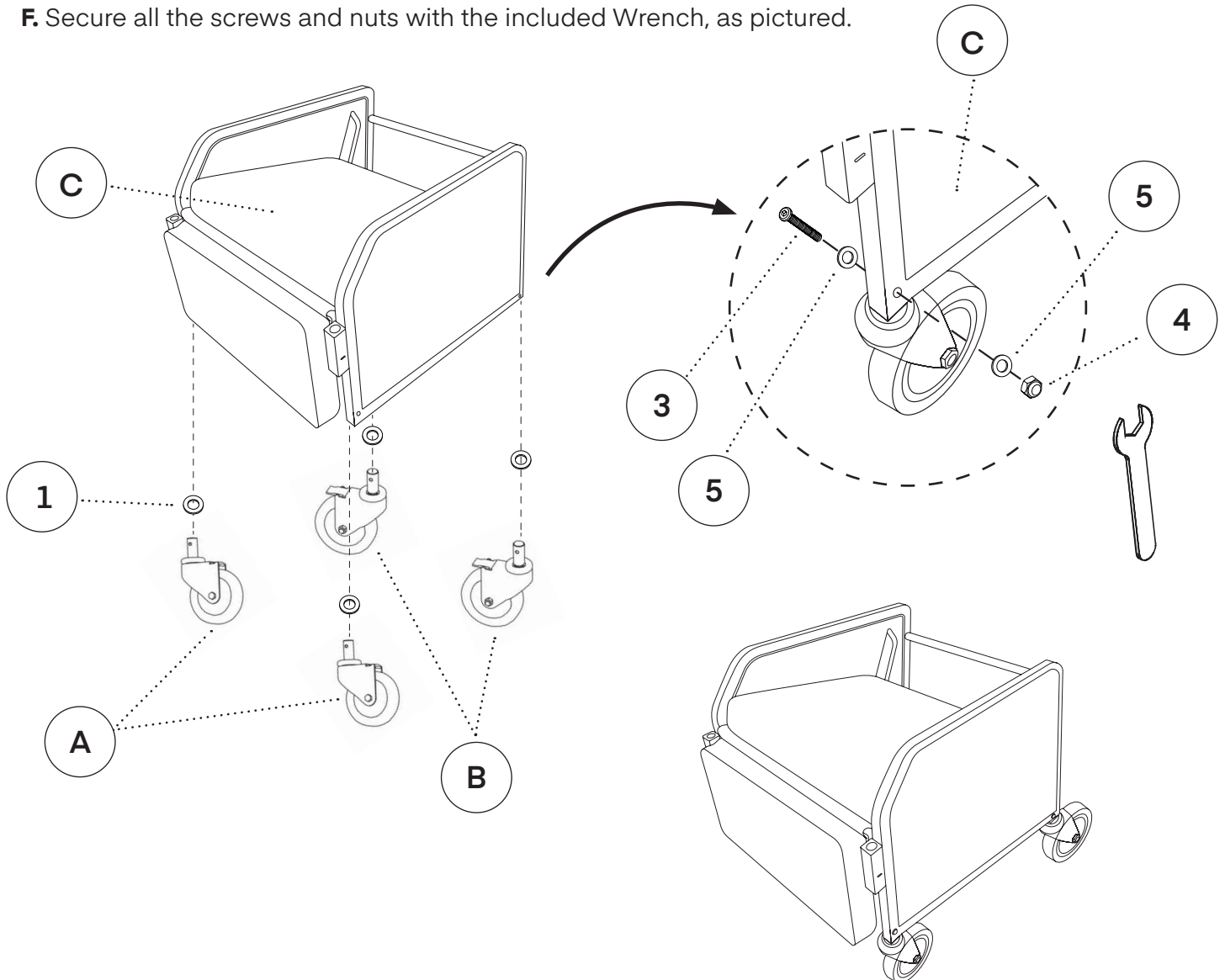


Step 2:

- A. Insert the end tube of Backrest Cushion (D) into the U-shaped groove located on the back of the Seat Cushion (C), as pictured.
- B. Insert two Hexagon Screw 2's (2) and then tighten it with the included Wrench.

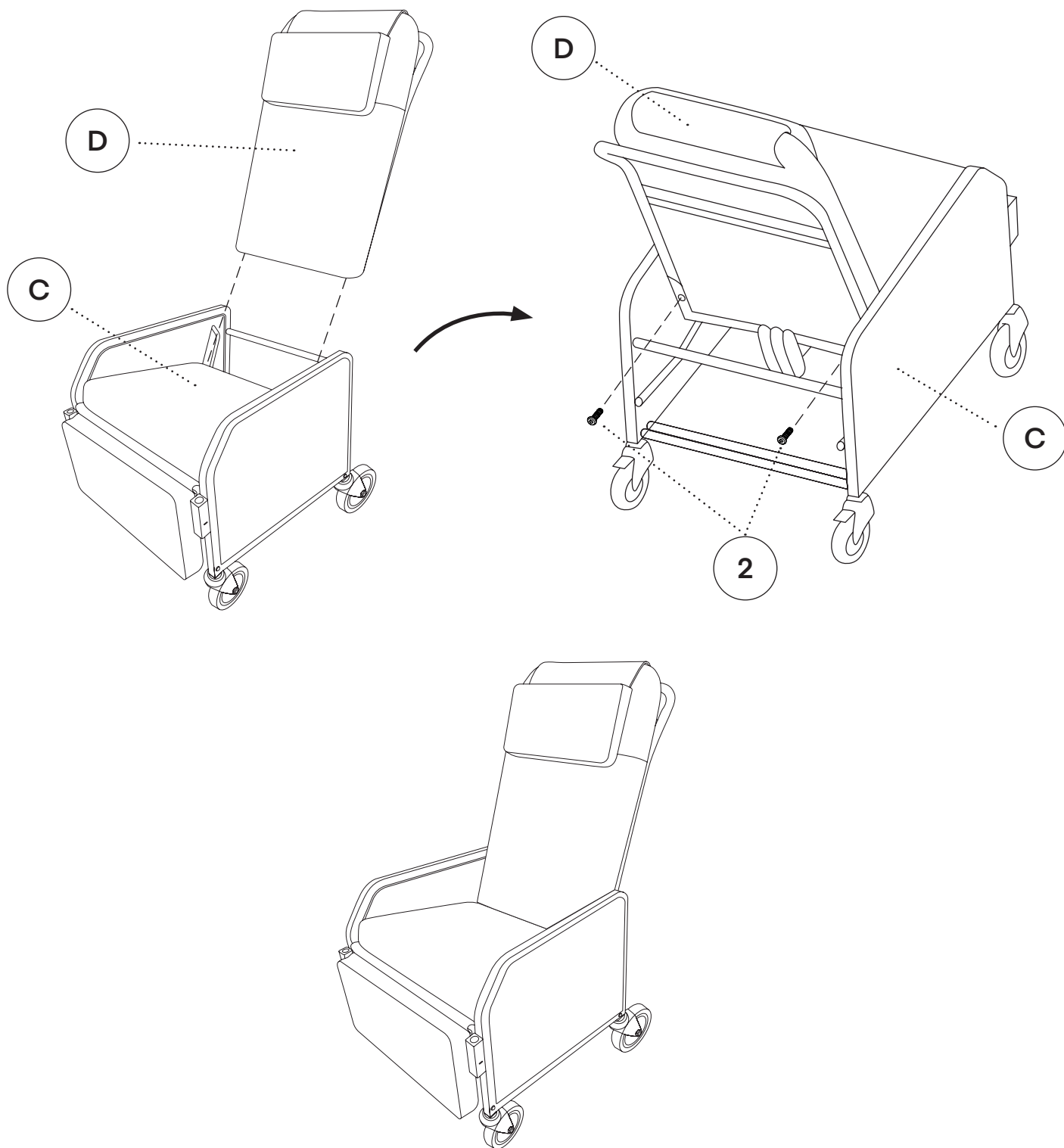


Step 3:

- A.** Lower the side tray down before installing the Left Armrest Frame (F).
- B.** Place the Left Armrest Frame (F) holes above the Seat Frame (C) holes, and insert four Hexagon Screw 1's (2) half into the holes.
- C.** Tighten the screws one by one with M5 Inner Hexagon Wrench, as pictured.

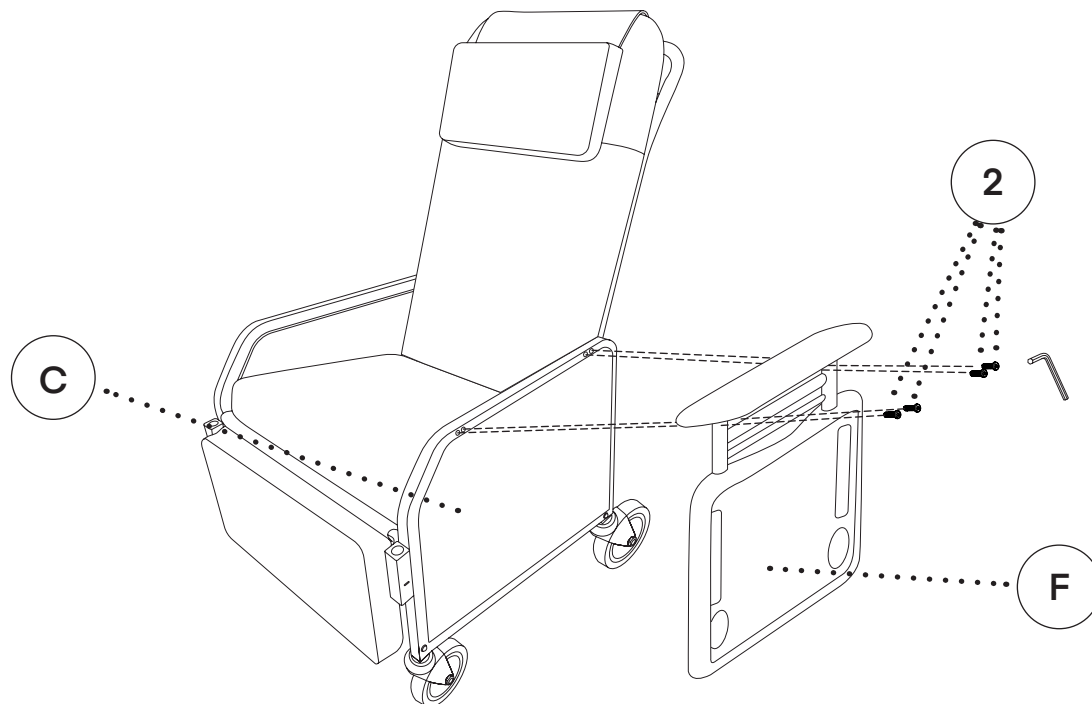


Side Tray Use:

- A.** When the Side Tray (F) is installed, you could raise the Side Tray (F) upright slightly.
- B.** The Side Tray (F) will be locked automatically when the tray is raised to the relative position.
- C.** When the Side Tray (F) is locked, you are now ready to use it.
- D.** When you finish using the Side Tray (F), push it backward and then lower it down.
- E.** Now the Side Tray (F) is tucked neatly under the left armrest.

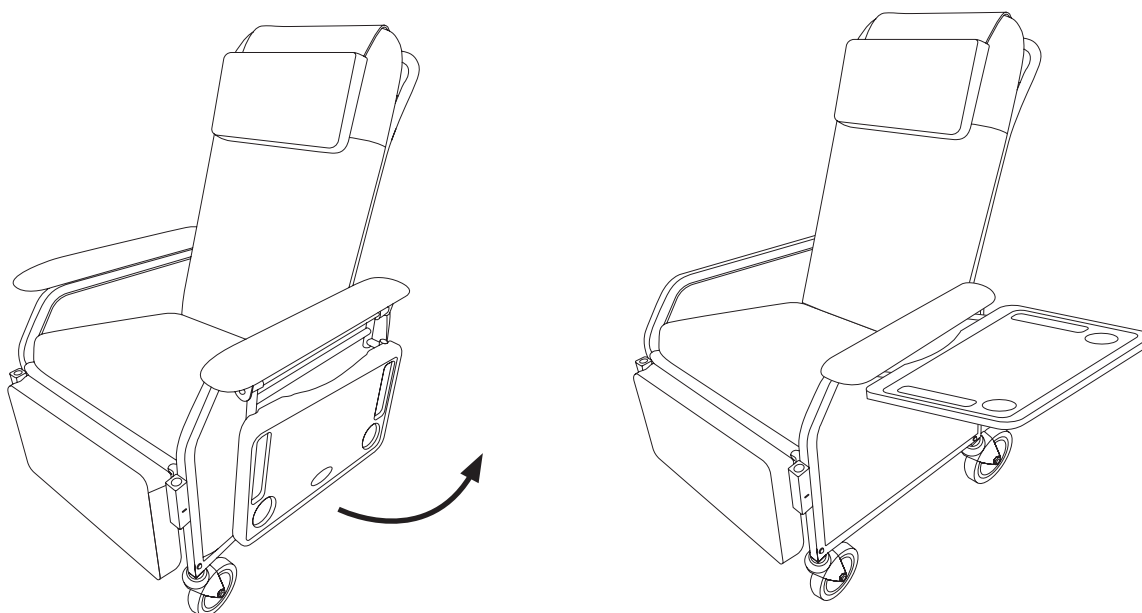

Step 4:


- A. Place the Right Armrest Frame (E) holes above the Seat Frame (C) holes, as pictured.
- B. Insert two Hexagon Screw 1's (2) halfway into the holes and then fix the screws one by one with the M5 Inner Hexagon Wrench.
- C. Make sure all screws are tightened and your chair is ready to use.

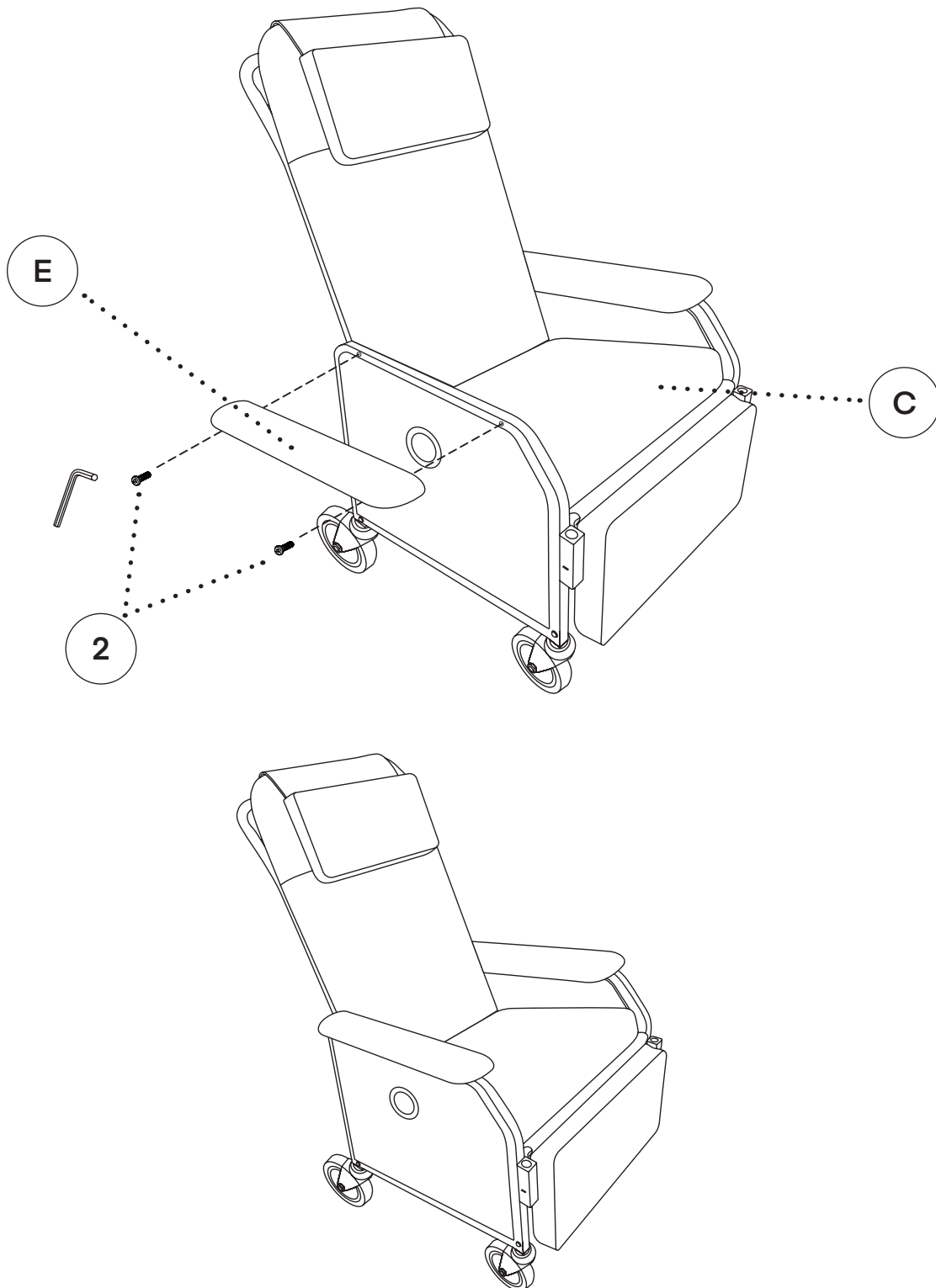

Step 5:


Front Tray Usage and Storage:

- A.** Take the Front Tray (G), lay it face down on a flat, clean surface.
- B.** Push the support bars, located under the transparent protective film, slightly outward before using it.
- C.** Open the support bars slowly, as pictured.

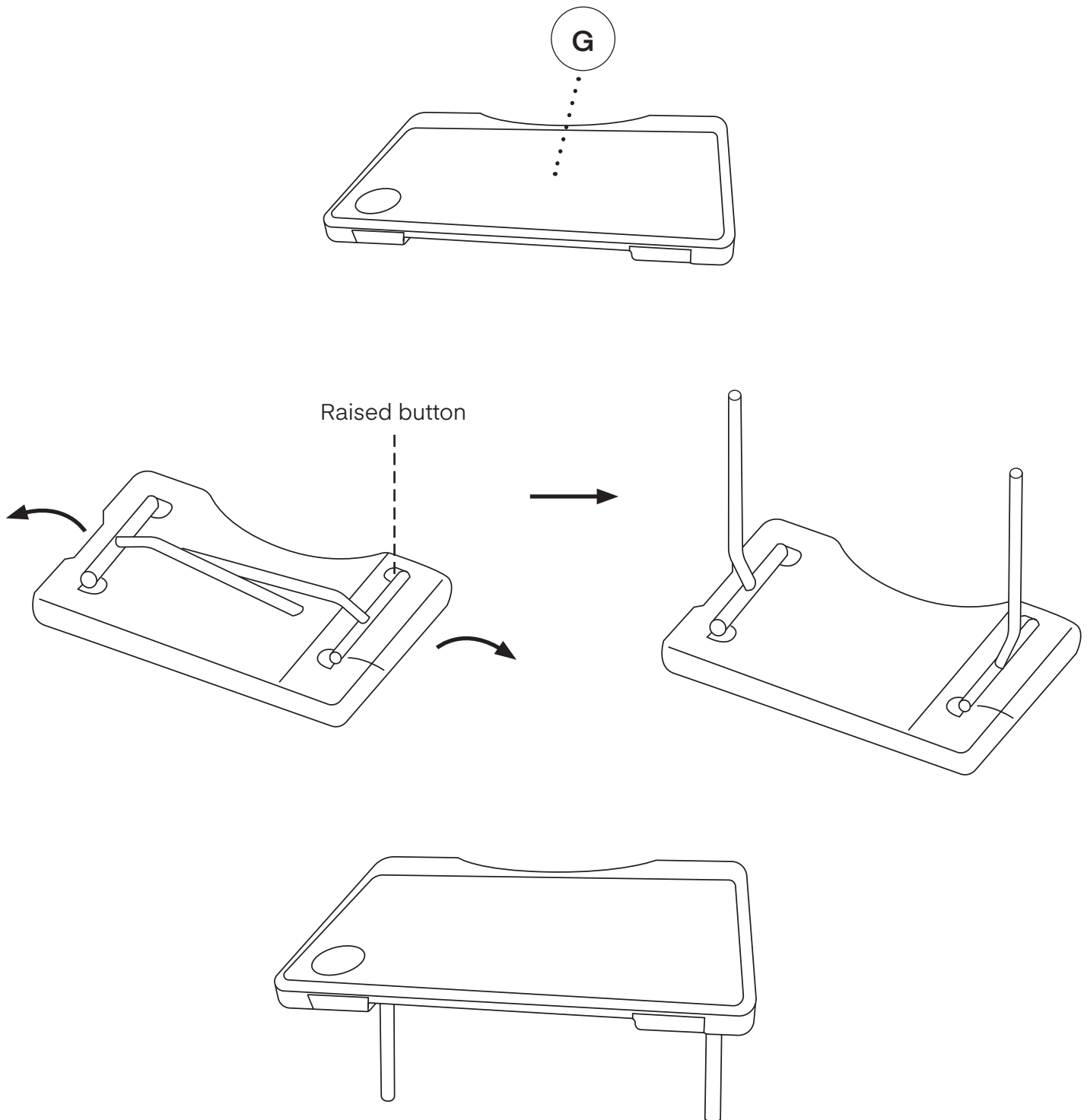


Step 6:

- A. Insert the support bars of the Front Tray (G) into the holes of the holder located at the front of the chair's of the chair frame, as pictured.
- B. Make sure the support bars reach at the bottom of the holders and it's ready to use.

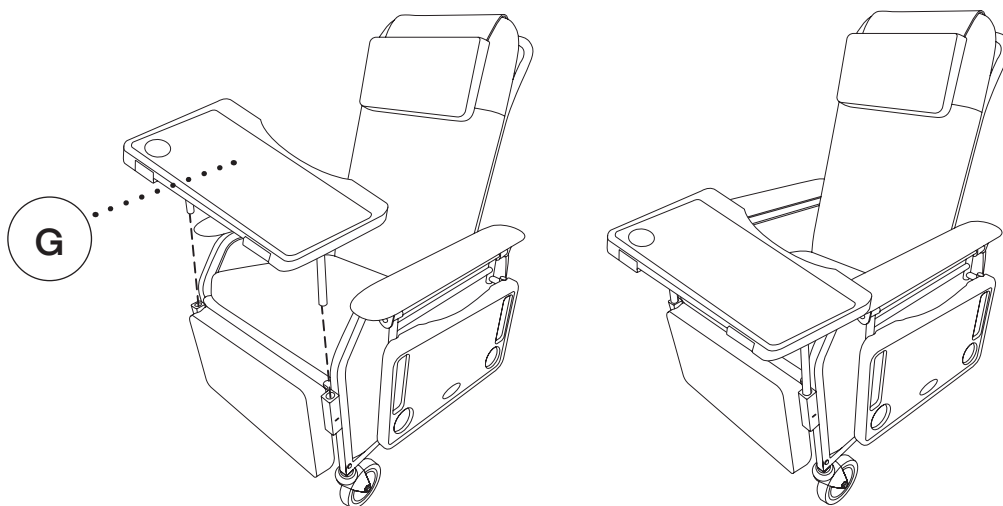


Step 7:

- A. When the Front Tray (G) is not in use, press the right button under the transparent protective film and then close the right support bar.
- B. Press the left button under the transparent protective film and then close the left support bar.
- C. Remove the tray, and hang it on the two small support holders located at the back of the backrest frame.
- D. Make sure the front tray back support bars hang firmly at the back of the Backrest Frame (D) for storage.

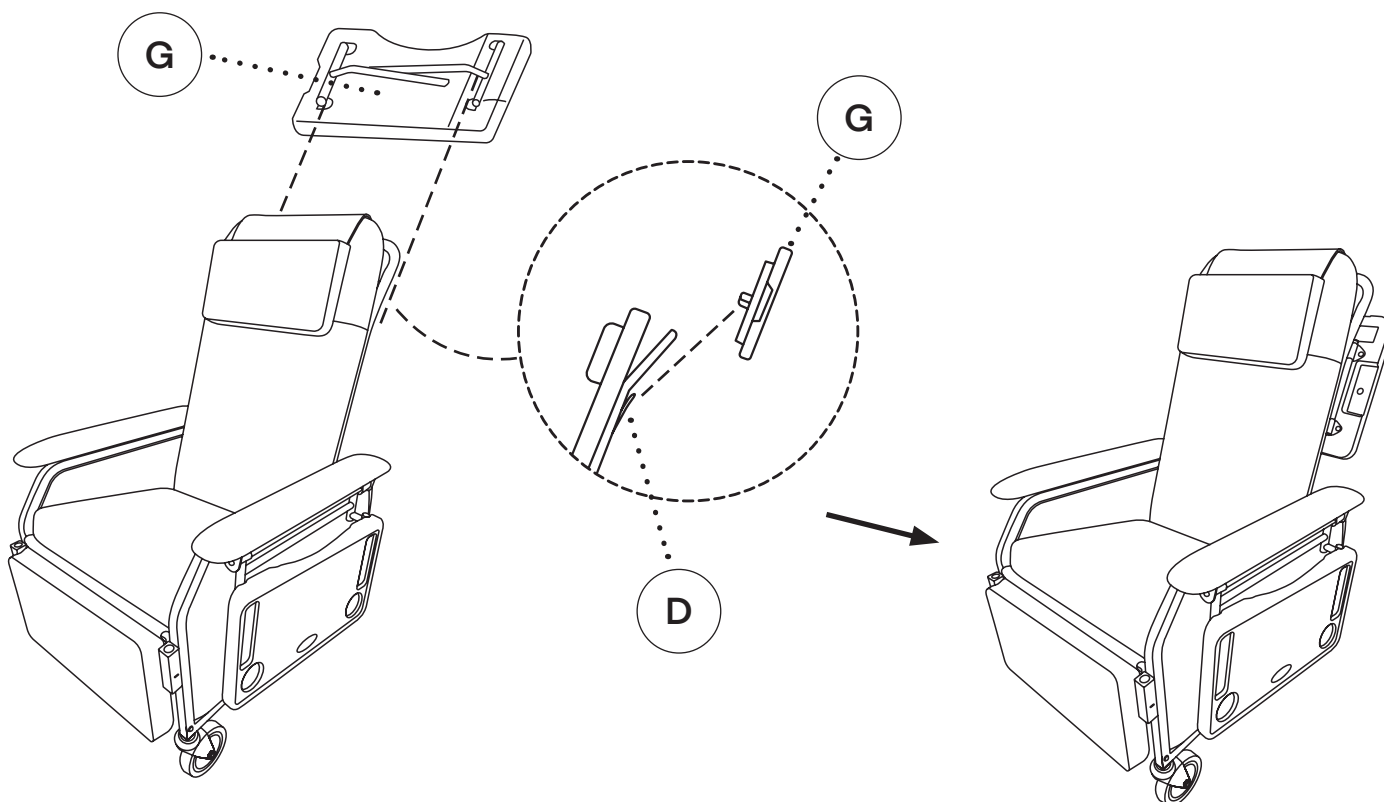



How To Adjust The Recliner Chair

- A.** To adjust the recliner chair to your desired position, press and hold the button located on the right side, while slowly leaning back on the backrest.
- B.** Release the button once you've found a comfortable position.
- C.** To return to the original sitting position, press the button again.
 - a.** The chair will automatically adjust when you press and hold the button.
- D.** Release the button once you've found a comfortable position.

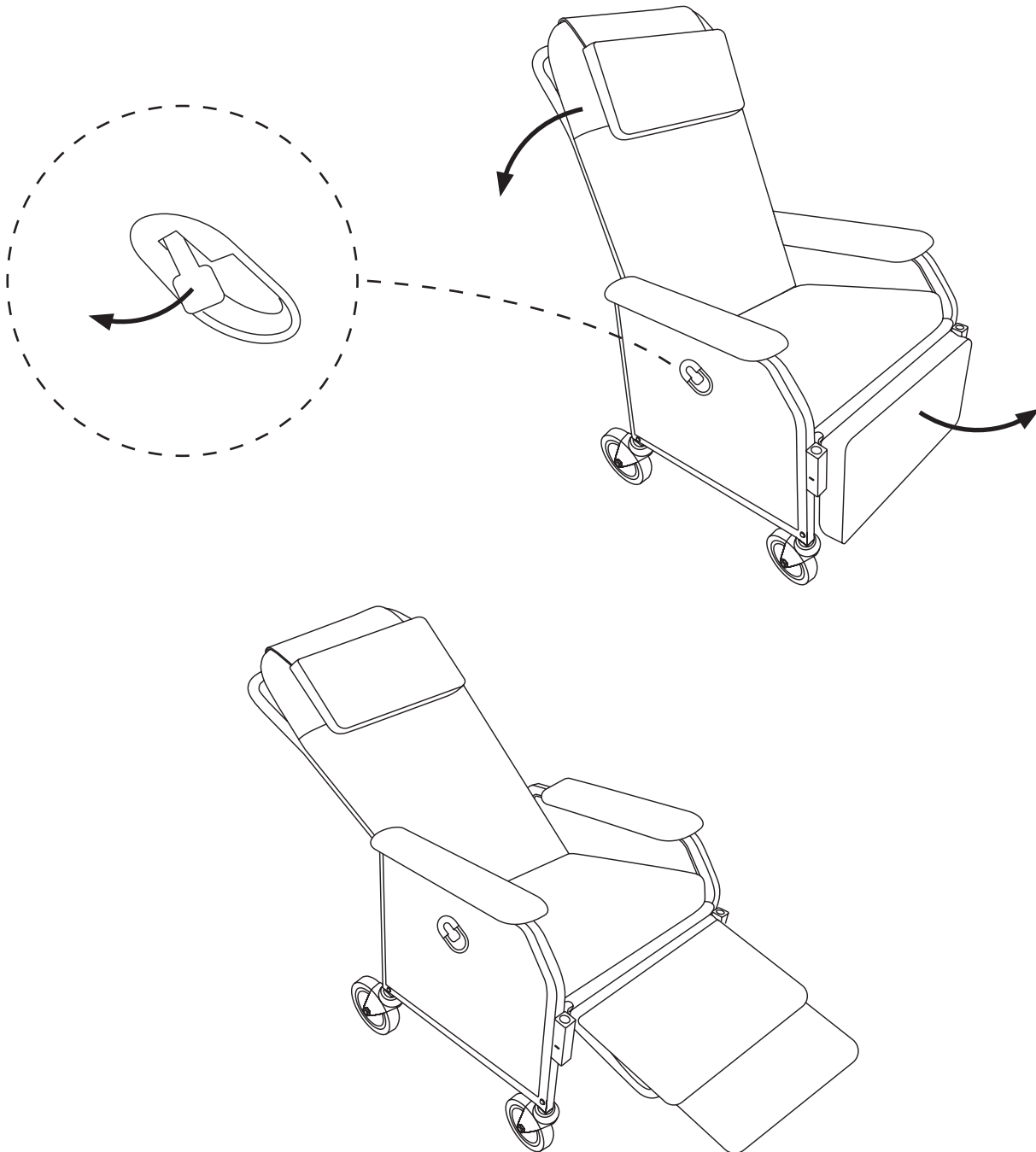


How To Move The Recliner Chair

- A.** Before moving the recliner chair, ensure the brakes of the rear swivel wheels are set to "ON."
- B.** To release the brakes, press the brake pedal to the "OFF" position with your foot, and you can then relocate the chair as needed.
- C.** Press the brakes back to the "ON" position to ensure they are engaged.
 - a.** The chair should be stable before sitting or lying in it.

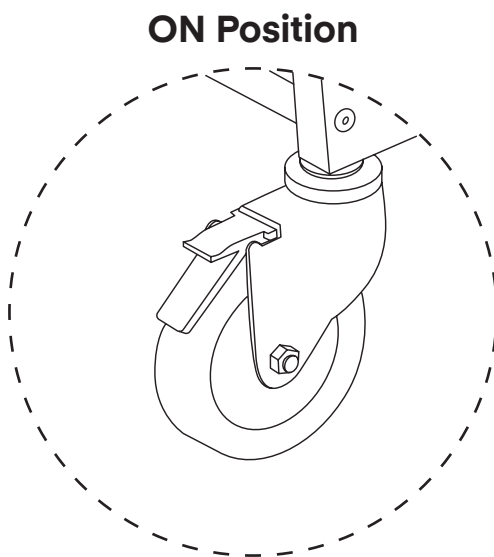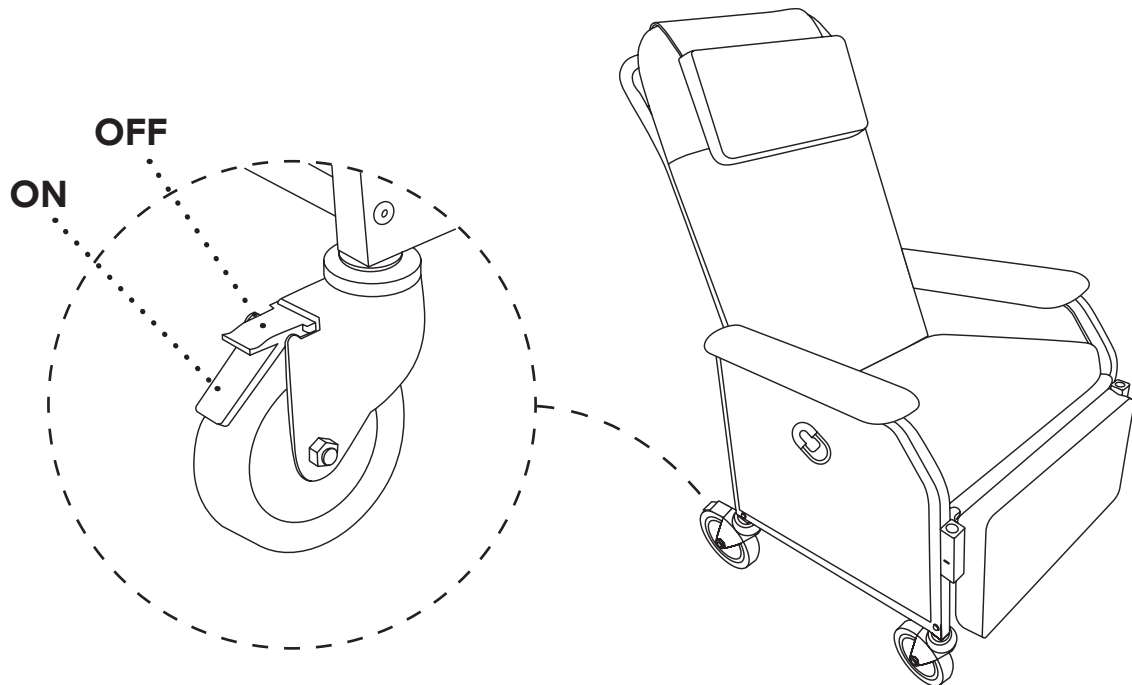

ON Position

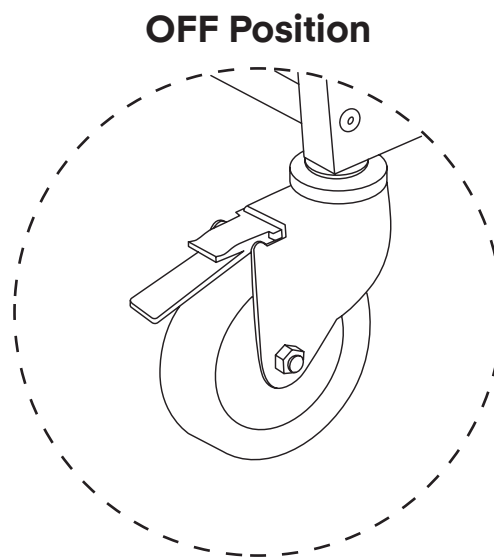

OFF Position

Care & Maintenance Guide

Disclaimer: Viva Comfort will not be held responsible for any damage to the upholstery or surfaces of our products due to cleaning attempts made, whether correctly or improperly conducted. We will not be liable for any damage to the upholstery or the product surfaces, such as discoloration or deterioration, from attempting to clean.

Thank you for purchasing a Viva Comfort product. We are honored to have your business and serve your medical equipment needs. To ensure your product(s) stay in optimal condition, even with consistent use, we have crafted a Care & Maintenance Guide to help you properly maintain and sanitize your product. Please carefully read these guidelines and follow the recommendations listed. Please contact our customer service representatives if you have any questions regarding these instructions.

Note: Be mindful when using sharp or pointed objects in the vicinity of upholstery and exam tables. They can cause damage to the exam table surfaces and puncture or tear the upholstery.

Note: DO NOT use pumice stones, scouring pads, steel wool, or textured cleaning pads to clean any surface of the exam table or the upholstery. We recommend a microfiber towel or a soft cloth.

Note: DO NOT store steel wool on/in your exam tables or on your upholstery.

Upholstery Cleaning:

Viva Comfort uses premium antimicrobial upholstery on all our exam tables. Consistent care should be taken to maintain the upholstery's integrity. We recommend that you always test the solutions below on an inconspicuous spot before using them on the entire upholstery. The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our upholstery.

The following Cleaners, Cleaning Materials, and Chemical Makeups will deteriorate and cause permanent damage to your upholstery.

Please Note: Should anything spill on your upholstery, wipe it up immediately and rinse it several times with water to avoid damage.




- Avoid using acidic or abrasive cleaners, as they can cause damage to your upholstery.
- Avoid using dyes (hair, textile, food) near your exam tables as they will permanently stain your upholstery. Should a dye spill occur, wipe it up as soon as possible with dishwashing detergent or an all-purpose cleaner. This is not guaranteed to solve the problem, it but can help reduce the chance of staining.
- Avoid using drainer cleaners or rust removers near your upholstery. Should drain cleaner spill, wipe it up as soon as possible, and rinse it several times with water. Should rust remover spill, wipe off all residue immediately, wash the area with dishwashing soap, and rinse it several times. Ensure you thoroughly dry the area.

- Avoid using the following products near or on your exam tables, as they will cause damage to the surface that may result in discoloration and/or deterioration.

CHEMICAL COMPOUND	AFFILIATED NAMES
Silicon Dioxide	Pumice/Pumice Stones, Degreaser
Sodium Hydroxide	Caustic Soda, Caustic, Lye, Soda Lye, Degreaser, Drainer Cleaner
Hydrofluoric Acid	Rust Remover, Toilet Bowl Cleaner, Concrete Cleaner, Metal Polishes
Phosphoric Acid	Tile Cleaner, Toilet Bowl Cleaner, Rust Remover, Concrete Cleaner
Sulfuric Acid	Oleic Acid, Oil of Vitriol, Oleum, Drain Cleaner, Heavy-Duty Acidic Cleaner
Hydrochloric Acid	Muriatic Acid, Hydrogen Chloride, Toilet Bowl Cleaner, Calcium, Lime and Rust Remover

Below are the recommended cleaning instructions for our antimicrobial upholstery.

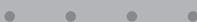Please Note: DO NOT use any alcohol-based cleaning agents, as this will degrade the upholstery.

MILD SOIL LEVEL 	<p>If the surface is lightly soiled, we recommend using a solution comprised of mild dish soap (10%) mixed in warm water (90%). Saturate a soft towel and clean the upholstery gently. In a separate bucket, have clean water and a soft towel. Thoroughly rinse the upholstery with clean water, as well as dry it completely.</p>
MODERATE SOIL LEVEL 	<p>If the surface is moderately soiled, we recommend using a solution comprised of Fantastik® (50%) and warm water (50%). Saturate a soft towel and clean the upholstery gently. In a separate bucket, have clean water and a soft towel. Thoroughly rinse the upholstery with clean water, as well as dry it thoroughly.</p>
SEVERE SOIL LEVEL 	<p>If the surface is severely soiled, we recommend using a solution comprising of bleach (10%) and warm water (90%). Saturate a soft, white towel and clean the upholstery gently. In a separate bucket, have clean water and a soft, white towel. Thoroughly rinse the upholstery with clean water, as well as dry it completely. Since this solution utilizes bleach, please only use it when necessary; ensure all the bleach solution is removed from the bed, do not let the solution sit on the upholstery, and understand that discoloration may occur.</p>

-
- Fantastik® is a registered trademark of DowBrands, Inc.®, Registered trademark of the Dow Chemical Co.®

Exam Bed Surface Cleaning:

The following Cleaners, Cleaning Materials, and Chemical Makeups will deteriorate and cause permanent damage to your exam beds' surfaces.

Please Note: Should anything spill on your exam table's surface, wipe it up immediately and rinse it several times with water to avoid damage.



- Avoid using acidic or abrasive cleaners, as they can cause damage to your exam table and its upholstery.
- Avoid using dyes (hair, textile, food) near your exam tables as they will permanently stain your table. Should a dye spill occur, wipe it up as soon as possible with dishwashing detergent or an all-purpose cleaner. This is not guaranteed to solve the problem, it but can help reduce the chance of staining.
- Avoid using drainer cleaners or rust removers near your exam table. Should drain cleaner spill, wipe it up as soon as possible, and rinse it several times with water. Should rust remover spill, wipe off all residue immediately, wash the area with dishwashing soap, and rinse it several times. Ensure you thoroughly dry the area.
- Avoid using the following products near or on your exam tables, as they will cause damage to the surface that may result in discoloration and/or deterioration.

CHEMICAL COMPOUND	AFFILIATED NAMES
Silicon Dioxide	Pumice/Pumice Stones, Degreaser
Sodium Hydroxide	Caustic Soda, Caustic, Lye, Soda Lye, Degreaser, Drainer Cleaner
Hydrofluoric Acid	Rust Remover, Toilet Bowl Cleaner, Concrete Cleaner, Metal Polishes
Phosphoric Acid	Tile Cleaner, Toilet Bowl Cleaner, Rust Remover, Concrete Cleaner
Sulfuric Acid	Oleic Acid, Oil of Vitriol, Oleum, Drain Cleaner, Heavy-Duty Acidic Cleaner
Hydrochloric Acid	Muriatic Acid, Hydrogen Chloride, Toilet Bowl Cleaner, Calcium, Lime and Rust Remover

Compatible Household Cleaners For Exam Bed Surfaces

Below is a list of compatible cleaners to use on the exam bed surfaces. We recommend that you always test the solutions below on an inconspicuous spot before using them on the entire surface. The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our exam bed surfaces. We will not be liable for any damage to your product surfaces, such as discoloration or deterioration, from attempting to clean.

(These cleaner recommendations are only for exam bed surfaces; DO NOT use the below cleaners on the upholstery. For instructions on how to clean your upholstery, refer to our "Upholstery Cleaning" section.)



Dawn®

Fantastik®

Windex®

Clorox®

Lysol®

Formula 409®

Glass Plus®


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

Lestoil®

Laminate Surface Cleaning:

Note: Though the tips below are recommended for cleaning soiled and stained areas, we will not be liable for any damage to the product surfaces, such as discoloration or deterioration, from attempting to clean.

Note: The below solutions still require you, the user, to follow proper and safe handling of the products. Carefully review all warnings on the labels of any cleaning agents you utilize and ensure it's compatible with our product.

Note: Be sure to utilize proper protective equipment before handling the below cleaning solutions.

MILD SOIL LEVEL 	If the surface is lightly soiled, we recommend using a soft cloth or microfiber towel dampened with a mild soap and water mixture to clean the area.
MODERATE SOIL LEVEL 	If the surface is moderately soiled, we recommend using a solution comprised of a mild household cleaner and baking soda. Create a paste consistency with the recommended cleaning agents, take a nylon bristle brush, and lightly scrub the affected area. DO NOT use a metal bristle brush nor aggressively scrub, or you will cause permanent damage to the surface. Thoroughly rinse the area with clean water, as well as dry it thoroughly.
SEVERE SOIL LEVEL 	If the surface is severely soiled, we recommend using undiluted bleach. Use a cotton pad/ball with bleach on it to gently rub the affected area for a minute or so. Thoroughly rinse the area with clean water, as well as dry it completely. Since this solution utilizes bleach, please only use it when necessary; ensure all the bleach solution is removed from the area, do not let the solution sit on the surface, and understand that discoloration may occur.


- Fantastik® is a registered trademark of DowBrands, Inc.®, Registered trademark of the Dow Chemical Co.®
- Dawn® is a registered trademark of The Procter & Gamble Company®.
- Windex® is a registered trademark of S.C. Johnson & Son, Inc.®.
- Clorox® is a registered trademark of The Clorox Company®.
- Lysol® is a registered trademark of Reckitt Benckiser LLC®.
- Formula 409® is a registered trademark of The Clorox Company®.
- Glass Plus® is a registered trademark of Reckitt Benckiser LLC®.
- Lestoil® is a registered trademark of The Clorox Company®.

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 3 years from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

Notes

[illegible]

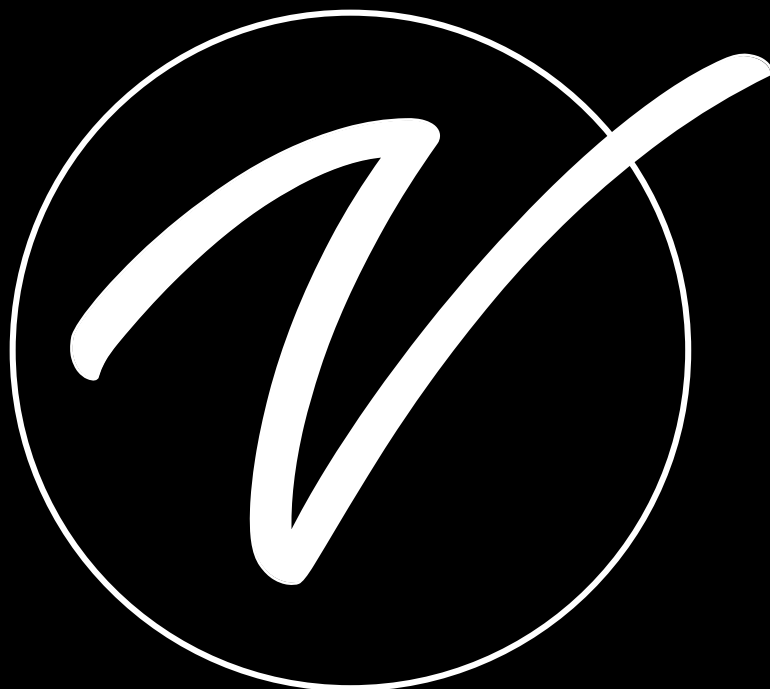

ADIME915-01-BLU

© 2024 Viva Comfort

vivacomfort.com