



Now 510k
Cleared

Invest in simplicity

For fast, clear and reliable COVID-19 results, when you need them the most.

Discover the portable, easy-to-use BD Veritor™ Plus System. It provides reliable results in just 15 minutes.¹

When your patients need reliable, rapid COVID-19 testing, turn to the BD Veritor™ Plus System. It offers dependable results at the point of care in a simple-to-operate portable instrument.



Learn more at bdveritor.com



Simplifying COVID-19 testing



Simplifies the testing process

- May reduce manual test processing errors with easy operations and single-button functionality
- Provides intuitive sample processing with prefilled, unitized reagent tubes



Achieves reliable, rapid results

- Displays easy-to-read digital results for COVID-19 in 15 minutes:¹
 - “CoV2: +” for positive; “CoV2: -” for negative
- Provides clear, objective results by correcting for nonspecific binding and detecting positives not recognized by the unaided eye¹



Delivers workflow efficiency

- Adapts easily to your workflow by offering 2 operational modes
 - **Walk Away:** Selected by double-clicking the power button, insert the test device into the Analyzer immediately after the sample has been added. The Analyzer will incubate for the appropriate time for each assay then perform the analysis and display the test result allowing staff to perform other duties while the sample incubates.
 - **Analyze Now:** The test device is inserted into the Analyzer after the recommended incubation time for each assay is complete, allowing for specimens to be processed in batches.* The Analyzer then performs the analysis and displays the test result.



Provides result traceability

- Captures or downloads the lot number, patient/specimen ID, operator ID, and test records using the BD Veritor™ InfoWifi module (optional add-on feature)

Clinical performance vs RT-PCR¹

	PPA [†]	NPA [‡]	Results in
COVID-19 (SARS-CoV-2)	84.0% (95% CI, 77.2%-89.1%)	99.7% (95% CI, 99.0%-99.9%)	15
[†] positive percent agreement [‡] negative percent agreement			COVID-19 results in 15 minutes

Ordering information	Cat no.	Qty
BD Veritor™ System for SARS-CoV-2 Now 510k Cleared	256092	30 tests
BD Veritor™ Plus Analyzer	256066	1 unit



Scan to **access training modules** on the BD Veritor™ website

Contact your authorized distributor partner or BD to learn more.

For expedited service, say **Veritor** at the prompt. **1.844.823.5433**

Reference: 1. BD Veritor™ System for SARS-CoV-2. Package insert. 500075511 (01) Becton, Dickinson and Company.

^{*}Any batch testing performed should be conducted at the discretion of the user, in accordance with local policies and clinical judgment.

