

Simplifying Needleless IV Access with a Truly Universal Device

The BD Twinpak is a truly universal IV access device, offering twice the performance of traditional methods. It features two cannulas in one shield: one red hub cannula for syringe filling and one clear blunt plastic cannula for safe split septum access. The Twinpak helps to streamline IV work flow and reduces the number of individual pieces, thereby saving time and reducing costs.



Twice the Performance, Twice the Convenience


BD Twinpak

Dual Cannula Device

The Self-Contained Solution

Working Together for Simplified, Convenient IV Access



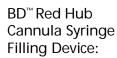

Provides Access to Vials and Ampules



BD Twinpak Shield: Holds Both Cannulas Securely



BD™ Blunt Plastic Cannula:

Allows for Safe, Needleless Access to Split Septum Injection Sites



Green Cap: Protects / Covers BD Blunt Plastic Cannula Prior to Use

Red Hub Cannula

(Syringe Filling Device):

 20G Metal Cannula provides versatility in syringe filling from singleand multidose vials, ampules, and for other

multiple access needs (e.g., reconstitution)

- Blunt Tip Design provides a safer alternative to a traditional, sharp needle, for syringe filling
- Red Hub easily identifies the syringe filling device

Blunt Plastic Cannula

(with Green Cap):

- Dual Side Ports provide greater fluid dispersion in flushing and drug delivery
- **Center-Point Design** features tapered tip for less resistance during insertion
- **Universal Compatibility** used with Interlink®, Abbott LifeShield®, and McGaw SafeLine™ injection sites as well as vials designed for needleless access
- **Green Cap** provides easy identification and a protective cover for clear BD Blunt Plastic Cannula and can later be used to cover the Red Hub Cannula



Secure Red Hub Cannula to the syringe. Remove shield.



After filling, passively re-shield the Red Hub Cannula.



Flip the BD Twinpak and attach the clear BD Blunt Plastic Cannula for split septum IV access.

BD Medical Systems 1 Becton Drive Franklin Lakes, NJ 07417

BD Medical Systems 2464 South Sheridan Way Mississauga, Ontario L5J 2M8 Canada www.bd.com

Catalog No.	Description	Case Quantity
303390	BD Twinpak	100 Per Box 1000 Per Case
303391 303392 303393	3ml Syringe with BD Twinpak 5ml Syringe with BD Twinpak 10ml Syringe with BD Twinpak	100 Per Box 800 Per Case 100 Per Box 400 Per Case 100 Per Box 400 Per Case

Interlink is a registered trademark of Baxter International, Inc. LifeShield is a registered trademark of Abbott Laboratories. SafeLine is a registered trademark of B.Braun Inc

BD, BD Logo and Twinpak are trademarks of Becton,

Printed in USA BD 8/00-0320