



Helping all people
live healthy lives

BD Integra™ Syringe **with *Retracting*** **BD PrecisionGlide™ Needle**

How to Use the BD Integra™ Syringe

1. Attachment

Attach the BD Integra™ needle onto the BD Integra™ syringe with a push and a clockwise twist. If pre-attached, tighten the needle to the syringe prior to use.



2. Aspiration

Pull the shield directly off of the needle. Draw up medication in accordance with established protocol.



3. Injection

Administer medication in accordance with established protocol, making sure that all medication has been dispensed.



4. Activation

Push plunger rod to activate the safety mechanism. Depress until you sense two "clicks".

Activation can occur before or after the needle is withdrawn from the patient.



5. Needle Retraction

The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete.

Discard immediately in the nearest sharps container.



Don't Forget...

Aseptic technique and proper skin preparation of the site are essential. Follow your institution's policy for safe disposal of all medical waste.

The BD Integra™ Syringe combines innovative technology with the BD quality and comfort you trust.

Prior to Activation

Single-Handed Activation



Tru-Lok™ Technology

Ensures low waste space and a reliable connection

Wide Flanges

For comfort and stability

Clear Barrel

For dosing accuracy

Detachable Needle

BD PrecisionGlide™ Needle Technology

Designed for patient comfort

After Activation




Needle Retracts into the Syringe!


Eliminates exposure to the needle after activation

Tru-Lok™ Technology helps to lower medication waste

The BD Integra™ Syringe, with Tru-Lok™ technology, is designed to have significantly lower waste space volume as compared to a traditional BD luer connection.* Less waste space can reduce drug wastage.

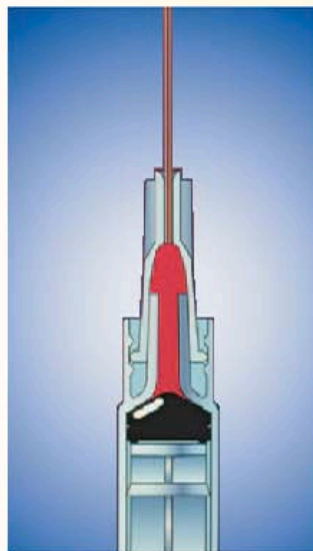

BD™ Conventional
3ml Syringe

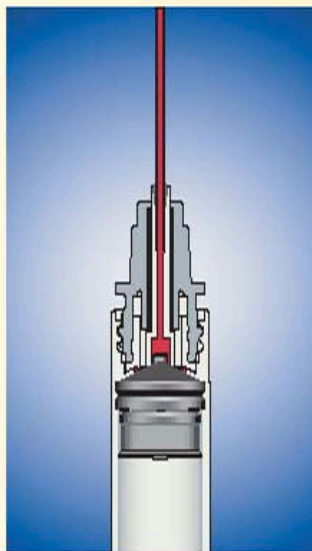

BD Integra™
3ml Syringe

The BD Integra™ Syringe Advantage



BD PrecisionGlide™ Needle Technology

- Designed for patient comfort
- Leading brand of hypodermic needle technology in the U.S.

Tru-Lok™ Technology

- Provides low waste space
- Creates a secure connection

Detachable Needle

- Allows clinicians to change needles for aspiration and administration

Safe, One-Handed Activation

- Retracting needle disappears into the syringe barrel
- Eliminates exposure to used needle
- Requires virtually no change in technique

Activation Choice

- Activates before or after needle is withdrawn from the patient
- Complete dosing is ensured prior to activation

* Data on file at BD

The BD Integra™ Syringe is available in an array of sizes for all SubQ and IM injections.

BD Integra™ 3mL Syringe with Detachable Needle			
Catalog #	Description	Gauge/Needle	Packaging
305269	3mL	25G x 5/8"	100/Box – 400/Case
305270	3mL	25G x 1"	100/Box – 400/Case
305271	3mL	23G x 1"	100/Box – 400/Case
305272	3mL	22G x 1½"	100/Box – 400/Case
305273	3mL	21G x 1"	100/Box – 400/Case
305274	3mL	21G x 1½"	100/Box – 400/Case
BD Integra™ Needles			
Catalog #	Description	Gauge/Needle	Packaging
305310	Retracting Ndl.	25G x 5/8"	100/Box – 1000/Case
305311	Retracting Ndl.	25G x 1"	100/Box – 1000/Case
305312	Retracting Ndl.	23G x 1"	100/Box – 1000/Case
305313	Retracting Ndl.	22G x 1½"	100/Box – 1000/Case
305275*	Blunt Fill Ndl.	18G x 1½"	100/Box – 1000/Case
305833*	Blunt Filter Ndl.	18G x 1½"	100/Box – 1000/Case
3mL Syringe Only			
Catalog #	Description	Gauge/Needle	Packaging
305283	3mL Syringe Only		100/Box – 400/Case

*18G Blunt Fill and Filter Needles have a non-retracting needle

REMEMBER:

Safety-Engineered Devices are required in your Custom and Standard Procedure Trays.

For more information call
1.888.237.2762



BD, BD Logo, and all other trademarks are property of Becton, Dickinson and Company. © 2013 BD.

BD 1/13-MSS0072

BD Medical
1 Becton Drive
Franklin Lakes, NJ 07417
www.bd.com