

BD Microtainer™ Tubes Tube Guide and Order of Draw

Catalog #/Color Closure	Additive	Mix by Inverting	Laboratory Use
-------------------------	----------	------------------	----------------



365973 Lavender

K₂EDTA

10x

For whole blood hematology determinations. Tube inversions prevent clotting.



365971 Green

Lithium Heparin

10x

For plasma determinations in chemistry. Tube inversions prevent clotting.



365958 Mint Green

Lithium Heparin and Gel for plasma separation

10x

For plasma determinations in chemistry. Tube inversions prevent clotting.



365956 Gold

Clot Activator and Gel for serum separation

5x

For serum determinations in chemistry.



365959 Gold



365957 Red

No additive

0x

For serum determinations in chemistry, serology and blood banking.



366270
Tube Extender



BD Vacutainer Technical Services 1.800.631.0174
BD Customer Service 1.888.237.2762
www.bd.com/vacutainer

BD Vacutainer Systems
Preanalytical Solutions
1 Becton Drive
Franklin Lakes, NJ 07417