



BD Nexiva™

Closed IV Catheter System—Single Port


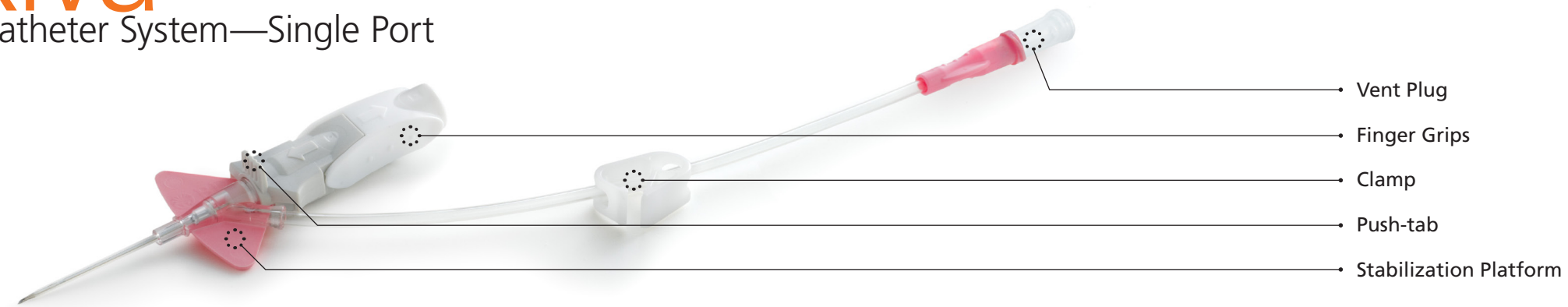


POINTS TO PRACTICE

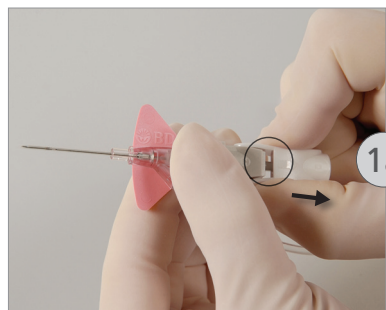

PREPARATION:

- > Secure vent plug.
- > Clamp should not be engaged.
- > Twist to remove needle cover.
- > Holding as shown, pull back approximately 1/8" on finger grips (*fig. 1a*).
- > Push finger grips back to their original position so the stabilization platform and finger grips are snugly together (*fig. 1b*).

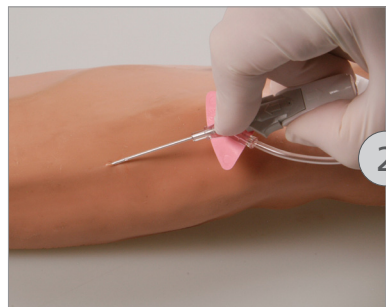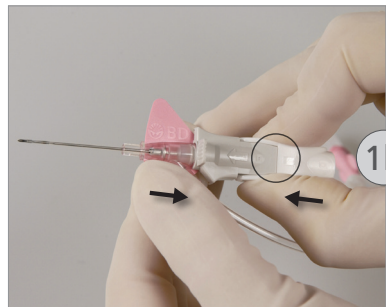

INSERTION AND FLASHBACK:

- > Hold the system as shown and access the vessel (*fig. 2*).
- > Initial blood return is along the catheter, then up the extension tube. Look at catheter for initial blood return.
- > Lower and advance the entire catheter and needle unit slightly to ensure the catheter tip is within the vessel.



ADVANCEMENT:

- > Place pad of index finger behind the push-tab and push the catheter off the needle into the vessel (*fig. 3*).


TIP: Do not pull back on the needle during advancement.



NEEDLE REMOVAL:


- > Stabilize the catheter system and pull back on the finger grips until the push-tab component releases from the stabilization platform (*fig. 4*).
- > Discard the shielded needle into a puncture-resistant, leak-proof sharps container.

TIP: Do not hold onto the push-tab component of the device as this will prevent the release of the needle shield.

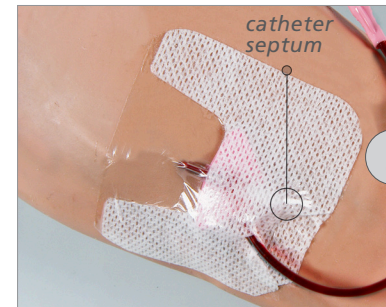

STABILIZATION:

- > Apply a transparent dressing to cover the septum, allowing maximum use of extension tubing (*fig. 5*).

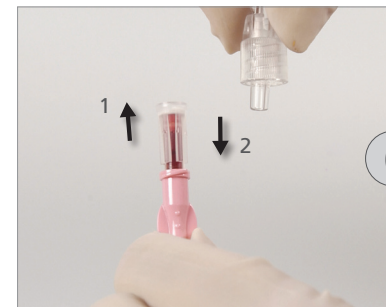

PREPARATION FOR USE:

- > Engage the clamp.
- > Remove the vent plug and attach administration tubing or access port (*fig. 6*).
- > Release the clamp to flush or infuse.
- > Complete securement of extension tubing.

Consult product insert for complete instructions, warnings and cautions.



BD Medical
 9450 South State Street
 Sandy, Utah 84070
 1.888.237.2762
www.bd.com/Nexiva