

Preemie Transport Blanket

Instructions for Use

Warning:

- Preemie Transport Blanket is a single use product only.
- Do not use on infants with abdominal wall defects, meningomyelocele, major congenital anomaly, or obvious skin disorder.
- DO NOT use if Preemie Transport Blanket has been tampered with.
- AVOID any contact with any heated surfaces.
- DO NOT place any portion of the Preemie Transport Blanket over the infants face. Ensure the airway is clear at all times.
- DO NOT wrap the infant too tightly.
- ALWAYS monitor infant body temperature.
- Remove the Preemie Transport Blanket once the infant begins to regulate his/her own temperature.

Transport Technique:

1. Prior to delivery, unfold the Preemie Transport Blanket and lay flat on a surface, bubble side down. Note location of important design features.




2. Position the infant in the center of the Preemie Transport Blanket with his/her head between the U-shaped shoulder flaps. NOTE: Monitor infant temperature at all times.



3. Fold the right side of the blanket over the infant first.



4. Remove the white tabs to reveal the adhesive backing on the Preemie Transport Blanket.

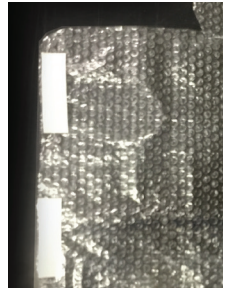

5. Fold the left side of the blanket over the right side, while gently pressing the adhesive backing to secure the blanket together.



6. Remove the white tabs on the U-Shaped shoulder flaps to reveal the adhesive backing on the Preemie Transport Blanket.



7. Lastly, fold the shoulder flaps over the infant, securing to shoulder flaps onto the blanket using the adhesive backing.



Note: Perforated Openings on both sides of the U-Shaped shoulder flaps give access to the infant without having to take the Preemie Transport Blanket off.





Preemie Transport Blanket

Instructions for Use

RM95-1125 Rev. 1/13/16 © 2016 Bionix Development Corp.
PATENT PENDING.

www.BionixMed.com

5154 Enterprise Blvd. • Toledo, Ohio 43612
Phone: 800.551.7096 • Fax: 800.455.5678

