

### **Igloo® Wound Irrigation Shield Instructions**

1. Prep and drape the wound in a sterile fashion. Prepare a basin with sterile saline or other desired irrigant solution.
2. Fill a standard luer lock syringe or other delivery device with the irrigation solution.
3. Attach the Igloo Wound Irrigator to the tip of the syringe or other delivery device.
4. Place the Igloo Wound Irrigator over the wound, lightly touching the skin of the patient. For large, spacious (or chronic) wounds, place Igloo just above the wound surface.
5. Irrigate the wound in the usual fashion; syringe may need to be refilled one or more times as needed.
6. After use, discard the Igloo Wound Irrigator appropriately.

**DO NOT RE-USE OR RESTERILIZE. SINGLE-USE ONLY.**

### **Igloo® Wound Irrigation System Instructions**

1. Prep and drape the wound in a sterile fashion. Prepare a basin and sterile saline bottle.
2. Straighten irrigation tubing.
3. Twist the Igloo Irrigation System adapter onto saline bottle until tight. Irrigation tubing should reach bottom of bottle.
4. Attach the Igloo Wound Irrigation Shield to the tip of the irrigation wand.
5. Place the Igloo Wound Irrigation Shield over the wound, lightly touching the skin of the patient. For large, spacious (or chronic) wounds, place Igloo just above the wound surface.
6. Irrigate the wound in the usual fashion.
7. After use, discard the Igloo Wound Irrigation System appropriately.

**DO NOT RE-USE OR RESTERILIZE. SINGLE-USE ONLY.**