

Cardinal Health™ *H. pylori* Antibody Rapid Test Cassette

Two easy steps for one rapid result

Offer your patients rapid, qualitative detection of IgG antibodies specific to *Helicobacter pylori* (*H. pylori*) in whole blood and serum.



Ease of use

- CLIA waived: fingerstick whole blood
- Moderately complex (nonwaived): serum
- Time to result: less than 7 minutes
- Simple two-step procedure

Precision


- Sensitivity (average value) 95.1%
- Specificity (average value) 94.1%

Convenient packaging


- 25 individually packaged test devices, each containing one disposable sample dropper
- Room-temperature storage
- 24-month shelf life from date of manufacture

Procedure

1. Place test device on a clean, dry, level surface.
2. For fingerstick whole blood or serum, apply 1 drop of specimen to the sample well. Allow 30 seconds for total absorption. Add 3 drops of wash buffer to the sample well.
3. Do not move test device until ready to read.
4. Read results at less than 7 minutes.

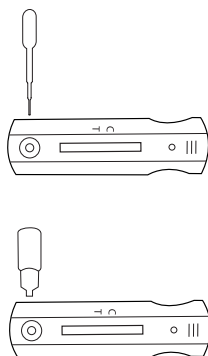

Interpretation of results

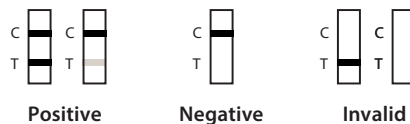

Note: If both the C line and the T line appear, the result indicates that the IgG antibodies specific to *H. pylori* are detected and the result is positive.

Ordering information

Catalog No.	Description	Qty.
CH25	Cardinal Health™ <i>H. pylori</i> Antibody Cassette	25 test cassettes/box

We are your lab advisor.

Expertise. Choice. Efficiency. Savings.

For more information or to order, contact your Cardinal Health sales representative, call **800.964.5227** or visit **cardinalhealth.com/rapidtestkits**

