


A REVOLUTION IN HERNIA REPAIR



Superior performance
from many angles^{1,†}

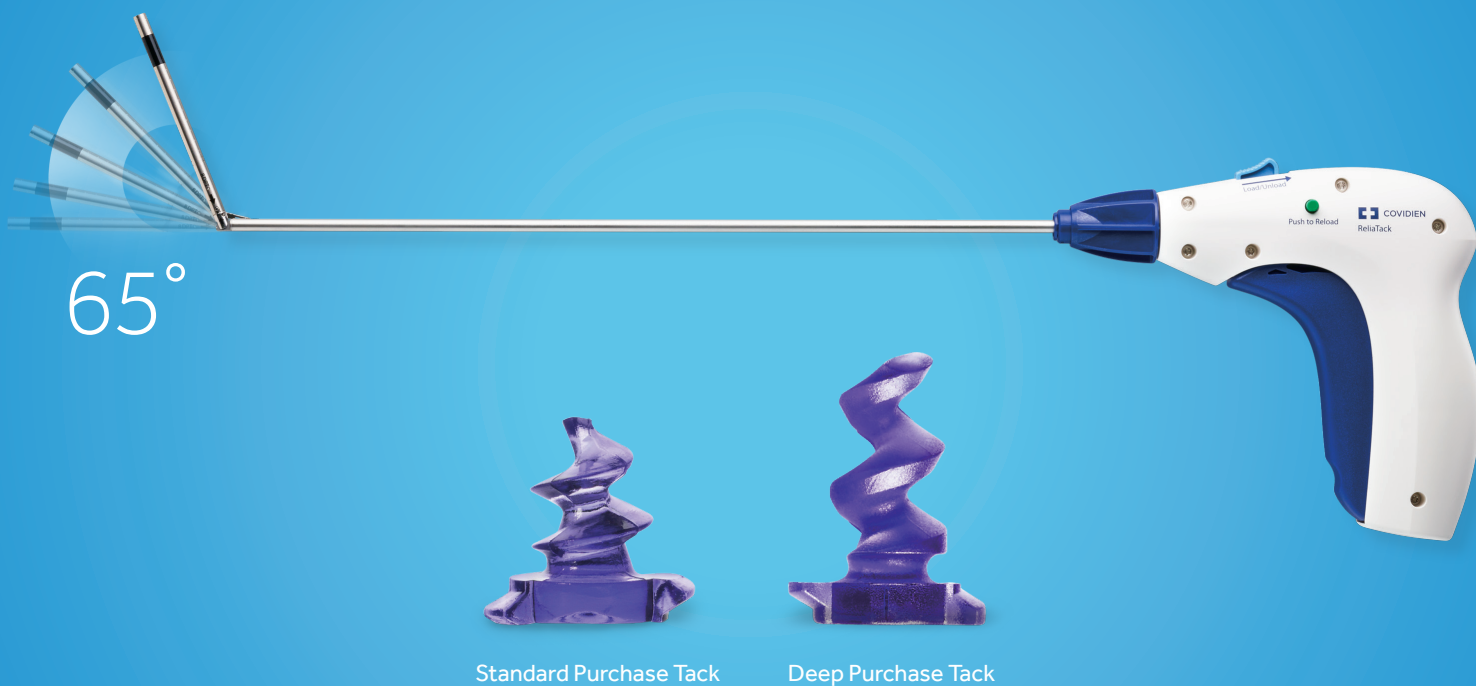

ReliaTack™ Articulating Reloadable Fixation Device

[†]Benchtop testing based on commercially available absorbable and permanent fixation devices, which include the SECURESTRAP™, SorbaFix™, OptiFix™, CapSure™, ProTack™, and ReliaTack™ devices with standard purchase tacks when the shaft is angled at 30, 45, 65, and 90 degrees. ReliaTack™ deep purchase tack shear pull test performed in synthetic foam. Results may not correlate to performance in animal or cadaveric tissue, or performance in humans.

Medtronic
Further. Together

A REVOLUTION IN HERNIA REPAIR.

You want every advantage when it comes to laparoscopic ventral hernia repair:

- Superior access^{2,€}
- Stronger fixation^{1,†}
- Lower cost of care^{3,§}

That's why we've designed our ReliaTack™ articulating reloadable fixation device to give you all three.

Plus, to enhance your procedural flow, we've added two new features:

- A single-turn knob for easier articulation[‡]
- An optimized shipping wedge for intuitive loading[‡]

It's part of our commitment to helping you improve patient outcomes with cost-effective solutions from our complete hernia repair portfolio.

Choose between standard or deep purchase tacks to meet your unique patient and procedural needs.



€Compared to AbsorbaTack™ fixation device.

†Benchtop testing based on commercially available absorbable and permanent fixation devices, which include the SECURESTRAP™, SorbaFix™, OptiFix™, CapSure™, ProTack™, and ReliaTack™ devices with standard purchase tacks when the shaft is angled at 30, 45, 65, and 90 degrees.


ReliaTack™ device deep purchase tack shear pull test performed in synthetic foam. Results may not correlate to performance in animal or cadaveric tissue, or performance in humans.

§Based on potential for eliminating contralateral ports and average savings when 31–50 tacks are needed.

‡Compared to the first generation ReliaTack™ device.

SUPERIOR ACCESS^{2,£}

- 65 degrees of articulation delivers consistent perpendicular tack deployment⁴
- Consistent tack purchase when fired at 30, 45, 65, and 90 degrees¹

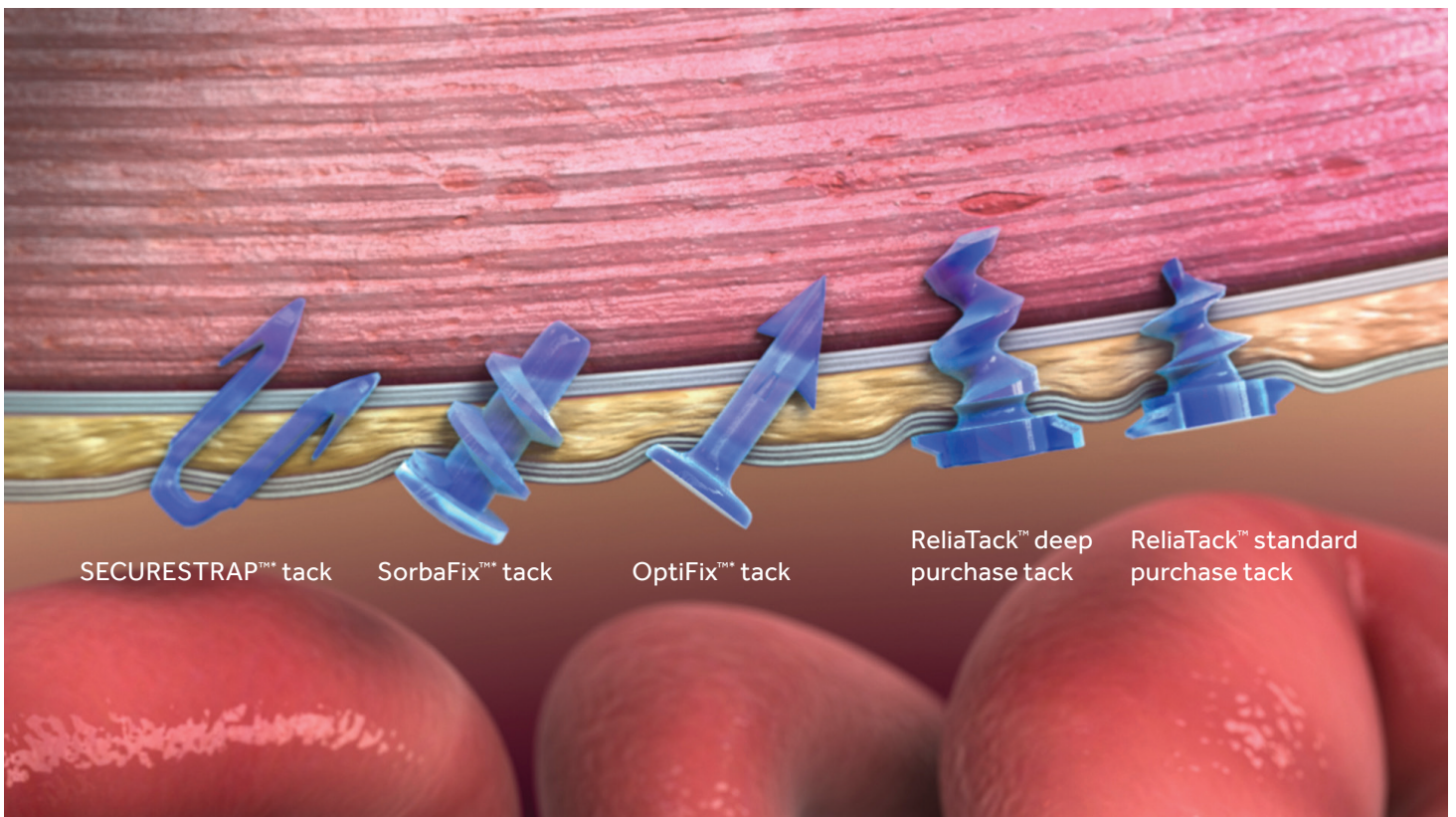

£ In comparison to AbsorbaTack™ fixation device.


STRONGER FIXATION^{1,†}

The ReliaTack™ device with deep purchase tacks demonstrates superior fixation strength compared to both absorbable and permanent fixation devices when the shaft is angled at 30, 45, 65 and 90 degrees^{1,†}

Deep Purchase Tacks Comparison Chart


Absorbable Fixation Devices

DPT is:

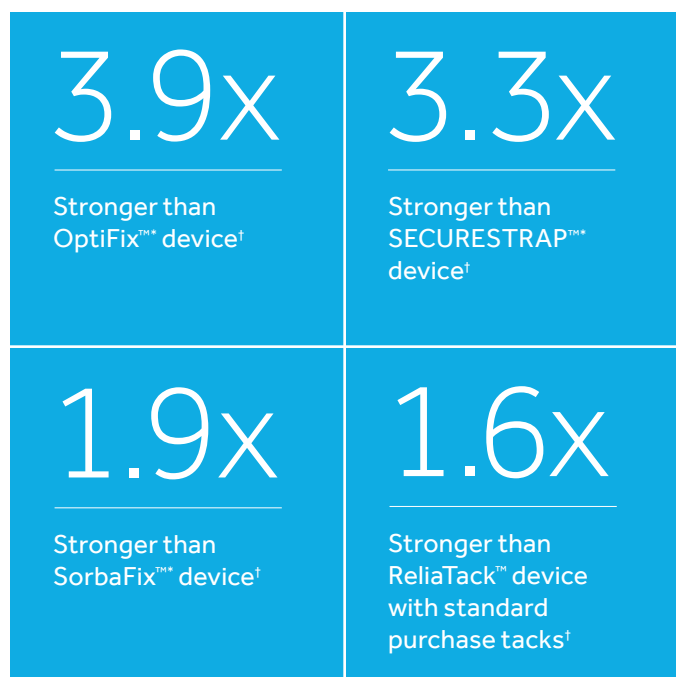


Permanent Fixation Devices

DPT is:

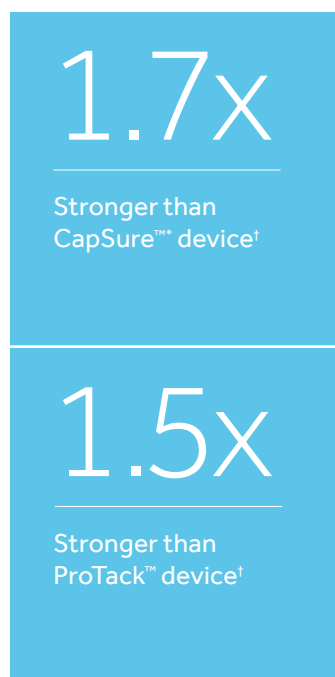

Deep purchase tack fixation strength

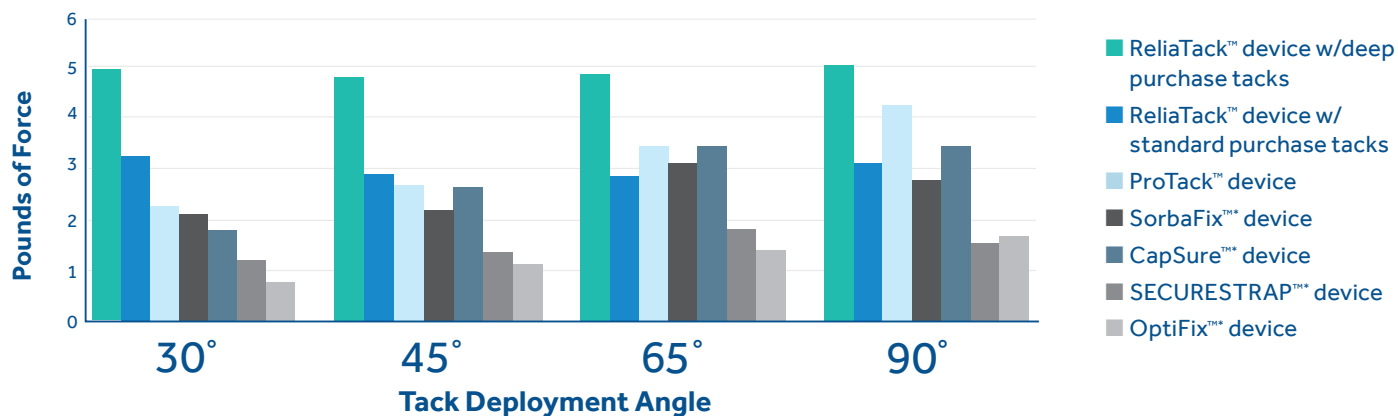


†Benchtop testing based on average fixation strength, when the shaft is angled at 30, 45, 65, and 90 degrees. The ReliaTack™ device deep purchase tack shear pull test performed in synthetic foam. Results may not correlate to performance in animal or cadaveric tissue, or performance in humans.

LOWER COST OF CARE^{3,§}

Up to sixty firings eliminates the need for multiple fixation devices per case[‡]

May save up to \$461 per case compared to SECURESTRAP[™]* device and SorbaFix[™]* device^{3,‡,¶}

**ReliaTack[™] device
with standard purchase tacks**

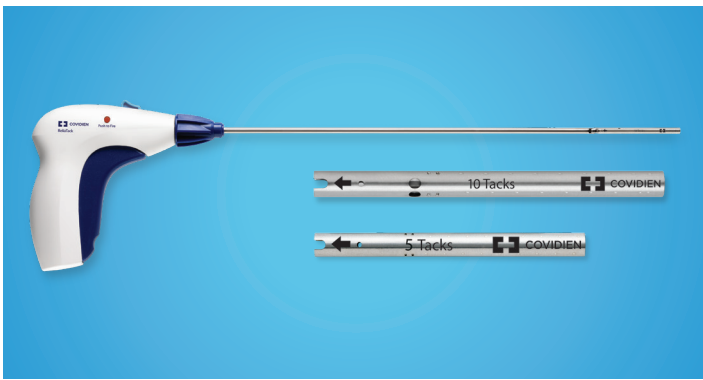


**ReliaTack[™] device
with deep purchase tacks**



SECURESTRAP[™] device



SorbaFix[™] device

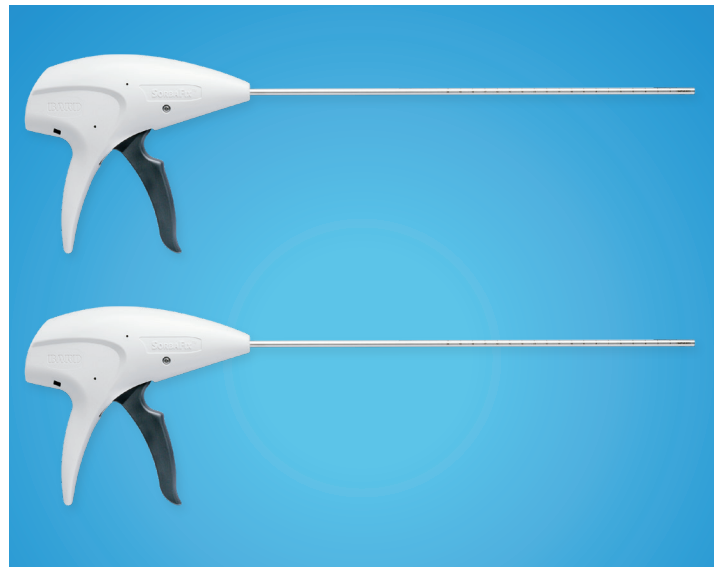

§ Based on potential for eliminating contralateral ports and average savings when 31-50 tacks are needed.

‡ When 30-60 tacks are needed.

‡ Average of greatest estimated per-case savings over SECURESTRAP[™]* and SorbaFix[™]* when 31-50 tacks are needed.

¶ Articulation may save up to \$100 per case by potentially eliminating contralateral ports.


ORDERING INFORMATION

ReliaTack™ Device with Standard Purchase Tacks



REL TACK3X10SW

One handle with three ten-tack standard purchase reloads

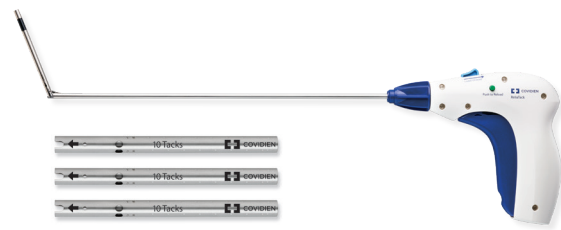

REL TACK10RSW

Ten-tack standard purchase reload




REL TACK5RSW

Five-tack standard purchase reload



ReliaTack™ Device with Deep Purchase Tacks

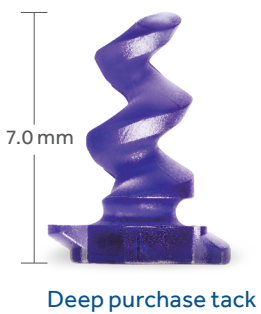

REL TACK4XDPTSW

One handle with three eight-tack deep purchase reloads and one five-tack deep purchase reload



REL TACK8RDPTSW

Eight-tack deep purchase reload



REL TACK5RDPTSW

Five-tack deep purchase reload



To order or learn more about the ReliaTack™ articulating reloadable fixation device visit www.medtronic.com/reliatack or contact your local Medtronic sales representative.

HERNIA CARE Mesh. Fixation. Permacol™. Dissection.

Our comprehensive product portfolio can enhance your hernia repair procedures.

References

1. Based on internal test report #RE00010135-1. ReliaTack™ device deep purchase tack shear pull test performed in synthetic foam. Results may not correlate to performance in animal or cadaveric tissue, or performance in humans. p-value=0.00. October 2015.
2. Based on mesh overlap claims testing report 2165-055-1. P-value= 0.007 March 2014.
3. ReliaTack™ DPT cost comparison, ReliaTack deep purchase internal average cost data compared to SECURESTRAP™ and SorbaFix™ IMS data.
4. Based on design verification articulation angle engineering report #2165-050. February 2014.

IMPORTANT: Please refer to the package insert for complete instructions, contraindications, warnings and precautions.

© 2017 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic.™™ Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. 10/2017 – P150042(3)a – [WF# 2112589]