

Connecting DETECTO® Scales to Welch Allyn Devices

Bariatric



6857DHR
Waist-High
Bariatric Scales

Wheelchair



FHD144II
Stationary
Wheelchair Scales

6550
Fold-up Portable
Wheelchair Scales

BRW1000
Bariatric
Wheelchair Scales

Digital Column



8430
Waist-High
Medical Scales

6437DHR
Eye-Level
Medical Scales

Low-Profile



GP400758C
Low-Profile Portable Scales



Connex® Vital Signs
Monitor 6000 Series



Connex® Integrated
Wall System



Easily Connect Your Welch Allyn Device to a DETECTO Scale



DETECTO SCALE



CONNECTIVITY KIT



WELCH ALLYN
DEVICE W/ LICENSE
ACTIVATED



EMR

Welch Allyn Connectivity to DETECTO Professional Scales

The following table outlines the specific DETECTO scales that have been approved for use with both the Connex® Vital Signs Monitor 6000 Series and the Connex Integrated Wall System (CIWS).

Model Number	Scale Description	Scale Family
6437DHR	Digital Physician Scale with Digital Height Rod	Digital Column
6449	Digital Scale with Height Rod and Handpost	Digital Column
6439	Digital Eye-Level Scale with Height Rod	Digital Column
6437	Digital Eye-Level Physician Scale	Digital Column
8437	Digital Waist-High Physician Scale	Digital Column
8430	Digital Waist-High Scale with Height Rod	Digital Column
8437S	Stainless Steel Scale, Waist-High Digital	Digital Column
8430S	Stainless Steel Scale, Waist-High Digital and Height Rod	Digital Column

6857DHR	Waist-High Stand-On Bariatric Scale - Digital Height Rod	Bariatric
6854-DHR	Waist-High Stand-On Bariatric Scale - Digital Height Rod	Bariatric

BRW1000	Portable Bariatric Wheelchair Scale	Wheelchair
6500	Portable Wheelchair Scale	Wheelchair
6550	Folding Portable Wheelchair Scale	Wheelchair
FHD-133-II	Geriatric/Bariatric Wheelchair Scale 36"X36"	Wheelchair
FHD-144-II	Geriatric/Bariatric Wheelchair Scale 48"X48"	Wheelchair
6495	Digital Stationary Wheelchair Scale	Wheelchair
CR-500D	"Roll-A-Weigh" Wheelchair Scale, 500 lb Capacity	Wheelchair
CR-1000D	"Roll-A-Weigh" Wheelchair Scale, 1,000 lb Capacity	Wheelchair

GP-400-758C	"Legal for Trade" Portable Digital Scale, 400 lb Capacity	Low-Profile
GP-600-758C	Portable Digital Scale, 600 lb Capacity	Low-Profile

Scale Connectivity Kits for the following Welch Allyn devices:

**Welch Allyn
Connex® Vital Signs
Monitors**



**Welch Allyn
Connex® Integrated
Wall System**



License Required
Yes License key included with purchase of connectivity kit from DETECTO
Yes License key included with purchase of connectivity kit from DETECTO

Important Notes Regarding Acquisition and Setup of Scale Connectivity:

- Welch Allyn will not sell scales or connectivity kits/licenses.
- Connex Vital Signs Monitors require software version 1.7 or later to access the scale connectivity feature.
- Connex® ProBP™ 3400 Digital Blood Pressure Device is not currently available with weight scale connectivity.
- Scale connectivity feature must be activated on Welch Allyn device using the Welch Allyn Service Tool and a license key shipped with the connectivity kit.
- Connectivity license must be purchased through DETECTO.

Important Notes Regarding Function of Welch Allyn Devices with DETECTO Scales:

- **Select DETECTO Scales:** Measure weight, allow for manual entry or digital capture of height and calculate BMI.
- **Select DETECTO Scales:** Have the ability to transfer values for weight, height, and BMI from a DETECTO digital scale to a Welch Allyn Device.
- **Connex® Devices with 1.7 Software:** Accepts and displays weight, height, and BMI when transmitted from the scale (the Connex monitor does not calculate BMI). The monitor also allows for manual entry of weight, height, and/or BMI.

Important Note Regarding EHR Connectivity:

EHR connectivity requires existing Welch Allyn device interface built by the specific EHR vendor for customer's specific EHR product and release. This interface must include each specific category, height, weight, BMI, for that information to transfer. Contact your connected device manager to research customer-specific scenarios.

For more information on DETECTO scales,
please call 800.641.2008 or visit decteto.com/wa.htm.