

# ***INSTRUCTION***



**Adjustment Range:  $\pm 6$ mmHg**

- 1. The attached small key is for adjustment of the manometer needle. Adjustment range: 6mmHg from center of zero position. If needle is beyond this range, it should be re-calibrated.**
- 2. Put the key into air intake hole, turn in clockwise direction; needle in the gauge will go left. Turn key in anti-clockwise direction; Needle will go right.**
- 3. Through such adjustment, the needle will be in exact position of zero. Remove key when adjustment is completed.**
- 4. Such adjustment will not cause any discrepancy in test result.**
- 5. Warning: small key cannot be turned full circle of manometer. Otherwise, mechanism will be damaged.**