



Savanna®

QUIDEL®




Quick Start Guide



Study the User Manual thoroughly
before using the QuickStart Guide.

Unpack Savanna


- Inspect the shipping container for obvious shipping damage prior to opening.
- Unpack the shipping container and inspect the unit for damage.

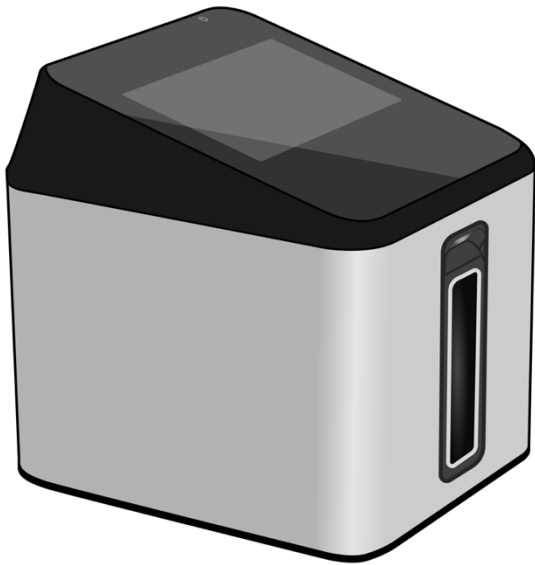

Savanna Instrument



Power Adapter

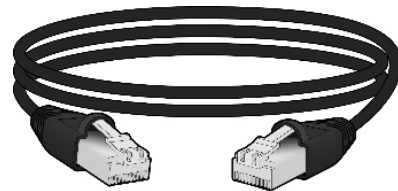

Ethernet Cable

Connect to Power and Turn On

Place the Savanna instrument on a countertop within reach of an electrical outlet. The unit is portable and may be moved to a suitable location for testing.

Step 1

Plug one end of the power adapter into the power port on the rear panel of Savanna, and the other end of the power adapter into an appropriate electrical outlet.



Power Port

Step 2

Turn on Savanna by pressing the power button located in the upper rear corner of the screen bezel. Once Savanna begins initial startup testing, an initialization notification will appear on the display screen, signaling that Savanna has been turned on.



Set Up Savanna

After the user logs into the Savanna system, there are two options to set up Savanna.

Option 1: Click the “**Next**” button on the touchscreen to skip data import and start the manual setup.


Option 2: Insert a USB drive into one of the USB ports located on the rear panel of Savanna, and follow the on-screen instructions to import configuration data from other Savanna instruments, if applicable.



Run QC

Periodic calibration is not required for Savanna. Run Quality Controls as specified by the package insert of the respective assay you intend to run.

Menu Structure (Main menu options vary by user role.)

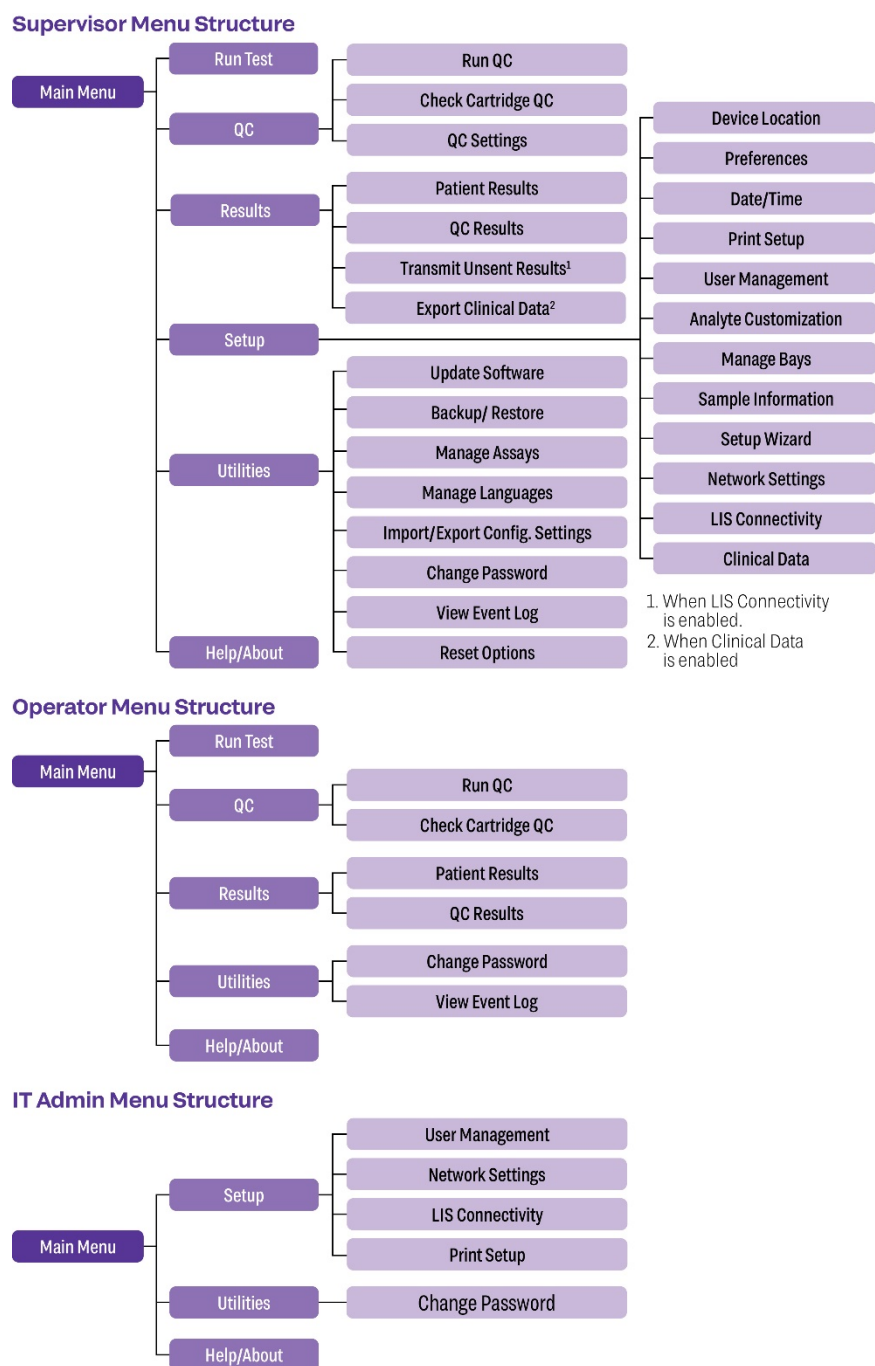

Assistance

If you have any questions regarding the use of this product or if you want to report a test system problem, please contact Quidel Technical Support at 800.874.1517, technicalsupport@quidel.com, or your local distributor.



Quidel Corporation
Athens, OH 45701 USA
quidel.com

