
PROPLUS[®] 2000KL

STRETCHER / IN-BED PATIENT WEIGHING SCALE
BALANZA CAMILLA PARA PESAJE DE PACIENTES
BALANCE CIVIÈRE POUR PESÉE DE PATIENTS

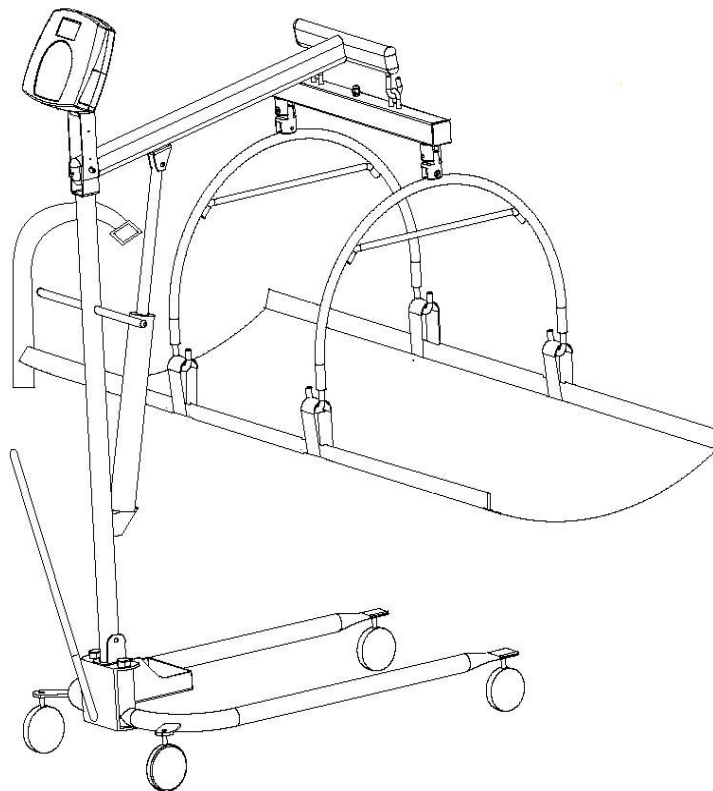USER INSTRUCTIONS

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MODE D'EMPLOI

Patents 7550682B2 and D508655 (US) Patent 0149984/1-8 and Patent Pending (EU)
Patents ZL200480031711X and 200430004551.2 (PRC) © Pelstar LLC 2009

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully and keep it for easy reference or training.

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TABLE OF CONTENTS

ENGLISH	1
ESPAÑOL	18
FRANÇAIS	34
CAUTION AND WARNING	2
SPECIFICATIONS FOR THIS SCALE	3
QUICK START INSTRUCTIONS	4
ASSEMBLY INSTRUCTIONS	6
SET UP INFORMATION / BATTERIES	12
MAINTENANCE	13
TROUBLESHOOTING	14
EXPLODED VIEW OF SCALE	15
PARTS LIST FOR MAINTENANCE	16
WARRANTY	17

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.homscales.com/customers/registration.aspx>

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CAUTION AND WARNING

To prevent injury and damage to your scale, please follow these instructions very carefully.

- To ensure proper operation of the ProPlus® Stretcher Scale, this operation manual should be reviewed carefully before using your scale. Keep this manual available for future reference, and also for use in the orientation or training of new personnel who will use this scale.
- If the scale becomes damaged, it should not be operated until properly serviced. All repairs should only be performed by authorized Health o meter® Professional service personnel.
- This scale is designed for static weighing of patients only. **No scale should be used for patient transfer. Pelstar, LLC assumes no responsibility for patient injury or scale damage should this caution be ignored by the scale operators.**
- **Do not leave patients unattended while on the scale.**
- For accurate weighing, the casters on the lifter base must be placed on a flat and stable surface with the weighing mechanism not touching the floor.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.
- Operating at other voltages and frequencies than specified could damage the equipment.
- If the “LOW BAT” indicator activates, replace the batteries or connect the scale to an AC power source, as soon as possible, for accurate weighing. See page 12 for battery replacement information.
- It is intended that this equipment be used with the assistance of a healthcare worker.
- Failure to comply with the following will void the scale warranty:
 - Do not gas or steam sterilize nor autoclave the scale.
 - Do not place liquids on top of the scale display console.
 - Do not exceed recommended weight limit of **400 lb/181kg** for this scale.
 - This scale contains delicate sensors. Do not bang into doors, elevators, etc., as damage may result.

SPECIFICATIONS

GENERAL

The Health o meter® Professional ProPlus® Stretcher Scale Model 2000KL uses highly sophisticated microprocessor technology. This scale is designed to provide accurate, reliable and repeatable weight measurements and features that make the weighing process simple, fast and convenient.

The scale is set up to use motion-sensing weighing technology, to determine the actual weight of a moving patient. The scale may be changed to measure live weight (please refer to page 13 of the ProPlus® Display Module User Manual for instructions on changing the scale setting).

The scale features a folding stretcher for easy mobility and a display assembly stand that allows the display head to swivel/pivot 180 degrees, for convenient reading from either side of the scale. The weight can be displayed in kilograms or in pounds (decimals, fractions of a pound, or lb/oz).

The unit can be operated using its AC adapter, or by 6 D-cell batteries (not included).

2000KL SCALE SPECIFICATIONS

Capacity and Resolution	400 Lb x 0.2 lb / ¼ lb / 4 oz (181 kg x 0.1 kg)	
Power Requirements	Adapter 120V AC - 9V DC – 60 Hz or 6 x D size batteries	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 30°F to 125°F (0°C to 50°C) Maximum Humidity: 85%	
Stretcher Dimensions	Closed: Length: 70 7/8" (180 cm) Width: 6" (15 cm) Weight: 17.6 lb (8 kg)	Fully extended: Length: 70 7/8" (180 cm) Width: 29 1/2" (75 cm)

QUICK START INSTRUCTIONS

Pre-weighing the Stretcher (Setting Stretcher Tare)

There are two ways to pre-weigh the stretcher using the TARE function. One is automatic (PUSH BUTTON TARE) and the other is manual (KEYBOARD or KEYPAD TARE) tare.

AUTOMATIC (PUSH BUTTON TARE)

1. With the stretcher off the scale, press the ON/OFF button to turn on the scale.
2. Wait until "000LB00OZ" and "ZERO" appear on the left side of the display.
3. Hook the stretcher to the scale. The display will show a value equivalent to the stretcher weight.
4. Press the TARE button. Once the tare is set, the display will zero out. "TARE" will appear in the upper right side of the display.
5. Remove the stretcher from the scale. The display will show a negative weight equal to the weight of the stretcher, once the stretcher is removed.

MANUAL (KEYBOARD OR KEYPAD TARE)

1. Take the stretcher off the scale or be sure it has been removed from the scale.
2. Press the ON / OFF button to turn on the scale.
3. Wait until "000LB00OZ" and "ZERO" appear on the left side of the display.
4. Press the TARE button. The user will be prompted to enter the tare value.
5. Use the keypad to enter the weight of the stretcher and then press ENTER.
6. The value entered will show as a negative value on the display.

Placing the Patient on the Stretcher

Lay the stretcher on a bed or any other flat, stable surface you wish to use to transfer the patient onto the stretcher, and then follow the following steps:

1. Carefully slide or move the patient and evenly place the patient onto the stretcher.
Note: The patient's center mass should be placed as close as possible to the center of the stretcher.
2. Wheel the lifter scale close to the stretcher, and then lower the boom hoops to within 5-10 inches (12-25cm) of the patient's body.
3. Connect the hanging loops of the stretcher to the hooks at the end of each set of stretcher hoops, making sure the loops hang properly and equally from the hooks.
Note: Make sure the stretcher load hoop / beam is properly attached to the boom hooks. If not, an inaccurate reading could result.
4. Tighten the hydraulic control valve knob by turning it clockwise. The control valve knob is located on the pump near the pump handle.
5. Use the Base Adjustment Lever to widen the lifter base. The base must be spread to the widest possible position to maximize the scale stability when lifting the patient.
6. SLOWLY pump the hydraulic pump handle to lift the stretcher and patient until the entire stretcher is hanging from the loops and is not touching any surface below or any object around it. Do not lock the wheels on the lifter base. If needed, the base will move slightly or adjust for balance. **Caution: NEVER move patient while on scale; never leave a patient unattended while he or she is on the scale.**

QUICK START INSTRUCTIONS (CONTINUED)

Taking a Weight Measurement

1. The scale will calculate the patient's weight. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight. Once a weight has been displayed, while the patient is still on the stretcher, we recommend performing a second weighing by pressing the REWEIGH button.
2. After the weight measurement has been taken and has been recorded, lower the stretcher and patient slowly back to their bed or other flat, stable surface:
 - A. Open the hydraulic pump control valve by turning the knob counterclockwise VERY slowly, to control the descent.
 - B. Guide the patient on the stretcher carefully down to the bed or flat, stable surface.
3. Release the hanging loops from the stretcher hoops, once the patient is resting on the bed.
4. Press the ON / OFF button to turn the scale off.

Calculating BMI

1. After obtaining the patient's weight on the scale, press the BMI button while the scale is on. The scale will prompt you to enter the patient's height. Use the keypad to enter the height.

Note: When weighing in pounds, the height is entered in 1/4" increments. To increase the fractional height of the patient, press "1" for 1/4", "2" for 1/2" and "3" for 3/4". When weighing in kilograms, the height must be entered in 1cm increments.

2. Once height is keyed in, press ENTER.
3. The display will show the patient's BMI (Body Mass Index).
4. Press ENTER again to return to the normal weighing mode of the scale.

Note: The scale will not calculate BMI for a patient that is 24 pounds (12 kg) or less. If a height is not entered for the patient within 30 seconds of pressing the BMI button, the scale will return to the normal weighing mode.

ASSEMBLY INSTRUCTIONS

Before Assembly

The Model 2000KL ProPlus® Stretcher Scale is shipped disassembled in three (3) separate cartons. Carefully inspect all cartons for shipping damage before unpacking. If damage is found, contact your shipper or Health o meter® Professional representative immediately. Claims must be filed with the shipper as soon as possible after receipt of the damaged package.

Remove each assembly from the carton and unwrap packing material carefully to prevent scratching the unit's parts. It is recommended that two people perform the assembly. **To avoid damaging the scale parts when unpacking it, do not use a box cutter, knife, scissors or any sharp object to open the packaging.**

The scale is shipped in three (3) separate cartons as follows:

- A. Large carton containing:
- (1) Hydraulic pump assembly
 - (1) Lifter mast
 - (1) Base adjustment lever
 - (1) Lifter base with casters
- B. Small carton containing:
- | | | |
|---------------------------|-------------------------------|-------------------------------|
| (1) Lifter boom | (1) Stretcher storage tray | |
| (1) Scale Display support | (1) AC Adapter holder | (1) Stretcher retaining strap |
| (1) Display assembly | (1) AC Adapter | (1) Hexagon screw w/ lock nut |
| (1) Load cell assembly | (1) Display rear cover | |
| (2) Stretcher hoops | (6) Screws for display | |
| (1) Stretcher canvas | (2) Shoulder screws / washers | |
- C. Long, narrow carton containing:
- (2) Stretcher bars

Tools required: Phillips screwdriver, adjustable wrenches, 7/16" Allen wrench.

Parts of Stretcher Scale:

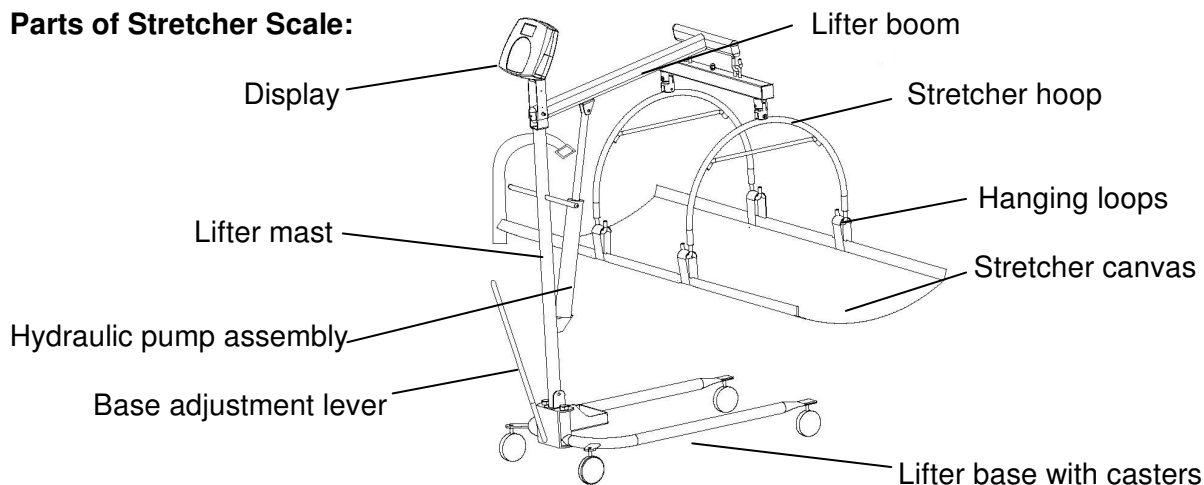
Figure 1

ASSEMBLY INSTRUCTIONS (CONTINUED)

Assembling the Patient Lifter

1. Place the Stretcher Storage Tray on the Lifter Base, as shown below, before attaching the Lifter Mast.

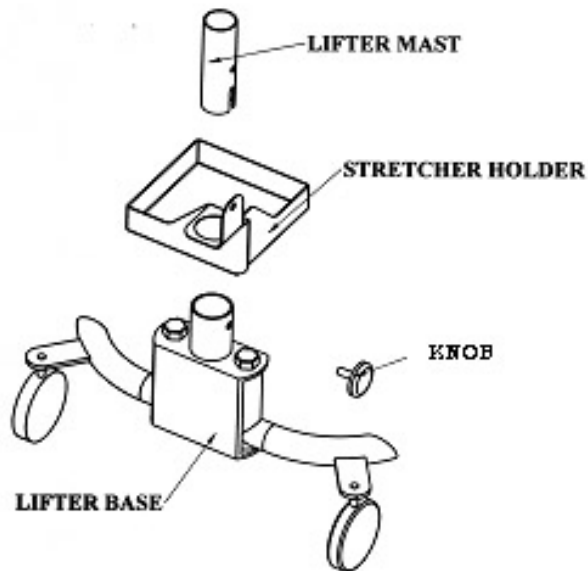
Figure 2

2. Remove black plastic protective caps from Lifter Mast. Insert the bottom of the Lifter Mast and Hydraulic Pump assembly into the mast sleeve (through the Stretcher Tray Holder) on the Lifter Base. The notch on the end of the mast fits over the round tube at the bottom of the Mast Sleeve. The mast must lock into position in the mast sleeve, making it impossible to rotate. Check that the mast is locked into position and that the hole in the mast is aligned with the threaded hole in the mast sleeve.
3. Insert the threaded locking device (the plastic knob and stud chained to the base) into the threaded hole in the mast sleeve. Tighten to lock the mast to the base. **Keep mast and base locked at all times except when removing mast from base for storage.**

ASSEMBLY INSTRUCTIONS (CONTINUED)

Attach the Display and Scale to the Patient Lifter:

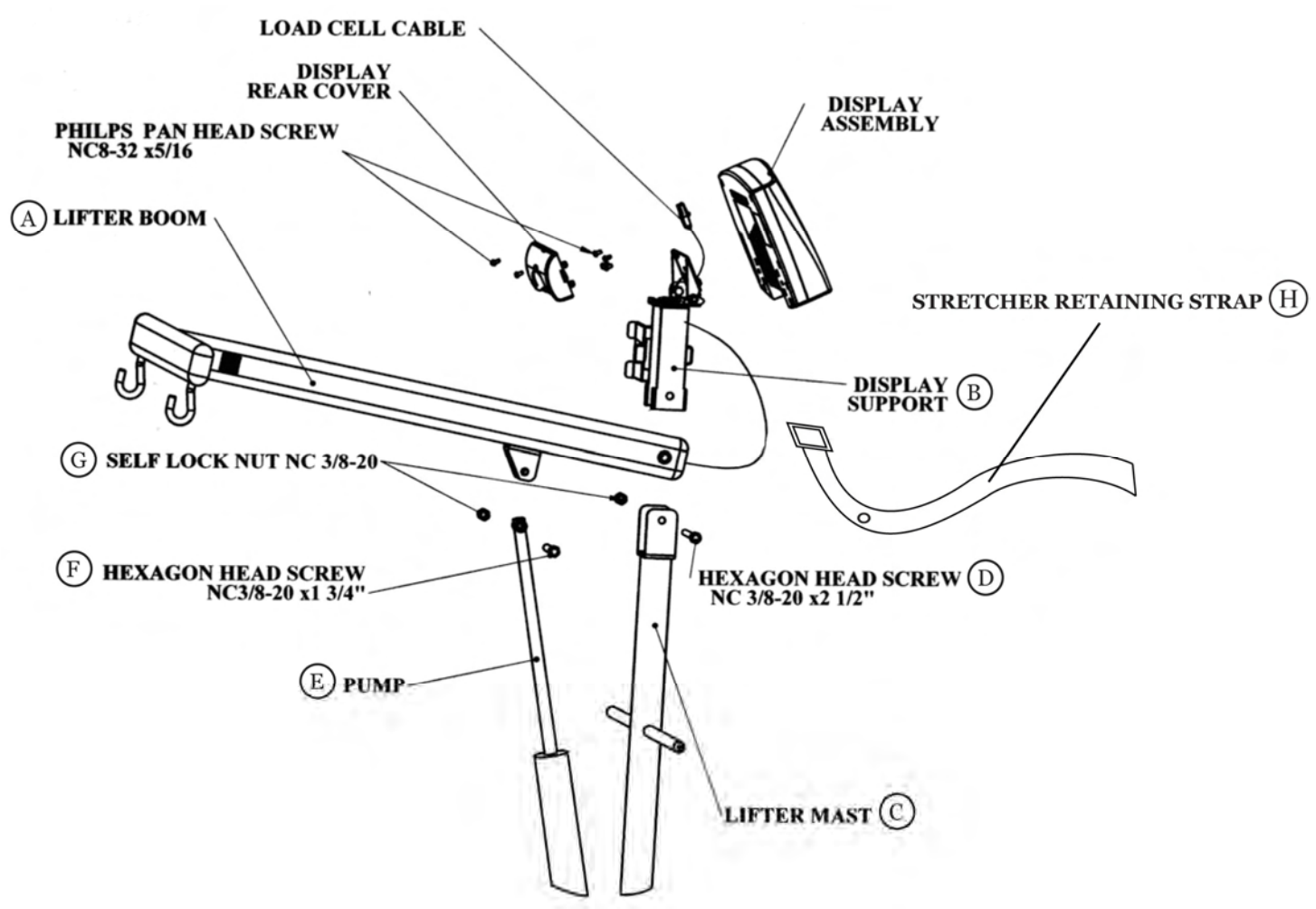
Figure 3

ASSEMBLY INSTRUCTIONS (CONTINUED)

4. Referring to Figure 3 on page 8, connect the Lifter Boom (A) and the Display Support (B) to the Lifter Mast (C) using the NC $\frac{3}{8}$ -20x2 $\frac{1}{2}$ " hexagon bolt and large self lock nut (D). Be sure to insert the NC $\frac{3}{8}$ -20x2 $\frac{1}{2}$ " hexagon bolt through the Stretcher Retaining Strap (H) eyelet hole before you attach the display support and lifter boom to the lifter mast. **Note:** This screw is on the axis of the Lifter Boom, so it is imperative that the three parts – Lifter Boom, Lifter Mast and Display Support – are assembled in the proper order. With the Boom hooks facing down, place the Lifter Boom (A) inside the two sides of the Lifter Mast (C) and align the holes. Place the Display Support (B) over the Lifter Mast (C) and Lifter Boom (A) and align the holes. Thread the screw through all three parts and tighten the self lock nut on the other side well.
5. Insert the shaft end of the Pump (E) into the eared bracket on the Lifter Boom (A) and align the holes. Slip the 1-3/4" screw (F) through the bracket and the Pump shaft hole. Tighten with a locknut (G).
6. Insert the Base Adjustment Lever into its socket at the back of the Lifter Base. Check to see that the bump on the outside of the socket fits securely in the notched adjusting plate. To adjust the base legs, pull back on the adjusting lever to unlock. Move the lever to the right to widen the base.
7. A wheel lock is provided on the Lifter Base for parking only. To lock caster, step on cam lever on side of caster. To unlock, step on higher cam lever. **NOTE: Never lock wheel lock or block wheels when lifting someone.** Wheel lock is provided for parking the Lifter only.
8. Check the operation of the Hydraulic Pump by pumping the handle to elevate the Boom. Make certain the control valve knob (located on the pump near the handle) is fully tightened (clockwise).
9. To lower the Boom, slowly turn the control valve knob counterclockwise. The rate of descent can be controlled by how much the knob is turned. The Boom may not lower readily without weight. To test, simply apply downward force to the Boom. **Note:** If bolts, mounting pump or Lifter Mast are too tight, the Boom may not lower properly.

ASSEMBLY INSTRUCTIONS (CONTINUED)

Display Assembly:

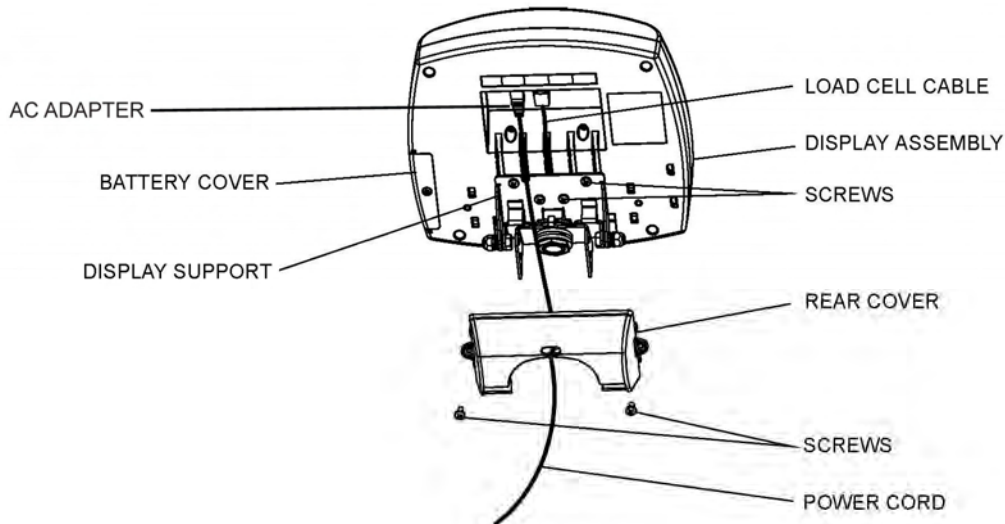
Figure 4

10. Position the Display Assembly close to the Display Support. Connect the load cell cable connector to the load cell port on the rear of the Display Assembly and insert and secure the cable into the center channel in the display. See Figure 4, for an illustration. **Note: Do not attach the power cord at this point—wait until step 13.**
11. Slide the display assembly onto the display support by inserting the two metal supports into the two slots in the rear display assembly. **Note: The load cell cable should now be secured in the channel between the display assembly and the display support.**
12. Insert the 4 screws into the display support and tighten.
13. Connect power cord to the power port in the display assembly and insert and secure the cable into the exposed portion of the second channel from the left in the back of the display. Do not plug the scale into a power source until the scale is fully assembled.
14. Insert the power cord into the retaining slot on the rear cover. Align the rear cover tabs with the slots on the rear of the display assembly. Apply pressure to snap the rear cover into place and secure with 2 screws.

ASSEMBLY INSTRUCTIONS (CONTINUED)

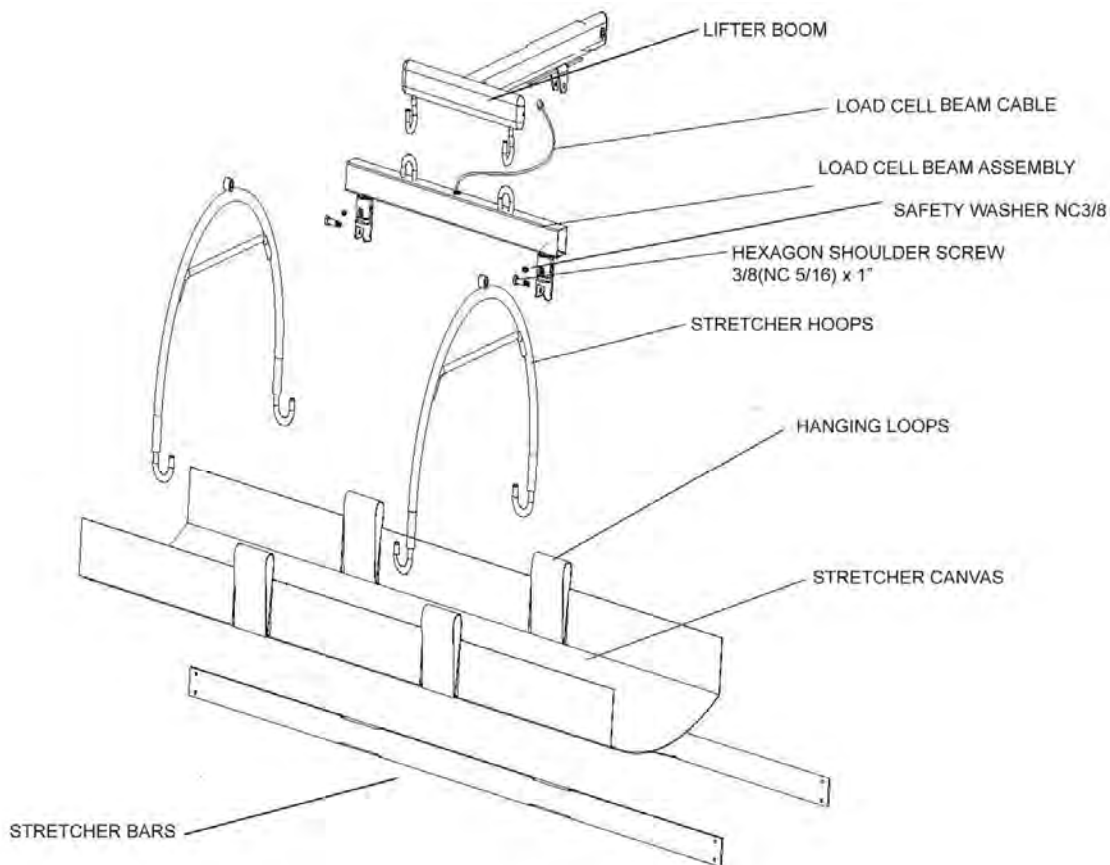
Figure 5

15. Hang the Load Cell Beam Assembly on the two hooks at the end of the Lifter Boom, using the loops on top of the Load Cell Beam Assembly (see Figure 5, above, for an illustration).
16. Connect the end of the Load Cell Beam Cable coming out of the center of the Load Cell Beam Assembly to its port on the side of the Lifter Boom.
17. Use the two washers and hexagon shoulder screws to connect the two Stretcher Hoops to the Load Cell Beam Assembly and tighten well. Make sure the stretcher hooks face in, toward the stretcher canvas, or inward toward where the patient will be.
18. Insert the stretcher bars into the stretcher channel pockets on either side of the stretcher canvas, making sure that the stretcher bars pass through the two hanging loops on each side. Make certain that both ends of the channel pockets are sealed by squeezing the Velcro strips together.

Note: The hanging loops are tethered to the stretcher fabric. Make certain that each stretcher bar passes through the portion of the hanging loops with the tether for each side. The other portion of the hanging loop will go over the hooks of the stretcher hoops.

SET UP

The Model 2000KL ProPlus® Stretcher Scale is shipped with the Menu Lock function enabled. The Menu Lock function disables certain buttons on the scale's keypad, thus preventing unauthorized users from making changes to your scale settings. The Menu Lock can be bypassed temporarily to allow you to customize your scale with the functionality you prefer. Please refer to page 7 of the ProPlus® Display Module User Manual for instructions on how to bypass the Menu Lock.

Preparing the Scale for Use

1. Remove protective plastic film from keypad and display.
2. Place batteries in the battery holder (see "Replacing Batteries" information in Figure 6, below).
3. Plug the scale's AC adapter into the power source.
4. Press the ON/OFF button to turn the scale on. The display will show "Health o meter ProPlus" and then "000Lb00oz". **Note:** To adjust the display backlight and/or contrast, please refer to page 13 of the ProPlus® Display Module User Manual.
5. Hang the stretcher on the Stretcher Hoops. The display will show "WEIGHING" and/or "REWEIGHING" until a stable weight is calculated. The weight of the stretcher will be displayed. (See page 4 on more information regarding pre-weighing the stretcher and the TARE function).
6. Press the TARE key and the scale will zero out.
7. Place a weight that is WITHIN the scale's capacity on the stretcher (see page 3 for more information). The display should read "WEIGHING" until the weight calculation is complete and the weight is displayed.
8. Press the KG/LB button to select the weight mode (pounds or kilos). To change the display mode in pounds to fractions or decimals, please refer to page 7 of the ProPlus® Display Module Manual.
9. Press the REWEIGH button; and the scale will perform the weighing process again (you do not have to remove the applied weight to repeat the weighing process).
10. Remove the weight from the stretcher. The scale will return to zero and the display will read "TARE" on the upper right side of the screen along with "000Lb00oz".
11. When you disconnect the scale's AC adapter from the power source, the scale will switch to battery power.

Note: If the set up procedure failed, refer to the troubleshooting instructions. If the problem is not corrected, refer to qualified service personnel.

Note: The scale will always default to the settings and units last used.

Replacing / Inserting Batteries

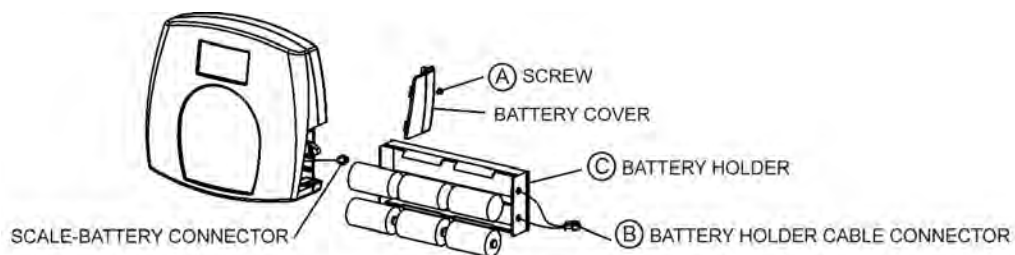
Figure 6 - Batteries

SET UP (CONTINUED)

1. Unplug the scale.
2. Remove the battery cover from the display assembly (A)--see Figure 6 on page 12.
3. Disconnect battery holder cable connector from the scale-battery connector (B).
4. Carefully remove the battery holder by sliding it out of the display assembly (C).
5. Replace the batteries with new ones. We recommend EVEREADY Energizer® e2™ batteries.
6. Carefully slide the battery holder into the display assembly.
7. Plug the battery holder cable connector to the scale-battery connector.
8. Attach the battery cover to the display assembly and install the screw.

Warning: If the scale will not be used for some time, remove batteries to avoid a safety hazard.

MAINTENANCE

GENERAL

This section provides instructions for maintenance, cleaning, troubleshooting and operator replaceable parts for the ProPlus® Stretcher Scale Model 2000KL. Maintenance operations other than those described in this section should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Inspect AC adapter for cord cracking or fraying or for broken or bent prongs.

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

Disconnect the scale from the AC power source.


1. Clean all external surfaces with a clean damp cloth or tissue. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solution.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.
4. Do not use abrasive cleaners.

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting qualified service personnel.

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	<ol style="list-style-type: none"> 1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply 	<ol style="list-style-type: none"> 1. Replace batteries 2. Use a different outlet 3. Replace adapter
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> 1. External object interfering with the scale 2. The weight of the stretcher tare was not set correctly (see p. 4) 3. The patient's center mass is not at the center of the stretcher and one of the hanging hooks of the lifter boom does not lie on the lifter beam 4. Scale is out of calibration 	<ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Take the patient and the stretcher off the scale, zero the scale, hang the stretcher on the scale and enter the tare weight. After the tare weight is set start weighing action. 3. Take the patient off the stretcher and re-position so that the center mass will lie between the two hoops. 4. Check weight with known weight value
Weighing is performed but the display shows "weigh" and "reweigh" every few seconds; the weighing process takes too long and no weight is displayed.	The patient is not still	Ask the patient to be still or change the scale to use the live weight setting (p.13 of the ProPlus® Display Module User Manual)
The display shows "Overload" message	The load on the scale exceeds the capacity (see page 3)	Remove the excess weight and use the scale according to its limits
The display shows "LOW BAT" message	The batteries are empty	Replace batteries according to instructions on page 12
The display shows "Load Cell Error" message	There is a problem with one or more load cells or the load cell cable is disconnected	Check load cell cable connection at the display and platform assembly ports (see page 10). If the problem is not corrected, refer to qualified service personnel to replace the defective load cell

EXPLODED VIEW OF 2000KL SCALE
VISTA DETALLADA DE LA BÁSCULA 2000KL
ÉCLATÉ DE LA BALANCE 2000KL

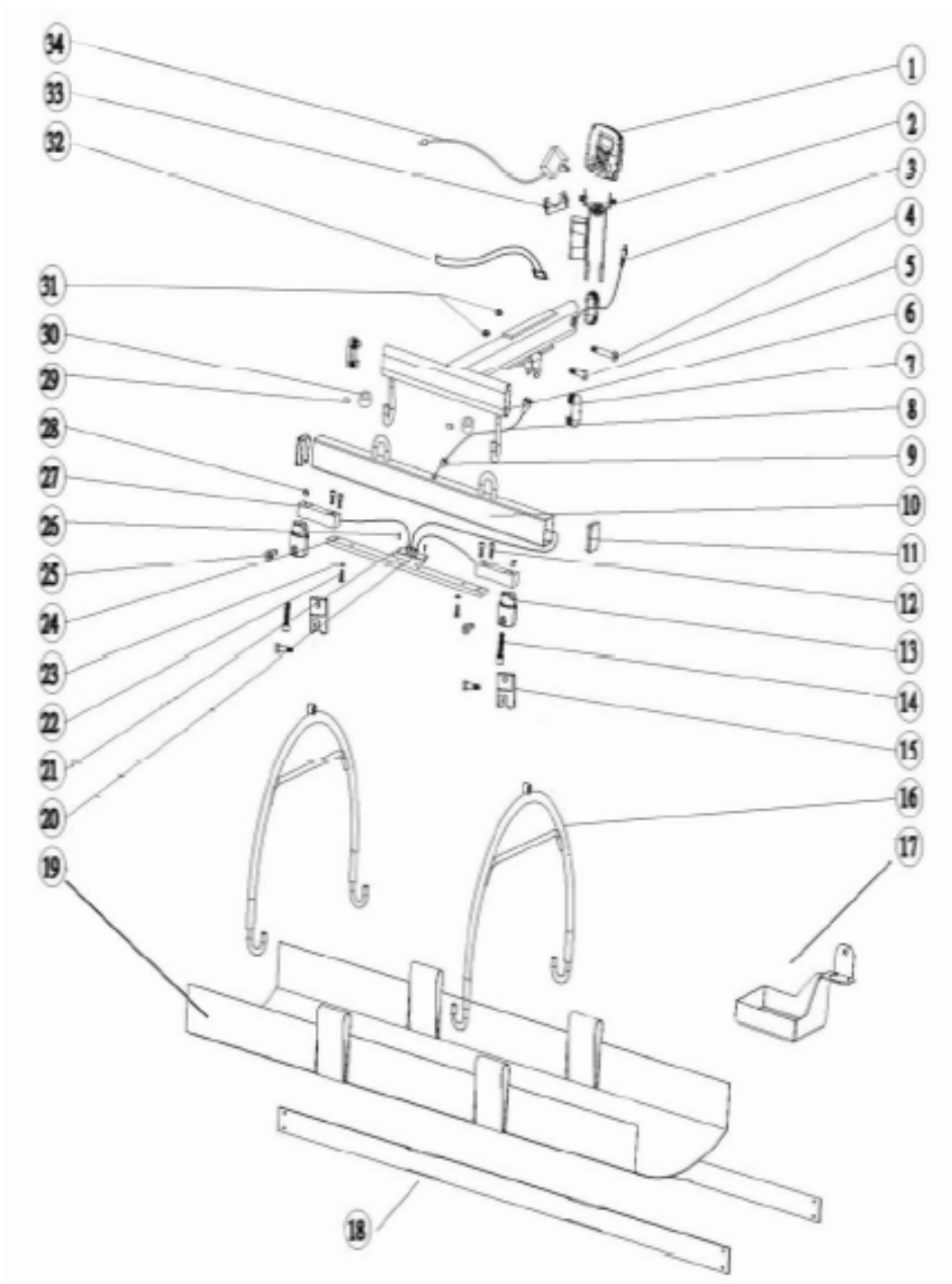2000KL PARTS LIST FOR MAINTENANCE

Key No.	Part No.	Description	Qty.
1	B1109403-0	DISPLAY ASSEMBLY	1
2		DISPLAY SUPPORT	1
3		LOAD CELL CABLE	1
4		HEXAGON HEAD SCREW NC 3/8-20 x 2-1/2	1
5		HEXAGON HEAD SCREW NC 3/8-20 x 1-3/4	1
6		LIFTER BOOM ASSEMBLY	1
7		ELLIPTICAL PIPE COVER	3
8		LOAD CELL BEAM CABLE	1
9		LOAD CELL WIRE CLIP	1
10		LOAD CELL BEAM ASSEMBLY	1
11		SQUARE PIPE COVER	2
12		HEXAGON HEAD SCREW	4
13		LOAD CELL ADAPTER	2
14		HEXAGON HEAD SCREW	2
15		FORK JOINT	2
16		STRETCHER HOOPS	2
17	B3835701-0	STRETCHER HOLDER	1
18	3866201-1	STRETCHER BAR	2
19	B401338	STRETCHER CANVAS	1
20		LOAD CELL CONNECT PCB	1
21		SCREW	2
22		HEXAGON PAN HEAD SCREW	2
23		WASHER	2
24		LOAD CELL MOUNT PLATE	1
25		HEXAGON SHOULDER SCREW 3/8 (NC511b) x 1	4
26		SCREW	6
27	B1304202-0	LOAD CELLS	2
28		WASHER	2
29		SCREW	2
30		LOCK RINGS	2
31		SELF LOCK NUT NC 3/8-20	2
32		STRETCHER RETAINING STRAP (FOR STORAGE)	1
33	B3822701-0	WIRE COVER	1
34	ADPT31	ADAPTER UNIT	1

WARRANTY

3- YEAR LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® ProPlus® scales are warranted from date of purchase against defects of materials or in workmanship for a period of three (3) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call 1 (800) 638-3722 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Repair Department
11800 South Austin Avenue, Unit B
Alsip, IL 60803

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval, prior to servicing or replacing.

**Pelstar, LLC
11800 South Austin Avenue, Unit B • Alsip, IL 60803 • USA
1-800-638-3722 or 1-708-377-0600**

**PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.homscales.com/customers/registration.aspx>**

Health o meter® Professional products are manufactured, designed and owned by Pelstar, LLC.

ProPlus® is a registered trademark of Pelstar, LLC.

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

We reserve the right to improve, enhance or modify Health o meter® Professional product features or specifications without notice.

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Health o meter® est une marque déposée de Sunbeam Products Inc., utilisée sous licence.
© Pelstar, LLC 2009