
Health o meter®

Professional

PROPLUS® 2101 SERIES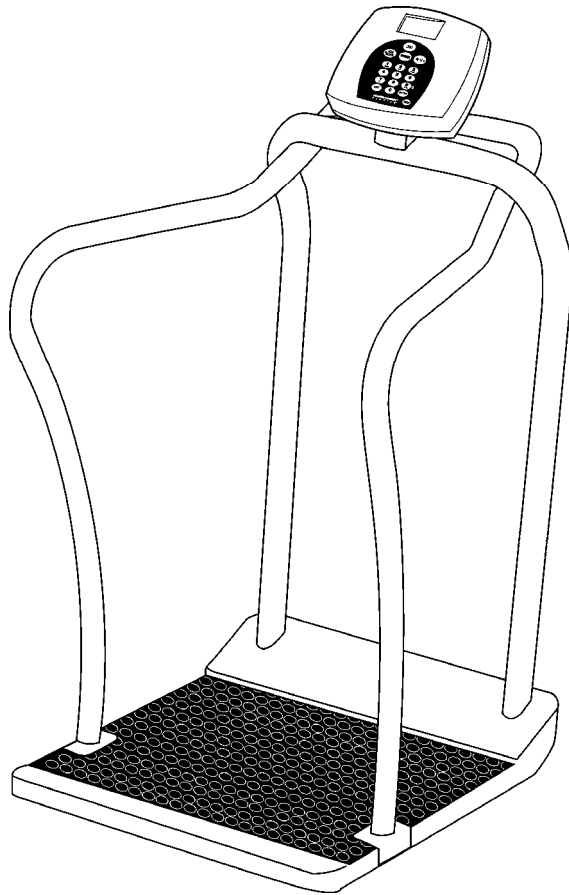

USER INSTRUCTIONS INSTRUCCIONES DE USO MODE D'EMPLOI

ISO13485:2003
CERTIFIED

Patents D523367, 7550682B2 and D508655 (US); Patents Pending and 0149984/1-8 (EU);
Patents 806658, 879866, ZL200480031711X and 200430004551.2 (PRC) © Pelstar, LLC 2010

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully and keep it for easy reference or training.

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DIGITAL BARIATRIC SCALE WITH HANDRAILS USER INSTRUCTIONS

CAUTION AND WARNING	2
SPECIFICATIONS FOR THIS SCALE	2
CERTIFICATIONS / CONNECTIVITY / DISPOSAL	3
ASSEMBLY INSTRUCTIONS	4
SET UP / BATTERIES	5
QUICK START INSTRUCTIONS	6
MAINTENANCE / CALIBRATION	8
TROUBLESHOOTING	9
DETAILED VIEW OF SCALE	10
PARTS FOR MAINTENANCE.....	11
WARRANTY	12

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.homscales.com/customers/registration.aspx>

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CAUTION AND WARNING

To prevent injury and damage to your scale, please follow these instructions very carefully.

- Do not transport the scale while the platform is loaded.
- For accurate weighing the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.
- Operating at other voltages and frequencies than specified could damage the equipment.
- If the “LOW BAT” indicator activates, for accurate weighing, replace the batteries or connect the scale to a power source, using the power adapter, as soon as possible.

SPECIFICATIONS FOR THIS SCALE

GENERAL

The ProPlus® 2101Series Digital Handrail Scale uses highly sophisticated microprocessor technology. Each precision scale is designed to provide accurate, reliable and repeatable weight measurements and features that make the weighing process simple, fast and convenient.

This scale is set up to use motion-sensing weighing technology, to determine the actual weight of a moving patient. The scale may be changed to measure live weight (see the ProPlus® Display Module User Manual for instructions on how to change the scale settings).

Weight can be displayed in pounds (decimals, fractions of a lb, lb/oz) or in kilograms. The scale features a wrap-around handrail for patient comfort and safety. It also has a 180° swivel display head for privacy and easy operator reading from either side of the scale. It has two wheels built in for easy mobility. The unit can be operated using its power adapter (included) or by 6-D cell batteries (not included).

2101 SERIES SPECIFICATIONS

Capacity and Resolution	2101KL: 1000 lb x 0.2 lb / ¼ lb / 4 oz (454 kg x 0.1 kg) 2101KG: Max 400kg / Min 1kg, e=0.1kg (Class III)	
Power Requirements	120V-240V AC, 50-60Hz - 9V DC adapter or 6 D-cell batteries	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 30°F to 125°F (0°C to 50°C) Maximum Humidity: 85% RH	
Physical Dimensions	Platform Length: 22" (56 cm) Width: 26" (66 cm) Height: 2-¼" (5.7 cm)	Assembled Product Size and Weight Length: 30" (76 cm) Width: 27-¾" (70.5 cm) Height: 49-2/5" (125 cm) Weight: 72 lb (32.5 kg)

CERTIFICATIONS / CONNECTIVITY / DISPOSAL

CERTIFICATION DESCRIPTIONS

	Scales carrying this symbol meet the Precision Class III standards of the Directive 90/384 EEC
	Products carrying this symbol meet the requirements of: 1. Directive for non-automatic weighing instruments 90/384/EEC 2. Directive governing medical devices 93/42/EEC
	Pelstar, LLC has been officially certified as a manufacture of Health o meter® Professional medical devices to be in compliance with stringent legal requirements. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.

2101 SERIES SCALE CONNECTIVITY INFORMATION

This scale is designed to connect to a computer, monitor or other devices via USB or serial ports, to allow reliable transmission of weight, height and BMI from the scale. This connectivity helps close a source of potential medical errors caused by incorrectly copying and then recording weight information taken. This scale is EMR-EHR compatible. For more information regarding how to connect this scale to devices please request Health o meter® Professional Scale Technical Support (available normal business days) at (+1) 800-815-6615 or (+1) 708-377-0600.

2101 SERIES SCALE DISPOSAL

Do not dispose of this device in domestic waste. This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal.

ASSEMBLY INSTRUCTIONS

Before Assembly:

The ProPlus® 2101 Series Digital Handrail Scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or Health o meter® Professional representative immediately. Claims must be filed with the shipper as soon as possible after receipt of the package, if there are damages. The following information details what you will find inside the main carton, as you unpack the parts for assembly.

Remove each assembly from the carton and unwrap packing material carefully, to prevent scratching the unit's parts. Set the carton aside, for storage. To avoid damaging the scale parts when unpacking it, **do not use a box cutter, knife, scissors or any sharp object to open the protective inner packaging.**

Parts List:

- (1) Base (Platform) Assembly
- (1) Center Handrail
- (2) Side Handrails
- (1) Handle
- (1) User information
- (1) Box Containing:
 - (1) Display Assembly
 - (1) Small, Long Box

The Small, Long Box Contains:

- (1) Box Containing the Power Adapter
- (4) Feet
- (1) Rear Cover for Display
- (1) Adapter Holder
- (3) Allen Wrenches—small, medium, large
- (1) Plastic Bag Containing:
 - (1) Front Cover for Display Support
 - (1) Small Rear Cover for Display Support

Tools required: Phillips head screwdrivers (not included), Allen wrenches (included).

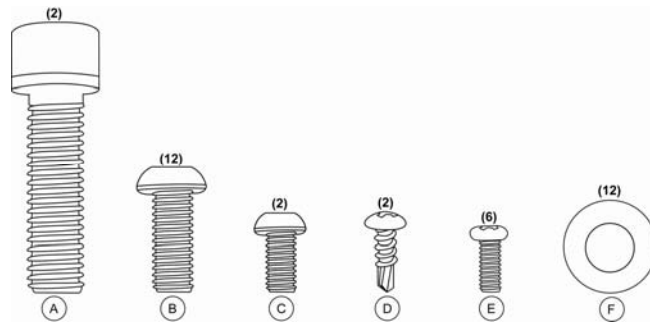
Figure 1

IMPORTANT: Please refer to the maintenance parts list on page 11 for more details about each part.

A detailed view of the scale is illustrated on page 10.

Assembling a 2101 Series Scale:

It is recommended that the assembly be done by two people.

1. Place scale base on a flat, level, dry floor.
2. Locate and pick up the center handrail, then find the load cell cable, untie it and pull it out the bottom.
3. Position the handrail over the platform and feed the cable through the handrail support in the base, being sure not to pinch the cable.
4. Slide the center handrail over the handrail supports on the base and fasten the handrail to the handrail supports using four (4) M8 1.25 x ¾ screws (B), four (4) M8 washers (F) and the medium Allen wrench (see Figure 1). Lightly tighten the screws. **DO NOT OVERTIGHTEN.** Wait to tighten until Step 6.
5. Attach the left / right side handrails to the center handrail and platform using the two (2) M10 1.5 x 1½ screws (A), eight (8) M8 1.25 x ¾ screws (B) with eight (8) M8 washers (F), and lightly tighten.
6. Starting at the top and working down the sides and base, fully tighten all 14 screws in the handrails.
7. Attach the handle to the back of the center handrail using two (2) medium M6 1.0 x ½ screws (C), and tighten.
8. Pull forward on the side handrails to tip the scale and gently place it face down on the floor.
9. Grasp the load cell cable protruding from the bottom of the base and feed it through the protective metal shield above the wheel in the base of the scale, then plug it into the load cell port in the center of the base.
10. Pull the excess cable through the top of the scale.
11. Thread the 4 feet into the underside of the base. Each foot should be completely hand tightened and then backed out 1-½ full turns. This allows for proper load cell operation for accuracy of the scale.
12. Carefully tilt the scale back to an upright position on the floor. **DO NOT DROP THE SCALE.** Adjust the feet, to level the scale. Be certain that the feet are **NOT** completely tightened into the base (see Step 11, above).

ASSEMBLY INSTRUCTIONS (CONTINUED)

Assembling a 2101 Series Scale (continued):

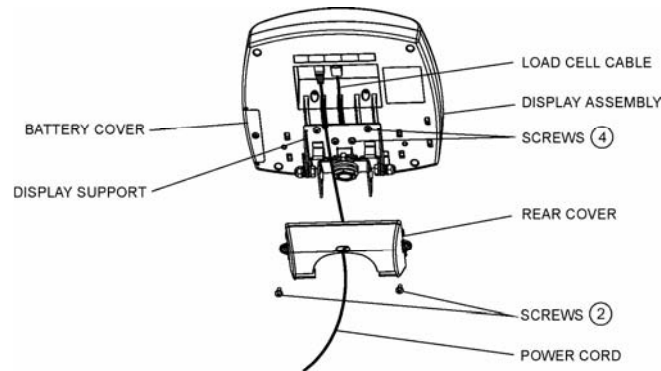
Figure 2

13. Rotate the display support 90° then position the Display Assembly close to the Display Support. Plug the load cell cable connector into the load cell port on the rear of the Display then insert and secure the cable into the center channel in the display. See Figure 2, above, for an illustration. **Note:** Do not attach the power cord at this point. Wait until Step 19.
14. Slide the display assembly onto the metal display support insuring that the two metal tabs go into the two slots in the rear of the display. **Note: The load cell cable should now be secured in the channel between the display assembly and the display support.**
15. Insert four (4) Phillips pan head M4 screws (E) into the metal display support, and tighten (see Figure 1).
16. Attach the Adapter Holder to the center rail using two (2) #8 x ½ (D) sheet metal screws, and tighten.
17. Connect the power adapter to the power port in the display, then insert and secure the cable into the exposed portion of the second channel from the left in the back of the display (see Figure 2, above). Hold the cord in place through the retaining slot on the rear cover and align the large rear cover tabs with the slots on the rear of the display assembly. Apply light pressure to snap the cover into place and secure it with two (2) Phillips pan head M4 screws (E). **Do not plug the scale into a power source until the scale is fully assembled.**
18. Rotate the display to align it with the center handrail then attach the front and small rear display support covers, applying light pressure to snap these covers into place.
19. Secure the power cord to the inside of the center handrail, using the 4 plastic cable clips, arranged to hold the cord out of the way. The adhesive backing on the clips will peel off then the clips can be fastened to the scale by applying light pressure.
20. The scale is now assembled. Plug the scale adapter into a power source and / or insert batteries.

SET UP

ProPlus® 2101Series scales are shipped with the Menu Lock function enabled. The Menu Lock function disables certain buttons on the scale's keypad, preventing unauthorized users from making changes to your scale settings. The Menu Lock can be bypassed temporarily to allow you to customize your scale with the functionality you prefer. Please refer to the ProPlus® Display Module User Manual for instructions on how to bypass the Menu Lock.

Preparing the Scale for Use

1. Remove protective plastic films covering the keypad and display.
2. Place batteries in the battery holder (see "Batteries" information in Figure 3, below).
3. Plug the scale's power adapter into the power source.
4. Be sure all objects are removed from the scale's platform, and then press the ON/OFF button to turn the scale on. The display will show "Health o meter ProPlus" and then "000Lb00oz" or "0.0KG". **Note:** To adjust the display backlight and/or contrast, please refer to the ProPlus® Display Module User Manual.
5. Press the KG/LB button to select the weight mode (pounds or kilos). To change the display mode in pounds to fractions or decimals, please refer to the ProPlus® Display Module User Manual.
Note: The scale will always default to the settings and units last used (LB or KG).
6. When you disconnect the scale's power adapter from the power source, the scale will switch to battery power, if batteries are installed. **Note:** If the set up procedure failed, refer to the troubleshooting instructions. If the problem is not corrected, refer to qualified service personnel.

SET UP / BATTERIES

Replacing / Inserting Batteries

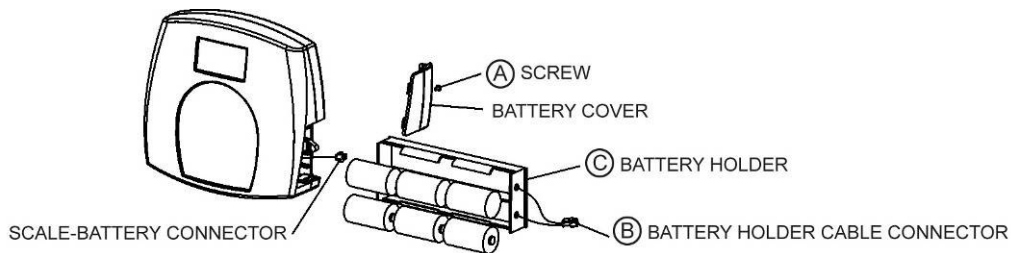
Figure 3 - Batteries

1. Unplug the scale.
2. Remove the battery cover from the display assembly—refer to Figure 3 (A) above.
3. Disconnect battery holder cable connector from the scale-battery connector (B).
4. Carefully remove the battery holder (C) by sliding it out of the display assembly.
5. Replace the batteries with new ones. We recommend EVEREADY Energizer® e2™ batteries.
6. Carefully slide the battery holder (C) into the display assembly.
7. Plug the battery holder cable connector to the scale-battery connector (B).
8. Attach the battery cover to the display assembly and install the screw (A).

Warning: If the scale will not be used for some time, remove batteries to avoid a safety hazard.

QUICK START INSTRUCTIONS

NOTE: This scale will always default to the settings and units last used (lb or kg), when it is turned on.

WEIGHING A PATIENT

1. Make sure there is no object on the weighing platform.
2. Press the **ON/OFF** button to turn the scale on.
3. Wait until "0LB 0OZ" or "0.0Kg" and "ZERO" on the left side of the display appear.
4. Ask the patient to step on the scale. The scale will calculate the patient's weight but the display will not show a weight until the stable weight of the person is determined. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.
5. Once a weight has been displayed, while the patient is still on the scale, we recommend performing a second weighing by pressing the **REWEIGH** button.
6. After the weight measurement has been taken and is recorded ask the patient to step off the scale.
7. Press the **ON / OFF** button to turn the scale off.

CALCULATING BMI

Body Mass Index is another name for Quetelet Index, a formula that is a convenient and reliable indicator of obesity. **NOTE:** The scale will not calculate BMI for a patient that is 24 pounds (12 kg) or less. If a height is not entered for the patient within 30 seconds of pressing the BMI button, the scale will return to the normal weighing mode.

1. Complete steps 1 to 4 above for "Weighing A Patient".
2. After obtaining the patient's weight on the scale, press the BMI (2) button while the scale is on.
3. The scale will prompt you to enter the patient's height. Use the keypad to enter the height.

NOTE: When weighing in pounds, the height is entered in 1/4" increments. To increase the fractional height of the patient, press "1" for 1/4", "2" for 1/2" and "3" for 3/4". When weighing in kilograms, the height must be entered in 0.1 centimeter increments.

QUICK START INSTRUCTIONS (CONTINUED)

CALCULATING BMI (CONTINUED)

4. Once height is keyed in, press **ENTER**.
5. The display will show the patient's **BMI** (Body Mass Index).
6. Press **EXIT** to return to the normal weighing mode of the scale.

TARE FUNCTION

When using the scale, the weight of an object such as shoes accompanying the patient can be subtracted from the total weight to determine the weight of the patient alone. The Tare Function automatically performs this subtraction. ProPlus® scales allow tare weight to be entered manually by using the keypad or automatically, as outlined in this section.

AUTOMATIC (PUSH BUTTON TARE)

1. With all weight off the scale, press the **ON/OFF** button to turn on the scale.
2. Wait until "0LB 0OZ" or "0.0Kg" and "ZERO" appear on the left side of the display.
3. Put the weight to be tared on the scale. The display will show a value for that weight.
4. Press the **TARE (9) button**. The scale will zero out and the word "TARE" will appear in the upper right side of the display to indicate there is a tare value for this patient.
5. Remove the weight from the scale. The display will show a negative weight equal to the weight of the shoes or other item, once that item is removed from the scale.
6. Place the patient and the tared object on the scale. The scale will automatically deduct the tared weight, as entered, from the gross weight of the patient and tared object.
7. The weight of the patient will appear on the scale.
8. The tared value is stored in memory until changed, cleared or the scale is turned off.
9. To remove the Tare, press the TARE (9) button again. The word "TARE" will disappear from the display and the tare value will be deleted from memory. The scale will resume normal operation.

MANUAL (KEYBOARD OR KEYPAD TARE)

1. Make sure there is no object on the weighing platform.
2. Press the **ON / OFF** button to turn on the scale.
3. Wait until "0LB 0OZ" or "0.0Kg" and "ZERO" appear on the left side of the display.
4. Press the **TARE (9) button**. The user will be prompted to enter the tare value.
5. Use the keypad to enter the weight of the shoes or other object and then press **ENTER**.
6. The value entered will show as a negative value on the display.
7. Place the patient and the tared object on the scale. The scale will automatically deduct the tared weight, as entered, from the gross weight of the patient and tared object.
8. The weight of the patient will appear on the scale.
9. The tared value is stored in memory until changed, cleared or the scale is turned off.
10. To remove the Tare, press the **TARE (9) button** again or **EXIT**. The word "TARE" will disappear from the display and the tare value will be deleted from memory. The scale will resume normal operation.

REMOVING THE TARE VALUE

Press the **TARE (9) button** again to delete a tare value from memory. The word "TARE" will disappear from the display and the tare value will be deleted from memory. You can also press the **EXIT** button. The scale will resume normal operation.

Additional, complete instructions regarding all of the display and keypad functions and menus can be found in the ProPlus® Display Module User Manual included with this scale.

MAINTENANCE

The following pages provide instructions for maintenance, cleaning, troubleshooting and operator replaceable parts for the ProPlus® 2101 Series. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Inspect the power adapter for cord cracking or fraying or for broken or bent prongs.

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

Disconnect the scale from the adapter power source.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solution.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.
4. Do not use abrasive cleaners.

2101 SERIES CALIBRATION

This scale has been factory calibrated and does not require calibration prior to use. For maintenance, the scale can be calibrated. Calibration of your ProPlus® scale is performed using kilos (kg) or pounds (lb), according to the units of measure used upon entering into the calibration path.


Operator Action	Display
1. Make sure the display is turned off, then press and hold the HOLD/RELEASE button while pressing the ON/OFF button.	“Enter load weight: 400.0LB “ or “200.0KG”
2. Using the ▲▼ keys and/or the numeric keypad, set the calibration load to at least half the capacity of the scale (for the best results and greater accuracy, use the full capacity of the scale).	“Enter load weight: 400.0LB “ or “200.0KG”
3. Make sure there is nothing on the scale, then press the ENTER button.	Zero calibration: Please clear the scale
4. Please wait 1-3 seconds, without touching or shaking the scale, to allow the Zero Calibration to finish.	Zero calibration: Please wait
5. No action required by operator.	Weight calibration: Put: XXX.X
6. Load the scale with the required weight and press the ENTER button. Please wait 1-3 seconds, without touching or shaking the scale, to allow the Calibration to finish.	Weight calibration Please wait
7. If calibration is successful, the display will show “Factor OK!”. If the calibration process failed, it will show “Factor Fail!”.	Factor Update... “Factor OK!” or “Factor Fail!”
8. If successful, the display will begin to reboot.	Rebooting... Please wait
9. The display will return to the start up message.	Health o meter ProPlus
10. Please wait until the scale resumes normal operation and then remove the weight from the scale.	Health o meter ProPlus

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting service personnel.

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	<ol style="list-style-type: none"> 1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply 	<ol style="list-style-type: none"> 1. Replace batteries 2. Use a different outlet 3. Replace adapter
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> 1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration 5. Improper Tare 	<ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Ask the patient to get off the scale, zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value 5. Place the item to be tared on the scale. Press REWEIGH. Once the weight of the item is displayed, press TARE. Place the patient and the tared item back on the scale. Press REWEIGH again
Weighing is performed but the display shows "weigh" and "reweigh" every few seconds; the weighing process takes too long and no weight is displayed.	The patient is not standing still	Ask the patient to stand still or you can change to live weight setting
The display shows "Overload" message	The load on the scale exceeds the capacity (1000 lb / 400 kg)	Remove the excess weight and use the scale according to its limits
The display shows "LOW BAT" message	The batteries are empty	Replace batteries according to instructions
The display shows "Load Cell Error" message	There is a problem with one or more load cells or the load cell cable is disconnected.	Check load cell cable connection at the display and platform assembly ports. If the problem is not corrected, refer to qualified service personnel to replace the defective load cell

DETAILED VIEW OF 2101 SERIES SCALE
VISTA DETALLADA DE LA BÁSCULA SERIE 2101
ÉCLATÉ DE LA BALANCE SÉRIE 2101

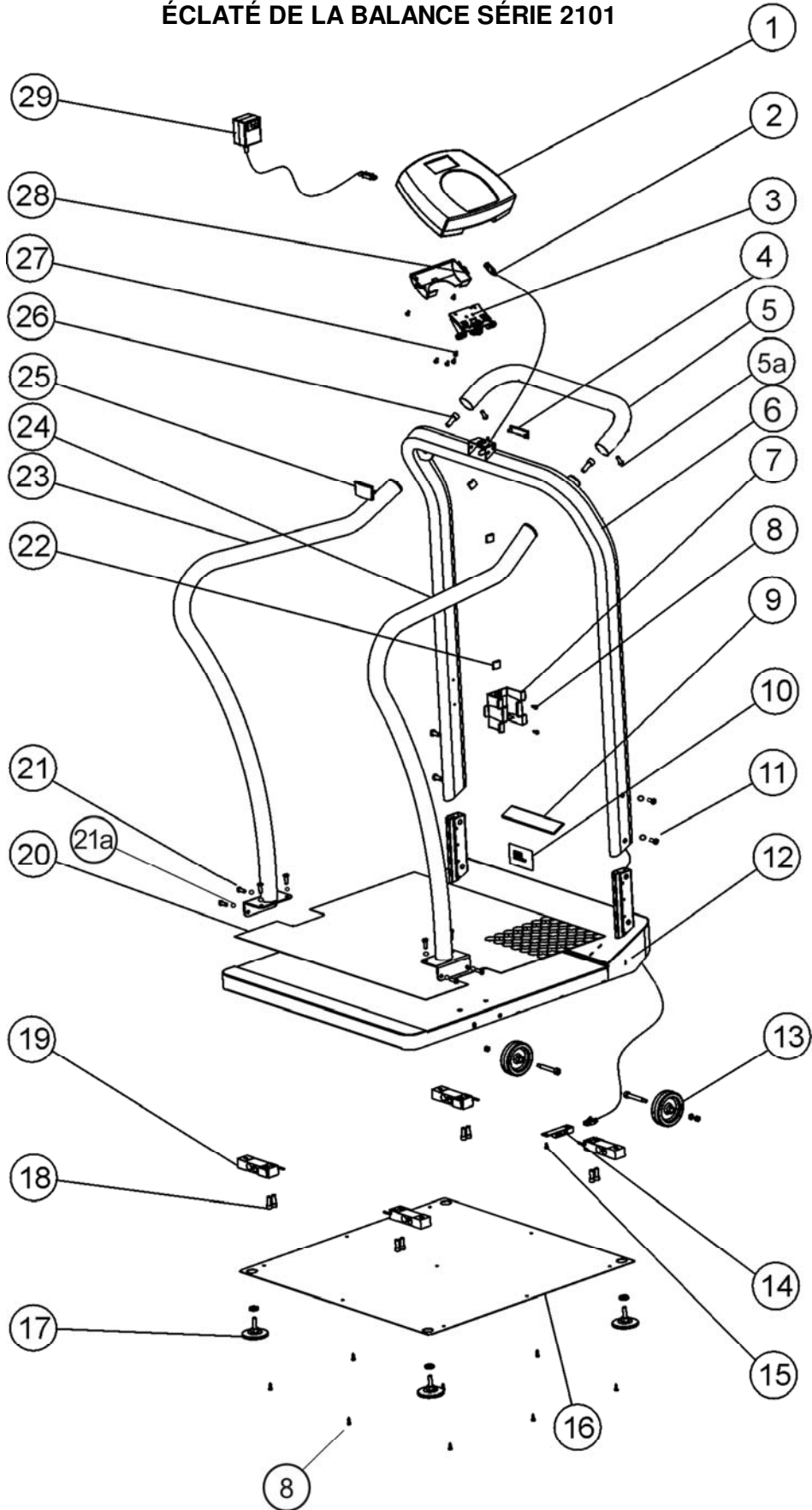2101 SERIES PARTS FOR MAINTENANCE

Key No.*	Part No.	Description	Qty.
1	B1109401-0	DISPLAY ASSEMBLY	1
2	B2033701-0	MAIN BOARD TO JUNCTION BOX LOAD CELL CABLE	1
3	B2266901-0	DISPLAY SUPPORT	1
4	B3822601-0	SMALL REAR COVER FOR DISPLAY SUPPORT	1
5	B3822901-0	HANDLE	1
5a		SOCKET HEAD CAP SCREW M6 1.0 x 1/2	2
6	B3822101-0	HANDRAIL	1
7	B3817001-0	ADAPTER HOLDER	1
8		SHEET METAL SCREW #8 x 1/2	11
9		HEALTH O METER LABEL	1
10		MODEL LABEL	1
11		SOCKET HEAD CAP SCREW M8 1.25 x 3/4	4
12	B3822001-0	BASE ASSEMBLY (PLATFORM ASSEMBLY)	1
13	B2267201-0	WHEEL ASSEMBLY	2
14	B2138901-0	CONNNECTOR BOARD	1
15		PHILLIPS PAN HEAD SCREW #6 x 1/4	1
16	B3822401-0	BOTTOM COVER	1
17	B2266701-0	PLASTIC FOOT ASSEMBLY	4
18		SOCKET HEAD CAP SCREW NC5/16-18 x 3/4	8
19	B1304202-0	LOAD CELL	4
20	B3823001-0	MAT	1
21		SOCKET HEAD CAP SCREW M8 1.25 x 3/4	12
21a		M8 WASHERS	12
22	B401305	PLASTIC CABLE CLIP WITH ADHESIVE BACK	4
23		LEFT SIDE HANDRAIL	1
24		RIGHT SIDE HANDRAIL	1
25	B3822501-0	FRONT COVER FOR DISPLAY SUPPORT	1
26		SOCKET HEAD SCREW M10 1.5 x 1 1/2	2
27		PHILLIPS PAN HEAD SCREW M4	6
28	B3822701-0	REAR COVER	1
29	ADPT30	POWER ADAPTER 120V-240V	1

* See page 10 for a detailed view of parts by number.

WARRANTY

3- YEAR LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® ProPlus® scales are warranted from date of purchase against defects of materials or in workmanship for a period of three (3) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Repair Department
11800 South Austin Avenue - B
Alsip, IL 60803

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval, prior to servicing or replacing.

Pelstar, LLC

**11800 South Austin Avenue - B • Alsip, IL 60803 • USA
1-800-638-3722 or 1-708-377-0600**

**PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.hom scales.com/customers/registration.aspx>**

ProPlus® is a registered trademark of Pelstar, LLC.
Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.
Health o meter® Professional products are manufactured, designed and owned by Pelstar, LLC.
We reserve the right to improve, enhance or modify Health o meter® Professional product features or specifications without notice. © Pelstar, LLC 2010

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Pelstar LLC
ProPlus® est une marque déposée de Pelstar, LLC. Health o meter® est une marque déposée de
Sunbeam Products Inc., utilisée sous licence. © Pelstar, LLC 2010