2595KL

**DIGITAL CHAIR SCALE
BÁSCULA ELECTRONICA DE SILLA
BALANCE ELECTRONIQUE DE SIÈGE**

**USER INSTRUCTIONS
INSTRUCCIONES DE USO
MODE D'EMPLOI**

Rev. 20100430

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully and keep it for easy reference or training.

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**PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.homscales.com/customers/registration.aspx>**

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CAUTION AND WARNING

- To prevent injury, do not transport the scale while the chair is loaded.
- For accurate weighing the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.
- Operating at other voltages and frequencies than specified could damage the equipment.
- If the “LO BAT” indicator activates, for accurate weighing, replace the batteries or connect the scale to an AC power source as soon as possible.

2595KL SPECIFICATIONS

GENERAL

The Health o meter® Professional 2595KL Electronic Chair Scale uses highly sophisticated microprocessor technology. This precision instrument is designed to provide accurate, reliable and repeatable weight measurements by providing features that make the weighing process simple, fast and convenient. The weighing is performed by motion-sensing weighing technology, to determine the actual weight of a moving patient. Weight can be displayed in pounds or in kilograms. The scale includes a padded seat with arm supports that can be folded up for easy entry/exit. Also included in the scale are two large wheels and a handrail for easy mobility. The unit can be operated using its power adapter or with 6 C-cell batteries.

2595KL SCALE SPECIFICATIONS

Capacity and Resolution	600 lb x 0.2 lb (270 kg x 0.1 kg)	
Power Requirements	Adapter 120V-240V AC, 50-60Hz – 9V DC (included) or 6 C-cell batteries (not included)	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 30°F to 125°F (0°C to 50°C) Humidity: 20%-95% RH	
Physical Dimensions	Seat Depth: 15-1/2" (39.4 cm) Width: 18-1/2" (47 cm) Arms Span: 19-1/8" (48.6 cm)	Entire Scale Length: 35" (89 cm) Width: 23-5/8" (60 cm) Height: 40-1/2" (102.9 cm) Weight: 80lb, 4oz (36.5kg)

2595KL SCALE CONNECTIVITY INFORMATION:

This scale is designed to connect to other devices via a USB port, to allow acquisition of weight directly from the scale, to help close a source of potential medical errors caused by incorrectly copying and then recording a weight taken. This scale is EMR-EHR compatible. For more information regarding how to connect this scale to your computer, a monitor, printer or other device please call Health o meter® Professional Technical Support at 800-638-3722.

2595KL ASSEMBLY INSTRUCTIONS

Parts List:

- (1) Chair Scale Assembly
- (2) Wheels
- (2) Feet
- (2) Wheel Screws
- (1) Power Adapter

Remove each assembly from its carton and unwrap packing material carefully to prevent scratching the parts.

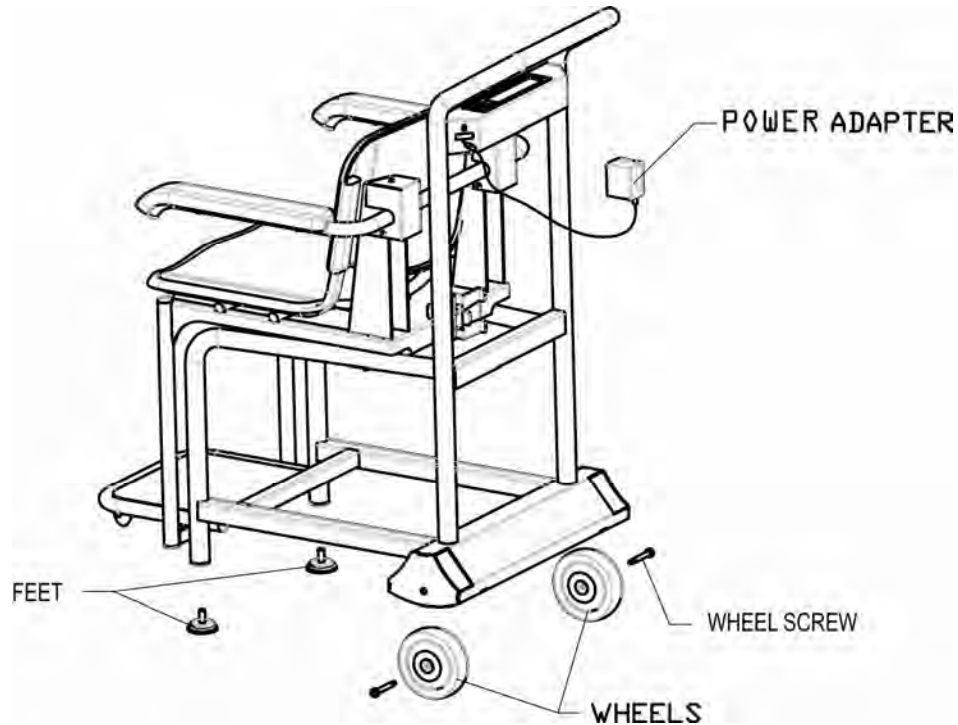
Figure 1. Assembly of the 2595KL

1. Place the chair scale assembly upside down on a level floor.
2. Unscrew 2 screws from both wheel supports. Attach the wheels to both sides of the chair assembly and replace the screws (use 3/16" or 5mm Allen wrench).
3. Screw the two plastic feet onto the chair assembly.
4. Turn the chair scale assembly right side up and adjust the feet for equal stabilization of the chair assembly.

NOTE: The scale may be operated by a power adapter (included) or by 6 C-cell batteries (not included). The scale will automatically switch to battery power when a power adapter is absent or not plugged in, if there are batteries in the scale.

SET UP / BATTERIES

1. Place batteries in the battery holder (see “Replacing Batteries” below) and then plug the scale’s power adapter into the chair scale assembly and into the power source.
2. Press the ON/OFF button to turn the scale on. The display will show “start”, then “0.0”.
3. When the self-test function is complete:
 - The digital display should read “0.0”.
 - “LB” should be indicated on the right side of the display. (To change the default units of the scale to kilograms (kg), please contact Health o meter® Professional Technical Support at 1.800.638.3722 or 1.708.598.9100).
4. Place a weight [not to exceed 600 Lb (270 kg)] on the scale. The digital display should blink “-----” until the weight calculation is complete.
5. Press the LB/KG button to select the weight mode, either in kilograms or pounds.
6. Press the REWEIGH button; the digital display should again read the weight of the item on the scale. Pressing the REWEIGH button will reweigh the patient without removing them from the scale.
7. Remove the weight from the scale. The digital display should continue to show the weight of the item that was on the scale.
8. Press the ZERO button; the digital display will return to zero.
9. Press the ON/OFF button to turn the scale off. The display will show “OFF” and then turn the scale off.
10. Disconnect the power source adapter from the scale, and then switch on the scale using the ON/OFF button. The scale should automatically change over to battery power.

NOTE: If the set up procedure failed, go to the troubleshooting instructions. If the problem is still not corrected, refer to qualified service personnel.

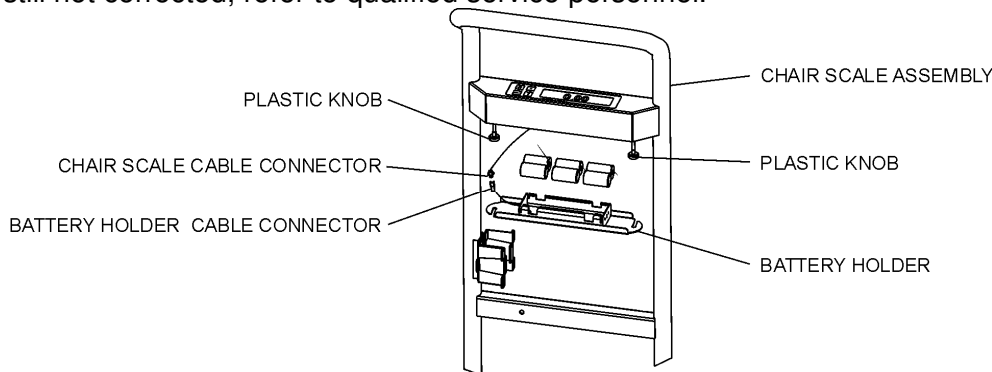
Figure 2. Replacing Batteries

Replacing Batteries

1. **Disconnect the scale from the adapter power source.**
2. Loosen two knobs found underneath the display until the battery holder is released from the chair scale assembly and rests on the knobs. Carefully pull out the cable to provide access to the connector. Disconnect the battery holder cable connector from the chair scale cable connector by pushing the button.
3. Raise the battery holder and simultaneously rotate it counterclockwise 1/4 turn. Carefully remove the holder from the chair scale assembly.
4. Replace the batteries with new ones.
5. Place the battery holder on the plastic knobs by rotating it 1/4 turn clockwise.
6. Plug the battery holder cable connector to the chair scale cable connector.
7. Hold the battery holder close to the chair scale assembly and tighten the two plastic knobs to secure in place. Be careful not to damage the cable.

SET UP

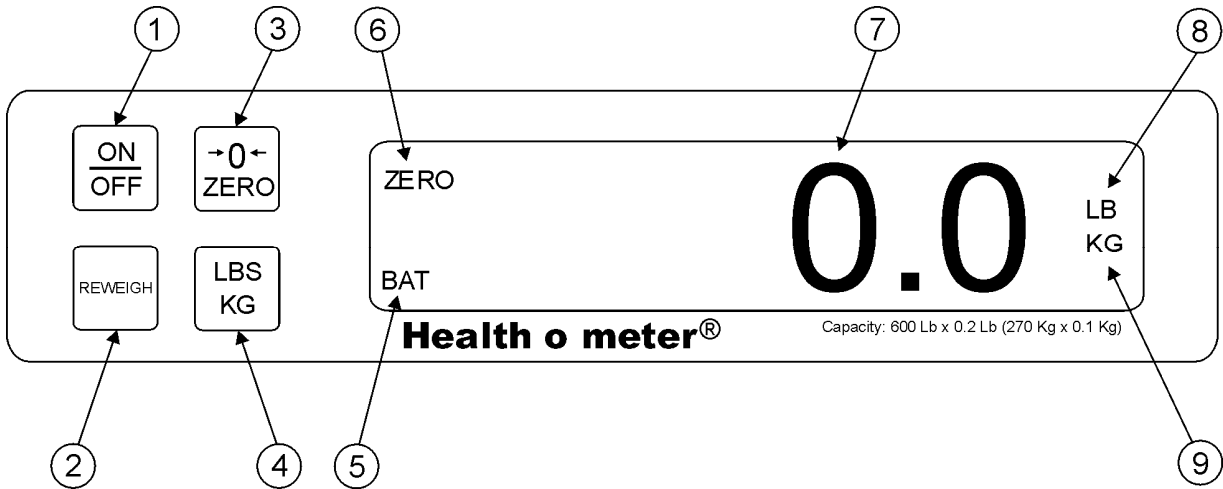
Figure 3. Control Panel

ITEM	DESCRIPTION	FUNCTION
1	ON/OFF button	Turns scale ON and OFF.
2	REWEIGH button	Allows repeated reweighing of person.
3	ZERO button	Zeros Digital Display prior to weighing. Enables the cancellation (tare) of weight if some object was placed on the scale.
4	LBS/KG button	Selects kilograms or pounds.
5	BAT indicator	When ON, indicates scale batteries are low. The batteries should be replaced.
6	ZERO indicator	When ON, indicates the scale is stable and ready for use.
7	Digital Display 6 Digit LCD	Displays weight in kilograms or pounds as shown by the “LB” or “KG” indicator on the right side of the display. When REWEIGH button is depressed, display will blink “-----” until patient’s weight is determined.
8	LB indicator	When ON, indicates displaying weight in pounds.
9	KG indicator	When ON, indicates displaying weight in kilograms.

KG LOCKOUT

The default weighing mode for this scale can be changed from LB/KG to KG only. To change to KG, turn the scale on and wait for the display to show “0.0”, then press and hold the LBS/KG button for 20 seconds until the scale beeps 6 times. The scale is now locked in the KG mode and the user cannot toggle between LB and KG. To disable the KG Lockout feature, press and hold the LBS/KG button for another 20 seconds until the scale beeps 6 times, indicating that the user can again toggle between pounds and kilograms.

QUICK START / USER INSTRUCTIONS

1. Press the ON/OFF button to turn the scale on.
2. Wait until "0.0" appears on the digital display.
3. Sit the patient on the scale. The display will blink "-----" until the weight of the person is determined.
4. If you wish to reweigh without asking the patient to get off and to sit on the scale again, press the REWEIGH button.
5. Ask the patient to get off the scale. The digital display should continue to read the person's weight until REWEIGH or ZERO button is pressed.

NOTE: After 2 minutes of inactivity, the scale will turn itself off to conserve power. To restart the scale, press the ON/OFF button.

MAINTENANCE

GENERAL

This section provides instructions for maintenance, cleaning, troubleshooting and operator replaceable parts for the 2595KL Electronic Chair Scale. Maintenance operations other than those described in this section should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Inspect the power adapter for cord cracking or fraying or for broken or bent prongs.

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

Disconnect the scale from the adapter power source.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the chair scale or arm support into cleaning or other liquid solution.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.
4. Do not use abrasive cleaners.


TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting service personnel.


SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	<ol style="list-style-type: none"> 1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply 	<ol style="list-style-type: none"> 1. Replace batteries 2. Use a different outlet 3. Replace adapter
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> 1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration 	<ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Ask the patient to get off the scale, zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value
The display shows "OL"	The load on the scale exceeds the capacity	Remove the excess weight and use the scale according to its limits
The display shows "UL"	The scale is in under-load condition	Make sure all seat parts and handles are installed in place and fixed properly
The display shows "LO BAT"	The battery is low	Replace batteries according to instructions

CALIBRATION PATH

This scale has been factory calibrated and does not require calibration prior to use. For maintenance, the scale can be calibrated. The modes are accessible by holding certain keys pressed after power-up. The keys must be pressed and held for at least three seconds.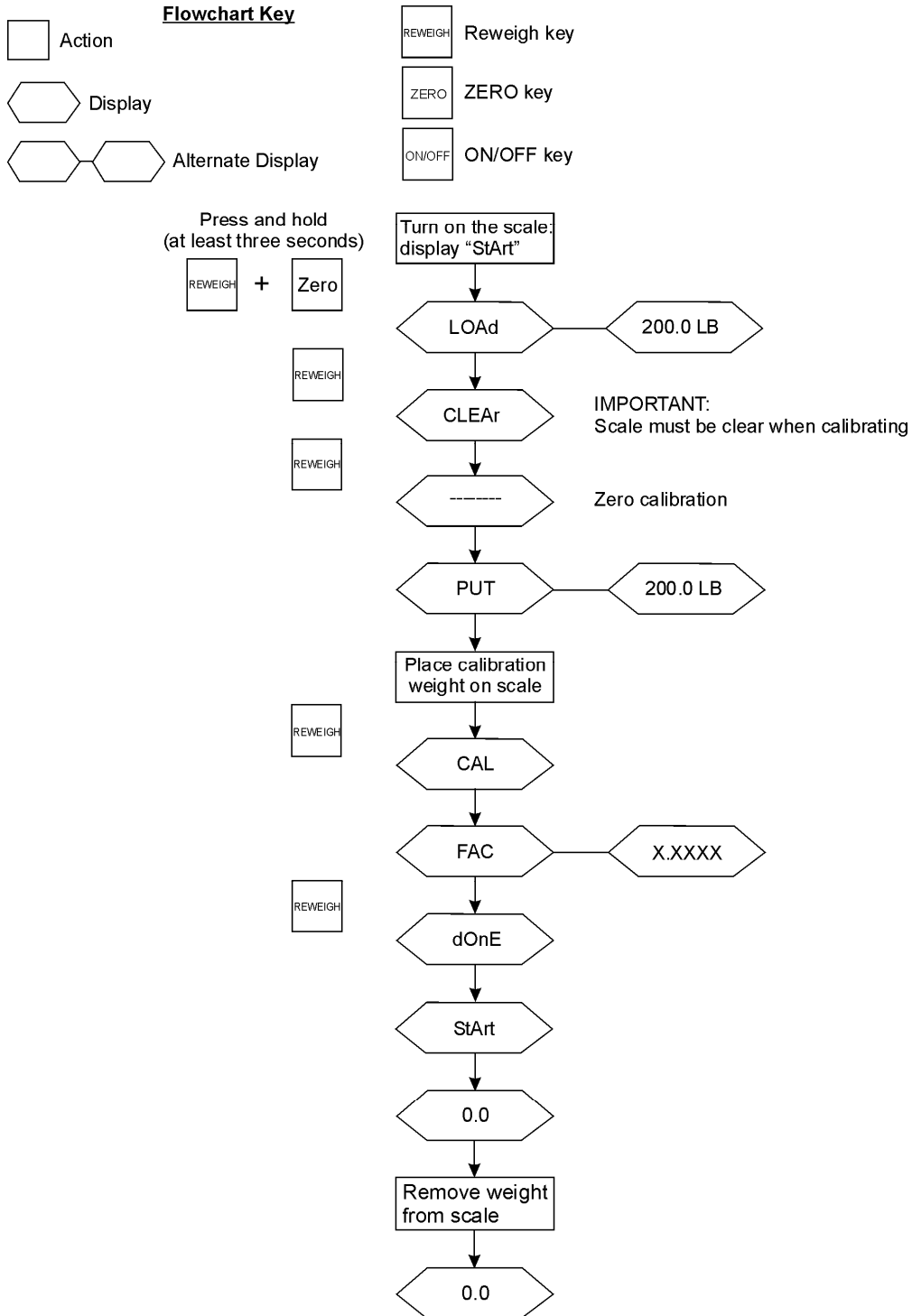

EXPLODED VIEW OF 2595KL SCALE
VISTA DETALLADA DE LA BÁSCULA 2595KL
ÉCLATÉ DE LA BALANCE 2595KL

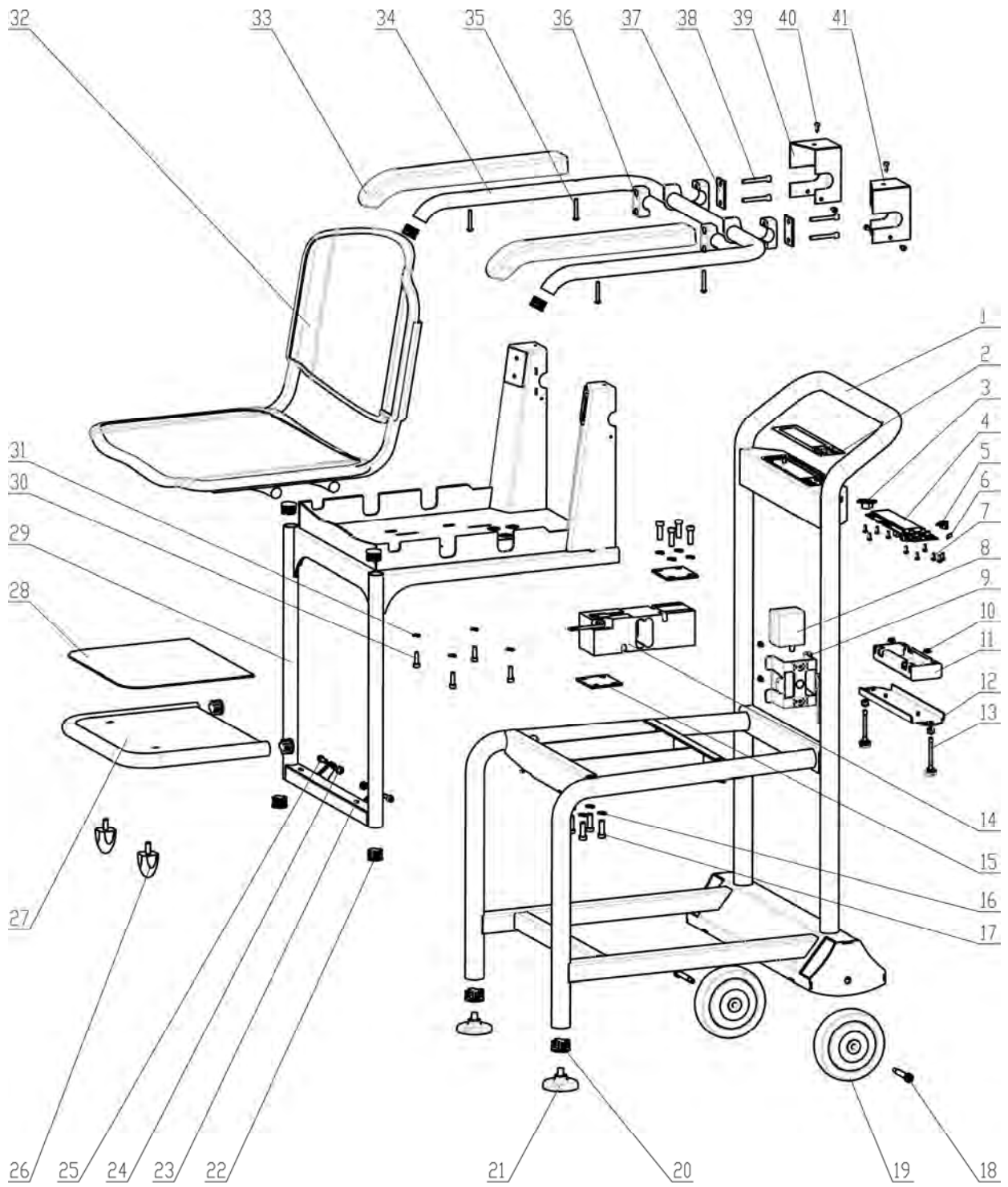2595KL PARTS LIST FOR MAINTENANCE

Key No.*	Part No.	Description	Qty.
1		CHAIR BASE	1
2		BUTTON LABEL	1
3		USB SOCKET PCB	1
4	B2138803-0	PCB ASM	1
5	B2033502-0	ADAPTER SOCKET PCB	1
6	B3245101-0	ADAPTER SOCKET LABEL	1
7		SCREW	10
8	ADPT30	ADAPTER UNIT 120V-240V AC, 50-60HZ - 9VDC	1
9	B3817001-0	ADAPTER BRACKET	1
10		FLANGE NUT	2
11	B2032801-0	BATTERY BOX ASM	1
12	B3821601-0	PCB COVER	1
13		SCREW	2
14	B411046	SENSOR	1
15	B3821501-0	SENSOR MAT PLATE	2
16		SPRING WASHER	8
17		BOLT, M8X25	8
18	B2266801-0	LOCATION BOLT	2
19	B2266801-0	WHEEL	2
20	B401280	PIPE PLUG 1#	2
21	B3810002-0	FOOT 1#	2
22	B401289	PIPE PLUG 2#	8
23		MAT	2
24		SELF-LOCK NUT	4
25		BOLT, M6X20	2
26	B3438901-2	FOOT 2#	2
27	B3819601-0	STEP ASM	1
28	B3819701-0	FOOT MAT	1
29	B3821301-0	CHAIR FRAME	1
30		SCREW, M6X25	4
31		WASHER	4
32	B401296	SEAT	1
33	B401294	HANDLE	2
34	B3820001-0	ENCLOSURE ASM	1
35		SCREW, M5X35	4
36	B401281	PIPE GRIPPER	4
37	B401281	PIPE GRIPPER MAT PLATE	2
38		BOLT, M6X50	4
39	B3819101-0	PIPE GRIPPER BOX 1#	1
40		SCREW, Φ 4.2X13	8
41	B3819001-0	PIPE GRIPPER BOX 2#	1

* See page 9 for a detailed (exploded) view of parts by number.

WARRANTY

3- YEAR LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® ProPlus® scales are warranted from date of purchase against defects of materials or in workmanship for a period of three (3) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call 1 (800) 638-3722 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Repair Department
11800 South Austin Avenue, Unit B
Alsip, IL 60803

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval, prior to servicing or replacing.

Pelstar, LLC
11800 South Austin Avenue - Unit B • Alsip, IL 60803 • USA
1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:
<http://www.hom scales.com/customers/registration.aspx>

Health o meter® Professional products are manufactured, designed and owned by Pelstar, LLC.
ProPlus® is a registered trademark of Pelstar, LLC.

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.
We reserve the right to improve, enhance or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2009
Health o meter® est une marque déposée de Sunbeam Products Inc., utilisée sous licence.