Health o meter[®]

P r o f e s s i o n a l

ASSEMBLY & OPERATING INSTRUCTIONS
INSTRUCCIONES DE INSTALACIÓN Y OPERACIÓN
INSTRUCTIONS DE MONTAGE ET MODE D'EMPLOI

CHAIR SCALE
MODEL: 445KL

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BALANZA CON SILLA
MODELO: 445KL

Assembly & Operating Instructions

Your scale is a precision instrument and should be handled accordingly. The Health o meter Professional Chair Scale is shipped completely assembled, so no additional assembly is required.

Some protective cardboard and plastic will need to be removed before using.

1. Remove metal bands attached to small skid
2. Remove cardboard at back and front end of chair
3. Remove protective foam and plastic sleeve over beam

Weighing Instructions

1. Make sure the scale is on a hard, level surface before using
2. Before placing patient on scale, slide small and large weight poises to zero, if necessary adjust scale to zero by rotating the zero adjustment screw left or right.
3. The model 445KL is designed to provide convenient and accurate weighing to the less mobile. The sidearms and footrest are movable for easy access and patient comfort (see Fig 1).

Safety Reminders

1. Do not leave patient unattended in the chair
2. Do not use chair scale in place of a wheelchair to transport patients
3. Do not bounce scale down stairs, as doing so may damage internal parts.

Cleaning

Wipe chair and scale surface with a mild soap and a soft, damp cloth.
DO NOT USE ABRASIVES.

Fig.1

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LIMITED WARRANTY

What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If scale fails to function properly, return the product, freight prepaid and properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service

Make sure you keep your sales receipt

or document showing proof of purchase.

Call 1 (800) 638-3722 or 1 (708) 377-0600 to receive a return authorization number. Attach proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar LLC
Attention R/A# _____
Repair Department
11800 S Austin Avenue.
Alsip, IL. 60803

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval prior to servicing or replacing.

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Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.
PELSTAR, LLC
Health o meter® es marca registrada de Sunbeam Products, Inc. usada bajo licencia.
Health o meter® est une marque déposée de Sunbeam Products, Inc.,. Utilisée sous licence.