## LIMITED WARRANTY

### What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

### Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

### What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

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Contactez le Service Garanties au 1-800-638-3722.

Another innovation from the people at

**Health o meter**<sup>®</sup>  
Professional

7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615

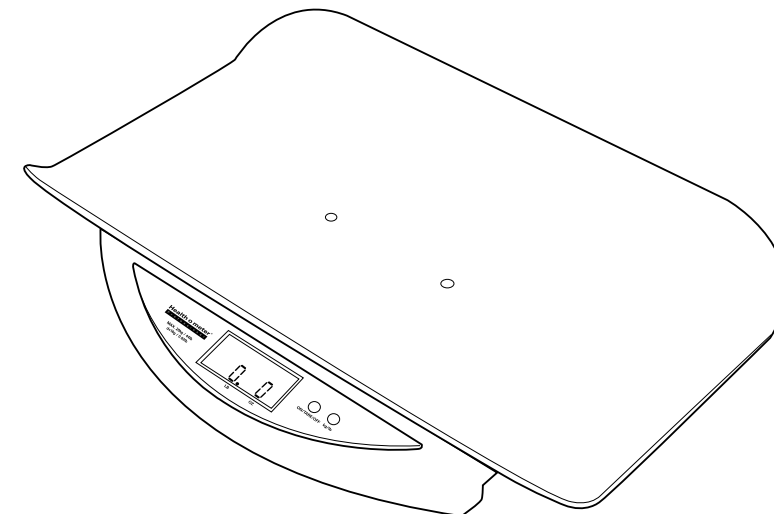
www.healthometermedical.com

Health o meter<sup>®</sup> Professional products are manufactured, designed and owned by Pelstar LLC.  
Health o meter<sup>®</sup> is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

**Health o meter**<sup>®</sup>

Professional

OPERATING INSTRUCTIONS  
INSTRUCTIONS D'UTILISATION  
INSTRUCCIONES DE OPERACIÓN

## DIGITAL PEDIATRIC SCALE

MODEL: 549KL

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MODÈLE: 549KL

## BALANZA PEDIÁTRICA DIGITAL

MODELO: 549KL

44 lb x 1/2 oz  
20kg x 0.01 kg

# DIGITAL PEDIATRIC SCALE MODEL: 549KL

## IMPORTANT

- To avoid injury, ensure that the tray is locked into place on the scale before use. Do not use the scale if the tray is not locked firmly in place.
- To prevent falls or injury, place the scale securely far from the edge of the counter or table that is used to hold the scale.

## USING THE SCALE

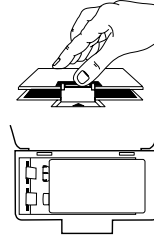
- 1: Install a new 9 volt alkaline battery (included) into the battery compartment in the back of the scale base.
- 2: Place the scale on a hard, flat surface for safety and accuracy.
- 3: Press the ON/TARE/OFF button to start the scale. 88.88 and then 0.0 will appear on the display, along with LB to the right of the display. To change the units to KG, press the LB/KG button once.
- 4: Lay the baby on the scale so that its weight is evenly distributed on the tray.
- 5: The baby's weight is displayed. Depending upon the units selected, the weight is displayed in pounds and ounces (LB) or in decimals (KG).
- 6: After flashing three times, the weight reading will remain on the display until the baby is removed.
- 7: If the display shows "Err" you have exceeded the capacity of the scale.

## TO TURN OFF THE SCALE

- 1: The scale will shut off automatically after 2 minutes of inactivity.
- 2: To manually turn off the scale, press and hold the ON/TARE/OFF key for about 3 seconds. The display will read "OFF" before turning off.

## INSTALLING BATTERY Use only 9V alkaline battery.

- 1: The battery compartment is located in the back of the scale base.
- 2: Place a new 9V alkaline battery into battery compartment. Make sure the '+' and '-' are positioned correctly, as shown on the bottom of the battery compartment.
- 3: Replace battery cover.

## Replace the battery when the display reads 'Lo'.

## TARE FUNCTION

The Tare Function allows you to place other items on the scale while weighing the baby without affecting the baby's weight reading.

- 1: Weigh the item you would like to tare.
- 2: Press and release the ON/TARE/OFF key to change the weight to 0.
- 3: Place the baby on the scale's tray and weigh normally.

**NOTE:** If the scale is empty and the display does not read 0, press the ON/TARE/OFF key once to return the display to zero.

## MAINTENANCE

- 1: To clean the scale, wipe the outside of the scale with a soft cloth and mild detergent. Do not use abrasive cleaners. Never place the scale in water or spray with liquid.
- 2: This scale contains electronic parts. Use it carefully. Do not drop, kick or jump on it.
- 3: Do not store scale near chemicals, heaters or radiators.
- 4: Do not store scale on its side.
- 5: This scale is an extremely sensitive weight device. Items such as clothing or towels placed or dropped on the scale may cause it to activate. In order to prevent run down of the battery, do not place anything on the scale.

## ONE YEAR LIMITED WARRANTY See back for details.

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