

Battery function - 2 lithium
Fonction pile : 2 piles lithium
Batteriefunktion - 2 Lithium
Batterijfunctie - 2 lithiumbatterijen
Batterifunktion - 2 litium
Batterifunktion - 2 litium
Funcione batteria - 2 batterie al litio
Función de pilas de litio
Funciona a pilhas - 2 pilhas de lítio
Pariston toiminta - 2 lithium paristoa.
Λειτουργία μπαταρίας - 2 μπαταρίες λιθίου
Pil işlevi - 2 lityum piller



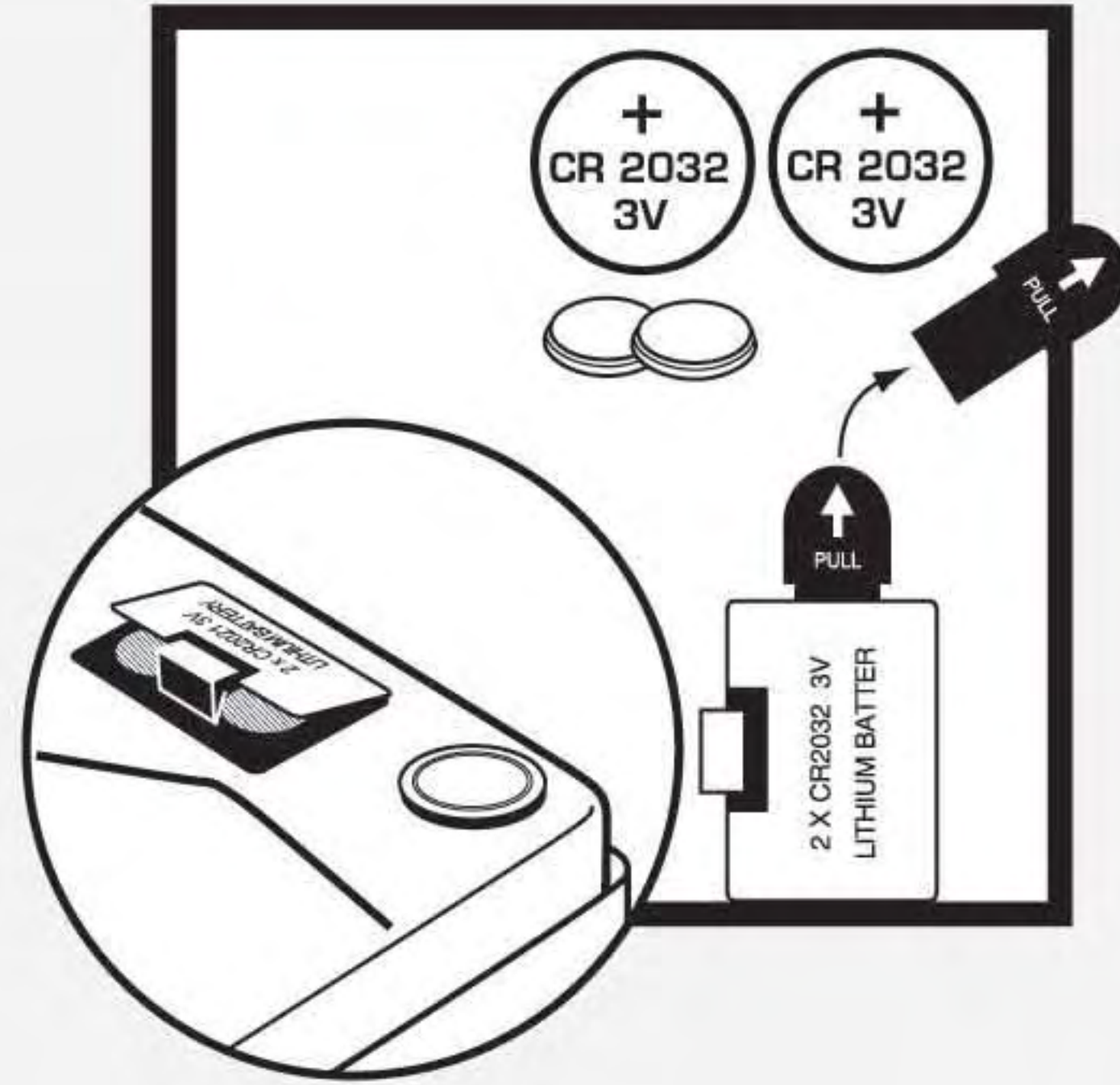
Remove battery protection
 Enlever la languette de protection des piles
 Bitte Isolierstreifen entfernen
 Verwijder de batterijbeveiliging
 Ta bort batteriskyddet
 Fjern batteribeskyttelsen
 Rimuovete la protezione della batteria
 Retire la protección de las pilas
 Retire a protecção da pilha
 Poista pariston suoja
 Αφαιρέστε την προστασία μπαταρίας
 Pil koruyucusunu çikart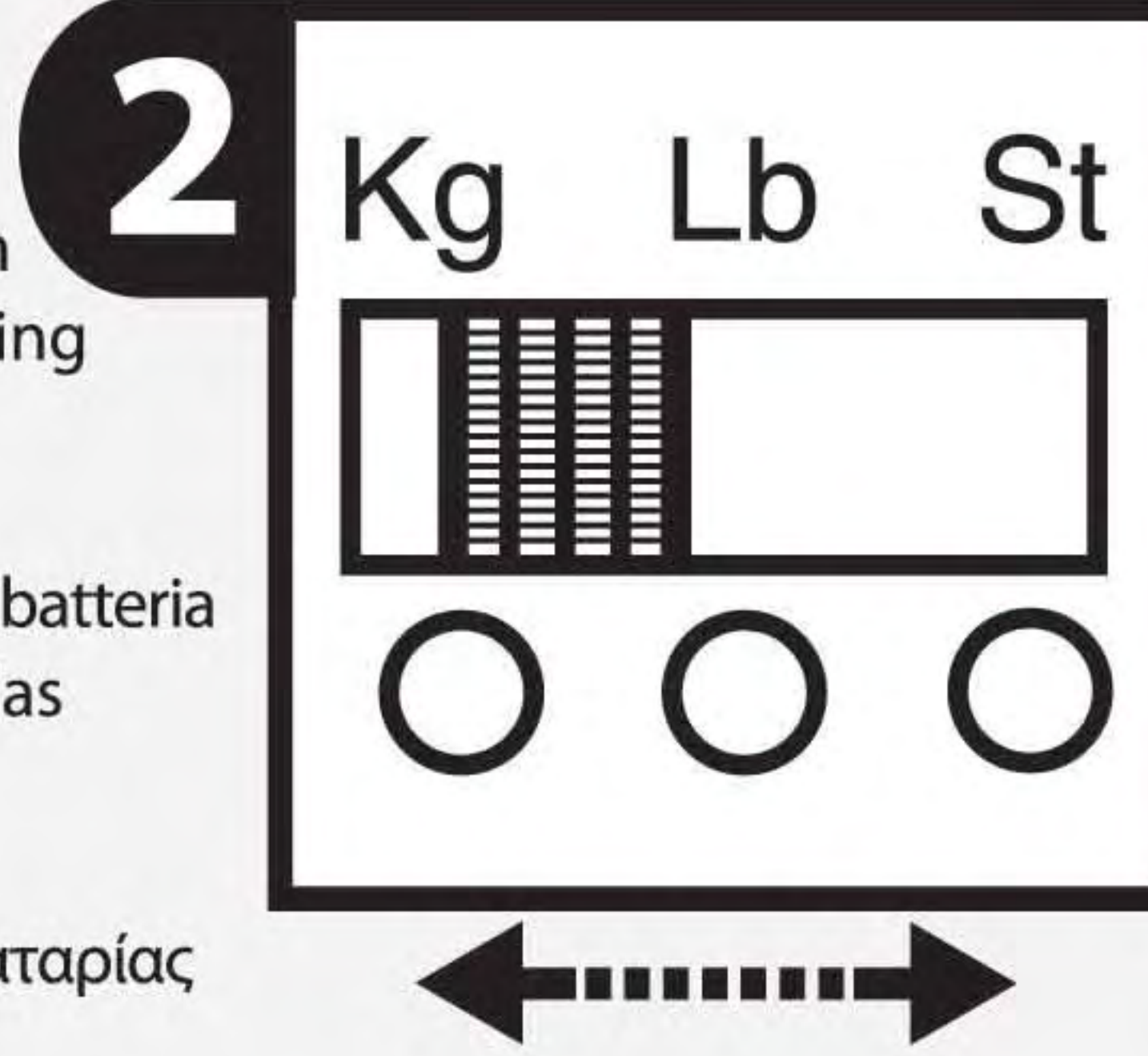




KG or LB or ST
 KG ou LB ou ST
 KG oder LB oder ST
 KG of LB of ST
 KG eller LB eller ST
 KG eller LB eller ST
 KG o LB o ST
 KG o LB o ST
 KG ou LB ou ST
 KG tai LB tai ST
 KG ή LB ή ST
 KG ya da LB ya da ST

To start - Tap on
 Mise en route - Appuyer
 Start - Tap-on
 Inschakelen - Aantikken
 Sätt igång - Tryck till
 Sådan tænder du - Tryk på
 Accensione - "Tap-on"
 Activar - Golpear báscula
 Pisar-para-ligar
 Kevyt polkaisu
 Για ενεργοποίηση - Ελαφρύ χτύπημα
 Başlama - Bas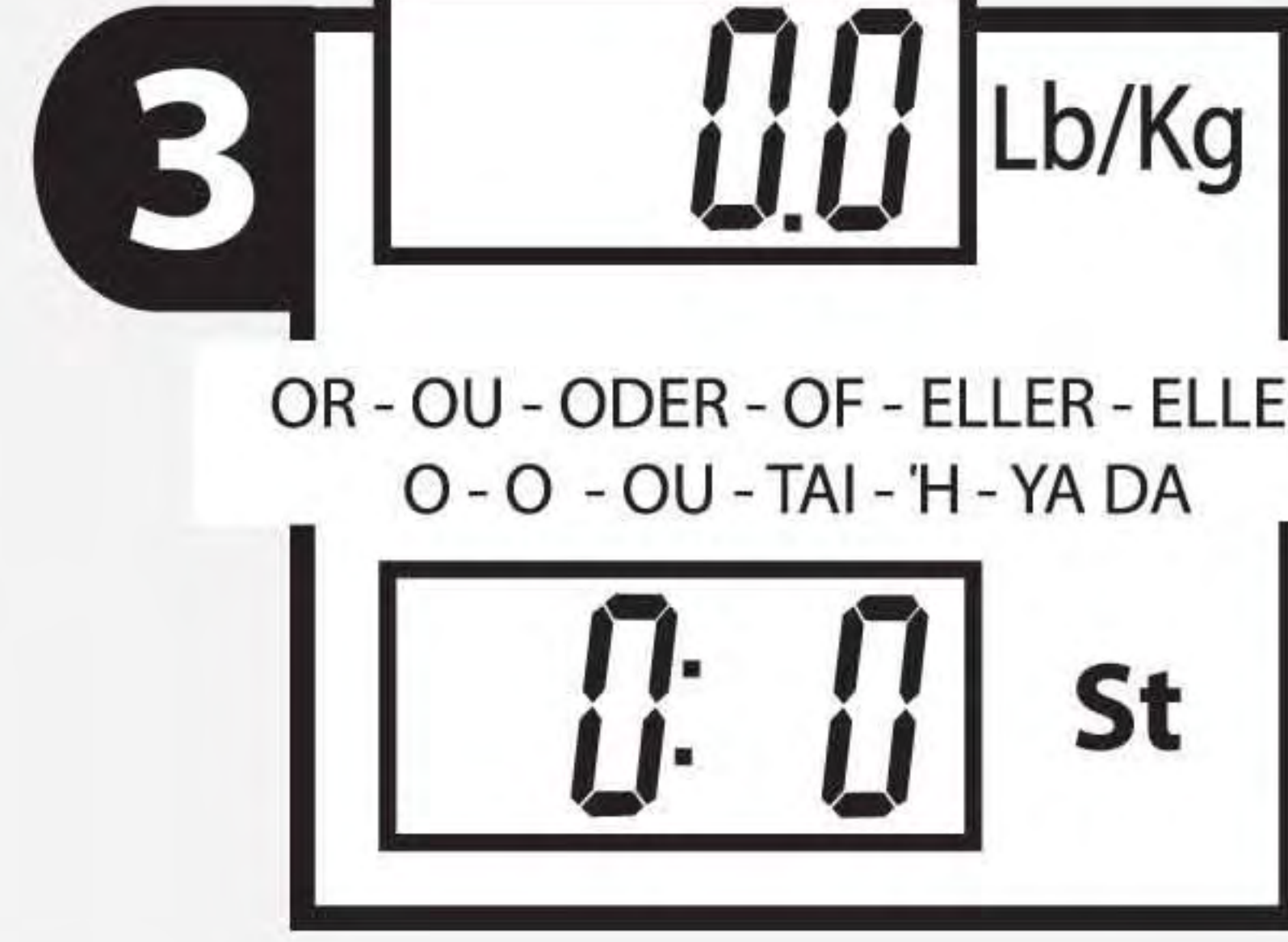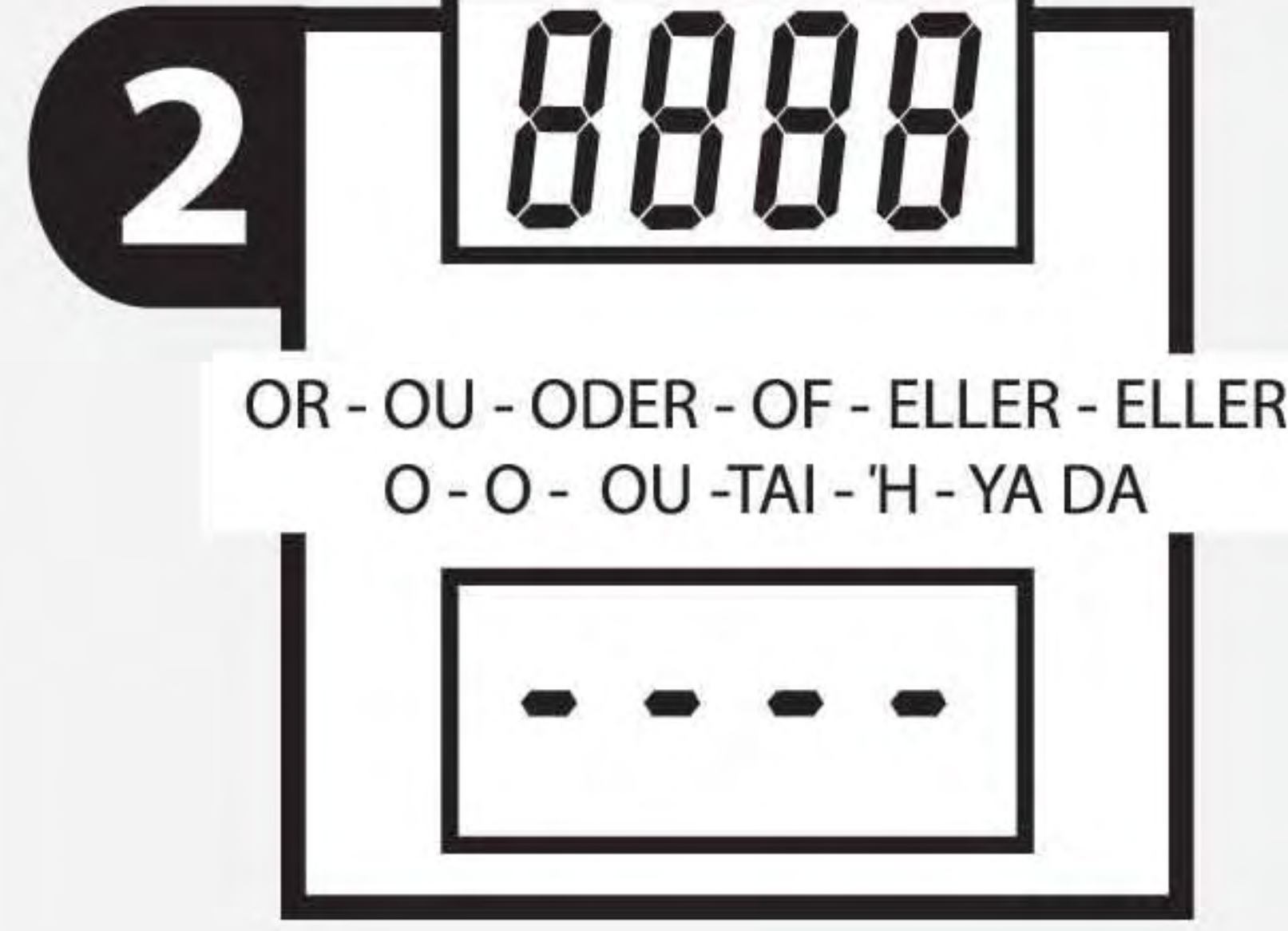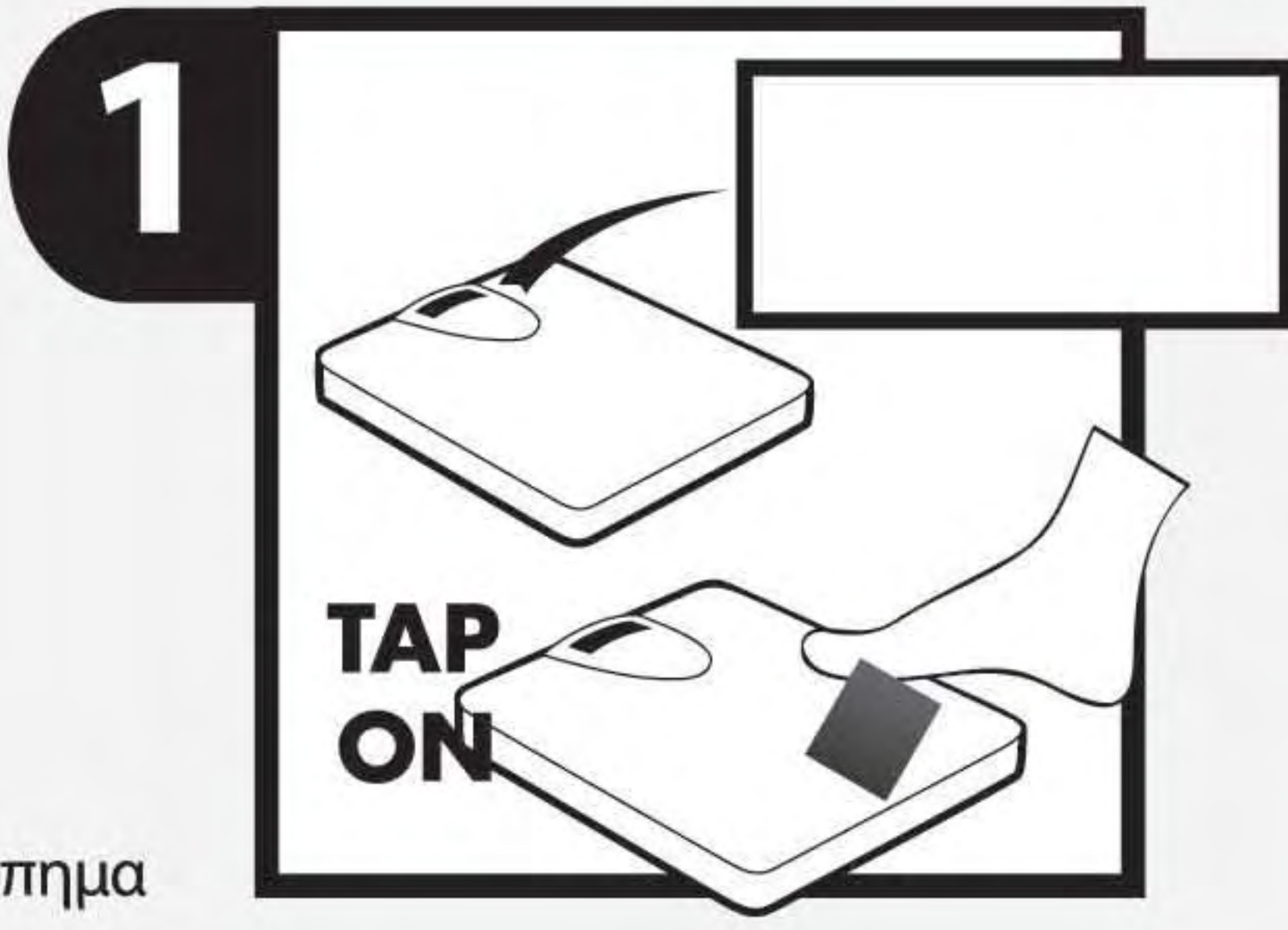

Wait for 0.0 or 0:0
 Attendre l'affichage de 0.0 ou 0:0
 Bitte warten Sie bis 0.0 oder 0:0 kommt
 Wacht op 0.0 of 0:0
 Vänta på 0.0 eller 0:0
 Afvent 0.0 eller 0:0
 Aspettate che appaia 0.0 o 0:0
 Espere hasta que aparezca 0.0 ó 0:0
 Espere por 0.0 ou 0:0
 Odata 0.0 tai 0:0 syttyvan
 Περιμένετε να εμφανιστεί η ένδειξη 0.0 ή 0:0
 0.0 veya 0:01 bekle


To weigh
 Pour peser
 Wiegen
 Weigen
 Wegen
 Att väga
 Sådan vejer du
 Pesatura
 Pesar
 Para pesar
 Punnitus
 Για ζύγιση
 Tartılmak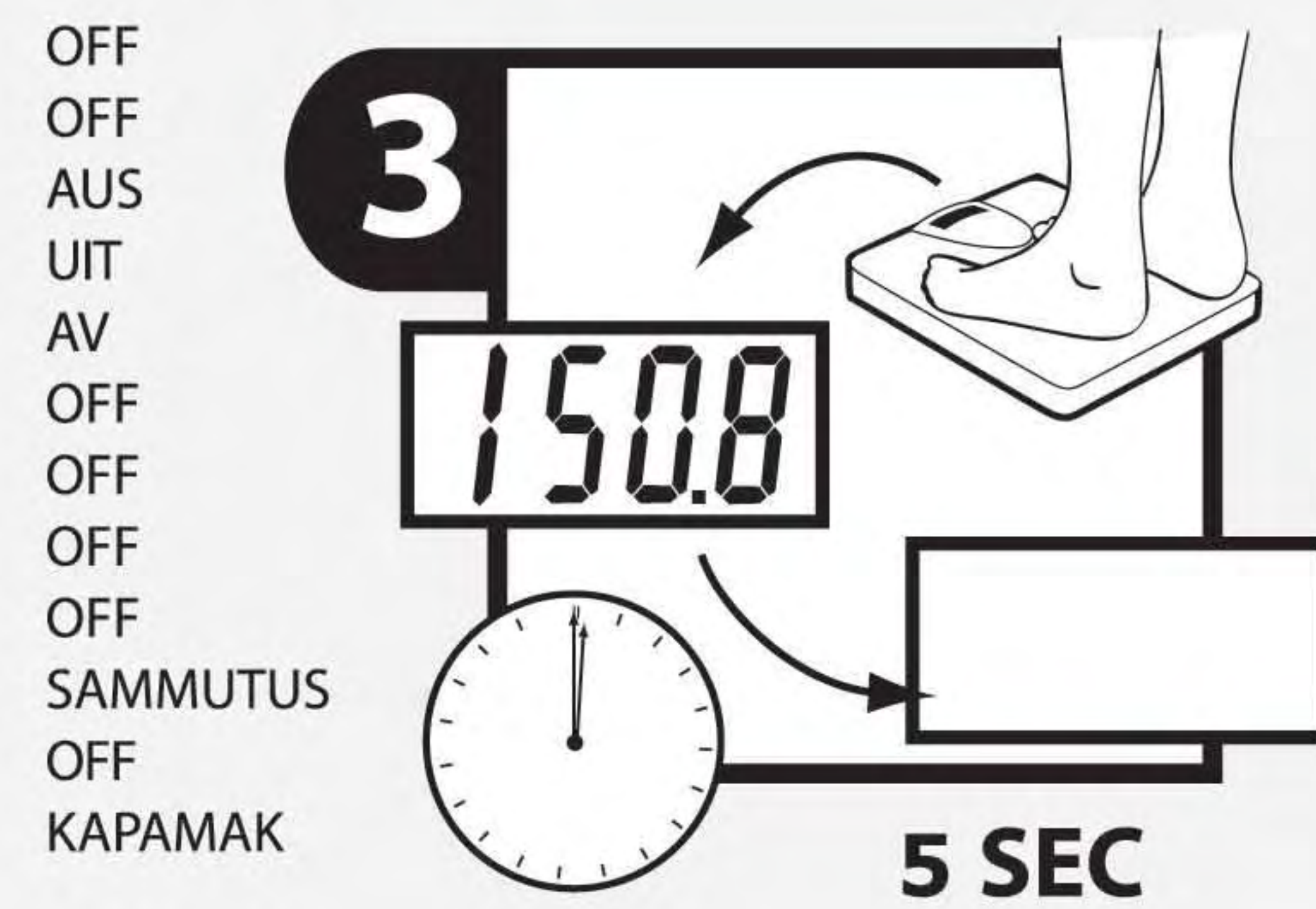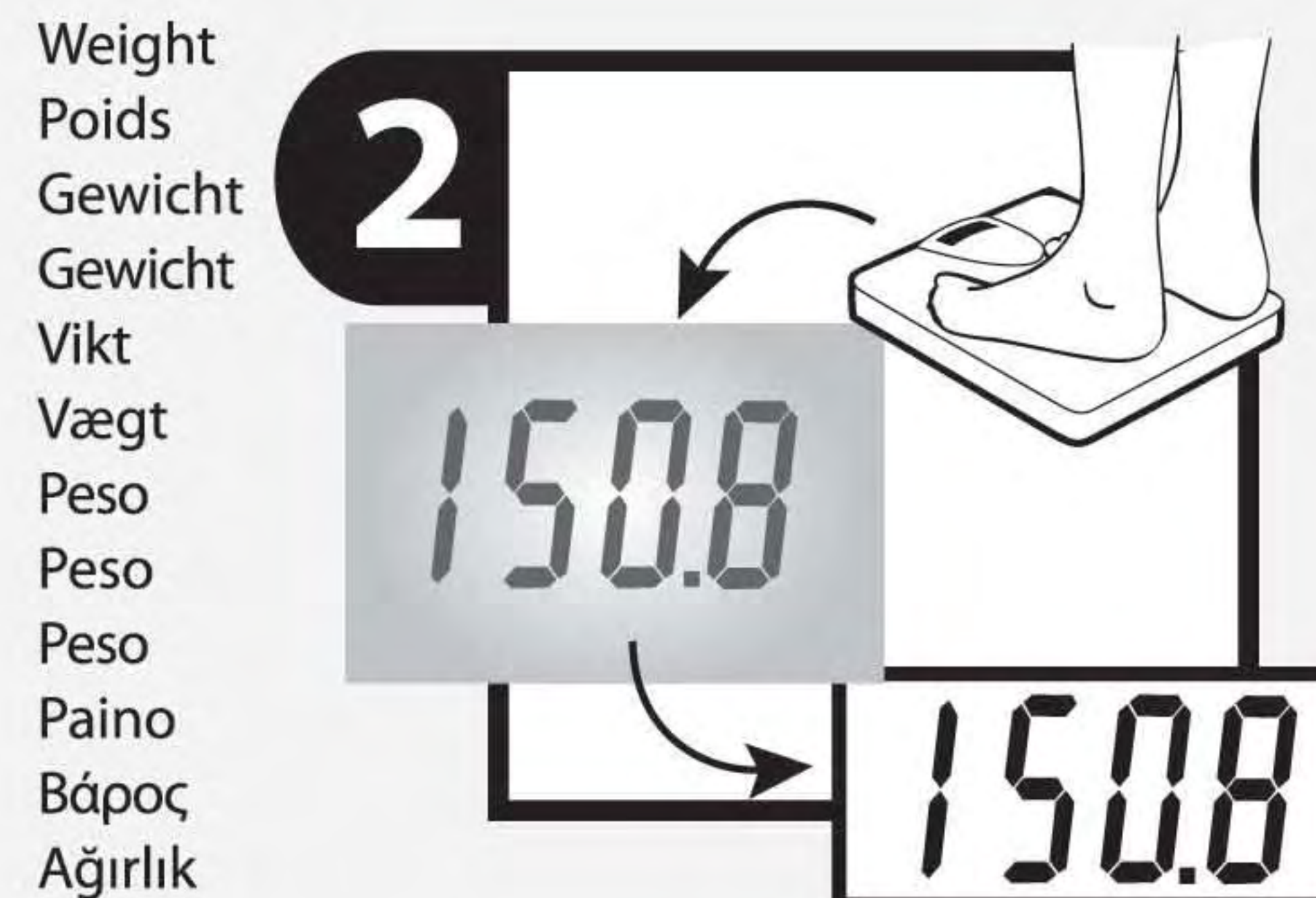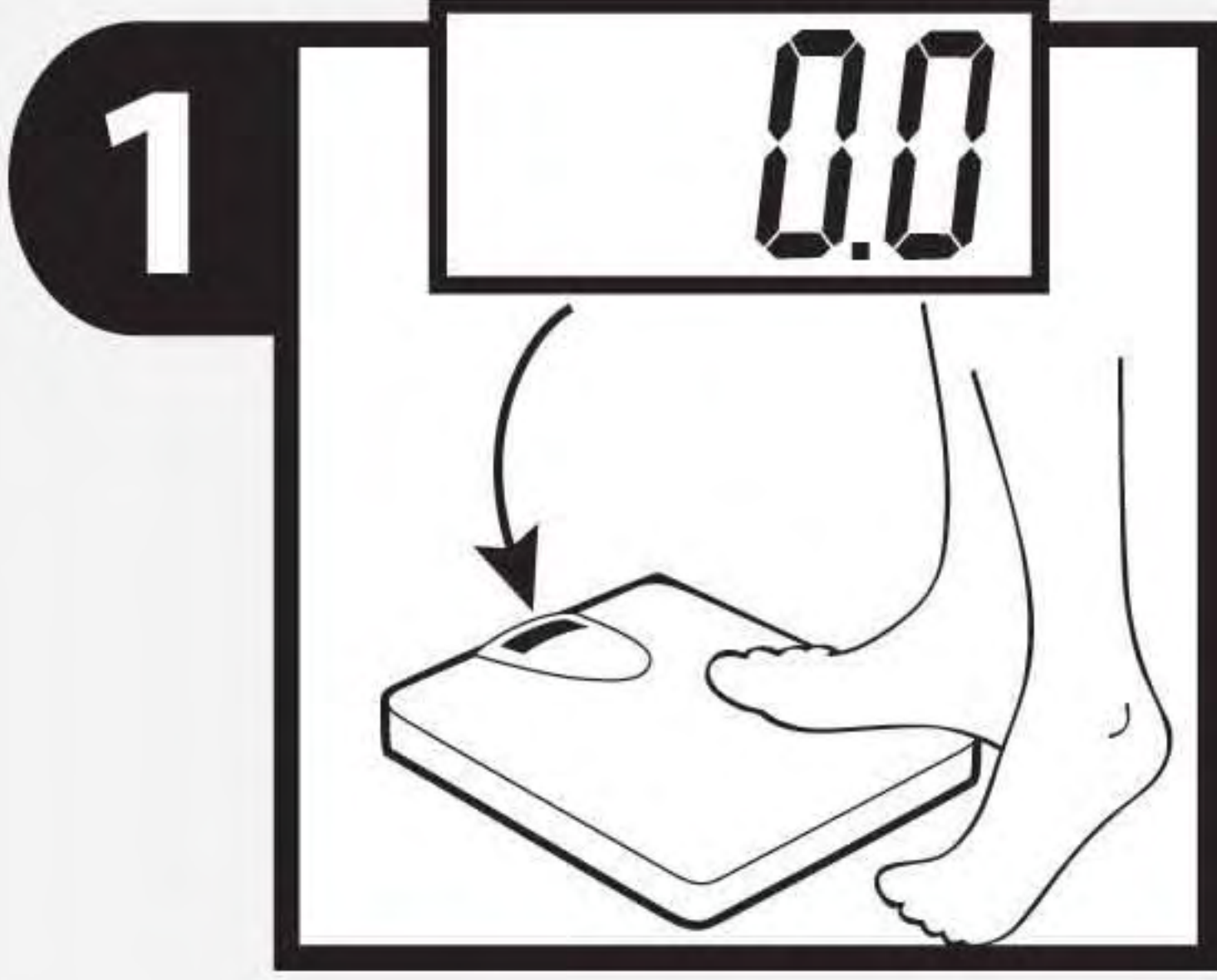


Weight
 Poids
 Gewicht
 Gewicht
 Gewicht
 Vikt
 Vægt
 Peso
 Peso
 Peso
 Paino
 Βάρος
 Ağirlik

OFF
 OFF
 AUS
 UIT
 AV
 OFF
 OFF
 OFF
 OFF
 OFF
 SAMMUTUS
 OFF
 KAPAMAK

Use & Care
 Utilisation et entretien
 Bedienung und Pflege
 Gebruik & onderhoud
 Användning & skötsel
 Brug og vedligeholdelse
 Uso e manutenzione
 Uso y mantenimiento
 Use & cuide
 Käyttö ja huolto
 Χρήση & προστασία
 Kullanım & Bakım

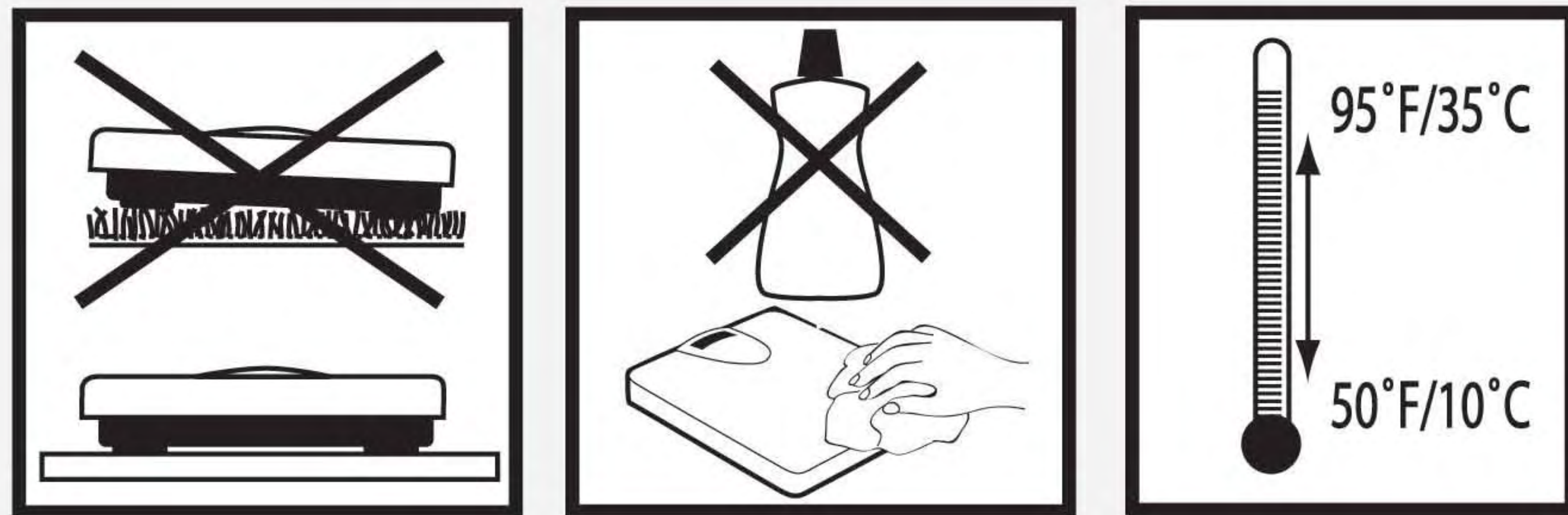
USB port (844KL only)
 Port USB (seul 844KL)
 USB-Port (nur 844KL)
 USB-poort (seul 844KL)
 USB-port (enbart 844KL)
 USB-port (eneste 844KL)
 Porta USB (solo 844KL)
 Puerto USB (sólo 844KL)
 Porta USB (só 844KL)
 USB-porttiin (ainoa 844KL)
 USB λιμάνι (μόνο 844KL)
 USB limanı (844KL yalnızca)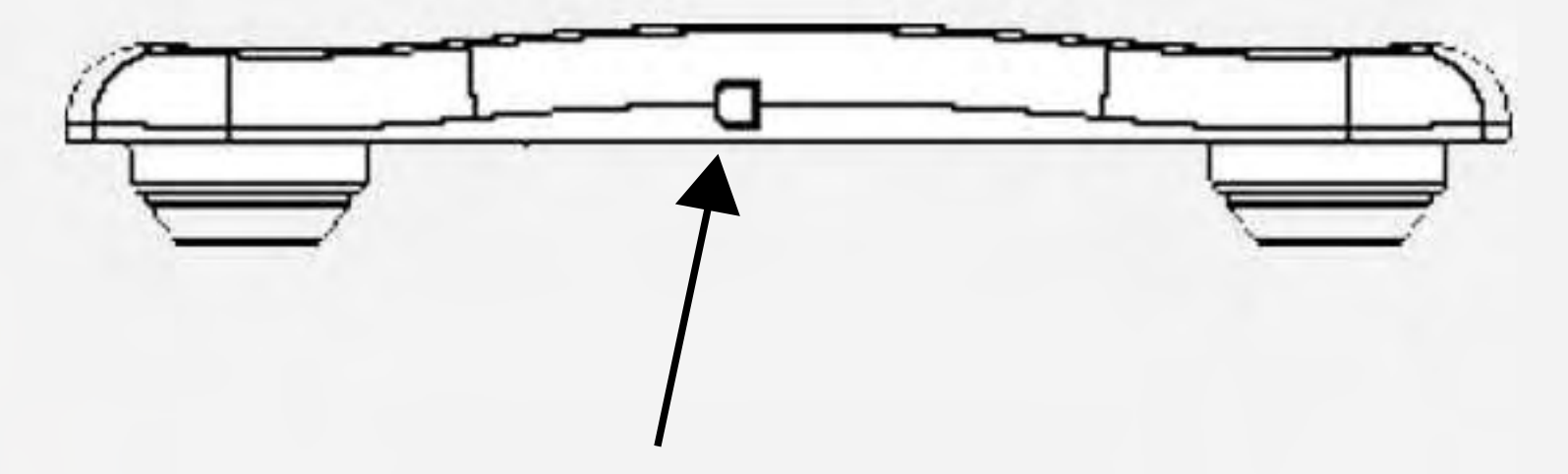


Problems
 Problèmes
 Probleme
 Problemen
 Problem
 Problemer
 Problemi
 Problemas
 Problemas
 Häiriöitä
 Προβλήματα
 Sorunlar

Change battery
 Changer la pile
 Batteriewechsel
 Batterij vervangen
 Byt batteri
 Udskiftning af batteri
 Sostituzione della batteria
 Sustituir pila
 Trocar a pilha
 Vaihda paristo
 Αλλαγή μπαταρίας
 Pil deęistir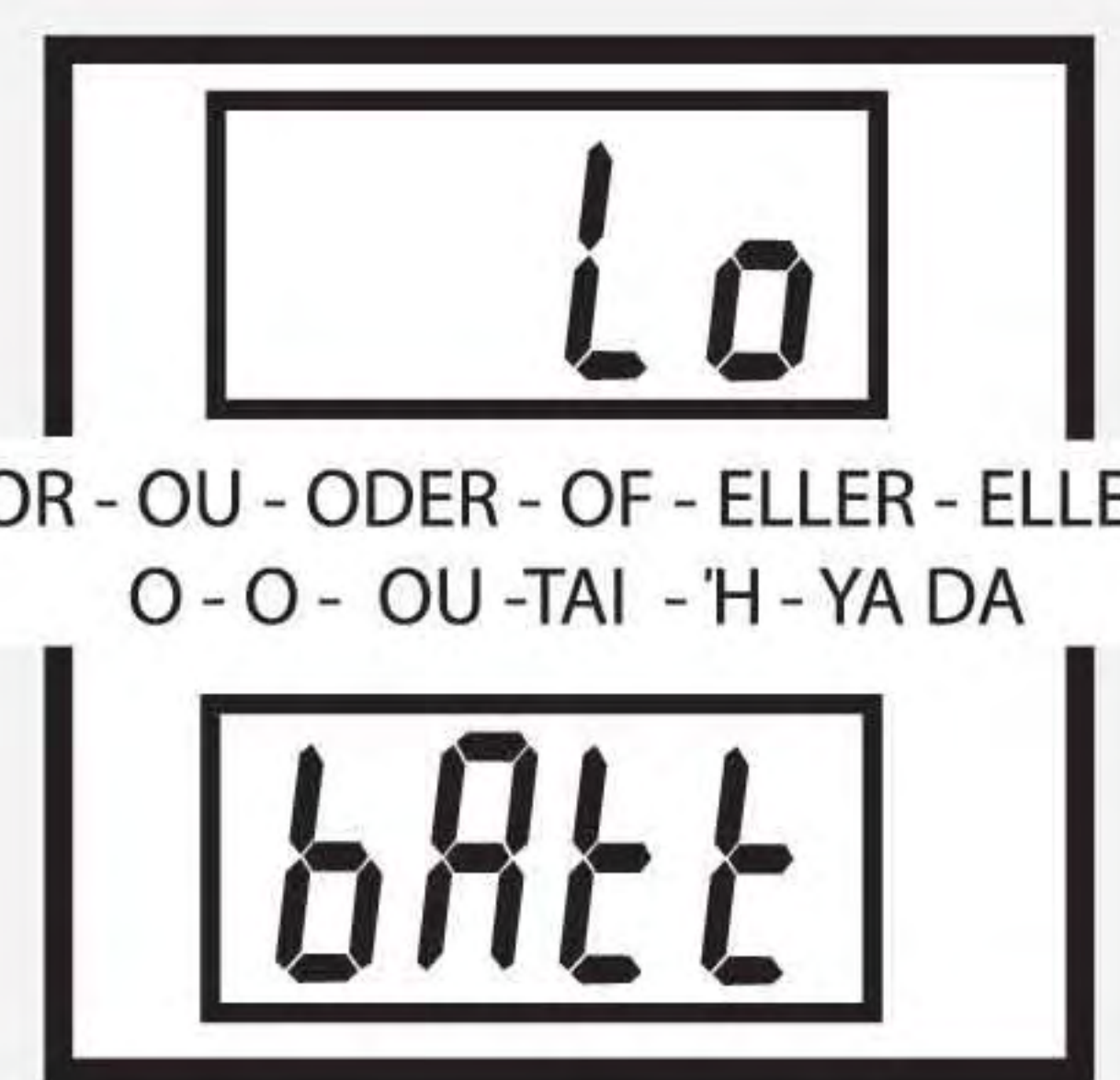

Scale overloaded
 Balance surchargée
 Überlastanzeige
 Weegschaal overbelast
 Vågen överbelastad
 Vægten er overbelastet
 Bilancia sovraccarica
 Báscula sobrecargada
 Ylikuormitus
 Υπερφόρτωση ζυγαριάς
 Terazı fazla yükli

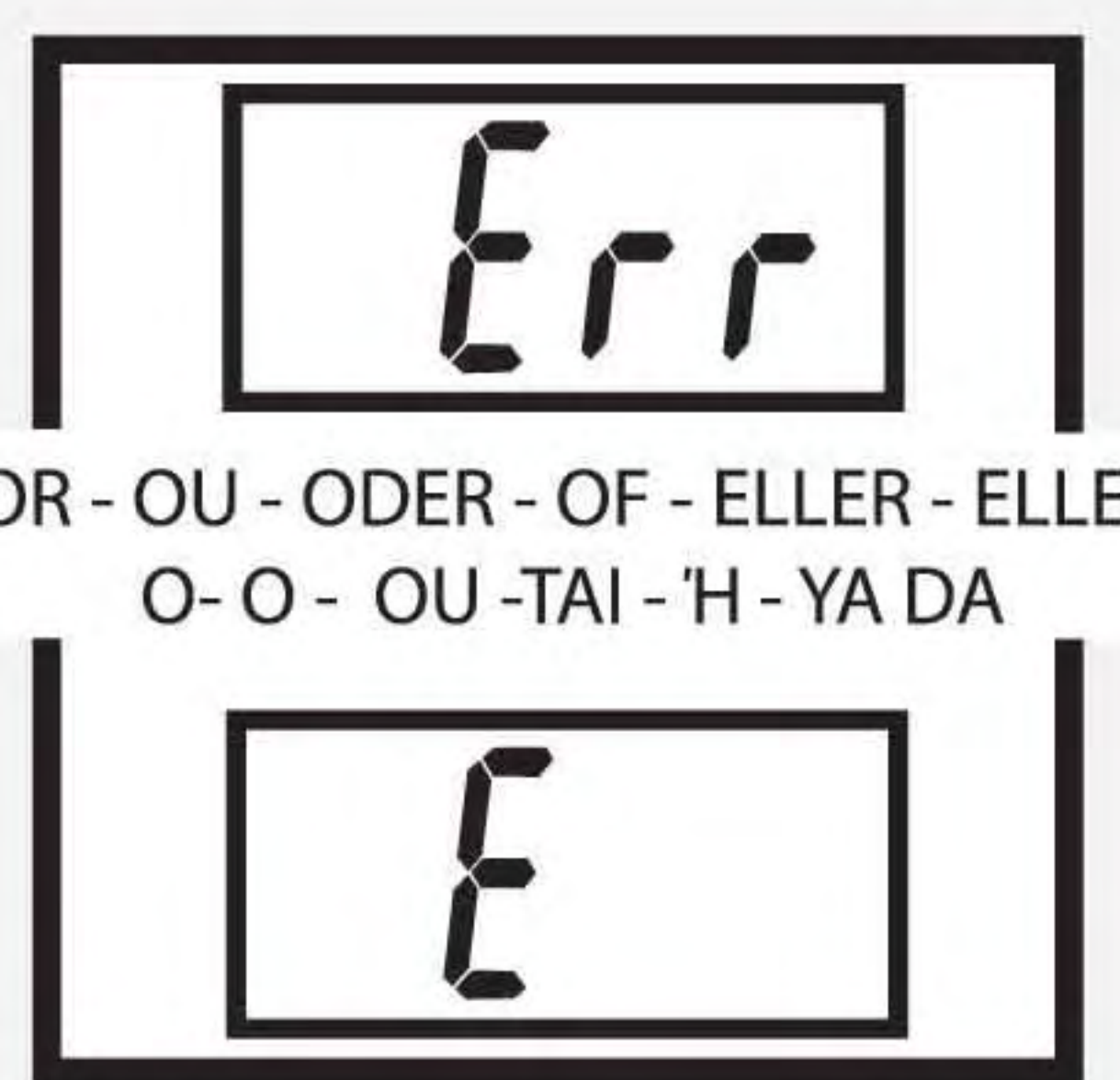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REGULATION FOR WASTE DISPOSAL 2002/96/CE

This product is encompassed by the category for electric and electronic components, which covers products that may contain dangerous substances for human health and the environment. The crossed dustbin symbol means that the product is subject to strict disposal arrangements under European legislation (Directive 2002/96/CE). The user must dispose of the product either by taking it to a shop that sells similar products (“Obligation of return”) or by putting it into the hands of a local refuse collection unit for selective recycling. Any product collected in this way will be dismantled, cleaned and disposed of in an environmentally safe way. Selective recycling helps to reduce the amount of waste and deals safely with any toxic substances. This decreases harmful effects on the environment in accordance with the Waste of Electric and Electronic Equipment (WEEE) Directive. To obtain more information about waste disposal procedures, please contact your local refuse collection unit.

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© Pelstar LLC 2009
Health o meter® is a registered trademark of
Pelstar LLC
Sunbeam Products Inc. used under license.