



THE ALL-PURPOSE CRYOSURGERY DEVICE

CryoPen sets a new standard for the cryogenic treatment of benign skin lesions, offering the ultimate in control, convenience, and cost-effectiveness – as well as an outstanding patient experience.

This high-performance, ergonomic cryotherapy pen dispenses a fine jet of nitrous oxide under high pressure, allowing you to work with millimeter precision.

The CryoPen O+ is an all-purpose model, suitable for regular daily use, and can take two different sizes of cartridge.



Key features

- Ergonomic design for superior control
- Leading-edge microtechnology ensures controlled gas flow
- Takes easy-to-load 8g or 16g N₂O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 4 autoclavable applicators
- Suitable for regular use
- Certified **CE1639** **FDA** **MDSAP**





APPLICATORS



Ø 1-3mm

Power: ■



Ø 2-6mm

Power: ■ ■



Ø 4-8mm


Power: ■ ■ ■




Ø 7-20mm

Power: ■ ■ ■ ■

Continuous flow time with a 16g cartridge

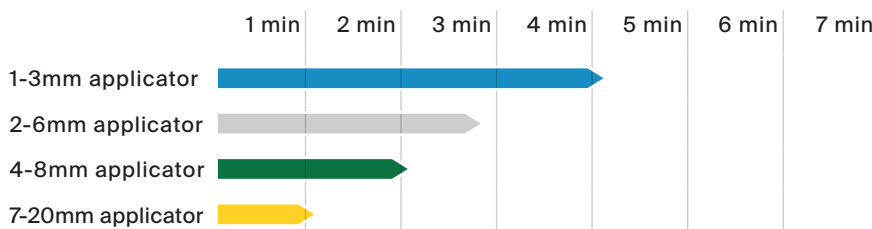

Continuous flow time with an 8g cartridge

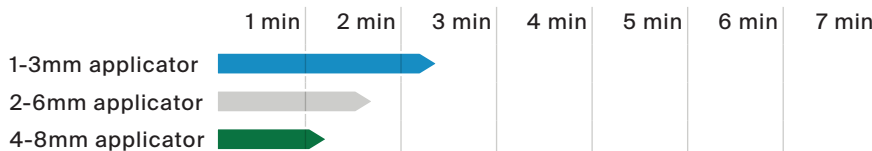

PRODUCT INFORMATION

Device

The CryoPen O+ is suitable for all treatment situations, whether more delicate or stronger freezing power is required. The device can be used with 8g or 16g cartridges, and with the complete range of applicators, making it a highly versatile instrument.

Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, cutaneous leishmaniasis, discoid lupus erythematosus, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, rosacea, lichen sclerosus, granuloma annulare, molluscum contagiosum, seborrheic keratosis and viral lesions.



CONTENTS OF THE STARTER SET

Device

CryoPen O+

Applicators

1-3mm applicator
2-6mm applicator
4-8mm applicator
7-20mm applicator

Cartridges

6x 16g cartridges

Accessories

Metal carrying case
Forceps
Converter for 8g cartridge
Device stand

Documentation

Treatment overview
User manual

BASICS

Dimensions

Device: 22.3 x 3.3 x 3.3 cm
Carrying case: 28.8 x 22.8 x 6.7 cm
Starter set: 38 x 30 x 7.5 cm

HS codes

90189084 (device)
28112930 (cartridges)

Weight

Device with a full cartridge: 300g
Starter set: 2,470g

