

## Under Buttocks Drape, Plastic Reinforced, Large

(89414 depicted)



1. Orient Under Buttocks Drape with patient and insert right and left hands as marked.

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


2. Slide drape under patient's buttocks.



3. Draping is complete.

40"



46"

### Features

- 40" x 46"
- Plastic Reinforced

### Draping Hints

- This drape doesn't have a pouch which is different from the drape depicted above.

### Stock #

89413

### Description

Under Buttocks Drape, Plastic Reinforced, Large, Sterile, 1 per package

### Items Per Case

50

◆ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

**Pack Stock #**  
**88521**

**Description**  
***Lithotomy Pack II***

**Packs/Case**  
**10**

**Component Stock #**

**Sterile**      Non-Sterile ♦

**Contents:**

<b>89618</b>	79498	<b>1 Back Table Cover, Zone-Reinforcement, 44" x 90"</b>
<b>89701</b>	79720	<b>1 Absorbent Towel, 15" x 26"</b>
<b>89731</b>	79731	<b>1 Utility Drape, 26" x 15", with Tape</b>
<b>89413</b>	79413	<b>1 Under Buttocks Drape, Plastic Reinforced, Large, 40" x 46"</b>
<b>89408</b>	79408	<b>2 Leggings, with 6" Cuffs</b>
<b>89101</b>	79101	<b>1 Half Drape, 60" x 44"</b>

**Pack Stock #**  
**88541**

**Description**  
***Lithotomy Pack IV***

**Packs/Case**  
**9**

**Component Stock #**

**Sterile**      Non-Sterile ♦

**Contents:**

<b>89618</b>	79498	<b>1 Back Table Cover, Zone-Reinforced, 44" x 90"</b>
<b>95111</b>	70111	<b>1 ULTRA Surgical Gown, Large, with Towel</b>
<b>89731</b>	79731	<b>1 Utility Drape, 26" x 15", with Tape</b>
<b>89413</b>	79413	<b>1 Under Buttocks Drape, Plastic Reinforced, Large, 40" x 46"</b>
<b>89408</b>	79408	<b>2 Leggings, with 6" Cuffs</b>
<b>89101</b>	79101	<b>1 Half Drape, 60" x 44"</b>

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. \*Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.

<Continued>

Pack Stock #	Description	Packs/Case
88222	Laparoscopy/Pelviscopy Pack	6

**Component Stock #**

Sterile	Non-Sterile ♦	Contents:
89618	79498	1 Back Table Cover, Zone-Reinforced, 44" x 90"
95111	70111	1 ULTRA Surgical Gown, Large, in Overwrap with Towel
95111	70111	1 ULTRA Surgical Gown, Large, with Towel
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"
89731	79731	1 Utility Drape, 26" x 15", with Tape
		1 Suture Bag
89413	79413	1 Under Buttocks Drape, Plastic Reinforced, Large, 40" x 46"
89408	79408	2 Leggings, with 6" Cuffs
89222	79222	1 Laparoscopy/Pelviscopy Drape, 104" x 35" x 104", with Armboard Covers <ul style="list-style-type: none"> <li>• 7" x 9" Trapezoidal Abdominal Fenestration</li> <li>• 3" x 6" Perineal Fenestration, with 15" x 21" Flap</li> <li>• Underleg Wings</li> <li>• Hook &amp; Loop Line Controls</li> <li>• CONTROL* Fabric Reinforcement</li> </ul>

Pack Stock #	Description	Packs/Case
88224	Laparoscopy/Pelviscopy Pack II	6

**Component Stock #**

Sterile	Non-Sterile ♦	Contents:
89618	79498	1 Back Table Cover, Zone-Reinforced, 44" x 90"
95111	70111	1 ULTRA Surgical Gown, Large, in Overwrap with Towel
95111	70111	1 ULTRA Surgical Gown, Large, with Towel
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"
89731	79731	1 Utility Drape, 26" x 15", with Tape
		1 Suture Bag
89413	79413	1 Under Buttocks Drape, Plastic Reinforced, Large, 40" x 46"
89224	79224	1 Laparoscopy/Pelviscopy Drape, 100" x 30" x 123", with Attached Leggings <ul style="list-style-type: none"> <li>• 7" x 9" Trapezoidal Abdominal Fenestration</li> <li>• 3" x 8" Perineal Fenestration, with 15" x 30" Flap</li> <li>• Hook &amp; Loop Line Controls</li> <li>• CONTROL* Fabric Reinforcement</li> </ul>

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.