



OB Pack IV



1. Orient Under Buttocks Drape with patient and insert right and left hands as marked.



2. Slide drape under patient's buttocks and open Fluid Collection Pouch with malleable wire to complete Under Buttocks draping.

(88841 depicted)



3. Next grasp legging where toe is marked with left hand while placing right hand within marked pocket.




4. Holding onto toe of legging, slip drape over stirrup. Repeat for other side.



5. Orient the stamped verbiage on the Abdominal Drape upright toward the patient's head and remove adhesive strip liner. Center on middle of abdomen. Unfold laterally, then toward head.



6. Draping is complete.

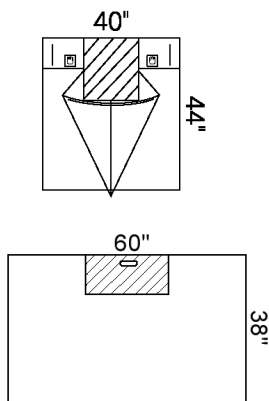

Features

Under Buttocks Drape with Fluid Collection Pouch

- 40" x 44"
- Absorbent Pad

Abdominal Drape

- 60" x 38"
- CONTROL* PLUS Fabric Reinforcement

Leggings with 6" Cuff

- 31" x 48"

Stock

88841

Description

OB Pack IV, Sterile

Items Per Case

5

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. *Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.

<See the following page for Pack Options>

Pack Stock #
88841

Description
OB Pack IV

Packs/Case
5

Component Stock #

Sterile Non-Sterile ♦

89618 79498

Contents:

1 Back Table Cover, Zone-Reinforced, 44" x 90"

1 Set-Up cover, 76" x 44"

95411 70411

1 ULTRA Zoned-Impervious Surgical Gown, Large with Towel

89701 79720

2 Absorbent Towels, 15" x 26"

1 Baby Blanket

89414 79414

1 Under Buttocks Drape, 40" x 44", with Fluid Collection Pouch
• **Absorbent Pad**

89408 79408

2 Leggings, with 6" Cuffs

79810

1 Abdominal Drape, 60" x 38"
• **CONTROL* PLUS Fabric Reinforcement**

1 Peri Pad

1 HOLLISTER* Cord Clamp

1 Vaginal Packing Sponge

1 DAVOL* Bulb Syringe – 2 oz.

10 Sponges, X-Ray Detectable, 4" x 4"

1 Placenta Basin with Lid

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. *Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.