

Pediatric Laparotomy Drape

(66887 depicted)



1. Orient body stamp on drape with patient and remove adhesive strip liners clockwise.




2. Center drape over operative site.



3. Unfold drape, first laterally, then toward feet.



4. Unfold drape toward head firmly seal sides of fenestration to patient's skin. Insert surgical lines and tubing through tube holders.

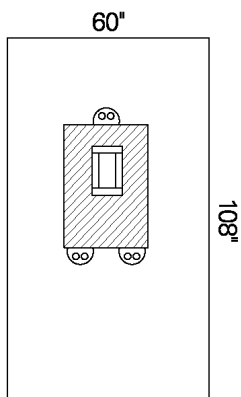

Features

- 60" x 108"
- 3" x 8" Fenestration
- CONTROL* Fabric Reinforcement
- Tube Holders

Draping Hints

- This drape has no armboard covers which is different from the drape depicted above.

Stock

Description

Items Per Case

89241

Pediatric Laparotomy Drape, Sterile, 1 per package

15

79240 ♦

Pediatric Laparotomy Drape, Non-Sterile, Bulk Pack

25

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.