

### Large Impervious Split Drape

(89331 depicted)



1. Locate the "Tails" and arrow markings on the drape and orient with the patient.



2. Center drape below extremity or area to be draped.



3. Unfold drape, first laterally, then toward feet to expose adhesive strip liners.




4. Unfold tails and remove adhesive strip liners.



5. Conform first one side of split around leg, then the other.



6. Draping is complete in preparation for a key drape.

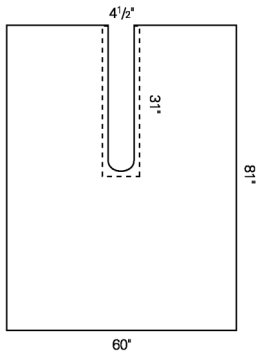

#### Features

- 60" x 81"
- 4 1/2" x 31" split

#### Stock #

79333 ♦

#### Description

Large Impervious Split Drape, Non-Sterile, Bulk Pack

#### Items Per Case

100

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. \*Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.

Pack Stock # 88452		Description <i>ORTHOARTS* Hip Pack with Pockets</i>	Packs/Case 3
<b>Component Stock #</b>			
<b>Sterile</b>	Non-Sterile ♦	<b>Contents:</b>	
	79006	1 Heavy Duty Back Table Cover, Reinforced, 60" x 90"	
95111	70111	1 ULTRA Surgical Gown, Large, in Overwrap with Towel	
95321	70321	2 ULTRA Zoned-Impervious Surgical Gowns, X-Large, with Towels	
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"	
		1 Suture Bag	
89821		1 Impervious Stockinette, Large, 12" x 48"	
89121	79121	1 Large Drape, 60" x 76"	
	79333	1 Impervious Split Drape, Large, 60" x 81", 4½" x 31" Split	
89345	79345	1 ORTHOARTS* Hip Drape, 114" x 88" x 126" <ul style="list-style-type: none"> <li>• Armboard Covers</li> <li>• Leg Pockets on Both Sides</li> <li>• Comfortable Teardrop Fenestration</li> <li>• Heavy Duty Fabric</li> <li>• CONTROL* PLUS Fabric Reinforcement</li> <li>• Tube Holders</li> </ul>	
		1 Yankauer Suction Tip	
		1 10' Suction Tubing	

Pack Stock # 88453		Description <i>ORTHOARTS* Hip Pack</i>	Packs/Case 3
<b>Component Stock #</b>			
<b>Sterile</b>	Non-Sterile ♦	<b>Contents:</b>	
	79006	1 Heavy Duty Back Table Cover, Reinforced, 60" x 90"	
95111	70111	1 ULTRA Surgical Gown, Large, in Overwrap with Towel	
95321	70321	2 ULTRA Zoned-Impervious Surgical Gowns, X-Large, with Towels	
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"	
		1 Suture Bag	
89821		1 Impervious Stockinette, Large, 12" x 48"	
89121	79121	1 Large Drape, 60" x 76"	
	79333	1 Impervious Split Drape, Large, 60" x 81", 4½" x 31" Split	
89346	79346	1 ORTHOARTS* Hip Drape, 114" x 88" x 126" <ul style="list-style-type: none"> <li>with Armboard Covers <ul style="list-style-type: none"> <li>• Comfortable Teardrop Fenestration</li> <li>• Heavy Duty Fabric</li> <li>• CONTROL* PLUS Fabric Reinforcement</li> <li>• Tube Holders</li> </ul> </li> </ul>	
		1 Yankauer Suction Tip	
		1 10' Suction Tubing	

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.