


Hip Drape with Clear Leg Pockets

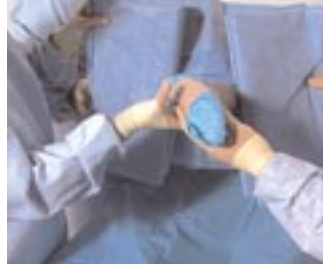

1. With an impervious split drape and impervious stockinette in place, locate and orient the body stamp on the drape with the patient. Remove the adhesive strip liner at the top of the fenestration.




2. Unfold the drape once and guide the leg through the conformable fenestration.



3. Advance the drape over the hip, positioning fenestration around surgical site.




4. Next, unfold the drape laterally, and then toward the patient's head, forming an anesthesia screen. Extend armboard covers. Unfold remaining portion of the drape toward the patient's feet.

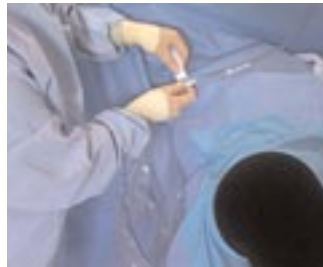

5. Insert surgical lines and tubing through the available hook-and-loop line control to complete draping.



6. Open the hook-and-loop fasteners on the pouch to expose the clear leg pockets when ready to place the leg within.

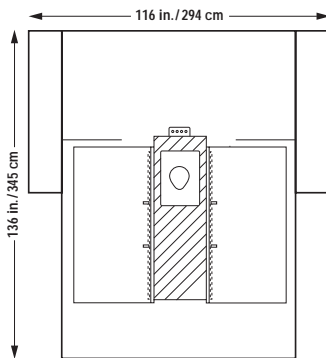

Features

- 116" x 136"
- Clear leg pockets
- Conformable tear-drop fenestration
- Hook-and-loop line controls
- CONTROL* Plus fabric reinforcement
- Armboard covers

Draping Tip:

- Reclosable hook-and-loop fasteners allow for the retrieval of the leg to the operative area as needed throughout the procedure.
- Both hands should be used when opening and closing hook-and-loop fasteners on the drape.
- May be used for left or right hip procedures.

Stock #	Description	Items Per Case
89349	Hip Drape with Clear Leg Pockets Sterile, 1 drape per package	9
79349*	Hip Drape with Clear Leg Pockets Non-sterile, bulk pack	10

◆ Except as noted, all non-sterile products are for custom trays and available only through your custom-tray supplier

† As with any drape or pack recommendation, it is the hospital's decision as to the appropriate draping technique.

* Registered Trademark or Trademark of Kimberly-Clark Worldwide or its affiliates.

© 2003 KCWW. All rights reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement.