

Status DS

One Step Immunoassay For The Qualitative Detection Of
Drugs Of Abuse And Their Metabolites In Urine



- Accurate results with a 97-99% correlation to GC/MS
- Fast turnaround time with results in as little as 5 minutes
- Flexible combination of drug panels from a single drug up to 10 drug panels
- Test menu includes:

-Tricyclic Antidepressants (TCA)

-Methadone (MTD)

-Amphetamine (AMP)

-Methamphetamine (MET)

-Marijuana (THC)

-Opiates (OPI)

-Phencyclidine (PCP)

-Barbiturates (BAR)


-Benzodiazepines (BZO)

-Cocaine (COC)


STEP 1
Collect urine sample.



STEP 2
Holding the dropper vertically, dispense 3 drops (110 µL) of the urine sample into each Sample well (s).

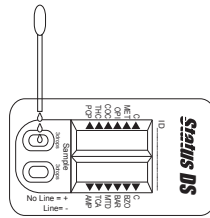


STEP 3
Read the results after 5 minutes, but within 10 minutes.

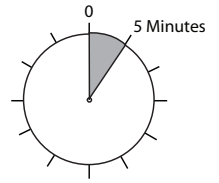

INTERPRETATION OF RESULTS

A line indicates a **negative** result for that drug

No line indicates a **positive** result for that drug

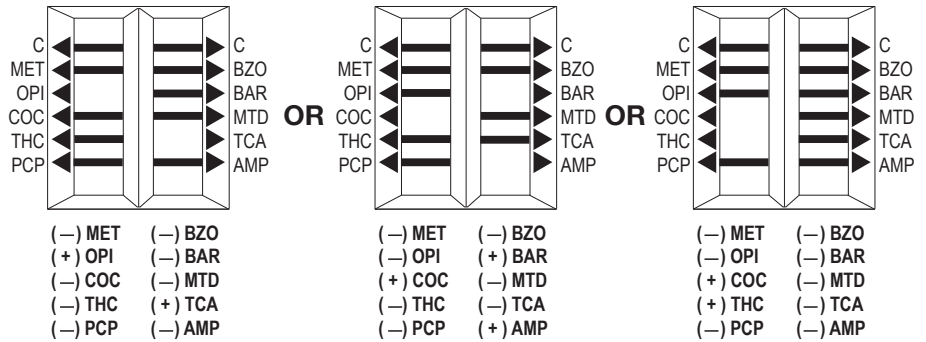

PRODUCT	# OF TESTS	ITEM #
Status DS TCA	35	20935
Status DS 8 Panel THC/OPI/COC/AMP/BZO/BAR/TCA/PCP	25	28025
Status DS 10 Panel MET/THC/OPI/COC/AMP/BZO/BAR/TCA/PCP/MTD	10	21010
Status DS 10 Panel MET/THC/OPI/COC/AMP/BZO/BAR/TCA/PCP/MTD	25	21025
Status DS Control Set	2 X 5 ml	21000



Infectious Disease • Drugs of Abuse • Cardiac Markers
GI Health • Women’s Health • Urology

Proudly made in the USA

For more information or to place an order, contact: