



Status™

lifeSign

COVID-19/Flu A&B

A Rapid Immunoassay for the Simultaneous
Direct Detection and Differential Diagnosis of SARS-CoV-2,
Influenza Type A and Type B Antigen from nasal and nasopharyngeal swab specimens

Infections with these viruses may present similar symptoms.
Can you tell them apart? **We Can.**

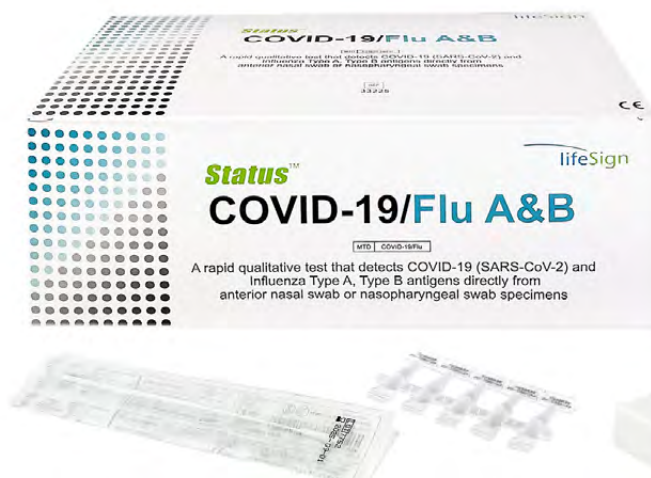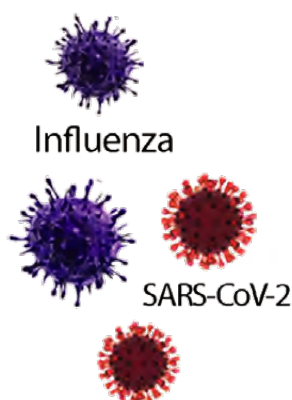

Learn More

1 test + 1 specimen = 3 Results

The additive power of **Status™ COVID-19/Flu A&B**

- **Sensitivity and Specificity**
 - COVID-19 - Sensitivity 93.9%, Specificity 100%
 - Flu A - Sensitivity 91.4%, Specificity 95.7%
 - Flu B Sensitivity 87.6%, Specificity 95.9%
- **FDA Emergency Use Authorization (EUA)**
- **Visually read in 15 minutes**
- **Flocked specimen swabs for superior specimen collection and patient comfort**

ORDERING INFORMATION		
PRODUCT	# OF TESTS	ITEM #
Status™ Covid-19/Flu A&B	25	33225



Proudly made in the USA

For more information or to place an order, contact:

DISTRIBUTED BY:



contactsales@carolinachemistries.com
carolinachemistries.com

877-722-8910



lifeSign

A PBM Group Company

85 Orchard Road • Skillman • New Jersey • 08558
800.526.2125 • 732. 246. 3366 • Fax: 732.246.0570 • 222.lifesignmed.com



Easy 3 Step Procedure

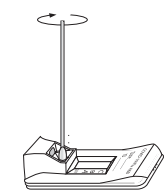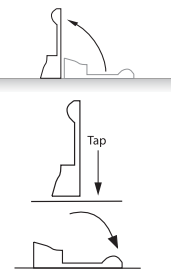	1 squeeze Tear the tab off the Extraction Reagent Capsule and dispense entire contents into the Extraction Well.
	2 Insert the specimen swab in the Swab Stand. • Rotate swab 3 times to mix the specimen. • Let Stand 1 minute. • Rotate swab 3 times again and discard the swab.
	3 Raise the device upright and let stand 102 seconds. Gently tap the device to ensure the liquid flows into the hole. Lay the device back down.
Set a timer for 15 minutes.	
4	Read test results at 15 minutes. NOTE: False positive or false negative results can occur if the test is not read between 15 and 20 minutes.

- Refer to our website at www.lifesignmed.com for complete test procedure details, training video and package insert.
- **Status™ COVID-19/Flu A&B** is an antigen test for use with nasal and nasopharyngeal swab specimens.
- **Status™ COVID-19/Flu A&B** test has been granted emergency use authorization (EUA) by the FDA for use at the point-of-care setting. During the duration of the emergency this test can be performed in a patient care setting that is operation under a CLIA Waiver, Certificate of Compliance or Certificate of Accreditation.

 **Proudly made in the USA**
For more information or to place an order, contact:

Printed in USA
All rights reserved
81581 - 07/07/2022

DISTRIBUTED BY:



contactsales@carolinachemistries.com
carolinachemistries.com

877-722-8910



A PBM Group Company

85 Orchard Road • Skillman • New Jersey • 08558
800.526.2125 • 732. 246. 3366 • Fax: 732.246.0570 • 222.lifesignmed.com