



CAVSWAB™ MID-TURBINATE



CavSwab™

Recommended for pediatric use, the MedSchenker® Mid-turbinate CavSwab™ with Allium™ flock technology has been engineered with a precision scalloped surface to produce 'cavums.'

Allium™ flock technology refers to the unique ability of our nylon fibers to adapt to the direction the swab is being pushed/pulled, resulting in less friction and an overall more pleasant experience for both the medical professional and the patient.

The notched 120mm shaft, with an 85mm breakpoint, increases grip and dexterity. It has also been marked for uncomplicated identification of required depth and a removable stopper indicates when that depth has been reached. The design of the stopper, the face of a bear, helps distract and put younger patients at ease.

CavSwab™ Mid-turbinate


Package	2000 units per master package
Tip	Nylon Fiber
Rod	Nylon
Sterility	Electron Beam
Total Length	120mm
Breakpoint	85mm

Age recommended stop points:

Swab tip: 0-2 years old

- ◀ Notch A: 2-6 years old
- ◀ Notch B: 6-12 years old
- ◀ Notch C: Additional notch for bigger kids.

Swab colors available:
Blue, Pink

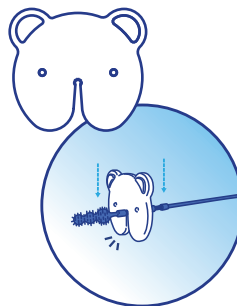

The bear clip ensures a quicker and more comfortable testing process by eliminating the need for the lab technician to manually measure the distance between the child's nose and nasal pharynx.

ADVANTAGES

- ❄️ Marked nylon shaft for uncomplicated identification of required depth.
- ❄️ Removable stopper, the face of a bear, helps distract and put younger patients at ease.
- ❄️ Precision scalloped surface of 'cavums' provide a greater surface area to absorb and collect larger amounts of specimen.
- ❄️ Allium™ flock technology - nylon fibers adapt to the direction the swab is being pushed/pulled.
- ❄️ Notched shaft increases grip and dexterity while also reducing the amount of stress on the practitioner.
- ❄️ Electron beam sterilized and individually wrapped in pouches to ensure 100% sterility.

AC3100-B-0002-0121 AC3100-P-0002-0121