722

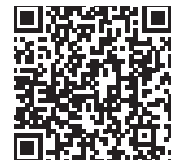
& 729 user manual

SETUP
VIDEO

OPERATION
VIDEO

USER
MANUAL

THIS PAGE IS INTENTIONALLY BLANK

Table of Contents

APPLICABLE MODEL(S).....	IV
REVISION HISTORY TABLE	IV
GENERAL INFORMATION	1
Purpose of this Manual.....	1
Intended Use.....	1
Training Options.....	1
Symbols and Warnings.....	2
Safety Instructions	4
Electromagnetic Interference	5
Electromagnetic Compatibility (EMC).....	5
Electrical Requirements	8
Contraindications.....	9
Transportation and Storage Conditions	9
Disposal of Equipment	9
EQUIPMENT FEATURES	10
INSTALLATION	11
Required Tools	11
Explanation of Instructions	11
Unpacking and Set-up.....	12
OPERATION	13
Operating Conditions	13
SmartTech™	14
Smart Controller™ Technology.....	14
ComfortSync™	14
User Program Mode.....	14
Hibernation State.....	15
Smart Safety™ Technology.....	15
Safety Lockout.....	15
Shutdown.....	15
Controls.....	16
Headrest Operation	18
Armrest Operation.....	19
LIST OF ACCESSORIES.....	20
MAINTENANCE	22
Disinfection	22
Cleaning	23
Preventative Maintenance.....	23
Power Cord Removal Instructions.....	23
Fuse Replacement Instructions	23
Scuff Cover Replacement Instructions (722 Only)	23
Calls for Service.....	23
LIMITED WARRANTY	24
TROUBLESHOOTING GUIDE.....	25
GUIDE TO INDICATOR LIGHTS AND SOUNDS	27
SPECIFICATIONS.....	28
IN-SERVICE CHECKLIST	30

APPLICABLE MODEL(S)

This manual is for the following product numbers and serial numbers:

PRODUCT NUMBER	DESCRIPTION	EFFECTIVE S/N
722-001-XX	722 Oral/Dental Surgery Chair, Std Base, 100-240V	251201-XXXXXX to Current
729-001-XX	729 Oral/Dental Surgery Chair, Std Base, 100-240V	251201-XXXXXX to Current

REVISION HISTORY TABLE

DATE	REVISION	CO#/ECO#	DETAILS
10/01/2025	A	CO-0596	Initial release
1/10/2026	B	CO-0594	Added instructions for scuff cover replacement, edited In-Service Form, and other minor edits.

User Manual

MTI 722 & 729

GENERAL INFORMATION

Purpose of this Manual

This manual covers the safety, installation, instructions for use, and maintenance of this equipment. It is intended that it be used by the service professional installing the equipment and by the medical professionals who have been trained to use it. No repair information is included with this manual because no operator repairs are intended.

Intended Use

This equipment is intended to be used as a chair/table for general examinations and procedures performed by a qualified medical professional in a medical office environment.

Training Options

Training is based on the intended use of the equipment and the proficiency of the medical professionals who will be using the equipment. Training is accomplished either through reading the installation and operation sections of the user manual or through remote training provided by MTI Technical Support. To request training, contact MTI.

PRODUCT INFORMATION

Date of Purchase: _____

Model Number: _____

Serial Number: _____

(see "EQUIPMENT FEATURES" on page 10 for location of serial number label)

Symbols and Warnings

This manual contains Notices, Cautions, and Warnings, which are meant to call attention to particular parts or sections of this manual. Since safety is of our utmost concern, this manual should be read by all personnel who will be operating this medical equipment.

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WARNINGS/AVERTISSEMENTS

Are used to call attention to condition, practice, or procedure, which could cause personal injury if not followed correctly. Please become familiar with all warnings and observe them at all times.

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Are used to call attention to a condition, practice, or procedure which could cause product damage if not used correctly.

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NOTICES/AVIS

Are used to call attention to helpful information regarding a condition, practice, or procedure.

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Safety Instructions

WARNING/AVERTISSEMENT

Read this manual before installation and use.

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WARNING/AVERTISSEMENT

No modification of this equipment is allowed.

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WARNING/AVERTISSEMENT

Do not operate equipment in the presence of flammable anesthetics.

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WARNING/AVERTISSEMENT

Do not leave the equipment unattended during use of programmable buttons (Home/P1-P4) or manual actuation of a function.

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WARNING/AVERTISSEMENT

Ensure equipment, patient, and other objects are in a safe condition before operating the equipment.

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WARNING/AVERTISSEMENT

Verify patient arms, headrest and other adjustable equipment features and accessories are locked and in a safe and secure position before use.

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WARNING/AVERTISSEMENT

Ensure controls, cords, and other parts of the equipment do not create a tripping or other safety hazard for operator or patient.

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WARNING/AVERTISSEMENT

To prevent unintentional movement of the equipment caused by accidental activation of the equipment controls, electromagnetic interference, or radio frequency interference from other equipment, use the On/Off Button (only on select models) on the hand control or hand switches. For equipment without the On/Off Button, use the alternate security method by activating the Safety Lockout setting. See "Safety Lockout" on page 15. Failure to do so may result in personal injury. Using the On/Off Button or the Safety Lockout setting will not prevent non-powered motion.

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WARNING/AVERTISSEMENT

Exceeding the specified load limit of any equipment component may result in personal injury or property damage. See "Specifications" on page 28.

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WARNING/AVERTISSEMENT

Backrest, seat, footrest, armrest, headrest, and other parts of the equipment should only be used for their intended purpose. MTI is not responsible for use of components outside of their intended use.

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WARNING/AVERTISSEMENT

This product can expose you to chemicals including nickel, which

is known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov and see www.mti.net/faqs for details on California Proposition 65.

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Electromagnetic Interference

This equipment is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this equipment, do one or more of the following:

1. Ensure other device is operating within its specifications.
2. Plug equipment into isolated circuit.
3. Remove interfering device from room.
4. Increase separation between equipment and interfering device.
5. Contact MTI if electromagnetic interference persists.

Electromagnetic Compatibility (EMC)

WARNING/AVERTISSEMENT

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

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WARNING/AVERTISSEMENT

During use, portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the chair, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

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WARNING/AVERTISSEMENT

Use of this equipment adjacent to other electrical equipment should be avoided since it could result in improper operation. If such use is necessary, verify normal chair operation before use with patients.

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NOTICE/AVIS

The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

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This chair has been tested to comply with the requirements of EN 60601-1-2:2014 Ed.4 for electromagnetic compatibility (EMC). The chair was designed to be operated within the electromagnetic environment specified below.

Guidance and manufacturer's declaration - electromagnetic emissions		
Emissions Test	Compliance	Electromagnetic Environment - Guidance
RF emissions CISPR 11	Group 1	The 722 & 729 MTI Surgery Chairs use RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The 722 & 729 MTI Surgery Chairs are suitable for use in all establishments other than domestic and those directly connected to public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 6100-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND THE MTI SURGERY CHAIRS			
The MTI Surgery Chair is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the MTI Surgery Chair can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the MTI Surgery Chair as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter Watts (W)	Separation distances according to frequency of transmitter meters (m)		
	150 kHz to 80 MHz $d = (1.17)(\sqrt{P})$	80 MHz to 800 MHz $d = (1.17)(\sqrt{P})$	800 MHz to 2.5 GHz $d = (2.33)(\sqrt{P})$
0.01	0.12	0.12	0.23
.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.70	11.70	23.30
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

GUIDANCE AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC IMMUNITY

The MTI Surgery Chair is intended for use in the electromagnetic environment specified below. The customer or the user of the MTI Surgery Chair should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Conducted RF IEC 61000-4-6	3V rms 150kHz to 80 MHz	3V	Portable and mobile RF communications equipment should be used no closer to any part of the MTI Surgery Chair, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = (1.17)(\sqrt{P})$ 150 kHz to 80 MHz $d = (1.17)(\sqrt{P})$ 80 MHz to 800 MHz $d = (2.33)(\sqrt{P})$ 800 MHz to 2.5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3V/m 80MHz to 2.7 Ghz	3V/m	

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^aNOTICE: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the MTI Surgery Chair is used exceeds the applicable RF compliance level above, the MTI Surgery Chair should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the MTI Surgery Chair.

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^bOver the frequency range 150kHz to 80MHz, field strengths should be less than 3 V/m.

GUIDANCE AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC IMMUNITY			
The MTI Surgery Chair is intended for use in the electromagnetic environment specified below. The customer or the user of the MTI Surgery Chair should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 8 kV air	± 8 kV contact ± 2, 4, 8, 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast Transient / burst IEC 61000-4-4	± 0.5, 1, 2 kV for power supply lines ± 1 kV for input/output lines	± 0.5, 1, 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5, 1 kV line(s) to line(s) ± 0.5, 1, 2 kV lines(s) to earth	± 0.5, 1 kV ± 0.5, 1, 2 kV	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 Tested at 230, 220, 127 and 120 VAC.	0% U_T (100% dip in U_T) for 0.5 cycle 0% U_T (100% dip in U_T) for 1 cycle 70% U_T (30% dip in U_T) for 25 cycles 0% U_T (100% dip in U_T) for 250 cycles	0% U_T (100% dip in U_T) for 0.5 cycle 0% U_T (100% dip in U_T) for 1 cycle 70% U_T (30% dip in U_T) for 25 cycles 0% U_T (100% dip in U_T) for 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the MTI Surgery Chair requires continued operation during power mains interruptions, it is recommended that the MTI Surgery Chair be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m, 50 / 60 Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: U_T is the a.c. mains voltage prior to application of the test level.			

Electrical Requirements

WARNING/AVERTISSEMENT

To avoid risk of electrical shock, this equipment must be connected to a supply mains with protective earth.

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WARNING/AVERTISSEMENT

Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".

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⚠ WARNING/AVERTISSEMENT

When using endocardial catheters or high frequency surgical/ procedure devices, use non-conductive material to insulate the patient from metal portions of the equipment. Failure to comply may result in electric shock or burns to patient.

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⚠ WARNING/AVERTISSEMENT

Connecting electrical equipment to multiple socket-outlet effectively leads to creating a medical electronic system and can result in a reduced level of safety.

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⚠ WARNING/AVERTISSEMENT

Equipment must be positioned in normal use such that the power cords can be easily accessed to unplug.

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⚠ WARNING/AVERTISSEMENT

Equipment must be connected to an appropriate power source when loss of power source would result in an unacceptable risk.

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⚠ WARNING/AVERTISSEMENT

Do not use a three to two prong adapter to plug the equipment into a two conductor outlet, as grounding reliability is not maintained.

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⚠ WARNING/AVERTISSEMENT

Do not plug this equipment into a low power or auxiliary outlet. This equipment must be connected to a 100-240 Volt AC (VAC), 47-63 Hertz (Hz) outlet. If this equipment is connected to an insufficient power source, it will cause irreparable damage to this device's electronics.

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⚠ NOTICE/AVIS

The power cord may be used to simultaneously isolate both poles of the power supply by removing the power cord at the equipment input or wall receptacle.

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Contraindications

None.

Transportation and Storage Conditions

	Ambient Temperature	-10°C to +50°C (+14°F to +122°F)
	Relative Humidity	20-90% (non-condensing)
	Atmospheric Pressure	700 hPa to 1060 hPa (0.69 atm to 1.05 atm)

Disposal of Equipment

At the end of product life, the equipment, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

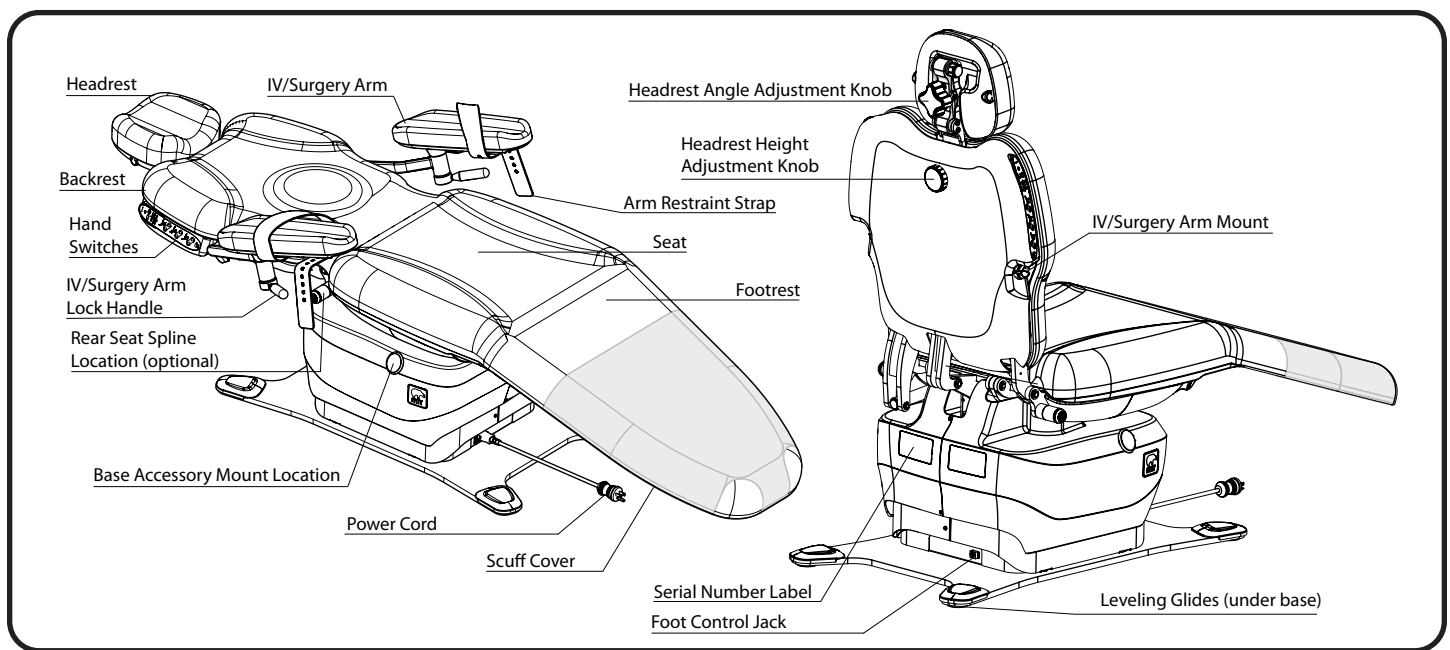
EQUIPMENT FEATURES

Description of 722/729 Chair


The 722 Oral/Dental Surgery Chair has power lift, back, and tilt functions. The 722 Surgery Chair has a 725 lb. (328.8 kg) patient capacity. The 722 Surgery Chair is 22" (55.9 cm) wide with an entry height of 18 3/4" (47.6 cm). All 722 Surgery Chairs are Standard Base models. The 722 Surgery Chair will rise to a height of 39" (99 cm).

The 729 Oral/Dental Surgery Chair has power lift, back, tilt, foot extension, and footrest functions. The 729 Oral/Dental Surgery Chair has a 725 lb (328.8 kg) patient capacity. The 729 Oral/Dental Surgery Chair is 22" (55.9 cm) wide with an entry height of 18 3/4" (47.6 cm). All 729 Surgery Chairs are Standard Base models. The 729 Surgery Chair will rise to a height of 39" (99 cm).

722 ORAL/DENTAL SURGERY CHAIR

729 ORAL/DENTAL SURGERY CHAIR

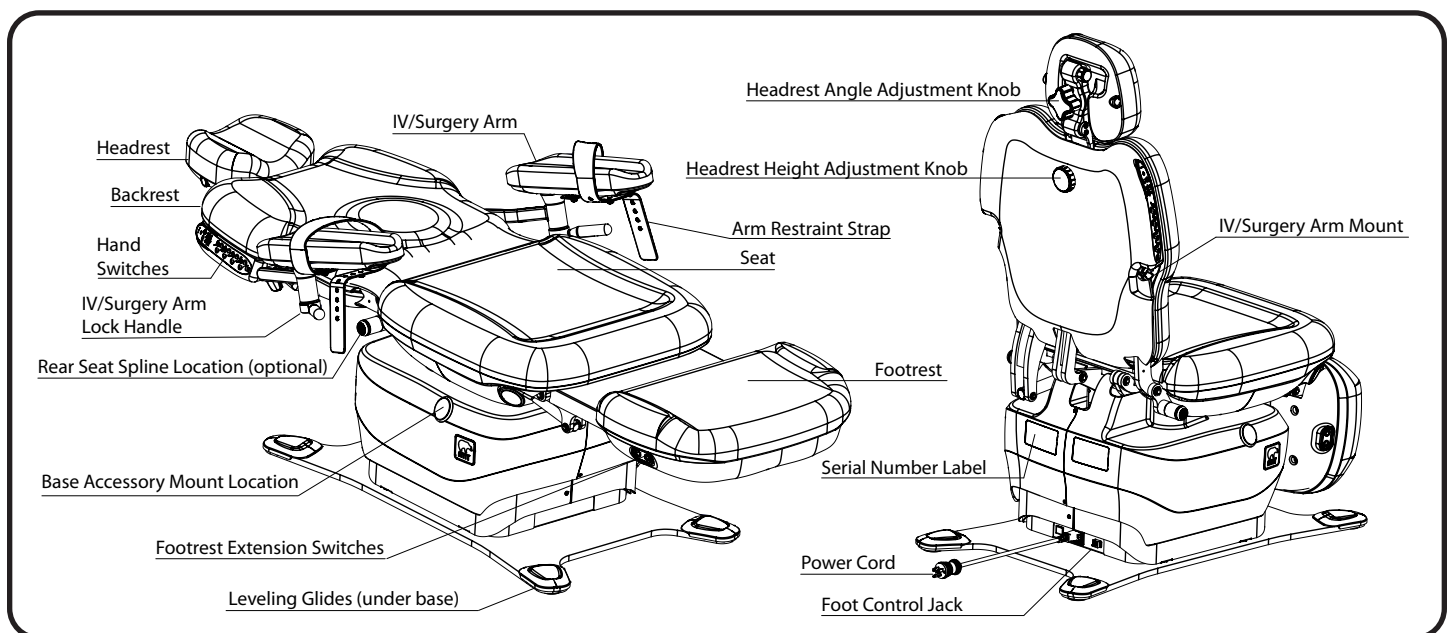INSTALLATION

WARNING/AVERTISSEMENT

This equipment is heavy. Get help to remove it from the shipping pallet. Use proper lifting procedures when lifting to avoid serious back injury.

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WARNING/AVERTISSEMENT

After equipment is installed, verify programmable positions (Home/P1-P4) are safe and known positions.

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CAUTION/PRÉCAUTION

To move equipment, lift only from the equipment's main structure.

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NOTICE/AVIS

If this product has been exposed to low temperatures, allow adequate time for it to warm to room temperature before putting it into service.

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NOTICE/AVIS

Inspect the shipping boxes for visible damage. Record on bill of lading, photograph, report any damage to the carrier immediately, and contact MTI Technical Support. Request an inspection by the carrier if any concealed damage is noted.

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
Required Tools

	Adjustable Wrench		Utility Knife		Flat Head Screw Driver		7/16" Wrench
	Set of Standard US Allen Wrenches (for Accessories)		Phillips Screwdriver				

Explanation of Instructions

Instructions have steps which are designated by a number. Each step may also have sub-steps designated by a letter. For example, an instruction to fasten an accessory may have 4 steps (1-4) and each of those steps may have sub-steps (A-B) to be performed before moving onto the next step.

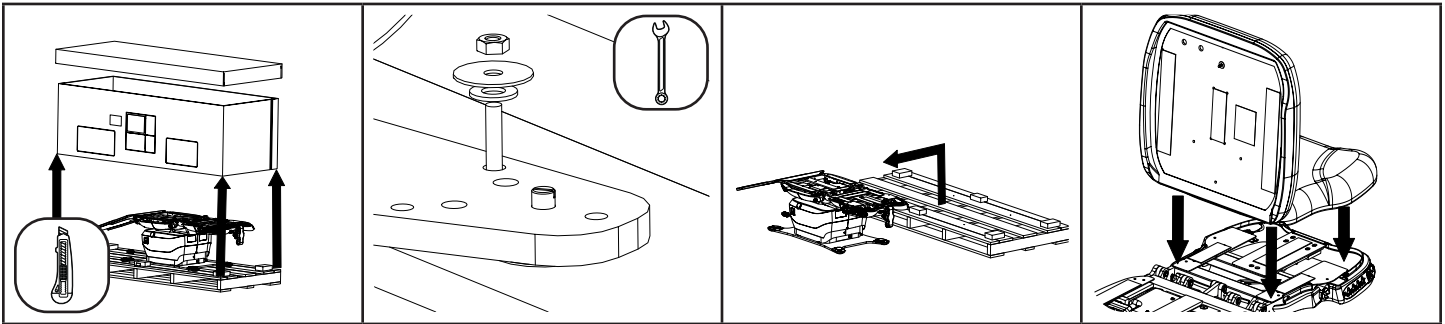Unpacking and Set-up


1. Remove shipping straps, shipping box, and other boxes from the pallet.

2. Remove (4) shipping bolts with an adjustable or 7/16" end wrench.

3. Slide the chair off the pallet. To move equipment, lift only from the equipment's main structure. Recycle packaging.

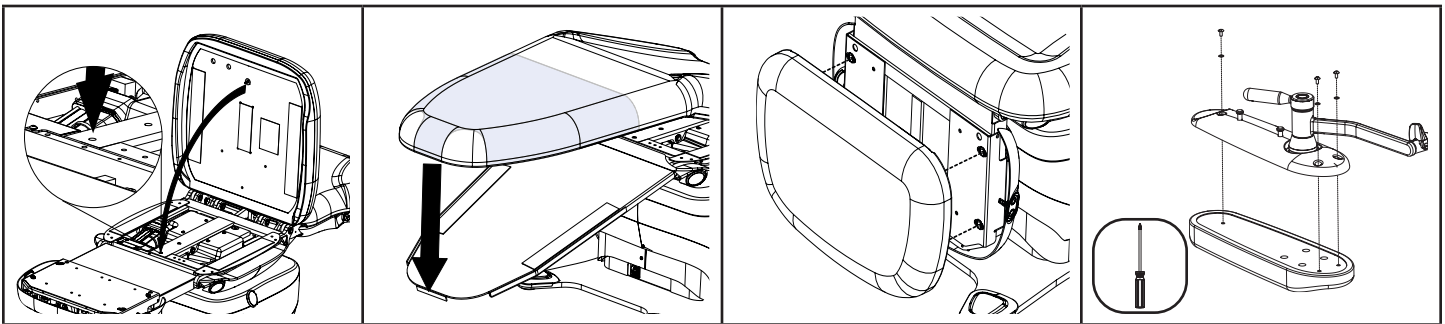4. With the back and seat sections of the chair in the flat position, center the back portion of the seat/back cushion on the back section of the chair while holding up the seat portion of the cushion.


5. **729 Model Only:** Lower the seat portion of the seat/back cushion into place. Ensure that the ball stud enters the mating hole in the seat section of the chair.

6. **722 Model Only:** Lower the seat and footrest portion of the seat/back/footrest cushion centering it on the footrest with the lower lip of the cushion no more than 1/4" away from the metal frame of the footrest.

7. **729 Model Only:** Attach the foot cushion by aligning the ball studs to the receptacles and pressing evenly to snap all 4 ball studs in place at the same time.

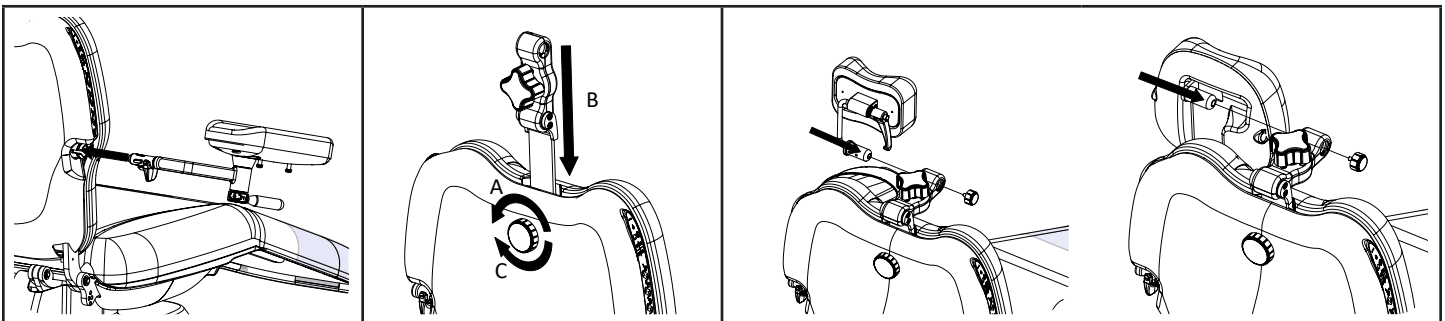
8. Using a Phillips screwdriver, attach the arm upholstery to each arm using the provided (3) screws and (3) washers.

9. To install the IV/Surgery Arms on the chair see "Armrest Operation" on page 19. Verify that the arm being installed is the correct arm for each side of the chair.

10. A. Loosen the knob on the back of the back section of the chair.
 B. Insert the headrest lock mechanism into the slotted opening in the top of the backrest.
 C. Tighten the knob on the backrest.

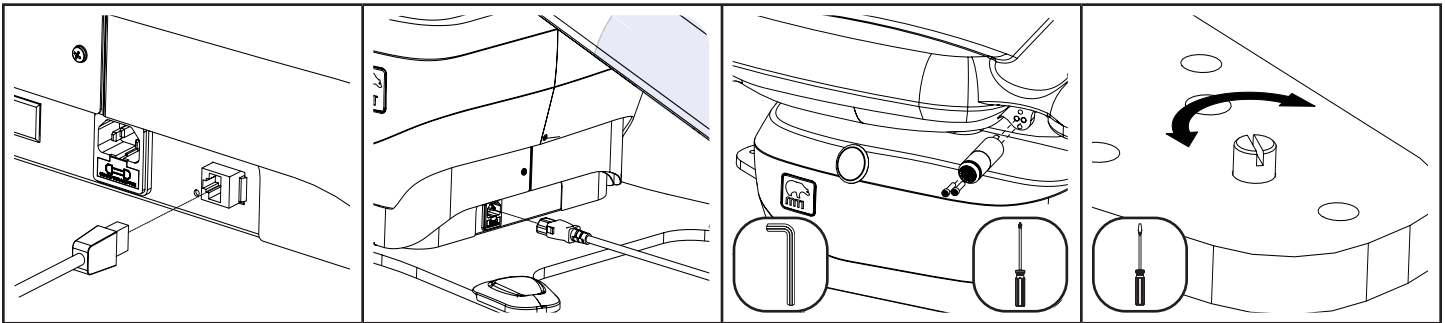11. To install the headrest in the headrest lock mechanism see "Headrest Operation" on page 18.


12. Plug in the foot control at the base of the chair. The foot control jack is located at the rear of the chair.

13. Insert the power cord at the base of the chair. The power cord is connected at the front of the 722 chair and at the rear of the 729 chair. Press firmly until it bottoms out. Plug the opposite end of the power cord into an appropriately marked receptacle.

14. If applicable, attach any accessories. See the accessory's Instruction Guide for installation instructions.

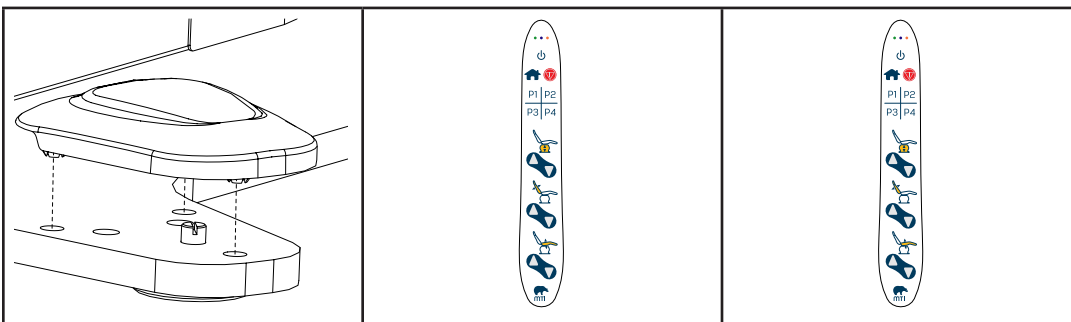15. Position the chair in the desired location. Level the chair by adjusting the height of the individual leveling glides with a flat head screwdriver.

16. Install the (4) leveling glide covers over the feet of the baseplate. Align the snap features over the three holes and press down.

17. Verify all functions of the chair are operational. Set the Smart Controller™ settings for the Home and Program Buttons. See "Smart Controller™ Technology" on page 14 for instructions.

18. Save Home and Program Buttons' positions to desired locations. See "Controls" on page 16 for programming instructions.

OPERATION

Operating Conditions

	Ambient Temperature	+10°C to +40°C (+50°F to +104°F)
	Relative Humidity	30% to 75% (non-condensing)
	Atmospheric Pressure	700 hPa to 1060 hPa (0.69 atm to 1.05 atm)

Maximum surface temperature of applied part chair surfaces will not exceed 42°C.

SmartTech™

SmartTech™ is the combination of MTI’s Smart Controller™ and Smart Safety™ technology using a centralized computer controller that enhances safety, reduces energy consumption, and provides smoother movement, additional memory options, and more user preferences.

Smart Controller™ Technology

The Smart Controller™ technology is the stack of technology features built into the chair, allowing the chair to move softly and smoothly with soft start/stop technology, a hibernation state to conserve power and a User Program Mode allowing users to change how some of the buttons function.

ComfortSync™

The 722 and 729 chair’s ComfortSync™ Arm system is the combination of features that increases caregiver access to the patient and a compensating arm system that positions the patient’s arms as the backrest is lowered and raised to reduce the need to reposition the patient’s arms during procedures.

User Program Mode

The User Program Mode allows users to set how the Home and Program Buttons function. For user preference these button settings can be changed by entering the User Program Mode to make the setting changes described below.

Home Button and Program Buttons Settings	
Momentary Button Setting	This setting is the factory default setting for the Home Button and Program Buttons. It allows the user to press and release the Home Button or Program Buttons to automatically move the chair to the button’s saved position. Press any button, except the same button that was pressed to initiate movement, to stop automatic motion.
Maintain Button Setting	This setting allows the user to press and hold the Home Button or Program Buttons to move the chair to the button’s saved position. Release the button to stop movement of the chair.

To enter the Smart Controller™ User Program Mode:

1. Press and Hold the **Stop** button for 5 seconds. The chair will beep once and the blue and orange LEDs on the hand switch will flash twice. The Foot Control does not have LEDs, but there are corresponding lights on the chair PCA inside the chair.
2. Press the following sequence of buttons (within 30 seconds from the time the Stop Button is released) to enter the Smart Controller™ User Program Mode: **Back Up, Back Up, Back Down, P2, Stop**. Each button press will beep and, if the correct sequence is entered, the chair will sound a low-high tune. If the wrong sequence of buttons is entered, the chair will sound three low beeps and return to normal operation.
3. Within a subsequent 30 seconds, press the following sequences of buttons to change the Home Button and Program Buttons Settings.

Home Button and Program Buttons Settings	
Momentary Button Setting	Back Up, Back Up, P1
Maintain Button Setting	Back Up, Back Up, P2

4. The chair will sound a high-low-high tune to accept the setting and the blue LED will flash twice. If the wrong buttons are pressed, the chair will sound three low beeps and the blue LED will not flash.
5. To exit the Smart Controller™ User Program Mode, press the Home Button. The chair will sound a high-low tune and only the green light will be on. The chair will also automatically exit the Smart Controller™ User Program Mode after 30 seconds if no buttons are pressed.
6. To confirm if the chair is in the desired setting (Momentary Button Setting or Maintain Button Setting), use the manual buttons

(Lift Up or Tilt Up) to move the chair to a position away from the Home Position. Press and release the Home Button. If the chair moves automatically towards the Home Position, the chair is in the Momentary Button Setting. If the chair stops moving as soon as the Home Button is released, the chair is in the Maintain Button Setting.

Hibernation State

To conserve power, after 60 minutes of non-use the chair will automatically hibernate. To wake the chair from the Hibernation State, press any button.

Smart Safety™ Technology

The Smart Safety™ technology is the safety features built into the Smart Controller™, making the chair safer and easier to use. The Smart Safety™ technology includes the Collision Avoidance System™, which establishes Safe Zones that will disable a function(s) when the chair encounters a Safe Zone boundary. When the Collision Avoidance System™ detects a Safe Zone boundary, the chair will sound a single tone and disable the chair function(s). To re-enable full chair functionality, move the chair function into the Safe Zone.

Safety Lockout

The Safety Lockout disables all commands to prevent unwanted or inadvertent movements.

To enable the Safety Lockout, press the following sequence of buttons (within 1 second between presses): **Stop, Stop, Back Up, Stop.**

To disable the Safety Lockout, press the following sequence of buttons: **Stop, Stop, Back Down, Stop.**

Shutdown

To shutdown the 722 or 729 Surgery Chair, press the On/Off Button on the hand switches on either side of the back section of the chair or unplug the chair from the outlet or disconnect the power cord from the chair. See “Power Cord Removal Instructions” on page 23.

Controls

ITEM	DESCRIPTION	FUNCTION
FOOT CONTROLS AND HAND SWITCHES		
1	Lift Up/Down Buttons	Raises or lowers the chair top by pressing the desired direction.
2	Back Up/Down Buttons	Raises or lowers the backrest by pressing the desired direction.
3	Tilt Up/Down Buttons	Decreases or increases the angle of the seat by pressing the desired direction.
4	Foot Up/Down Buttons (729 Model only)	Raises or lowers the footrest by pressing the desired direction. The footrest will automatically extend while pressing Foot Up and, when held, will continue to extend the foot fully. The Foot Down Button will automatically retract the foot as it approaches the floor.
5	Stop Button (Emergency Stop Button)	Stops all movement of chair.
6	Swivel Button	This button is non-functional in the 722 and 729 Surgery Chairs.
7	Home Button To determine if the chair is in the Momentary Button Setting or the Maintain Button Setting see "User Program Mode" on page 14.	<p>The Home Button moves the chair to either a Programmed Home Position or an ADA Compliant Home Position. The Programmed Home Position is user programmable and can be saved in any preferred position.</p> <p>Momentary Button Setting (Default): Press and <u>release</u> the Home Button to move the chair to the Programmed Home Position. Press and release the Home Button twice within 1 second to move the chair to the ADA Compliant Home Position (Lift Down, Tilt Down, and Back Up (and Foot Down and Retracted on 729 Model)). To save a new Programmed Home Position, position the chair to the desired position then press and hold the Home Button for 10 seconds. Chair will play a high-low-high tune after the Programmed Home Position has been saved.</p> <p>Maintain Button Setting: Press and <u>hold</u> the Home Button to move the chair to the Programmed Home Position. After arriving at Programmed Home, release the Home Button and then press it again within 1 second to move the chair to the ADA Compliant Home position (Lift Down, Tilt Down, and Back Up (and Foot Down and Retracted on 729 Model)). To save a new Programmed Home Position; position the chair to the desired position, press and release the Stop Button twice, then hold the Home Button for 10 seconds. Chair will play a high-low-high tune after the Programmed Home Position has been saved.</p>
8	Program Buttons (P1, P2, P3, P4) To determine if the chair is in the Momentary Button Setting or the Maintain Button Setting see "User Program Mode" on page 14.	<p>The Program Buttons move the chair to a user defined Program Position. NOTE: The chair has no pre-existing Program Positions upon arrival and will only beep and not move until Program Positions have been saved.</p> <p>The Program Buttons have two button settings that can be changed by the user.</p> <p>Momentary Button Setting (Default): Press and <u>release</u> to automatically move the chair to the Program Position. To save a Program Position, position the chair to the desired position then press and hold a Program Button for 5 seconds. Chair will play a high-low-high tune after the position has been saved.</p> <p>Maintain Button Setting: Press and <u>hold</u> the button to move the chair to the Program Position. To save a Program Position; position the chair to the desired position, press and release the Stop Button twice, then hold the Program Button for 5 seconds. Chair will play a high-low-high tune after the position has been saved.</p>
9	On/Off Button	Turns the chair on and off (Prevents accidental actuation of the chair).
10	LED Indicators	Used to provide user feedback on conditions, inputs, and errors. Left to right are green, blue, and orange LEDs. See "GUIDE TO INDICATOR LIGHTS AND SOUNDS" on page 27 for details.
11	Foot Extend/Retract Buttons (729 Only)	Independently extends or retracts the foot section of the chair by pressing the desired direction. NOTE: Footrest will not extend until foot has been raised at least 20° and moved away from Safe Zone boundary. When foot is raised, footrest retraction is limited to prevent interference with seat.

! NOTICE/AVIS

For additional safety, any button will stop all automatic motions except for the same button that was pressed to initiate movement.

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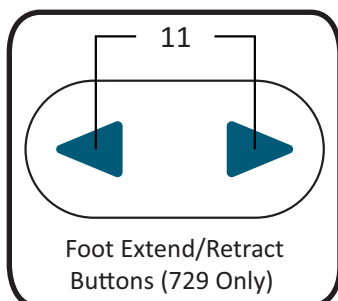
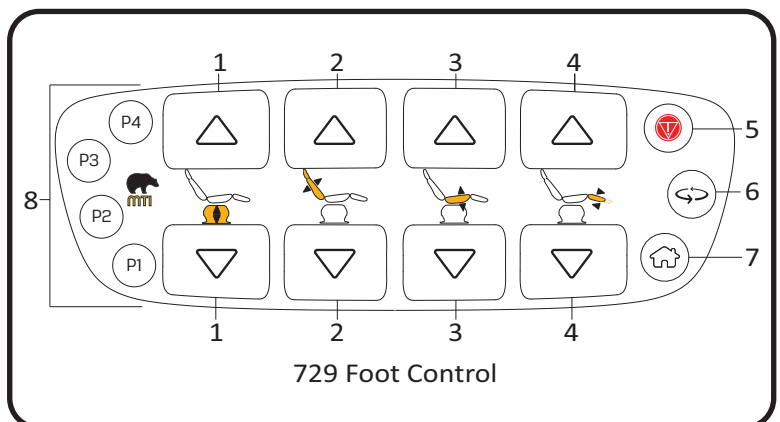
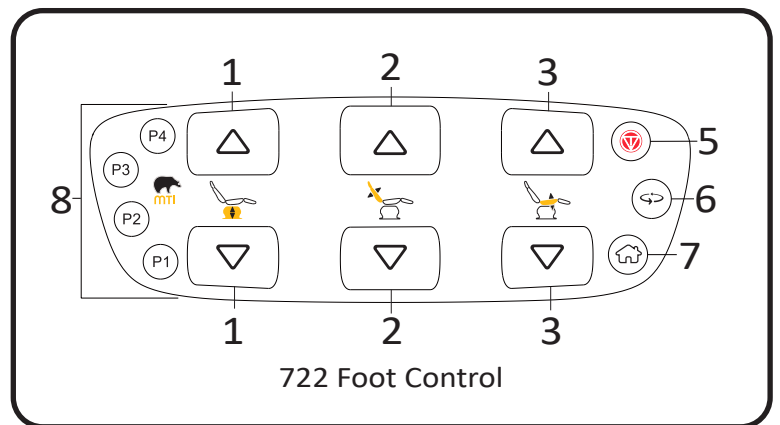
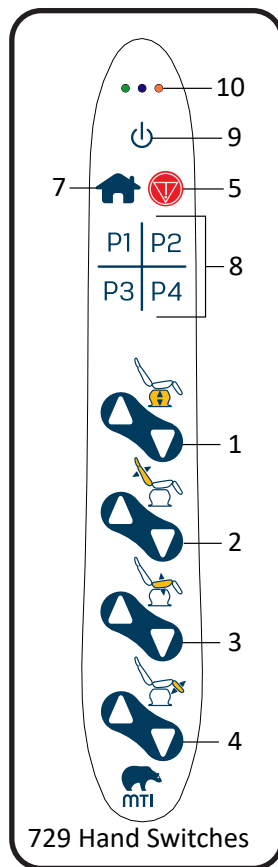
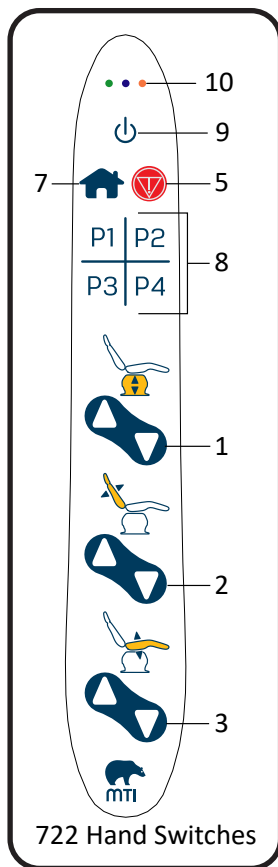
! WARNING/AVERTISSEMENT


Do not press combinations of three or more hand switch buttons at one time. Failure to comply may result in involuntary movement

and may result in personal injury. If involuntary movement occurs, press the Stop button immediately and contact MTI Technical Support Department.

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Headrest Operation

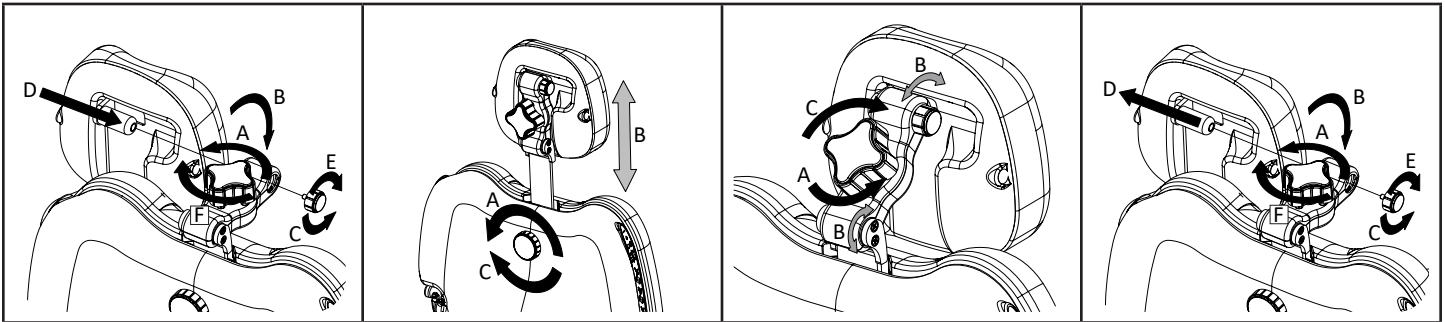1. To attach a headrest.

- A. Loosen the knob on the articulating headrest lock mechanism.
- B. Rotate the articulating headrest lock mechanism forward.
- C. Remove the small knob on the headrest pintle (turn counterclockwise). For the prosthetic headrest see Step 5 for more information.
- D. Insert the headrest pintle into the articulating headrest lock mechanism.
- E. Reattach the small knob into the end of the headrest pintle (turn clockwise).
- F. Reposition the headrest to a desired location and tighten the knob on the articulating headrest mechanism.

2. To adjust the height of the headrest:

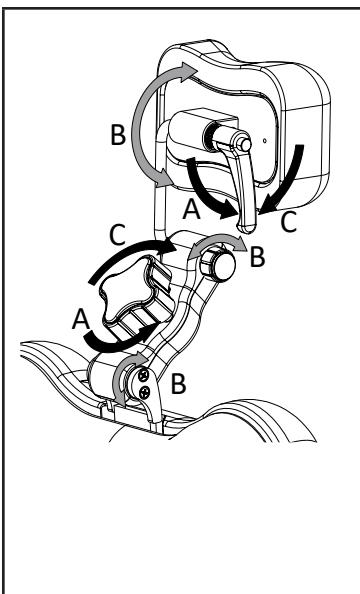
- A. To loosen, turn the knob on the back of the chair counterclockwise.
 - B. Adjust the height of the headrest.
 - C. To secure, turn the knob clockwise.
- The headrest can be completely removed, if desired, when the knob is loosened.

3. To adjust the angle of the deep oval headrest:

- A. To loosen, turn the knob on the articulating headrest lock mechanism counterclockwise.
- B. Adjust the position and angle of the headrest.
- C. To secure, turn the knob clockwise.


4. To remove a headrest:

- A. Loosen the knob on the articulating headrest lock mechanism.
- B. Rotate the articulating headrest lock mechanism forward.
- C. Remove the small knob on the headrest pintle (turn counterclockwise).
- D. Pull the headrest completely out of the headrest lock mechanism.
- E. Reattach the small knob to the end of the headrest pintle to keep it from being lost (turn clockwise).

5. To adjust the angle of the prosthetic headrest:

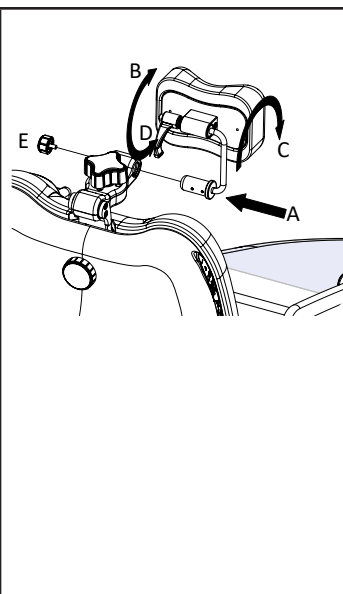- A. To loosen, turn the knob on the articulating headrest lock mechanism counterclockwise and/or loosen the handle behind the headrest cushion.
 - B. Adjust the position and angle of the headrest.
 - C. To secure, turn the knob and/or handle clockwise.
- NOTE:** To reposition the handle on the back of the headrest, pull on the handle while pressing on button, rotate and release.


6. The prosthetic headrest can be positioned with the handle on either side of the headrest.

- A. Follow instructions in Step 1A through 1D, but insert the headrest from the right.
 - B. Turn the handle on the back of the headrest cushion counterclockwise.
 - C. Reorient the cushion to face forward.
 - D. Turn the handle clockwise.
 - E. Reattach the small knob into the end of the headrest pintle (turn clockwise).
- To reposition the handle, see note in step 5.**

Armrest Operation

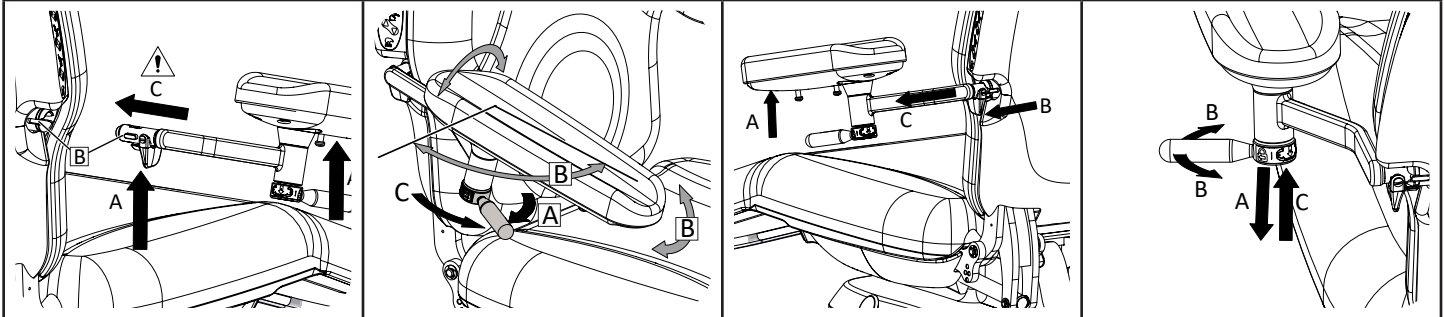IV/Surgery Arms

1. To attach the arm:

Verify the arm is the correct arm for the side of the chair. The arm should always mount facing forward. Images above for Steps 1 and 2 show the right arm. Images for Steps 3 and 4 show the left arm.

A. With one hand, hold the metal end of the arm with fingers on the locking clip. With the other hand, support the weight of the arm.

B. Orient the arm to match the key with the slots in the IV/Surgery Arm Mount.

C. Insert metal end of the arm into the IV/Surgery Arm Mount.

2. To adjust the arm:

A. While holding the front of the arm, rotate the arm lock handle away from the front and center of the chair.

B. Position the arm in the new desired position.

C. Tighten the arm lock handle by turning it towards the front and center of the chair.

! *The arm does not rotate more than 5 degrees inward and 90 degrees away from the chair.*

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3. To remove the arm:

A. With one hand, hold the metal end of the arm with fingers on the locking clip. With the other hand, support the weight of the arm.

B. Pull back on the lever at the metal end of the arm.

C. Pull the arm straight out.

4. To reposition the arm lock handle:

A. Pull down on the hub portion of the handle.

B. While pulling down on the handle, turn the handle to desired location.

C. Release the handle.

! WARNING/AVERTISSEMENT

Confirm that no part of the patient is under the IV/Surgery arm while operating the chair.

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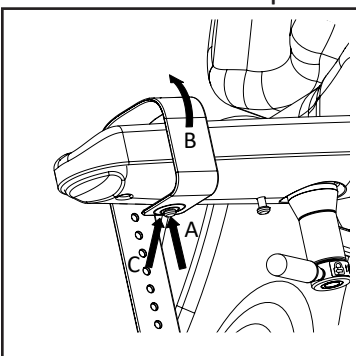
! WARNING/AVERTISSEMENT

Confirm that the locking clip snaps into the groove on the IV/Surgery Arm Mount to prevent the arm from coming loose.

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Arm Restraint Strap


1. To attach the arm restraint strap:

A. While facing the side of the chair, install the single-hole end of the strap on either strap mounting stud.

B. Wrap the strap up over the IV/Surgery Arm and over the patient's arm, moving toward the patient.

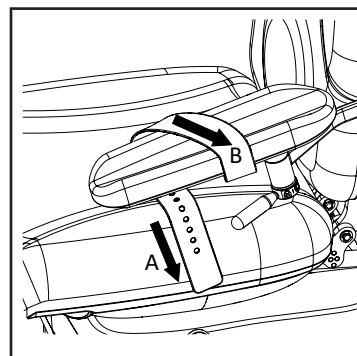C. Attach the multi-hole end of the strap to either strap mounting stud. Both ends of strap can be secured on the same stud if desired.

2. To remove the arm restraint strap:

A. Grab the multi-hole end of the strap and pull the strap off the stud.

B. Unwrap the strap from the patient's arm.

C. The strap can remain attached between patients or removed completely as desired.

LIST OF ACCESSORIES

DESCRIPTION	USE	RESTRICTIONS
Accessory Mount, Base	Provides a base mount for Mayo Trays, IV Rods, and Lights.	
Bed Rail & Transfer Support, ADA Compliant	Provides a support for entry and exit into the chair. Also used as flip-up bed rail to secure a patient.	Cannot be inserted while an IV/Surgery Arm or Transfer Support, ADA Compliant w/ Adapter is installed on the same side of the chair. Requires Spline Assy, A Size (Set of 2) mounted in Rear Seat Spline Location.
Caster Feet, Mobile, Lkg (set of 4)	Provides field installable casters to make chair mobile.	Not available on 729. Not compatible with 2" Leveling Glides.
Casters, Lkg Chair (set of 4)	Provides field installable casters to make chair mobile.	Not available on 722. Not compatible with 2" Leveling Glides.
Deep Oval and Prosthetic Headrests	Provides an articulating surface with various pillow shapes/sizes to support patient's head.	Headrest model included with chair is determined by selected upholstery kit.
Exam Light, Lumerus LED, Flex/Rigid Neck, 12-24V DC	Provides lighting during patient procedures.	Requires Mayo Tray w/Light Adptr or Post Assy w/Light Adptr.
Foot Control Slant Mount Assy, Std Base	Provides a slanted surface to mount a foot control at the back of the chair.	Not compatible with Caster Feet or Casters.
Foot Control, 3 Func (L,B,T)	Provides a foot control to manipulate the position of the chair.	Available only with 722 Model Chair.
Foot Control, 4 Func (L,B,T,F)	Provides a foot control to manipulate the position of the chair.	Available only with 729 Model Chair.
Forearm Support f/Deep Oval Headrest Assy (set of 2)	Provides a forearm support during chinning procedures.	Requires Deep Oval Headrest.
Forearm Support f/Prosthetic Headrest Assy (set of 2)	Provides a forearm support during chinning procedures.	Requires Prosthetic Headrest.
Hand Control Kit, Corded	Provides a corded hand control to manipulate the position of the chair.	Hand and Foot Controls cannot be connected to the chair simultaneously.
IV Rod Assy, Adj	Provides an adjustable IV rod.	Requires Accessory Mount, Base.
IV Rod Assy, Adj, Post Mtd	Provides an adjustable IV rod mounted to the post on a base mounted accessory.	Requires Mayo Tray Assy or Mayo Tray with Light Adptr Assy.
Leveling Glide Assy, 2" (set of 4)	Provides taller leveling glides than standard.	Not compatible with Caster Feet or Casters, Lkg Chair (set of 4).
Mayo Tray Assy, Base Mount Mtd	Provides a tray for the placement of small instruments and other items.	Requires Accessory Mount, Base.
Mayo Tray w/Light Adptr Assy, Base Mount Mtd	Provides a tray for the placement of small instruments and other items and a mount for an Exam Light, Lumerus LED, Flex/Rigid Neck	Requires Accessory Mount, Base.
Outlet Kit, Duplex, 115V ONLY	Provides power for 115V accessories.	
Post Assy w/Light Adptr, Base Mount	Provides a post for mounting an Exam Light, Lumerus LED, Flex/Rigid Neck, 100-240V.	Requires an Accessory Mount, Base.

LIST OF ACCESSORIES

DESCRIPTION	USE	RESTRICTIONS
Power Cord, IEC w/Hospital Grade 90 deg Plug, 3 ft	Provides a 3' power cord with a 90 degree plug.	
Spline Assy, A Size (set of 2)	Provides 2 splines to mount Transfer Support/Bed Rail, Flip-up Ambidextrous to the rear of the chair.	
Strap, Body Restraint	Provides a strap to secure the patient's body.	
Strap, Leg Restraint	Provides a strap to secure the patient's legs.	
Transfer Support, ADA Compliant w/ Adapter	Used as a support to assist the patient in transferring to the transfer surface.	Cannot be inserted while an IV/Surgery Arm or Bed Rail & Transfer Support, ADA Compliant is installed on the same side of the chair.

MAINTENANCE

WARNING/AVERTISSEMENT

There are no user serviceable parts inside. Refer all servicing to a qualified service technician.

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WARNING/AVERTISSEMENT

After equipment is serviced, verify programmable positions (Home/P1-P4) are safe and known positions.

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WARNING/AVERTISSEMENT

The Control of Hazardous Energy (lockout/tagout) 29 CFR §1910.147 procedure must be followed to disable equipment, preventing the release of hazardous energy while performing servicing and maintenance activities.

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WARNING/AVERTISSEMENT

MTI recommends cleaning the equipment frequently. Follow the current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities or other local guidelines for disinfecting.

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CAUTION/PRÉCAUTION

This equipment may be damaged by solvents and dyes. Immediately remove any fluids spilled on the equipment.

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CAUTION/PRÉCAUTION

Contact with sharp or pointed objects can damage the upholstery. Do not place hard surface objects on the upholstery surfaces as temporary indentations or impressions in the upholstery cover will result. Prolonged contact with a hard or irregular shaped object

may make the impression semi-permanent.

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CAUTION/PRÉCAUTION

Any sprayed disinfectant or cleaning solution should be wiped up immediately after application and not let to stand. A small sample area should be tested before any large application of any cleaner/disinfectant.

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CAUTION/PRÉCAUTION

Avoid prolonged direct sunlight exposure to preclude decolorization.

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CAUTION/PRÉCAUTION

Replace damaged upholstery to prevent cross contamination.

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Disinfection

Follow the current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities or other local guidelines.

Upholstery

MTI understands that many customers prefer to use a pre-mixed disinfecting solution in a spray or wipe. It is MTI's recommendation not to use sprays because over time the overspray will cause damage to the surfaces if not immediately wiped up. See "MTI Suggested Disinfectants Policy" for more details on disinfecting solutions which have been tested on MTI's products. It is available for download at www.mti.net/DisinfectantsPolicy. In the U.S. & Canada, call (877) 875-4990, or International call (801) 875-4990, or email technical.support@mti.net to request a copy.

Cleaning

Upholstery should be cleaned weekly with a mild liquid soap and water mixture. Afterwards, wipe upholstery with clean damp cloth.

Painted and Plastic Surfaces

All painted metal parts have been electrostatically painted and the paint is baked on for durability. Weekly, those surfaces should be wiped down with a quality grade cleaner and a soft cloth or a damp cloth with mild soap.

Preventative Maintenance

Periodically inspect the following areas:

1. All fasteners and pins should be in place and tightened securely.
2. Power cord should be free of cuts or other visible damage.
3. All mechanical functions should operate properly.

Have an authorized service technician inspect your equipment once a year.

Power Cord Removal Instructions

1. Pull back on the red tabs of the power cord. See “Unpacking and Set-up” on page 12 for installation instructions.

Fuse Replacement Instructions

1. See Figure 1. Pull back on the red tabs of the power cord at the base of the chair to remove the power cord.
2. Using a flat head screwdriver, open the fuse drawer in the power cord jack.
3. Remove the burnt fuse(s).
4. Insert new fuse(s) into the fuse drawer with same size fuse (5mm x 20mm 4 Amp 250V Time-Delay Glass tube type) removed in step 3 (See Figure 1).
5. Reinsert the fuse drawer. Ensure it is completely inserted.
6. Reinsert the power cord into the base of the chair.


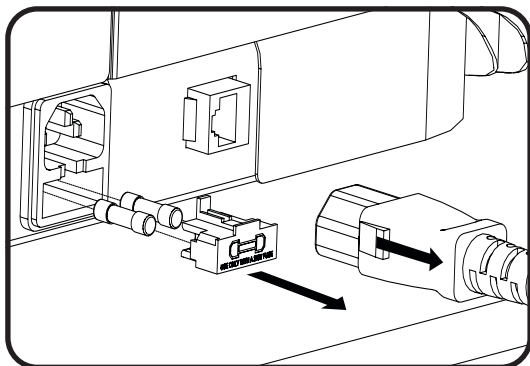
Figure 1

Scuff Cover Replacement Instructions (722 Only)

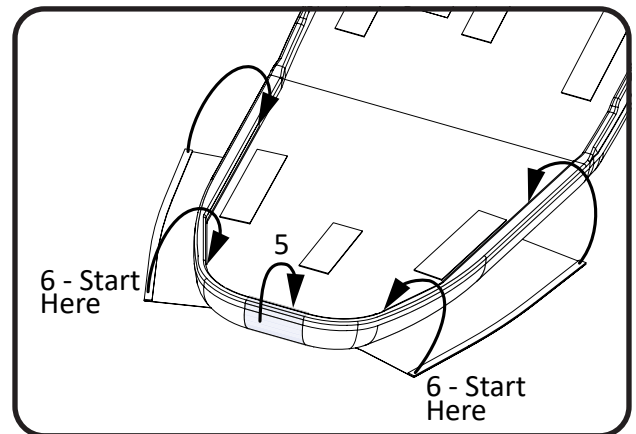1. Remove the back/seat/foot section cushion from the chair by pulling forwards on the back section of the cushion and then upwards on the seat and foot sections of the cushion with moderate force.
2. Place the cushion on a clean flat surface with the upholstery facing down. Remove the scuff cover from the foot cushion by pulling on all velcro tabs around the sides and end of the scuff cover.
3. Place the replacement scuff cover on a flat surface with the velcro facing up.
4. Place the back/seat/foot cushion on the scuff cover in the orientation shown in the image below. Remove any debris between the cushion and the scuff cover. Center the cushion left to right with the scuff cover.
5. Attach the velcro tab at the end of the foot section first (see image below). If needed, pull the scuff cover towards the seat to tighten the scuff cover at the foot end.
6. Starting near the bottom of the foot cushion, gradually fold over the left and right velcro tabs simultaneously, attaching the velcro to the inside walls of the cushion, working towards the seat cushion (see image below). Pulling on both sides keeps the scuff cover centered. Pull tightly to remove wrinkles and gaps. Pull up velcro and reapply as needed.

Calls for Service

Equipment’s model and serial number information are required when calling for service.

Serial number is indicated by the **SN** symbol located on the serial number tag.

If service is required, contact MTI.

U.S. & Canada (877) 875-4990

International (801) 875-4990

Fax (801) 952-0548

technical.support@mti.net

www.mti.net

LIMITED WARRANTY

Medical Technology Industries, Inc. (hereinafter referred to as MTI), shall repair or replace products of its manufacture, which prove to be defective in material and/or workmanship for a period of twelve (12) months from the date of shipment to the customer (the “Warranty Period”). MTI will, at its option, provide parts to the customer or repair the defective part at MTI’s factory or authorized repair facility. Except as it may otherwise specifically agree in writing, MTI shall not be liable for transportation or labor charges for repairs or adjustments or other work done on any of its products by MTI’s authorized dealers or service organizations.

Customer shall notify MTI of any alleged claim or defect within fourteen (14) days from the date customer discovers, or upon reasonable inspection should have discovered, such alleged claim or defect (but in any event before the expiration of the Warranty Period). All repair parts are warranted for Ninety (90) calendar days from the date of shipment to the customer. Warranty is non-cumulative.

This warranty does not apply to any part or product which upon examination by MTI or its authorized agents indicates the product has been repaired or altered in any way by persons other than MTI or its authorized agents, has been misused, abused or used in a manner contrary to any instructions issued by MTI. Any parts or products claimed under this warranty must be properly packaged by customer and returned to MTI’s factory with no liability to MTI for the parts, products or transportation charges thereon.

In no event, shall MTI have any liability for consequential, indirect, incidental, special, exemplary, punitive or enhanced damages, lost profits or diminution in value arising out of or relating to a product defect or the repair or replacement of any products manufactured or sold by it.

THE FOREGOING LIMITED WARRANTY IS MADE IN LIEU OF ALL WARRANTIES EXPRESSED OR IMPLIED, AND MTI MAKES NO OTHER WARRANTY WHATSOEVER WITH RESPECT TO THE PRODUCTS INCLUDING BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

TROUBLESHOOTING GUIDE

NOTICE/AVIS

See “GUIDE TO INDICATOR LIGHTS AND SOUNDS” on page 27 for additional solutions. For more information on this equipment, including circuit diagrams, parts and service manual (Item No. 130-4413), and other helpful information, please contact MTI.

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#	SYMPTOMS	PROBLEM CAUSE	SOLUTION
1	Chair will not function. Hand switch lights are off. No green light visible on power supply through back of chair.	Chair power cord is not securely connected.	Verify power cord is securely connected at both ends.
		Facility power or receptacle problem. Facility power circuit breaker problem.	Check facility’s power receptacle with a known working device of same voltage specifications. Also check facility circuit breaker to verify it is not tripped if no power to known working device.
		Facility supply voltage is incorrect voltage.	Verify voltage is per specifications on chair serial number/product tag.
		Fuses have blown.	Verify fuses are not burnt. Replace fuse(s) if necessary. See “Fuse Replacement Instructions” on page 23.
		Chair Power Supply is faulty.	Contact Technical Support.
2	Chair will not function. Hand switch lights are off. Green light visible on power supply through back of chair.	Chair is off	Press the “On” Button on a hand switch or unplug the chair for 30 seconds and plug it back in.
		Faulty hand switches	If possible, test a foot control from similar equipment. Otherwise, contact Technical Support.
		Chair PCA isn’t functioning correctly.	Contact Technical Support.
3	Lift, Back, Tilt, or Foot functions will not function.	An actuator is experiencing excessive loads.	Remove or reposition patient, move the chair in opposite direction, then intended direction to continue normal operation.
		An actuator is not functioning or Chair PCA isn’t functioning correctly.	Contact Technical Support.
4	Foot Extension will not extend and beeps. (729 Model only)	The footrest is at the safe zone boundary. The footrest is too close to the floor or is too close to vertical.	Change the angle of the footrest.
5	The chair consistently beeps when retracting the foot section.(729 Model only)	The footrest is at the safe zone boundary. The footrest is avoiding colliding with the seat cushion.	This is normal operation.
6	The chair beeps when pressing Lift Down or Tilt Down and stops moving. (729 Model only)	The footrest is too close to the floor.	Press the foot retract or foot up button to move it away from the floor.

TROUBLESHOOTING GUIDE

NOTICE/AVIS

See “GUIDE TO INDICATOR LIGHTS AND SOUNDS” on page 27 for additional solutions. For more information on this equipment, including circuit diagrams, parts and service manual (Item No. 130-4413), and other helpful information, please contact MTI.

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#	SYMPTOMS	PROBLEM CAUSE	SOLUTION
7	The chair will not function. The green light and orange light are lit on the hand switches or Chair PCA.	The Safety Lockout Feature is enabled.	To exit Safety Lockout press Stop, Stop, Back Down, Stop.
8	The chair does not move when a Program Button is pressed.	The Program Button Setting is set to the Maintain Button Setting.	See “Smart Controller™ Technology” on page 14 for instructions to change the Program Button Setting.
		The Program Button does not have a saved location.	See “Controls” on page 16 for instructions on saving a Program Button location.
9	Home Button does not move the chair to the expected location.	The saved location for the Home Button has been set by another user/ inadvertently.	Move the chair to the desired location. See “Controls” on page 16 for instructions on saving a Home Button location. For further instructions about the operation of the button, see “Smart Controller™ Technology” on page 14.
10	The chair consistently beeps while moving to a programmed position.	An actuator is malfunctioning.	Contact Technical Support.
11	Chair will not function or a function is erratic.	Chair PCA isn’t functioning correctly.	Contact Technical Support.
12	Chair PCA relays turn on/off quickly and excessively (“chatters”).	Chair PCA isn’t functioning correctly.	Contact Technical Support.
13	Lift function is slower than usual and only goes up about halfway. Chair beeps while moving.	One of the lifting column motors is not functioning or Chair PCA isn’t functioning correctly.	Contact Technical Support.
14	Motor is running after chair stops.	A motor gear has stripped.	Contact Technical Support for motor replacement.

GUIDE TO INDICATOR LIGHTS AND SOUNDS

#	LIGHTS						SOUND	STATUS OR CONDITION
	GREEN		BLUE		ORANGE			
1		Solid		Off		Off	Start Up Tune	Chair has been plugged in.
2		Slow Flash		Off		Off	None	Chair is in hibernation state to conserve power. To exit hibernation press any button.
3		Solid		Rapid Flash		Off	None	Chair is moving to the Programmed Home or a Program Position or manual button (Lift Up/Down, Tilt Up/Down, Back Up/Down, Foot Up/Down, or Foot In/Out) is being pressed.
4		Solid		Rapid Flash		Rapid Flash	None	Chair is moving to the ADA Compliant Home position.
5		Solid		Off		Off	High Beep when a Program Button is pressed	No Program Position is saved.
6		Solid		Flash Twice		Off	High-Low-High Tune	Program Position or Programmed Home Position has been saved.
7		Solid		Off		Mod. Flash for 10 sec.	High Beep during motion of specific actuator	Actuator/motor Problem (Contact Technical Support).
8		Solid		Off		Flash Twice	High Beep	Chair has been moved to a Safe Zone boundary. (ex. Excessive Trendelenburg or Footrest near Seat Cushion or floor). Move in opposite direction to resume full functionality.
9		Solid		Off		Flash Twice	High Beep	Excessive load on a motor. Remove or shift excess load and move in the opposite direction momentarily to resume.
								Actuator/motor reaches end of stroke prematurely. NOTE: If beeping occurs frequently, contact Technical Support.
10		Solid		Off		Rapid Flash	None	Swivel has been unlocked. NOTE: even though the 722 and 729 Chairs do not swivel, the orange LED will still flash when the Swivel button is pressed on the Foot Control. Press any button to turn it off.
11		Solid		Off		Solid	2 High Beeps (upon activation)	Safety Lockout feature is enabled. (To exit press Stop, Stop, Back Down, Stop) Orange LED will flash twice each time button is pressed while Safety Lockout feature is enabled.
12		Solid		Off		Slow Flash	High Beep with every button press	No actuators respond to motion commands. Chair ID Mismatch (Contact Technical Support).
13		Mod. Flash		Mod. Flash		Mod. Flash	High Beep with every button press	No actuators respond to motion commands. Chair ID Cable Invalid (Contact Technical Support).
14		Off		Off		Off	None	Loss of power (chair was unplugged).
15		Off		Off		Off	Shut Down Tune	Chair has been powered off. Press the On/Off Button on hand switch to resume operation.

GUIDE TO INDICATOR LIGHTS AND SOUNDS

#	LIGHTS						SOUND	STATUS OR CONDITION
	GREEN		BLUE		ORANGE			
16		Solid		Flash Twice		Flash Twice	Low Beep	Stop Button has been pressed for 5 seconds awaiting sequence to enter Smart Controller™ User Program Mode or Calibration Mode.
17		Solid		Flash once		Off	High Beep	High Beep with each button press until sequence has been entered for Smart Controller™ User Program Mode or Calibration Mode.
USER PROGRAM MODE								
18		Solid		Solid		Slow Flash	Low-High Tune then Low Beep every 10 sec.	Chair has entered Smart Controller™ User Program Mode. Exits automatically after 30 seconds of no activity.
19		Solid		Off		Slow Flash	High Beep each button press	Button press while in Smart Controller™ User Program Mode.
20		Solid		Flash Twice		Slow Flash	High-Low-High Tune	New setting accepted.
21		Solid		Solid		Slow Flash	Three Low Beeps	New setting not accepted.
22		Solid		Off		Off	High-Low Tune	Chair has exited Smart Controller™ User Program Mode.
CALIBRATION MODE								
23		Off		Off		Mod. Flash	High Beep when an individual motor button is pressed.	One or more actuators is out of calibration. Do not operate the chair. Contact Technical Support.
24		Slow Flash		Slow Flash		Slow Flash	Low-High Tune then Low Beep every 10 sec.	Chair is in Calibration Mode. Do not operate the chair. Contact Technical Support.

SPECIFICATIONS

DESCRIPTION	SPECIFICATION
Load Capacity ¹	
Lift	725 lb (328.9 kg)
Back	400 lb (181.4 kg)
Tilt	725 lb (328.9 kg)
Foot (footrest fully extended) (729 Model)	150 lb (68 kg)
Foot (footrest retracted)(729 Model)	200 lb (90.7 kg)
Footrest (722 Model)	200 lb (90.7 kg)
Armrest	100 lb (45.4 kg)
Weight of Chair ²	
722	303 lb (138 kg)
729	312 lb (142 kg)

SPECIFICATIONS

DESCRIPTION	SPECIFICATION
Shipping Weight of Chair ³ 722 729	362 lb (165 kg) 374 lb (170 kg)
Shipping Weight of Upholstery Kit 722 Seamless/Designer Upholstery Kit 729 Seamless/Designer Upholstery Kit	29 lb (13.2 kg) 26 lb (11.8 kg)
Table Top Range 722/729	18.75-39 in (47.6-99 cm)
Back Section Range	0° ± 2.2° (horizontal) to 85° ± 2.2°
Table Top Tilt Range	0° ± .6° to 30° ± .6° Range is limited when back section is at or lower than 20° below horizontal to avoid collision.
Foot Section Range 722 729	Fixed at 20 degrees to seat. 0° ± 2.2° (horizontal) to 90° ± 2.2°
Foot Extension Range (729 Model Only)	7 in (17.8 cm) Range is limited depending on Tilt angle, Lift, and footrest angle.
Foot Control	IPX6 Rated
General Dimensions (w/headrest & footrest fully extended) 722 729	22 in (55.9 cm) wide x 78.5 in (199.4 cm) long 22 in (55.9 cm) wide x 70.5 in (179.1 cm) long
Power Cord	Plugs: U.S.A. standard 3 prong and IEC 60320-C13 (self-retaining)
Electrical Ratings	100-240VAC (47-63 Hz); 7 Amps
Fuse Replacement	(2) 5mm x 20mm 4 Amp 250V Time-Delay Glass Tube Fuse
Duty Cycle	Intermittent Operation, 10% Duty Cycle Example: (1) minute on (9) minutes off
Classification	Class 1, Type B Applied Part, Ordinary Equipment, Intermittent Operation. Applied parts include: Back, Seat, Leg rest, and Footrest Upholstery, Arms, and Headrest.
Listing/Approvals	IEC 60601-1:2005 Ed.3+A1;C1:2014 AAMI ES60601-1:2005+A1 CSA C22.2#60601-1:2014 Ed.3 IEC 60601-1-2:2014 Ed.4

¹Load Capacity will vary depending on placement and distribution of load on the section of the chair.

²Weight may vary due to product variation.

³Weight may vary due to variation in weight of shipping materials.

In-service Checklist for G2 Products with SmartTech™

Model Number	Description	Serial Number(s)

The following in-service needs to be performed on this MTI product:

Set Smart Controller™ Settings

Enter the User Program Mode in the Smart Controller™ to change how the Program Buttons operate. See “User Program Mode” on page 14.

Button Setting Momentary Maintain

[Factory default] Momentary Button Setting: press and release to automatically move chair to the programmed position.

Maintain Button Setting: press and hold to move chair to the programmed position. Once button is released, chair will stop.

Saved Programmable Home
 Save by positioning chair to desired location. If buttons are set to Momentary Button Setting, press and hold the Home Button for 10 seconds. If buttons are set to Maintain Button Setting, press the Stop button twice and then press and hold the Home Button for 10 seconds. Once saved, audible and visual confirmation will be given.

Reviewed ADA Compliant Home
 ADA Compliant Home position is Lift Down, Tilt Down, and Back Up (and Foot Down and Retracted on 729 only). To access: if buttons are set to Momentary, press and release the Home Button twice (within 1 second). If buttons are set to Maintain, press and hold the Home Button to move to Programmable Home Position, then release and press and hold the Home Button again (within 1 second) until the chair stops moving.

Saved (4) User Program Buttons (P1-P4)
 Save by positioning chair to desired location. If buttons are set to Momentary Button Setting, press and hold the desired program button for 5 seconds. If buttons are set to Maintain Button Setting, press the Stop button twice and then press and hold the desired program button for 5 seconds. Once saved, audible and visual confirmation will be given.

Reviewed Safety Lockout
 Locks out all commands once this state has been enabled. To enter, press the following sequence of buttons: Stop, Stop, Back Up, Stop. Once enabled, audible and visual confirmation will be given. To exit, press the following sequence of buttons: Stop, Stop, Back Down, Stop. Once disabled, audible and visual confirmation will be given.

Reviewed Controls and Operation sections of the User Manual

If you would like to receive occasional emails regarding MTI promotions and product announcements, fill out the sales form at www.mti.net/contact-us.

THIS PAGE IS INTENTIONALLY BLANK

**Specialized Equipment
for Speciality Healthcare**
Medical Technology Industries

MTI, Inc.
3655 W Ninigret Drive
Salt Lake City, Utah, 84104-6572
800-924-4655 • call/text 801-875-4999 • Fax 801-952-0548
www.mti.net

© MTI All rights reserved. MTI reserves the right to make any product changes without notice. The MTI brand and Bear logo design are registered trademarks of MTI, Inc. in the United States.