



# Metrix® Platform for the Point-of-Care

A MOLECULAR LAB ANYWHERE AND EVERYWHERE


 aptitude  
**Metrix®**  
COVID-19 Test

Provided by  
**SEKISUI**  
DIAGNOSTICS



# METRIX®

## THE SWEET SPOT IN DIAGNOSTIC TESTING

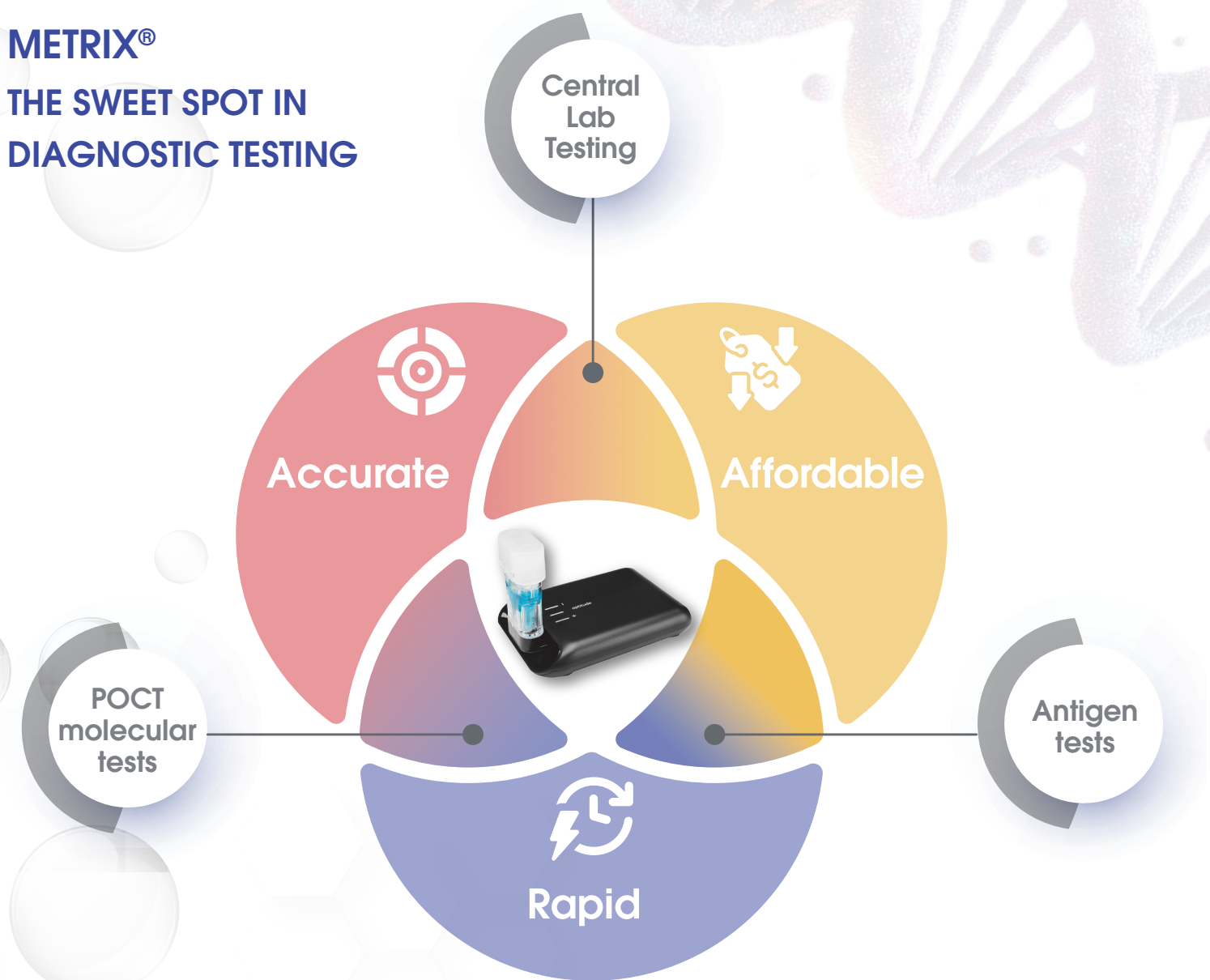


### Experience THE 3-S RULE to sample processing: SAMPLE, SNAP, SLIDE



#### 1. SAMPLE

Collect a nasal swab sample for the best accuracy.

Saliva can also be used by depositing into the collector.

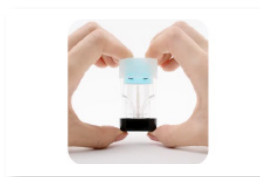

#### 2. SNAP

Insert the swab into the collector, snap off and discard the swab handle.

Snap the cap onto the collector and then onto the sensor.




#### 3. SLIDE

Slide the sensor onto the reader.  
Read result within 30 minutes or less.

This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA; This product has been authorized only for the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens. The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b) (1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb 3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

## GET THE RIGHT RESULTS THE FIRST TIME!

The Metrix® platform utilizes proven isothermal molecular technology in a sophisticated design, allowing you to provide an enhanced point-of-care testing experience for your patients.

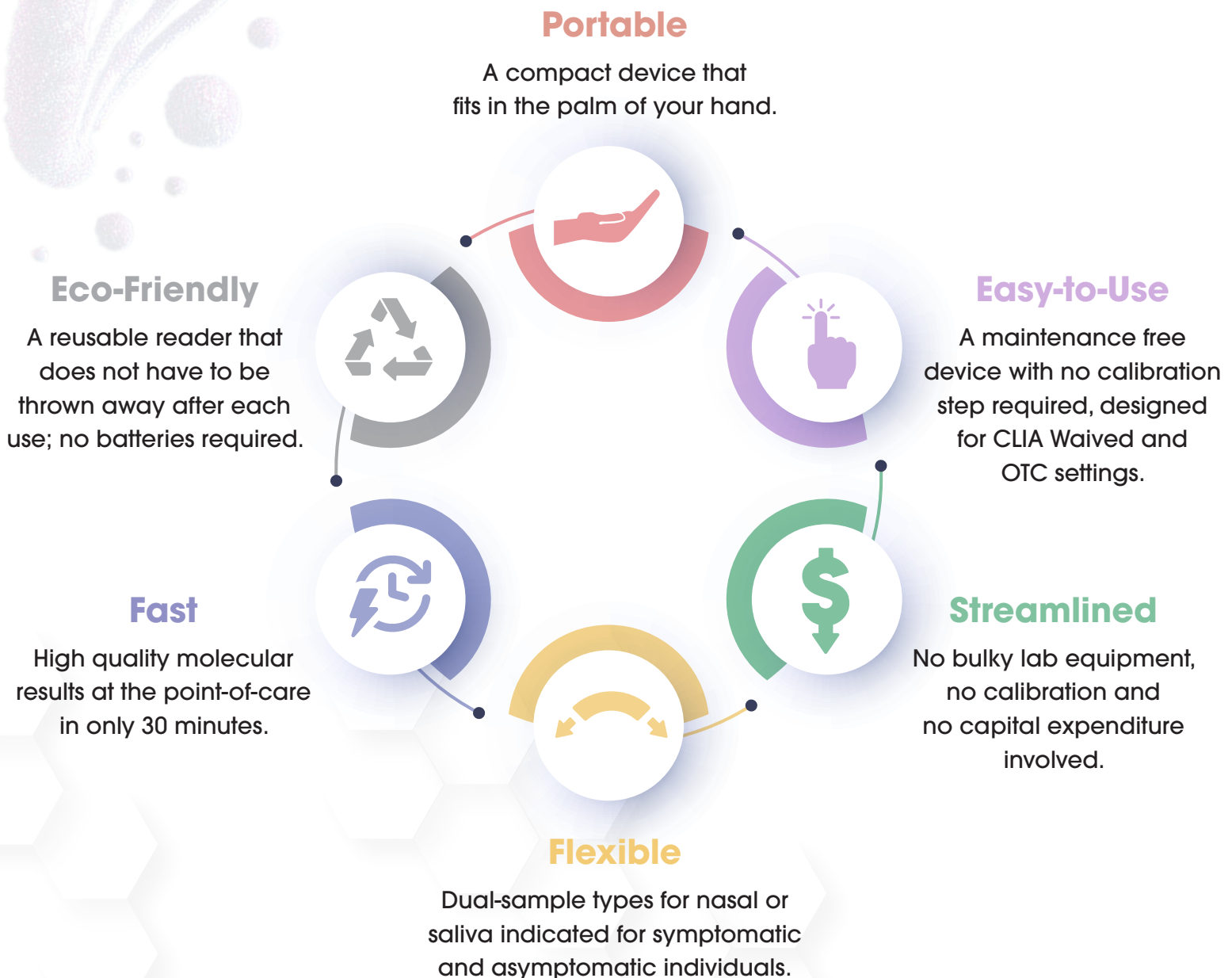

# ACCESSIBLE MOLECULAR TESTING AT THE POINT-OF-CARE



Ordering Information			
Test Description	Configuration	Catalog Number	CPT Code
Metrix Reader	1 Reader	MTRX-RDR	
Metrix COVID-19 Test	25 Tests	MTRX-C19-25PK	87635QW
Metrix Molecular Bundle	1 Reader + 50 Tests	MTRX-C19-B1	87635QW
Microbix COVID-19 Controls (for use with Metrix COVID-19 Test)	1 SARS-CoV-2 Positive Swab, 1 SARS-CoV-2 Negative Swab	RED-S-19/99-P-01	

To learn more about the Metrix COVID-19 Platform  
scan the QR code or go to [metrixmolecular.com](https://metrixmolecular.com)



**Provided by**  
SEKISUI Diagnostics, LLC  
One Wall Street  
Burlington, MA 01803  
Phone: 800 332 1042  
Email: [questions@sekisui-dx.com](mailto:questions@sekisui-dx.com)  
[sekisuidiagnostics.com](https://sekisuidiagnostics.com)

**Designed and manufactured by**  
Aptitude Medical Systems Inc.  
125 Cremona Drive, Suite 100  
Goleta, CA 93117

